

# Customization

The Complete Lounger (SL-6) Steel Lift Chair

IN JUST A FEW EASY STEPS, YOU CHANGE THE LIFT CHAIR IMAGE.



Clean hook & loop fasteners on steel frame



Attach seat box cover to front hook & loop strip



Attach hook & loop strip to seat box & pull back tight



Slip seat back over seat back frame



Pull seat back bottom flaps under bottom of seat



Attach seat back flap to hook & loop on seat rail



Install footrest side covers



Lay chaise pad over seat box



Attach chaise pad to footrest



Install arm foam over arm covers



Slide arms over metal arm rails & push tight



Push quick release control wire into side pocket



Pull fabric tight & attach all hook & loop fasteners



Plug hand controller into quick release cable

**Note:** The Complete Lounger arrives fully assembled with choice of fabric and back.



1-800-800-8586 (U.S.) 1-888-570-1113 (Canada)

[www.pridemobility.com](http://www.pridemobility.com)