
INSTALLATION AND CARE INSTRUCTIONS

Synergy[®] Spectrum Air[™]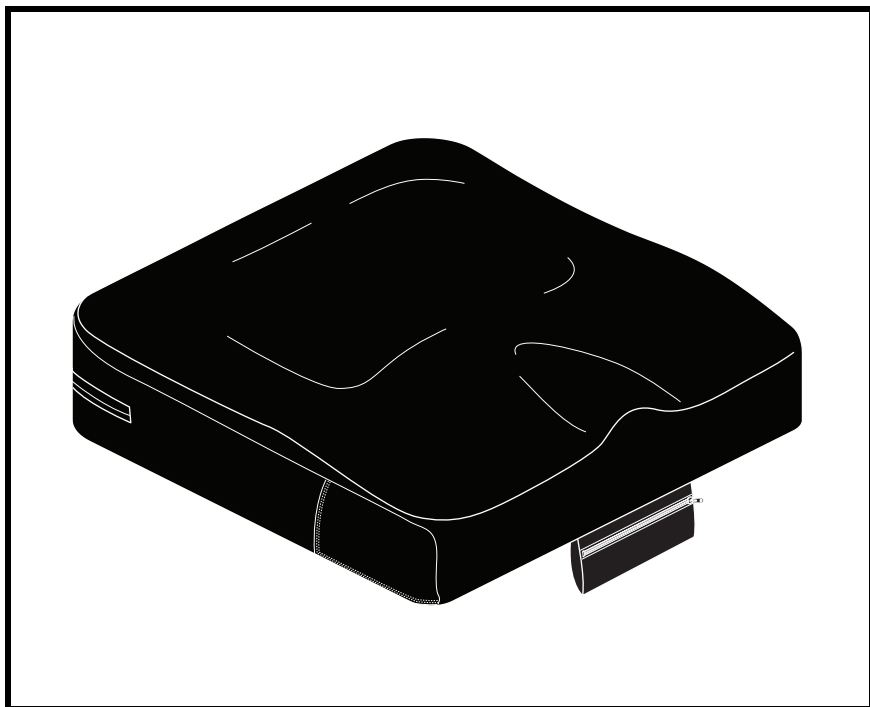

QUANTUM[®]
REHAB

A Division of Pride Mobility Products[®] Corp.

I N F M A N U 3 8 0 5

SAFETY GUIDELINES

WARNING! A Quantum Rehab Provider or a qualified technician must perform the initial setup of this product and must perform all of the instructions in this manual.

The symbols below are used throughout this owner's manual and on the product to identify warnings and important information. It is very important for you to read them and understand them completely.

WARNING! Indicates a potentially hazardous condition/situation. Failure to follow designated procedures can cause either personal injury, component damage, or malfunction. On the product, this icon is represented as a black symbol on a yellow triangle with a black border.


MANDATORY! These actions should be performed as specified. Failure to perform mandatory actions can cause personal injury and/or equipment damage. On the product, this icon is represented as a white symbol on a blue dot with a white border.

PROHIBITED! These actions are prohibited. These actions should not be performed at any time or in any circumstances. Performing a prohibited action can cause personal injury and/or equipment damage. On the product, this icon is represented as a black symbol with a red circle and a red slash.

NOTE: These instructions are compiled from the latest specifications and product information available at the time of publication. We reserve the right to make changes as they become necessary. Any changes to our products may cause slight variations between the illustrations and explanations in this manual and the product you have purchased. The latest/current version of this manual is available on our website.

NOTE: This product is compliant with WEEE, RoHS, and REACH directives and requirements.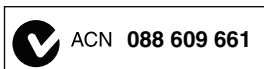

Copyright © 2010
Pride Mobility Products Corporation
INFMANU3805/Rev B/December 2010

TABLE OF CONTENTS

LABEL INFORMATION	4
INTRODUCTION	5
THE SYNERGY SPECTRUM AIR.....	7
INFLATING AND DEFLATING THE AIR INSERT.....	8
CLEANING	9
ASSEMBLING INNER CUSHION COMPONENTS	10
REPOSITIONING THE AIR HOSE.....	13
CARE AND MAINTENANCE	14
WARRANTY.....	15

LABEL INFORMATION

PRODUCT SAFETY SYMBOLS

The symbols below are used on the product to identify warnings, mandatory actions, and prohibited actions. It is very important for you to read and understand them completely.

Read and follow the information in the owner's manual.

Do not expose to heat sources such as open flame or sparks.

No smoking.

Disposal and recycling - Contact your Quantum Rehab Provider for information on proper disposal and recycling of your Pride product and its packaging.

Cover only — Maximum washing temperature 140°F (60°C)

Do not tumble dry.

Do not iron.

INTRODUCTION

WELCOME to Quantum Rehab, a division of Pride Mobility Products (Pride). The product you have purchased combines state-of-the-art components with **safety**, comfort, and styling in mind. We are confident that the design features will provide you with the conveniences you expect during your daily activities. Understanding how to **safely** use and care for this product should bring you years of trouble free operations and service.

Read and follow all instructions, warnings, and notes in this manual before attempting to operate your product for the first time. You must also read all instructions, warnings, and notes contained in any supplemental instructional booklets for the controller, front riggings, and/or seating system that accompanied your power chair before initial operation. Your **safety** depends upon you, as well as your provider, caretaker, or healthcare professional in using good judgement.

This manual is to be used in addition to the owner's manual that came with your chair. If there is any information in this manual which you do not understand, or if you require additional assistance for setup or operation, please contact your Quantum Rehab Provider. **Failure to follow the instructions, warnings, and notes in this manual and those located on your Pride product can result in personal injury and/or product damage and will void Pride's product warranty.**

PURCHASER'S AGREEMENT

By accepting delivery of this product, you promise that you will not change, alter, or modify this product or remove or render unusable or unsafe any guards, shields, or other safety features of this product; fail, refuse, or neglect to install any retrofit kits from time to time provided by Pride to enhance or preserve the safe use of this product.

INFORMATION EXCHANGE

We want to hear your questions, comments, and suggestions about this manual. We would also like to hear about the safety and reliability of your new Pride product, and about the service you received from your Quantum Rehab Provider. Please notify us of any change of address, so we can keep you apprised of important information about safety, new products, and new options that can increase your ability to use and enjoy your Pride product.

Contact Information

Pride Mobility Products Corporation
182 Susquehanna Avenue
Exeter, PA 18643-2694
USA

Pride Mobility Products Company
380 Vansickle Road Unit 350
St. Catharines, Ontario L2R 6P7
Canada

Pride Mobility Products Ltd.
32 Wedgwood Road
Bicester, Oxon OX26 4UL
UK

Pride Mobility Products Australia Pty. Ltd.
21 Healey Road
Dandenong, 3175
Victoria, Australia

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NOTE: If you ever lose or misplace your product registration card or your copy of this manual, contact us and we will be glad to send you a new one immediately.

THE SYNERGY SPECTRUM AIR

The Synergy Spectrum Air is a cushion system that contains an inflatable air insert and is available in both contour and flat versions. The cushion is labeled to assure proper placement on the chair.

WARNING! Altering the fitting position set by your healthcare professional can lead to pinching, skin abrasions, or bedsores/decupitis ulcers. Maintain the cushion in the prescribed position to ensure safe, comfortable use.

To place the Synergy Spectrum Air on the chair:

1. Locate the “Back Bottom” label on the underside of the cushion.
2. Place the cushion on the seat so that the “Back Bottom” label is located on the back of the seat, label side facing down. The zipper on the cushion will face the rear side of the seat. See figure 1.

WARNING! Extreme temperatures can affect the temperature of your cushion components and may cause skin irritation. Use caution when your power chair has been used or stored in an extremely hot or cold environment.

Do not exceed the weight capacity listed in your mobility product’s owner’s manual or 350 lbs. (158.76 kg), whichever is less.

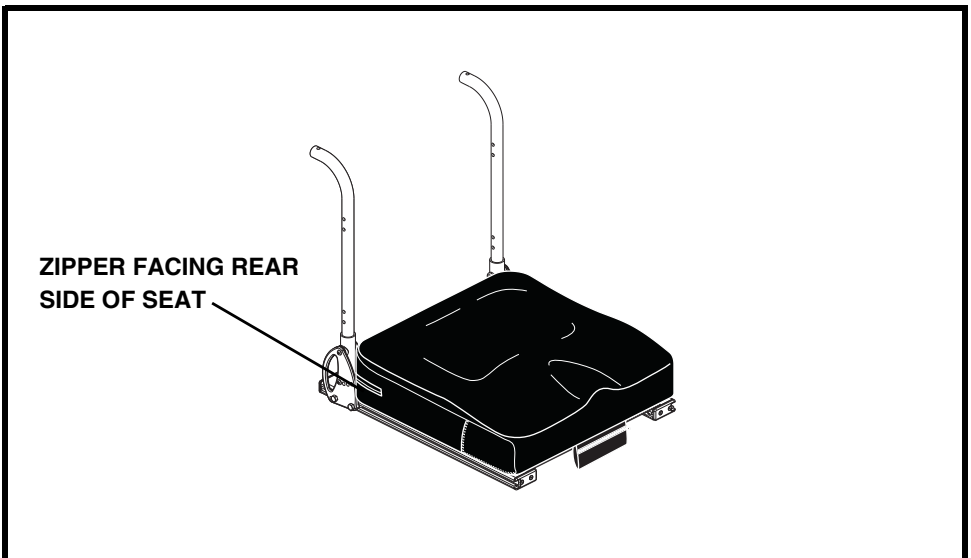
Figure 1. Cushion Placement

INFLATING AND DEFLATING THE AIR INSERT

The Synergy Spectrum Air can be easily inflated and deflated.

To inflate or deflate the cushion:

1. Remove the air pump from the storage bag. See figure 2.
2. Lift the access panel flap to expose the end of the air hose. See figure 2.
3. Press the end of the air pump onto the air hose. See figure 2.

NOTE: *The clear end of the pump is used to inflate the air insert and the black end of the pump is used to deflate the air insert. See figure 2.*

4. Squeeze the air pump until the desired firmness is reached.

NOTE: *If there is no change in inflation after pumping, check to ensure that the pump is completely inserted into the end of the air hose.*

5. Pull the air pump away from the air hose.
6. Cover the air hose by pressing on the access panel flap.
7. Return the air pump to the storage bag.

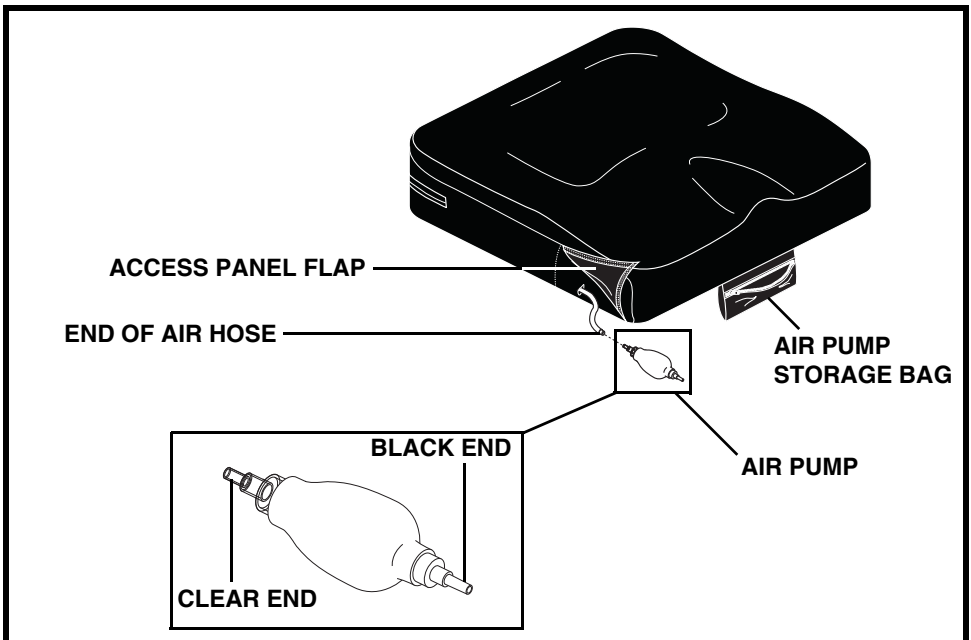
Figure 2. Synergy Spectrum Air

CLEANING

The Synergy Spectrum Air can be easily cleaned.

To clean the Synergy Spectrum Air:

1. Locate the zipper on the rear side of the outer cover.
2. Unzip the outer cover and pull out the assembled inner cushion components. **See figure 3.**
3. Wash the outer cover in a washing machine on the COLD setting.

WARNING! To prevent damage to the seat cover, do not use bleach when cleaning the outer cover.

4. The inner cushion components can be placed in a shower and washed with soap and warm water.
5. Allow the outer cover and inner cushion components to air dry.

WARNING! To prevent shrinking of the outer cover, do not dry it in a clothes dryer.

6. Once the cover and inner cushion components are dry, put the assembled inner cushion components back inside the cover. Zip the cover closed, and place the Synergy Spectrum Air back onto the seat of your chair.

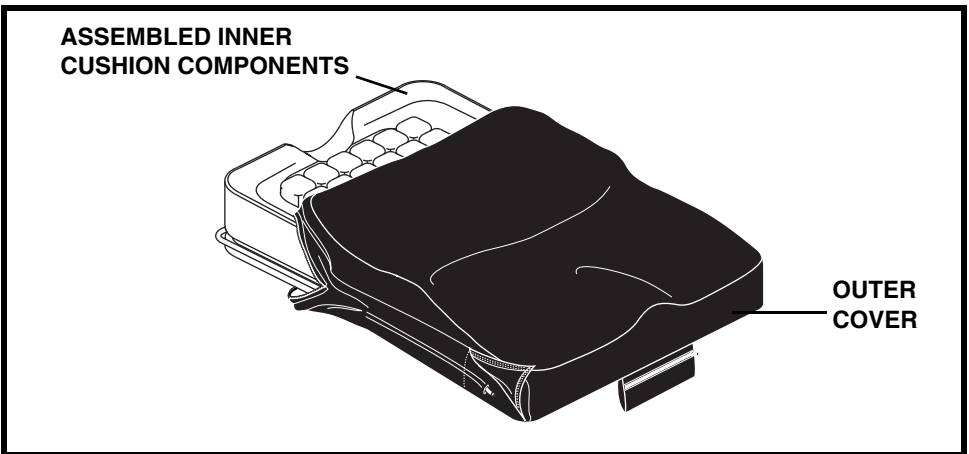
Figure 3. Synergy Spectrum Air

ASSEMBLING INNER CUSHION COMPONENTS

The Synergy Spectrum Air arrives assembled, but will need to be reassembled after cleaning. The contour and flat versions of the cushion are shown in **figures 4 and 5**, respectively.

To assemble the Synergy Spectrum Air:

1. Place the air insert on the seat base taking care to align the hook and loop fasteners on the front of the insert with those on the seat base. **See figure 4 or 5.**
2. Place the Sil-Air foam around the air insert. **See figure 4 or 5.**
3. Route the air hose under the foam and along the cut out in the foam. **See figure 4 or 5.**
4. Lift the access panel flap.
5. Place the assembled cushion into the outer cover with the front of the cushion going into the outer cover first.
6. Insert the air hose through the hole in the outer cover so that the end of the air hose is in the access panel.
7. Close the access panel.
8. Zip the outer cover closed.

NOTE: Pride highly recommends that the Synergy Spectrum Air be assembled and adjusted by your healthcare professional before using your cushion for the first time.

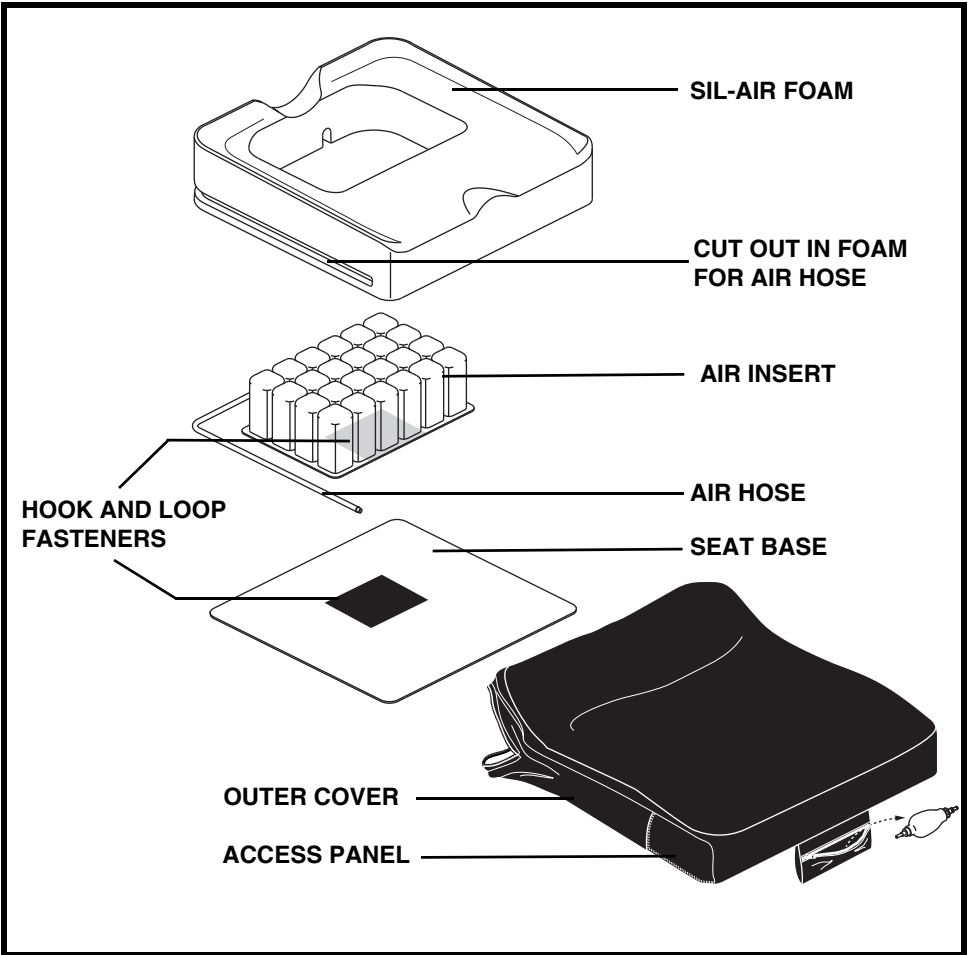
Figure 4. Contour Spectrum Air Cushion Components

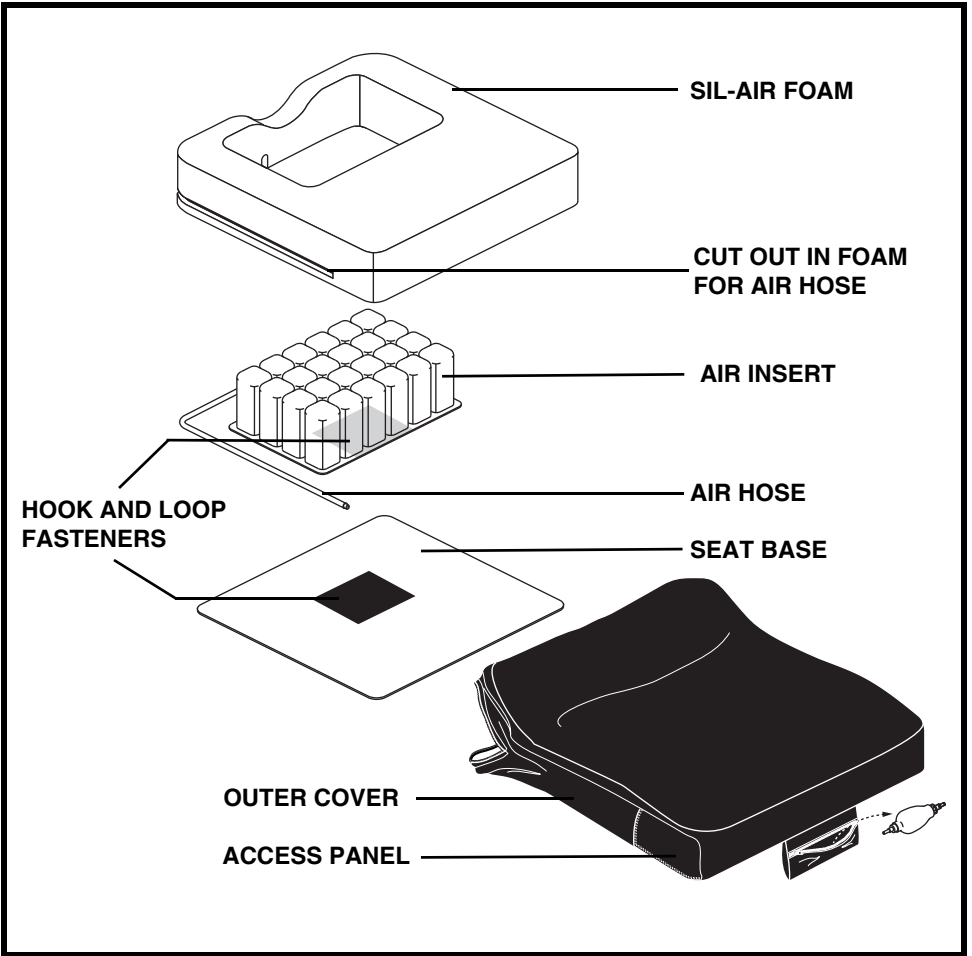
Figure 5. Flat Spectrum Air Cushion Components

REPOSITIONING THE AIR HOSE

The air hose can be placed on either the right or left side of the Synergy Spectrum Air.

To reposition the air hose:

1. Locate the zipper on the rear side of the outer cover.
2. Unzip the outer cover and pull out the assembled inner cushion components. **See figure 3.**
3. Remove the existing air tube and elbow valve.
4. Install the desired air tube on to the elbow valve (**see figure 6**) and route the air hose under the foam along the cut out in the foam. **See figure 4 or 5.**
5. Insert the air hose through the hole in the outer cover so that the end of the air hose is in the access panel. **See figure 4 or 5.**
6. Close the access panel.
7. Zip the outer cover closed.

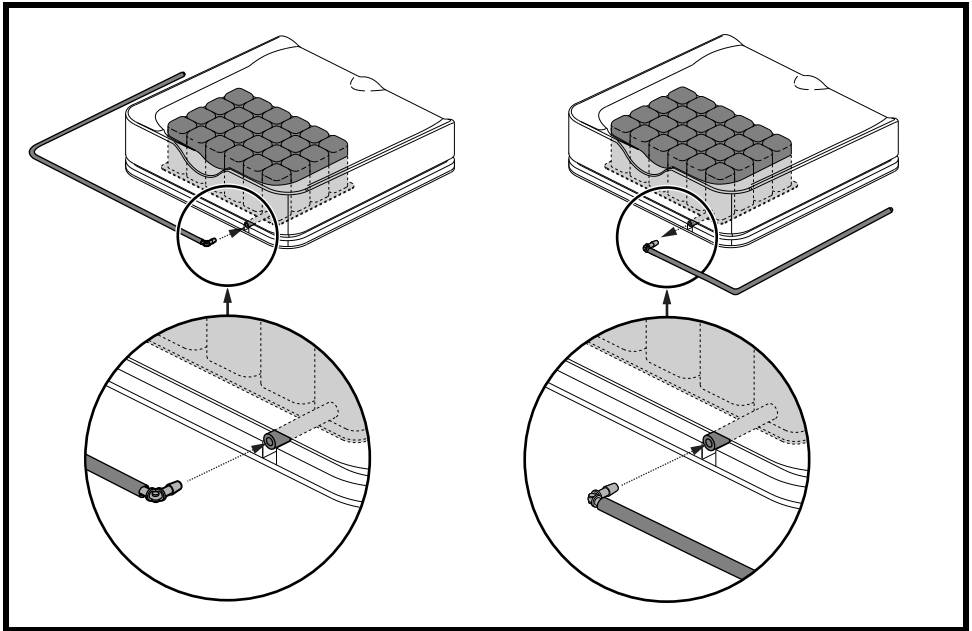
Figure 6. Left and Right Side Placement of Air Hose

CARE AND MAINTENANCE

Your mobility product must be disposed of according to applicable local and national statutory regulations.

MANDATORY! Contact your local waste disposal agency or authorized Pride Provider for information on proper disposal of mobility product packaging, metal frame components, plastic components, electronics, batteries, neoprene, silicone, and polyurethane materials.

The air pressure in your cushion system should be checked daily. Refer to “Inflating and Deflating the Air Insert” if you need to add or release air from the air insert.

Be aware of safety issues while smoking.

WARNING! Pride strongly recommends that you do not smoke cigarettes while seated on your cushion system although the cushion system has passed the necessary testing requirements for cigarette smoking. You must adhere to the following safety guidelines if you decide to smoke cigarettes while seated on your cushion system.

- Do not leave lit cigarettes unattended.
- Keep ashtrays a safe distance from the cushion system.
- Always make sure cigarettes are completely extinguished before disposal.

WARNING! Even though the plastics used on your mobility product have been tested and have passed flammability standards, Pride recommends that you do not expose your mobility product to open flames.

WARNING! Exercise extreme caution when using oxygen in close proximity to electrical circuits and combustible materials. Contact your oxygen provider for information on using oxygen safely.

WARNING! Replace worn or damaged upholstery immediately to reduce the risk of fire hazard.

WARNING! Be aware that washing of upholstered items may reduce the resistance to flammability of the fabric.

WARRANTY

The Synergy Spectrum Air cushion is designed and produced to the highest of standards. Our Synergy cushion is guaranteed to be free of defects in material and workmanship for a period of 24 months. If any defect in material or workmanship is found, Pride Mobility Products will repair or replace, at our discretion, the Synergy cushion. This warranty does not apply to cushions abused or misused by the user and deemed such by Pride Mobility Products.

Claims and repairs should be processed through the nearest authorized Pride Provider or Quantum Rehab Provider.

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A Division of Pride Mobility Products® Corp.