Including Models
Incluant les modèles
Incluidos los modelos

DE/TM/XL/
FULL

LP/FULL

Pride® Lifts & Ramps

MOBILITY MADE EASY. LA MOBILITE RENDUE FACILE.
MOVILIDAD HECHA FÁCIL.

Outlander Series Outlander Serie

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SAFETY GUIDELINES

WARNING! A Pride Provider or a qualified technician must perform the initial setup of this product and must perform all of the procedures in this manual.

The symbols below are used throughout this owner's manual and on the product to identify warnings and important information. It is very important for you to read them and understand them completely.

WARNING! Indicates a potentially hazardous condition/situation. Failure to follow designated procedures can cause either personal injury, component damage, or malfunction. On the product, this icon is represented as a black symbol on a yellow triangle with a black border.

MANDATORY! These actions should be performed as specified. Failure to perform mandatory actions can cause personal injury and/or equipment damage. On the product, this icon is represented as a white symbol on a blue dot with a white border.

PROHIBITED! These actions are prohibited. These actions should not be performed at any time or in any circumstances. Performing a prohibited action can cause personal injury and/or equipment damage. On the product, this icon is represented as a black symbol with a red circle and red slash.

Intended Use

The intended use of the Pride Mobility Products device is to aid in the transport of products that provide mobility to persons limited to a seated position that have the capability of operating a mobility device.

Quick Reference Information

Pride Provider: _____
Address: _____
Phone Number: _____
Purchase Date: _____

NOTE: This owner's manual is compiled from the latest specifications and product information available at the time of publication. We reserve the right to make changes as they become necessary. Any changes to our products may cause slight variations between the illustrations and explanations in this manual and the product you have purchased. The latest/current version of this manual is available on our website.

NOTE: This product is compliant with WEEE, RoHS, and REACH directives and requirements.

NOTE: This product meets IPX4 classification (IEC 60529).

NOTE: The Outlander Series Lift System and its components are not made with natural rubber latex. Consult with the manufacturer regarding any after-market accessories.

CONTENTS

I.	INTRODUCTION	4
II.	SAFETY	5
III.	INSTALLATION	10
IV.	OPERATION	15
V.	BATTERY CHARGING.....	20
VI.	CARE AND MAINTENANCE.....	22
VII.	TROUBLESHOOTING	24
VIII.	WARRANTY	25

I. INTRODUCTION

SAFETY

Welcome to Pride Mobility Products (Pride). Congratulations on the purchase of your new lift system. The lift system design combines the most advanced state-of-the-art components with modern, attractive styling. We are certain that the design features and trouble-free operation will add convenience to your daily living and ensure complete satisfaction.

At Pride, your safety is important to us. **Please read and follow all instructions in this manual before operating your lift system for the first time.** These instructions were produced for your benefit. Your understanding of these instructions is essential for the safe operation of your new lift system.

Pride is not liable for damage to property or personal injury arising out of the unsafe use of a lift system. Pride is also not liable for any property damage or personal injury arising out of the failure of any person and/or user to follow the instructions and recommendations set forth in this manual or any other instructions or recommendations contained in other lift system related literature issued by Pride or contained on the lift system itself.

INTERNET AND PRIVATE PURCHASES

If you purchased your product over the Internet or from a previous owner and you have any questions about the safe use and/or maintenance of the product, please visit our web site www.pridemobility.com or contact your authorized Pride Provider.

PURCHASER'S AGREEMENT

By accepting delivery of this product, you promise that you will not change, alter, or modify this product or remove or render inoperable or unsafe any guards, shields, or other safety features of this product; fail, refuse, or neglect to install any retrofit kits from time to time provided by Pride to enhance or preserve the safe use of this product.

SHIPPING AND DELIVERY

Before using your lift system, make sure your delivery is complete as some components may be individually packaged. If you do not receive a complete delivery, please contact your authorized Pride Provider immediately. Where damage has occurred during transport, either to the packaging or content, please contact the delivery company responsible.

NOTE: If you ever lose or misplace your copy of this manual, contact us and we will be glad to send you a new one immediately.

II. SAFETY

PRODUCT SAFETY SYMBOLS

The symbols below are used on the lift system to identify warnings, mandatory actions, and prohibited actions. It is very important for you to read and understand them completely.

Do not open the swing-away hitch adapter with a mobility device on the lift.

Crush Hazard—Pinch/crush points created during assembly and operation. Keep hands clear.

Crush Hazard—Moving lift can crush. Keep feet clear during operation.

Read and follow the information in the owner's manual.

Maximum lifting capacity.

Do not use as a step.


Lift system intended for use with unoccupied mobility device. Do not use product to lift people.

II. SAFETY

Removal of grounding prong can create electrical hazard. If necessary, properly install an approved 3-pronged adaptor to an electrical outlet having 2-pronged plug access.

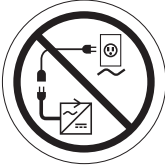Do not connect an extension cord to the charger cord.

Do not allow unsupervised children to play near the lift system while the batteries are charging.

Avoid exposure to rain, snow, ice, salt, or standing water whenever possible. Maintain and store in a clean and dry condition.

Keep tools and other metal objects away from battery terminals. Contact with tools can cause electrical shock.

Battery posts, terminals, and related accessories contain lead and lead compounds. Wear goggles and gloves when handling batteries and wash hands after handling.

Explosive conditions exist!

II. SAFETY


Disposal and recycling - Contact your authorized Pride Provider for information on proper disposal of your Pride product and its packaging.

Use only AGM or Gel-Cell batteries to reduce the risk of leakage or explosive conditions.

Corrosive chemicals contained in battery.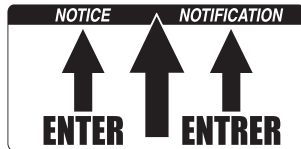

NOTICE! Enter platform where noted.

II. SAFETY

LIFTING CAPABILITIES

The lift system is an electromechanical device designed to lift and secure various types of mobility devices for the purpose of vehicle transport. The Outlander is designed for use with Class II, Class III, or Class IV hitches. Check the hitch capabilities of your vehicle prior to use. Under no circumstances should the lift system be made to exceed this weight limit. Subjecting the lift system to the strain of hoisting more than it is designed to lift may cause it to fail, resulting in damage to the mobility device and/or injury to the lift operator. Refer to the mobility device owner's manual for information on the overall weight of the mobility device before lifting.

WARNING! Adding accessories, oversize batteries, or a different seat will increase the weight of your mobility device. Verify with your authorized Pride Provider that the total weight of your mobility device, including additions, does not exceed the maximum hitch allowance.

INSTALLATION

The lift system is designed to be mounted to a vehicle hitch system. Before lift installation, ensure that the hitch shows no signs of rust or wear and that the front and rear vehicle suspension is acceptable. If any vehicle alterations are necessary, make sure that you fully understand the instructions for drilling or cutting before making any alterations.

WARNING! Before drilling or cutting into the vehicle, make absolutely certain through visual inspection that there are no obstructions, such as the fuel tank, exhaust pipes, or electrical wires, in the path of the drill bit.

LIFTING NON-PRIDE PRODUCTS

The lift system is an extremely versatile device, which users may employ to lift items other than Pride products. Pride has no control over such use, nor can Pride anticipate every possible use to which a lift system may be put. Lifting non-Pride products with the lift system is done at the operator's own risk, and Pride accepts no liability for damage or injury resulting from such use.

PRELIFT INSPECTION

Before operating the lift system, inspect your surroundings to ensure that the lift system has a clear path of operation. Remove any possible obstructions before operating the lift and do not allow children to operate or play near the lift.

OPERATOR POSITIONING

The operator of the lift should stand a safe distance from the unit being lifted/lowered to ensure that his/her feet are never positioned under the raised lift platform. The operator should also keep his/her hands clear of the lift system to avoid the potential for injury.

TRANSPORT VEHICLE POSITIONING

Be sure your vehicle is parked on flat, level ground before attempting to load, lift, or remove a mobility device with the lift system. Make sure the vehicle emergency park brake is engaged before loading or unloading a mobility device.

II. SAFETY

HITCH CLASS INFORMATION

Hitches come in a variety of sizes and are designated by class. The classes range from Class II to Class IV. A heavier load requires a larger class, which has a higher tongue weight. Tongue weight is the amount of weight (downward force) that the load puts on the hitch. **See figure 1.**

The combined weight of your lift and mobility device gives you the minimum tongue weight for your application. Exceeding the tongue weight of a hitch creates an unsafe driving situation and can damage the hitch and/or the vehicle. **See figures 2 and 3.**

WARNING! Adding accessories, oversize batteries, or a different seat will increase the weight of your mobility device. Verify with your authorized Pride Provider that the total weight of your mobility device, including additions, does not exceed the maximum weight allowance.

WARNING! Never exceed the maximum lifting capacity of a lift regardless of the class of hitch being used. Doing so may cause lift and/or vehicle damage.

WARNING! Before installing the lift and applicable hitch systems, check your vehicle owner's manual for hitch maximum tongue weight capacity. Do not exceed the maximum tongue weight capacity of your vehicle.

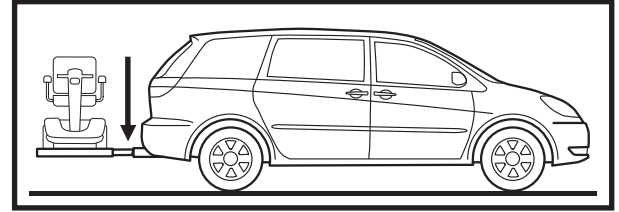
Figure 1. Tongue Weight Pressure Point

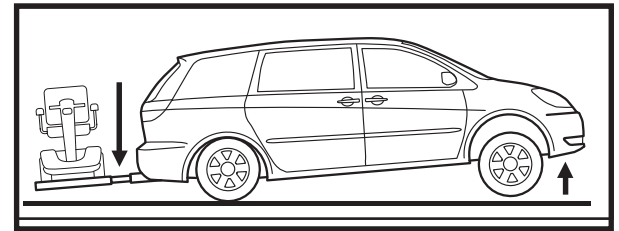
Figure 2. Unsafe Driving Situation

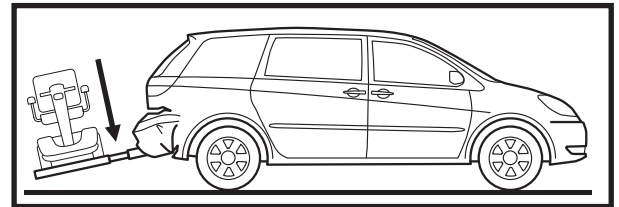
Figure 3. Vehicle Damage

HITCH CLASS			
Class		Lbs.	Kg
Class II	Hitch has a maximum tongue weight of	300 lbs.	136 kg
Class III	Hitch has a maximum tongue weight of	500 lbs.	227 kg
Class IV	Hitch has a maximum tongue weight of	1000 lbs.	455 kg

NOTE: Check vehicle manufacturer recommended tongue weight capacity prior to lift installation.

III. INSTALLATION

LIFT SYSTEM INSTALLATION

The Outlander is designed to be easily installed into any vehicle equipped with a Class II or Class III/IV hitch receiver. The hitch receiver should be installed no more than 21 in. (53.34 cm) from the bottom of the hitch tube to the ground.

NOTE: For applications above 21 in. (53.34 cm), please call your authorized Pride Provider for special installation instructions.

WARNING! The lift system should be installed only by an authorized Pride Provider or service technician.

To install the Scooter lock-down arm (Outlander FP only):

1. Lay the lift system flat on the floor.
2. Remove the black caps.
3. Remove hardware. **See figure 4.**
4. Position the lock-down arm onto the threaded bolts and align with the two holes. **See figure 4.**
5. Insert the washer.
6. Reattach hardware.
7. Secure the nut to the securement screw.
8. Reattach the black caps.

NOTE: If you purchase an Outlander TM, DE, or XL, the lock-down arm is installed.

NOTE: Any nylon insert lock nut removed during disassembly or adjustment of the lift system must be replaced with a new nut. Nylon insert lock nuts should not be reused as it may cause damage to the nylon insert, resulting in a less secure fit. Replacement nylon insert lock nuts are available at local hardware stores or through your authorized provider.

To install power chair hold-down bar:

1. Place the power chair on the lift.
2. Raise lift to the most upward position.
3. Place the power chair hold-down bar on the seat into the cushion. Once secured, attach to mast with supplied hardware. **See figure 5.**

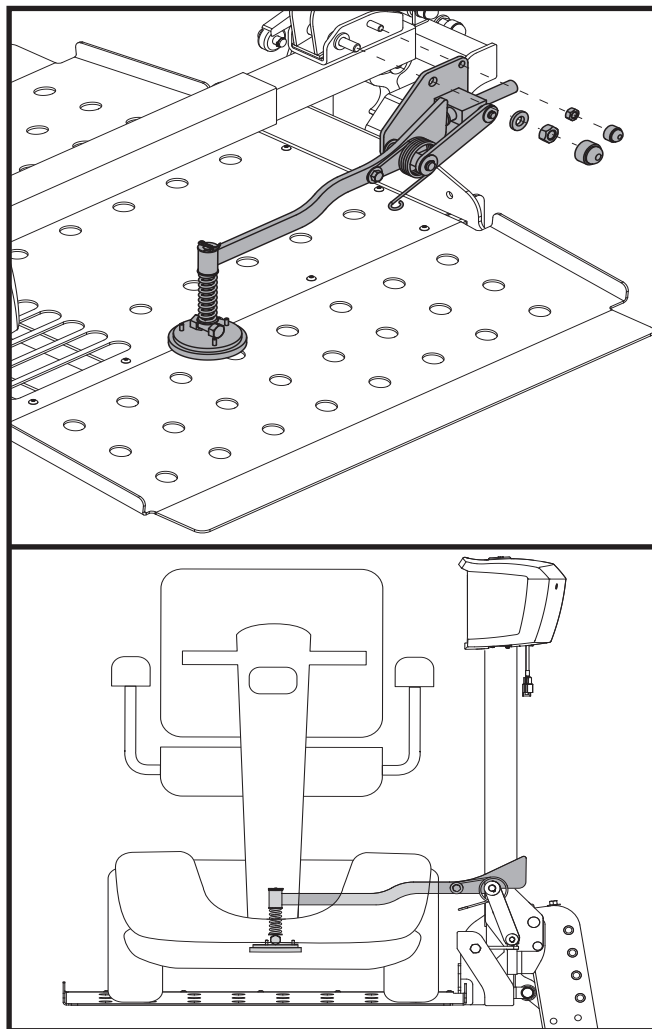
Figure 4. Lock-down Arm Installation

III. INSTALLATION

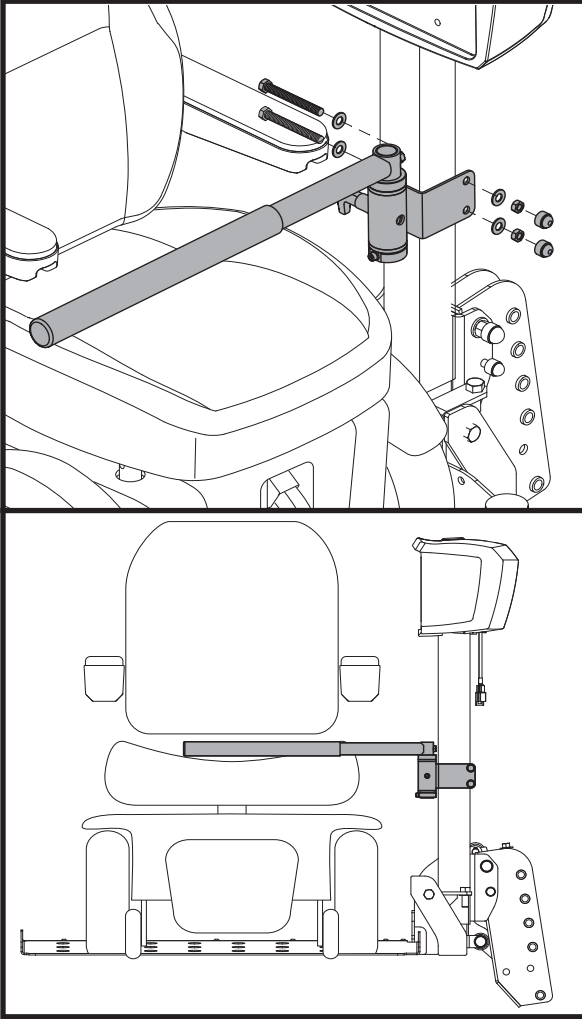
Figure 5. Hold-down Bar Installation

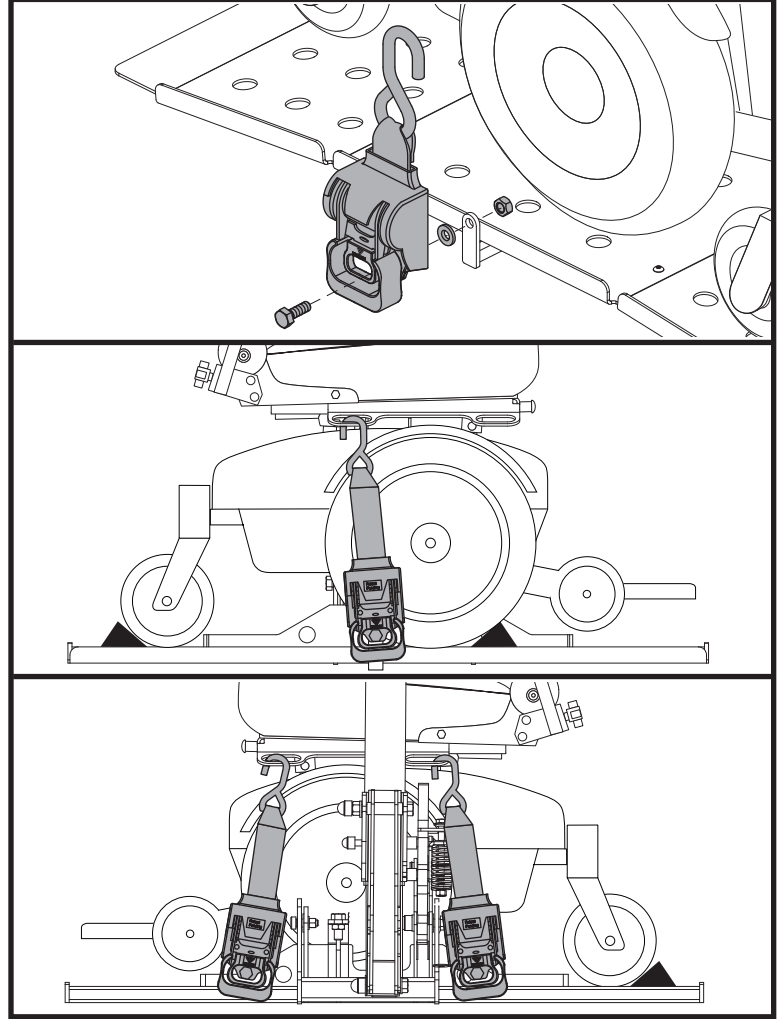
Figure 6. Retractable Tie-down Strap Installation

Steps for retractable tie-down strap installation:

1. Remove mounting hardware from strap.
2. Align the meeting position on strap to the desired mounting point hole on the full platform. **See figure 6.**
3. Install the hardware. **See figure 6.**
4. Repeat process for the second and third straps.

To install the lift system:

1. Remove the preassembled mounting hardware from the hitch tube.
2. Ensure that the hitch tube is properly oriented as shown in **figure 7**, and mount the hitch tube to the lift frame using the previously removed mounting hardware. Install the hardware in the direction indicated in **figure 7**.
3. Slide the hitch tube into the vehicle hitch receiver, aligning the mounting hole closest to the lift with the mounting hole in the hitch receiver. **See figure 7.**

WARNING! Lift system components are heavy. Ask for assistance if necessary when attaching the lift frame to the hitch receiver. Use proper lifting techniques and avoid lifting beyond your capacity.

WARNING! Avoid pinch points! Do not hold the lift frame by the pivot points when mounting the lift frame to the hitch receiver.

III. INSTALLATION

- Secure the hitch tube with the supplied bolt and nut, inserting the bolt from the threaded side of the hitch tube. **See figure 7.**

WARNING! Threading the bolt from the wrong side or allowing the bolt to turn while tightening the nut will dislodge the threaded insert in the hitch tube.

WARNING! Do not attach a Class II hitch tube to a Class III hitch receiver.

NOTE: Make sure the hitch bolt is tightened firmly before securing the nut to the hitch assembly to minimize the chance that the lift will tilt during transport.

NOTE: Make sure the hitch tube and lift platform are parallel to the ground before operating the lift system. An angled hitch tube may cause unreliable lift operation and/or damage to the lift system.

NOTE: Class II Hitch is compatible with swing-away.

- Route the lift system wiring harness through the vehicle. Refer to “Wiring Harness Installation.”

WARNING! Route the wiring harness through the vehicle rather than under the vehicle to avoid coming in contact with sharp edges, extreme temperatures, moving parts, and roadway debris, including road salt and other highly corrosive materials. Power shorts may occur if wires come in contact with hot exhaust parts or sharp edges.

WARNING! Never attach the wiring harness to a secondary power source. Do not attempt to use trailer wiring to power the lift system. The wiring harness must be connected directly to the vehicle battery.

NOTE: If your lift is equipped with an onboard battery, refer to “Onboard Battery Installation.”

- If equipped, install the wheel chocks to the lift platform. Refer to “Wheel Chock Installation.”

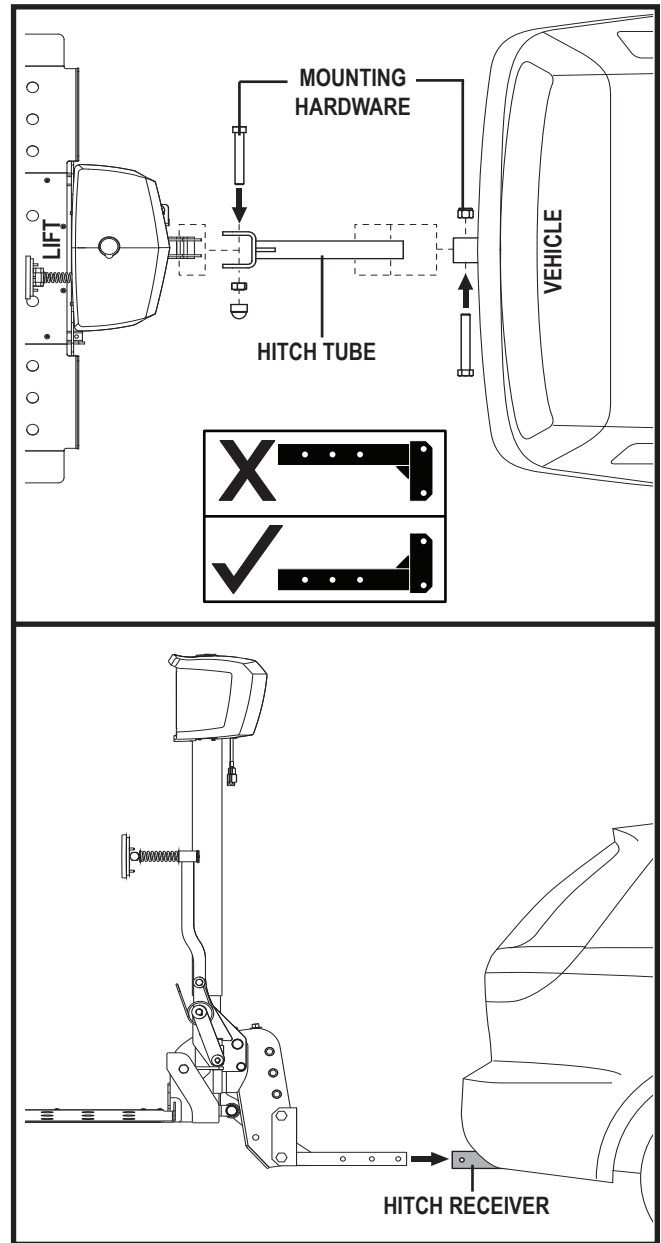
Figure 7. Hitch Tube Assembly

III. INSTALLATION

Onboard Battery Installation

The optional onboard battery is designed for easy installation as it does not require any wires to be routed through or under the vehicle.

To install the onboard battery:

1. Position the accessory bracket at desired location on Outlander mast.
2. Secure with supplied hardware. **See figure 8.**
3. Align the mounting holes in the battery case bracket with those in the accessory bracket. **See figure 9.**
4. Secure the battery case bracket to the accessory bracket with the supplied hardware. **See figure 9.**
5. Insert the onboard battery into the battery case, making sure that the connector on the bottom of the battery aligns and fully connects with the mating connector inside the battery case.
6. Plug the battery cord on the bottom of the battery case into the mating connector running from the motor. **See figure 9.**

Wiring Harness Installation

The wiring harness is approximately 25 ft. (7.62 meters) long and will accommodate most vehicles.

NOTE: You may wish to perform a practice run before installing the wiring harness. Route a piece of light rope (equivalent to the gauge of the wiring harness) along the anticipated path observing contact points, potential rubbing/chafing points, and any sharp edges. Remove sharp edges with a fine grade file, then treat the steel with a rust inhibitor or metal sealant.

To route the lift system wiring harness:

1. Route the lift system wiring harness (starting at the lift) through the interior of the vehicle until you reach the vehicle battery. **See figure 10.** You will typically find pass-through points at the rear lighting assemblies, which will enable you to route the cable through the trunk and under the back seat.
2. Conceal the harness behind or under the interior panels of the vehicle (there should be existing holes). Be certain that the harness is protected with a rubber grommet when passing it through the metal panels and into the engine compartment.
3. Inside the engine compartment, secure the harness to the firewall and the inner fender with plastic wire ties. Use care not to cause abrasions to the wiring harness. It is important to secure the wiring harness at various points along its run of the vehicle.

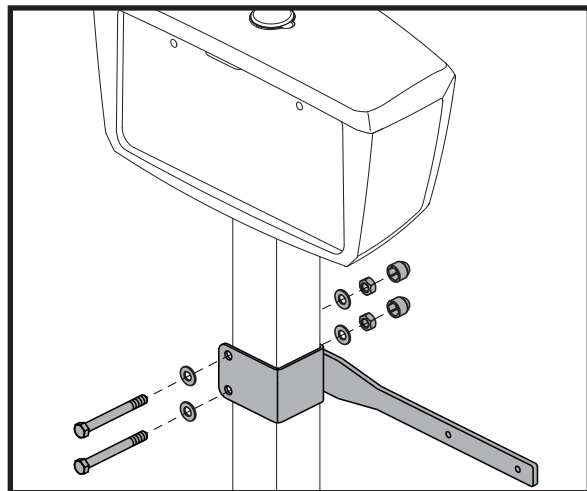
Figure 8. Accessory Bracket

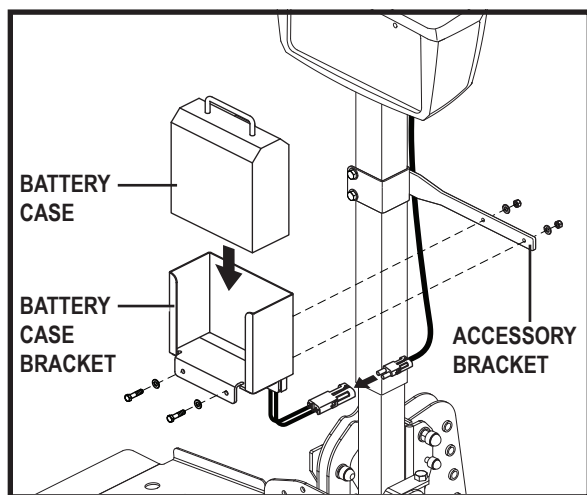
Figure 9. Onboard Battery Installation

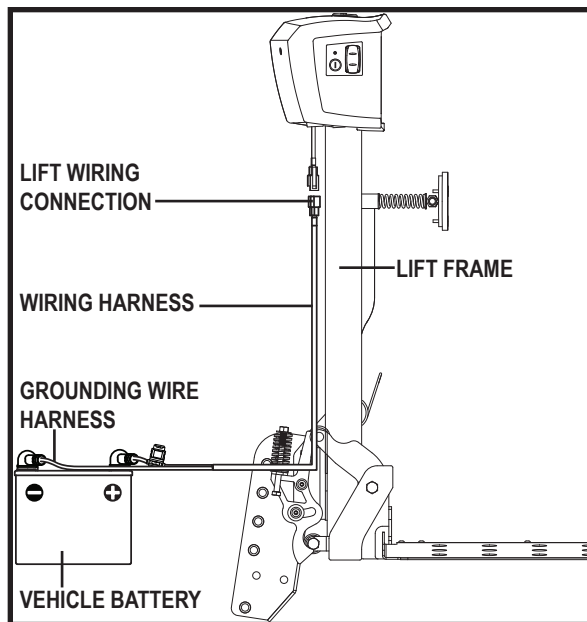
Figure 10. Wiring Harness Installation

III. INSTALLATION

WARNING! Do not cut or shorten the wiring harness. If the harness is too long, coil the excess wire and secure it with plastic wire ties.

4. Connect the red wire to the positive (+) terminal and the black (grounding) wire to the negative (-) terminal of the vehicle battery.

WARNING! Before operating the lift, inspect the wiring harness for proper routing and grounding. Improper routing can cause damage to the harness, and improper grounding can cause damage to the electrical system.

Wheel Chock Installation

Your lift will be supplied with four adjustable wheel chocks that, when installed, aid in centering your mobility device during loading.

To install the wheel chocks:

1. Drive your mobility device onto the lift platform, positioning it as near to the center as possible.
2. Position the wheel chocks on the platform.
 Scooters: Place a wheel chock in front of the front wheel(s) and behind the rear wheels or behind the front wheels and in front of the rear wheels. **Power Chairs:** Place a wheel chock in front of and behind the drive wheels.
3. Take note of the adjustment holes the wheel chocks align with on the platform and remove the chocks.
4. Remove the mobility device from the lift platform.
5. Raise the platform high enough to be able to install hardware from under it.
6. Position the wheel chock bottom on the underside of the platform, aligning it with the wheel chock top. **See figure 11.**
7. Secure the wheel chock sections to the platform using the supplied hardware. **See figure 11.**
8. Repeat these steps to install the remaining wheel chocks to the lift platform.

NOTE: The *Outlander™* and *LP* do not require wheel chock installation. The platform designs support the mobility device wheels so that wheel chocks are not required. See figure 12.

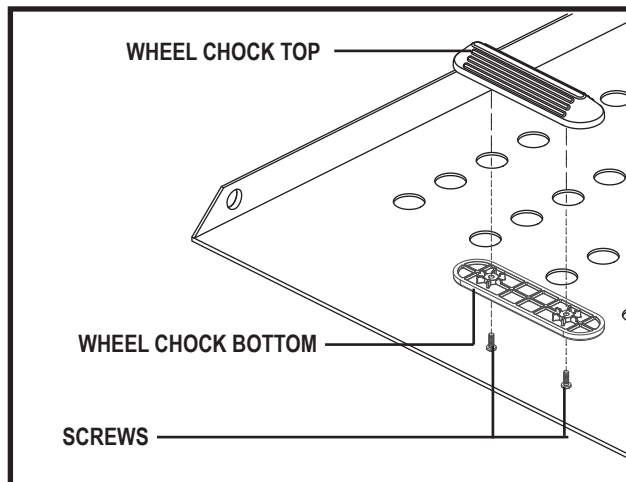
Figure 11. Wheel Chock Installation

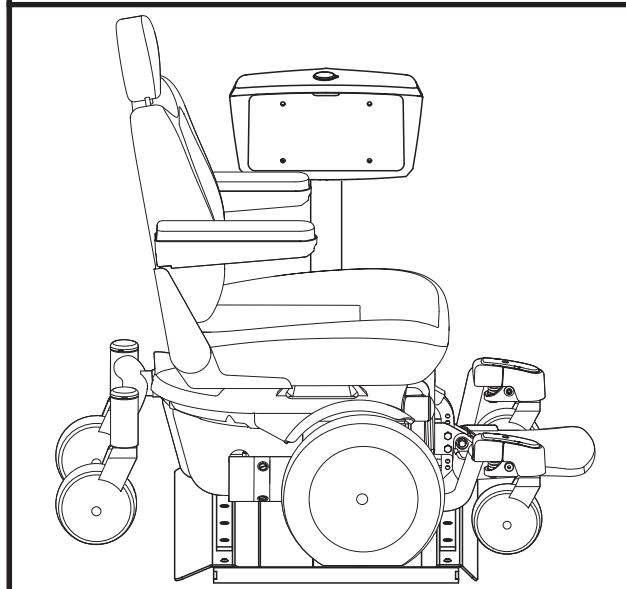
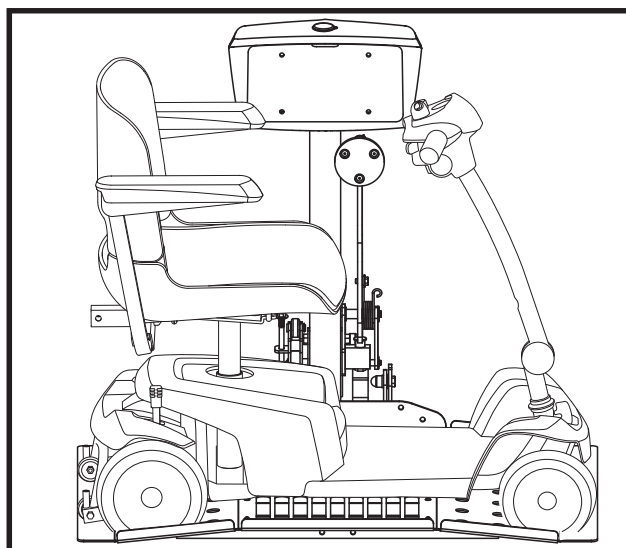
Figure 12. Wheel Placement

IV. OPERATION

LIFT SYSTEM OPERATION

WARNING! The lift system is intended for the transport of only mobility devices. The mobility device must be unoccupied before operating the lift.

WARNING! Do not exceed the load limits of your vehicle or hitch system as specified by the vehicle manufacturer.

WARNING! Use extreme caution when driving with the lift installed on your vehicle. Be aware that the lift extends away from the back of the vehicle and extra clearance is needed when backing up, turning, parking, or driving through driveway entrances/exits. Drive slowly when negotiating pot holes, road bumps, and railroad tracks.

To operate the lift system:

1. Insert the key into the key switch to turn on and off. The green light will illuminate when the lift system is on.
2. Use the toggle switch to raise or lower the lift system
See figure 13.

Loading and Securing Your Mobility Device

1. Set the speed control of the mobility device to the slowest setting and carefully drive your mobility device onto the lift platform, being sure to center the mobility device on the platform to the best of your ability.

NOTE: You may wish to load the mobility device while standing next to it using the manual freewheel feature. Refer to the mobility device owner's manual for more information on manual freewheel mode.

WARNING! If the lift is equipped with an automatic lock-down arm, load the product onto the lift platform from the driver's side of the vehicle to ensure proper contact of the automatic lock-down arm with the product floorboard. Loading the product from the passenger side will cause the automatic lock-down arm to interfere with the product seat when the platform is raised.

2. Shut down the power to the mobility device, remove the key if applicable, and make sure the unit is in drive mode (drive motors engaged). Refer to the mobility device owner's manual for more information on drive mode.

WARNING! Never leave a mobility device in manual freewheel mode during lift system transport, as the mobility device could roll.

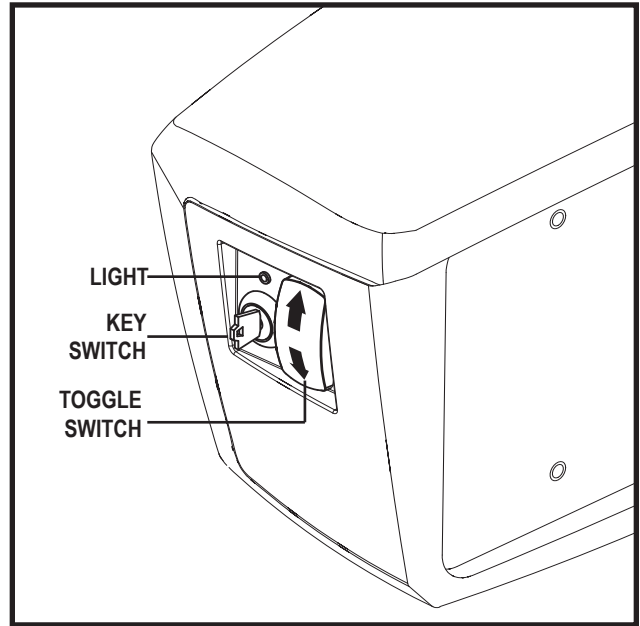
Figure 13. Key Switch and Toggle Switch Operation

IV. OPERATION

- Secure the mobility device to the platform using either the retractable tie-down straps or the hold-down bar for power chairs or either the automatic lock-down arm or straps for scooters.
 - Squeeze together the release and adjustment levers on the retractable tie-down strap and simultaneously bring both levers forward to a 90-degree position until they “click” then lock into place. This will unlock the tie-down strap. **See figures 14 and 15.**
 - Pull the strap out and secure it to the opposite side of the platform. Attach straps to the securement points of the mobility product. **See figure 16.**
 - Once each strap is secured in the proper position, squeeze together the release and adjustment levers and simultaneously bring both levers rearward to their original positions. This will lock the tie-down strap. **See figures 14 and 15.**
 - To tighten the strap, rotate only the adjustment lever forward and rearward in a ratcheting motion until the tie-down strap reaches an adequate level of resistance for mobility device securement.

NOTE: Inspect the tie-down straps before each use to ensure there are no signs of fraying or wear. Ensure that the mobility device is secured properly before transport. Pride is not liable for damage or loss resulting from improper securing of a mobility device.

WARNING! Never attempt to secure the tie-down straps over the seatback or armrests of a mobility device.

WARNING! For lift systems equipped with the automatic lock-down arm, do not attempt to adjust the pressure of the arm. This feature has been configured by your authorized Pride Provider to meet your needs and requires no further adjustment.

WARNING! If your product is equipped with pneumatic drive tires, make sure the tires are inflated to the recommended pressure labeled on the tire sidewall. This will ensure that the product is at the correct deck height for proper lock-down arm positioning.

- Raise the lift platform by pressing the toggle up. Continue to raise the lift until the motor stops, then release.

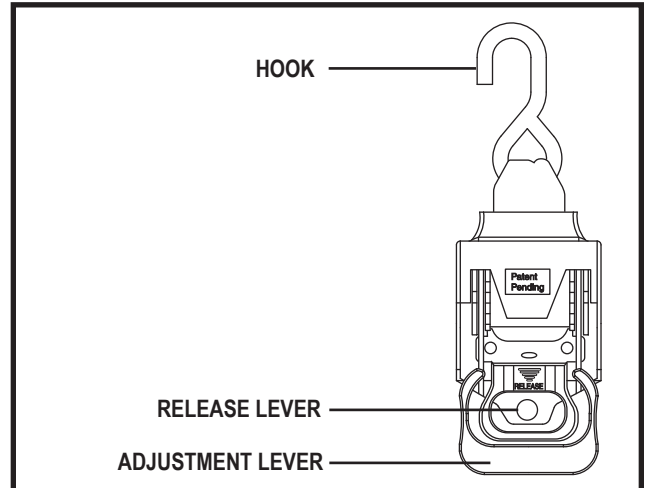
Figure 14. Retractable Tie-down Strap

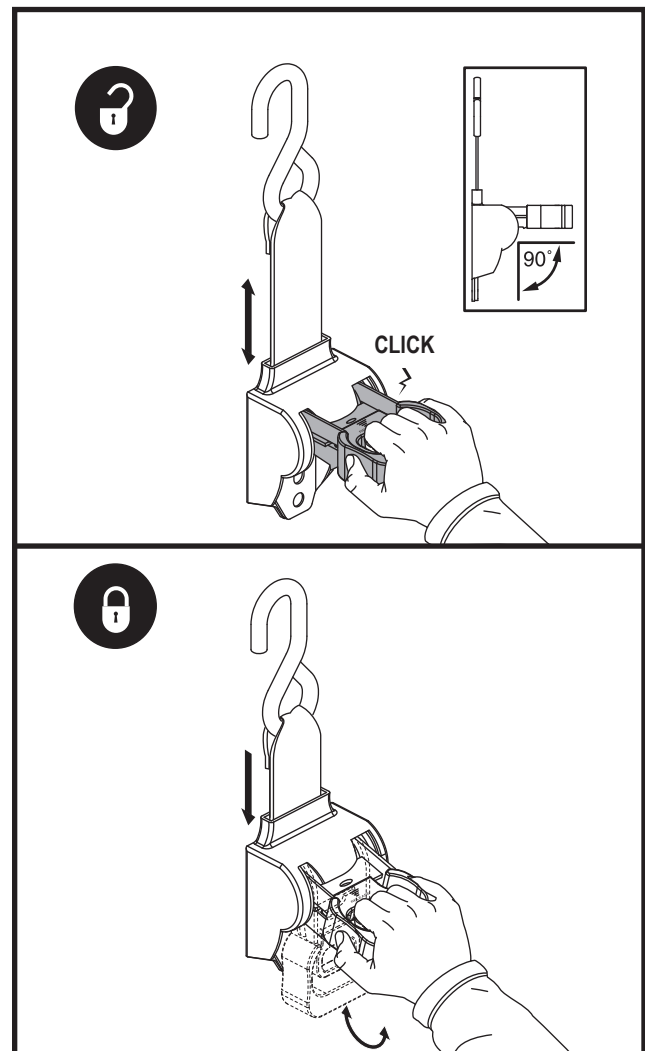
Figure 15. Unlocking and Locking Retractable Tie-down Strap

IV. OPERATION

NOTE: As the lift platform rises, the automatic lock-down arm activates to secure the product to the lift platform. You may need to adjust the product's tiller position to prevent interference with the smooth operation of the automatic lock-down arm.

NOTE: The motor will stop automatically at the top of its stroke and will emit a clicking sound. Release the key upon hearing this sound.

5. If equipped with product hold-down bar, position it into desired location.

NOTE: Ensure mobility device is secure.

NOTE: Pay attention to the path of the lift platform during operation. Make sure the lift does not rub against or interfere with the vehicle in any way. If you notice any contact between the lift platform and the vehicle, stop lift operation immediately and contact your authorized Pride Provider for assistance.

To remove a mobility device from the lift system:

1. Lower the lift platform to the ground.
2. Unfasten the tie-down straps or allow the lock-down arm to automatically return to the vertical position.
 - Unlock and loosen the tie-down straps.
 - Rotate only the adjustment lever on each strap forward and rearward in a ratcheting motion until each strap reaches its original furled position.
3. Unload the mobility device from the lift platform.

WARNING! Do not attempt to manually pull the lift platform down from the stowed position. Doing so will result in product damage and will void the product warranty.

4. Raise the lift platform for storage, noting that it will automatically fold into a stowed position.

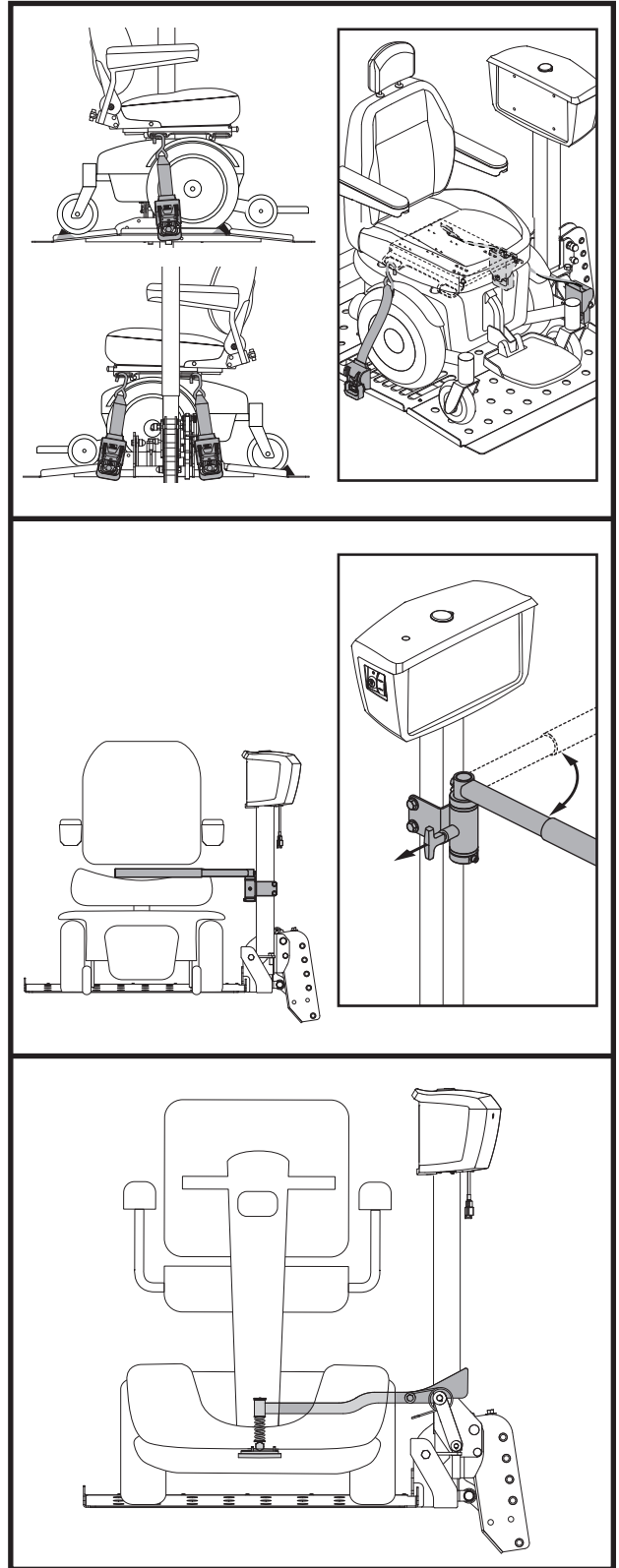
Figure 16. Securement of the Unit

IV. OPERATION

THE SWING-AWAY ADAPTER

The swing-away adapter allows the lift system to move away from the vehicle to permit easier access to vehicle.

To operate swing-away adapter:

1. Remove the safety pin.
2. If equipped, place the safety pin in the pin holder. **See figure 17.**
3. Lift up the latch to release.
4. Move lift system away from vehicle.

To secure swing-away adapter:

1. Move lift system toward the vehicle.
2. Align swing-away adapter over receiver and latch as shown in **figure 19.**
3. Reinstall the safety pin.

WARNING! Do not operate the swing-away lift assembly while the mobility device is on the exterior lift, or when the platform is in the lowered position. Do not open the exterior lift into traffic. Do not operate the exterior lift against an incline. Ensure all hardware is secure.

WARNING! Ensure the safety pin is engaged prior to using lift system or driving the vehicle.

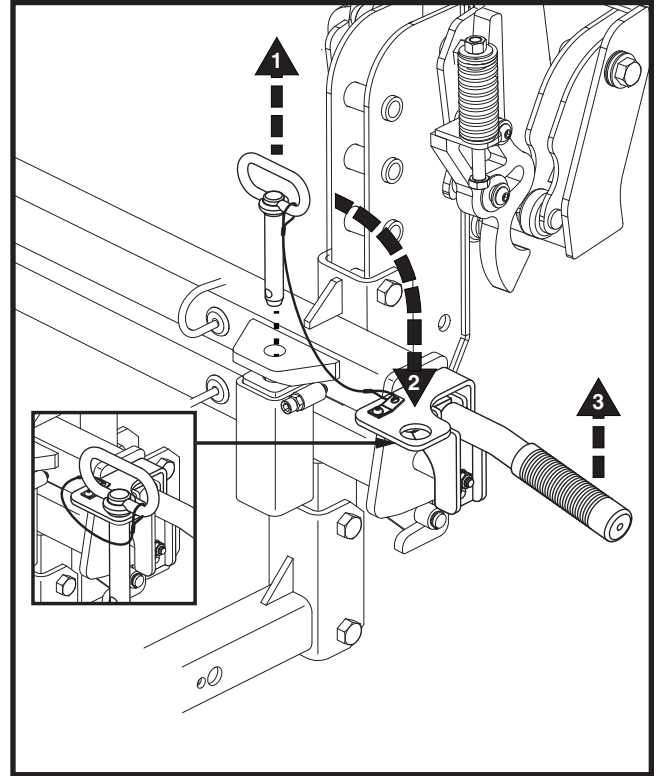
Figure 17. Operate Swing-away Adapter

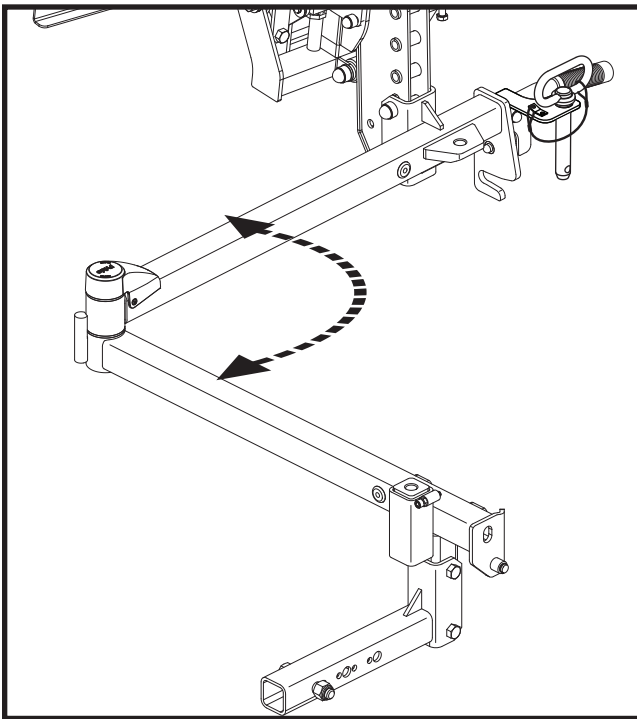
Figure 18. Move Lift System

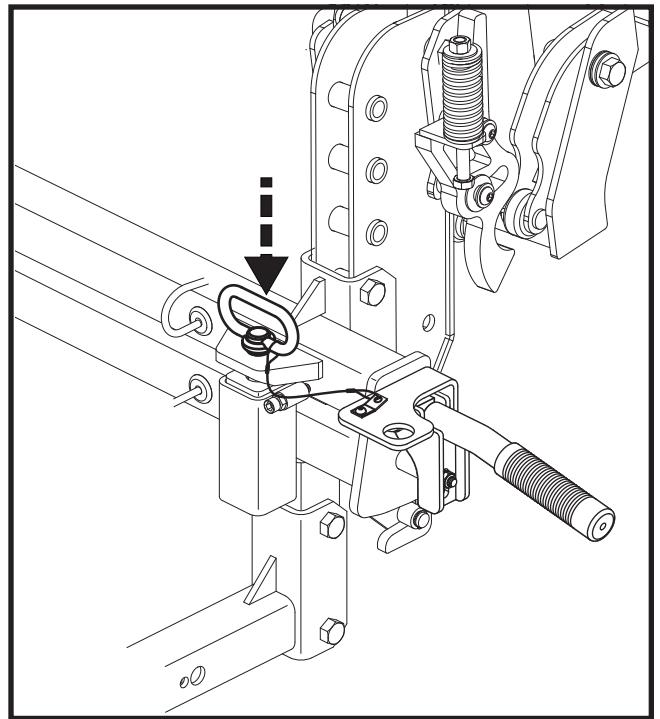
Figure 19. Secure Swing-away Adapter

IV. OPERATION

MANUAL LIFT SYSTEM OPERATION

The lift system is equipped with a manual crank that serves as backup in the event of a power failure. Using the supplied tool, rotate the manual crank (located on top of the motor housing) clockwise or counterclockwise to move the platform up or down. **See figure 20.**

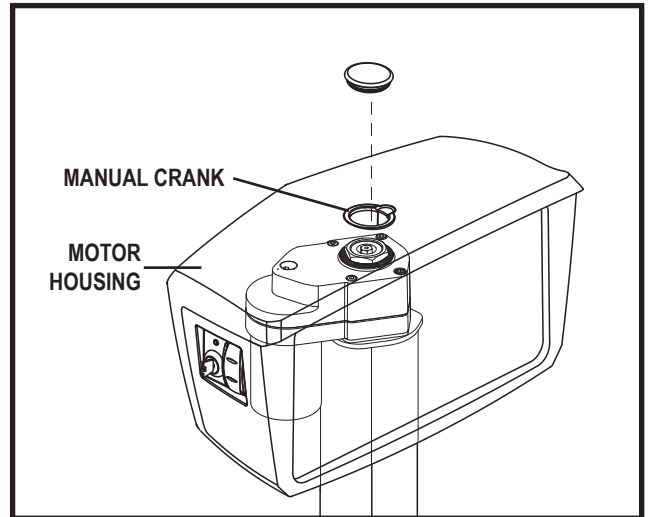
Figure 20. Manual Crank Operation

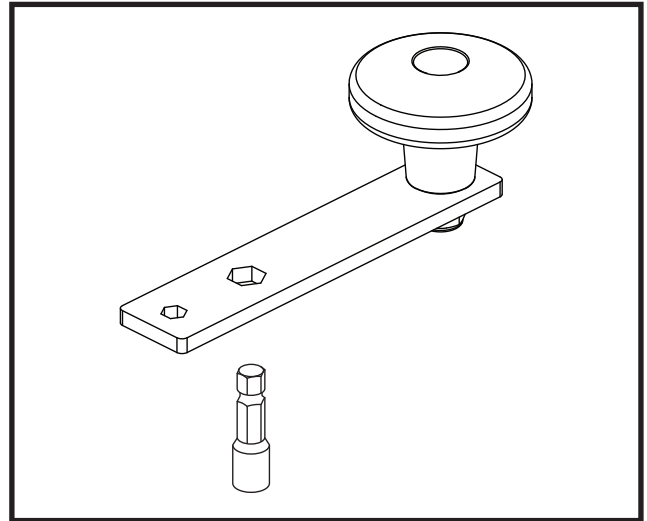
Figure 21. Manual Override Tool

V. BATTERY CHARGING

BATTERY CHARGING

To charge the onboard battery:

1. Remove the onboard battery case from the battery case bracket and place it near a standard electrical outlet.
2. Plug the charger cord into the mating socket located on the bottom of the battery case, then connect the charger cord to a standard electrical outlet. See figure 22.

NOTE: It is recommended that you replace the battery every 3–5 years. Store the battery in a moisture-free environment when not in use.

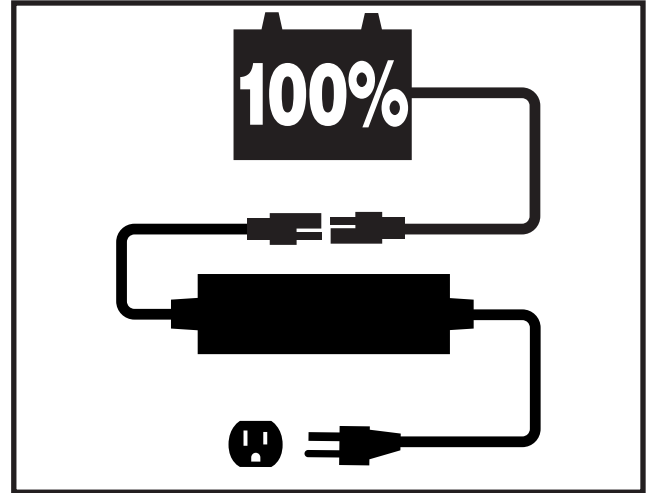
Figure 22. Battery Charging

MANDATORY! Read the battery charging instructions in this manual and in the manual supplied with the battery charger before charging the battery.

PROHIBITED! Never use an extension cord to plug in your charger cord. Plug the charger cord directly into a properly wired standard electrical outlet.

PROHIBITED! Removal of the grounding prong can create an electrical hazard. If necessary, properly install an approved 3-pronged adaptor to an electrical outlet having 2-pronged plug access.

PROHIBITED! Do not allow unsupervised children to play near the lift system or battery charger while the battery is charging.

PROHIBITED! Do not expose the battery charger to rain or other sources of moisture unless it has been tested for outdoor use. Refer to the manual supplied with the battery charger for more information.

WARNING! Explosive gases may be generated while charging the battery. Keep the battery and the battery charger away from sources of ignition such as flames or sparks and provide adequate ventilation when charging the battery.

WARNING! Do not attempt to charge the battery and operate the lift system at the same time.

WARNING! Do not attempt to wire the lift system directly to the vehicle battery if the lift system is equipped with an onboard battery.

WARNING! You must recharge the lift system battery with the supplied battery charger. Do not use an automotive-type battery charger.

WARNING! If the battery is exposed to adverse or extreme weather conditions, then it must be allowed to adjust to the difference in environmental conditions before being charged.

WARNING! Inspect the battery charger, wiring, and connectors for damage before each use. Contact your authorized Pride Provider if damage is found.

WARNING! Do not attempt to open the battery charger case. If the battery charger does not appear to be working correctly, contact your authorized Pride Provider.

V. BATTERY CHARGING

BATTERY REPLACEMENT

WARNING! Battery posts, terminals, and related accessories contain lead and lead compounds. Wear goggles and gloves when handling batteries and wash hands after handling.

MANDATORY! Use only AGM or Gel-Cell batteries to reduce the risk of leakage or explosive conditions.

WARNING! Corrosive chemicals contained in battery.

WARNING! Keep tools and other metal objects away from battery terminals. Contact with tools can cause electrical shock.

WARNING! Connect the battery harnesses in the proper manner. RED (+) cables must be connected to positive (+) battery terminals/posts. BLACK (-) cables must be connected to negative (-) battery terminals/posts. REPLACE cables immediately if damaged. Protective caps must be installed over all battery terminals.

NOTE: If you encounter a damaged or cracked battery, immediately enclose it in a plastic bag and contact your local waste disposal agency or your authorized Pride Provider for instructions on disposal and battery recycling, which is our recommended course of action.

To replace the onboard battery (see figure 23):

1. Unfasten the battery securement strap located at the bottom of the battery box.
2. Remove the hardware from the battery securement bracket and remove the bracket.
3. Remove the hardware from the battery connector.
4. Slide the battery out of the battery box to expose the battery harness.
5. Disconnect the battery harnesses from the battery terminals.
6. Connect the battery harnesses to the new battery.
7. Slide the new battery into the battery box.
8. Reinstall the hardware to secure the battery connector and tighten.
9. Reinstall the battery securement bracket and tighten the hardware.
10. Refasten the battery securement strap.

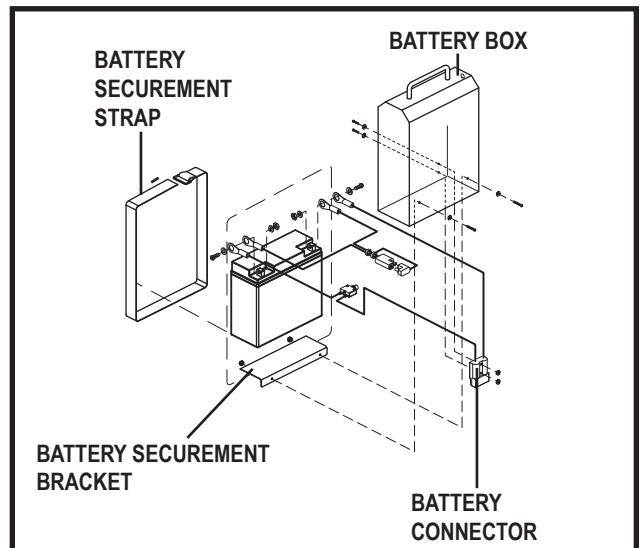
Figure 23. Battery Replacement

VI. CARE AND MAINTENANCE

CARE AND MAINTENANCE

Your lift system requires a minimal amount of care and maintenance. If you do not feel confident in your ability to perform the maintenance listed below, you may schedule inspection and maintenance at your authorized Pride Provider. The following areas require periodic inspection and/or care and maintenance.

Daily Maintenance Schedule

This lift system may come equipped with any of the securement devices listed below. Inspect these devices for loose or damaged parts on a daily basis. Have your authorized Pride Provider repair or replace any problem parts before using the lift system.

- Retractable tie-down straps
- Scooter lock-down arm
- Power chair hold-down bar

90-Day Maintenance Schedule

- Clean and lubricate the T-bar inner telescoping tube. Use only all-weather lithium grease. **See figure 24.**
- Lubricate all pivot points on the lift. Use only all-weather lithium grease.
- Inspect and clean the power-supply connections as necessary.
- Inspect all hardware and tighten or replace as necessary.
- Visually inspect lift system for loose or damaged parts. Have your authorized Pride Provider repair or replace any problem parts before using the lift system.

Five-Year Maintenance Schedule

Inspect the “Spring-activated Mechanical Switch” and lubricate or adjust spring tension as necessary. **See figure 25.**

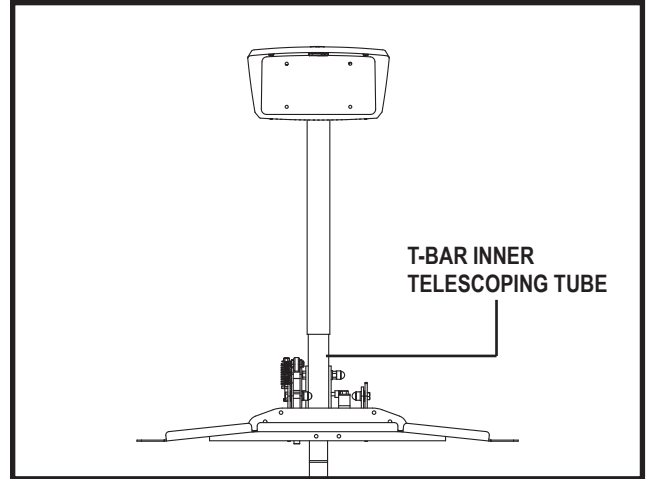
Figure 24. Lift Frame Maintenance

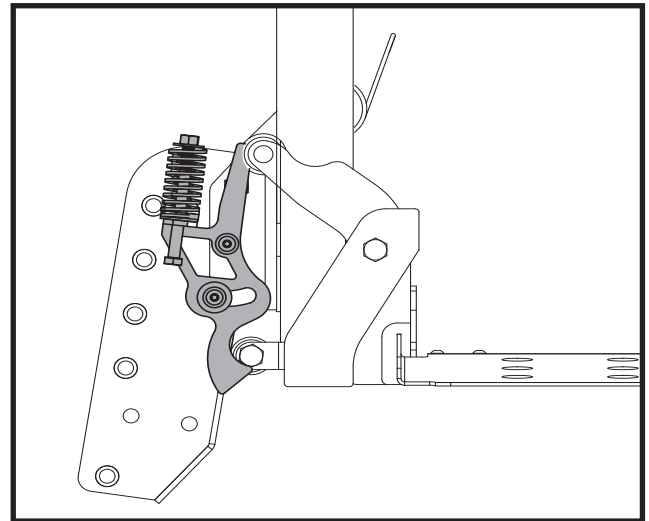
Figure 25. Spring-activated Mechanical Switch

VI. CARE AND MAINTENANCE

General Cleaning Guidelines

The lift system can be hand washed along with the vehicle with soap and water. However, automatic drive-through car washes should be avoided as they may interfere with and cause damage to the lift.

Wiring Harness

The wiring harness should be checked periodically for corrosion and/or decay. Contact your authorized Pride Provider if the wiring harness needs to be replaced.

Lift Platform/Metal Components

- Check the lift platform for wear or damage. Contact your authorized Pride Provider for repair or replacement.
- The lift platform surface may become slippery if exposed to moisture (ice, snow, rain, mist/spray conditions, etc.). Use caution when loading/unloading a mobility device in these conditions and wipe off any excess moisture from the lift platform.
- Regularly inspect the metal lift components for scratches. Treat any problem areas with a rust inhibitor or metal sealant.

If your lift system is exhibiting wear that you feel is questionable in any way, contact your authorized Pride Provider immediately. Do not attempt to repair, adjust, or modify the manufacturer settings and materials.

Lift Removal

- Raise platform to stowed position.
- Disconnect power source.
- Lubricate hitch mount bolt with WD-40® prior to removal.

VII. TROUBLESHOOTING

TRUBLESHOOTING

Any electromechanical device occasionally requires some troubleshooting. However, most of the problems that may arise can usually be solved with a bit of thought and common sense. Many of these problems occur because the battery is not fully charged or because the battery is worn down and can no longer hold a charge.

What if the motor on my lift system will not operate?

- Make sure the onboard battery or vehicle battery is fully charged.
- Make sure all connections to the motor and battery are secure.
- Check the wiring harness for signs of wear or decay. Replace the harness if necessary.

What if the lift system operates slowly up or down?

- Make sure the onboard battery or vehicle battery is fully charged.
- Make sure all connections to the motor and battery are secure.
- Check the wiring harness for signs of wear or decay. Replace the harness if necessary.

What if the lift platform does not fold into the stowed position?

- Make sure there are no obstructions in the T-bar pivot points.

What if the lift platform does not lower completely to the ground?

- Verify that the hitch receiver is at the proper height. The distance from the bottom of the hitch to the ground should not exceed 21 in. (53.34 cm).
- Make sure that the area under the lift platform is free from obstructions.

What if the lift bottoms out when driving over bumps or driveway entrances?

- Verify that the hitch receiver is at the proper height. The distance from the bottom of the hitch to the ground should be 21 in. (53.34 cm). Installing the riser tube adapter will provide an additional 4 in. (10.16 cm) of rise to the lift, keeping the lift from bottoming out.

VIII. WARRANTY

THREE-YEAR TRANSFERABLE LIMITED WARRANTY

For three (3) years from the date of purchase, Pride Mobility Products will repair or replace at our option, free of charge, any mechanical or electrical component found upon examination by an authorized representative of Pride to be defective in material and/or workmanship.

This warranty does not extend to those items, which may require replacement due to normal wear and tear.

- Labor, service calls, shipping, and other charges incurred for repair of the product, unless specifically authorized by Pride Mobility Products IN ADVANCE, are excluded.

Exclusions also include components with damage caused by:

- Contamination
- Abuse, misuse, accident, or negligence
- Battery fluid spillage or leakage
- Commercial use or use other than normal
- Improper operation, maintenance, or storage
- Repairs and/or modifications made to any part without specific consent from Pride Mobility Products
- Circumstances beyond the control of Pride

Pride cannot be held responsible for any defects in the motor vehicle in which the lift system is installed, defects in the lift system and/or securement kit caused by defects in any part of the motor vehicle, or damage to the motor vehicle caused by installation or operation of the lift system.

ONE-YEAR WARRANTY

The battery is covered by a separate one (1) year warranty provided by the battery manufacturer. The batteries are not warranted by Pride Mobility Products.

SERVICE CHECKS AND WARRANTY SERVICE

An authorized Pride Provider must perform warranty service. Do not return faulty parts to Pride Mobility Products without prior written authorization. All transportation costs and shipping damage incurred while submitting parts for repair or replacement is the responsibility of the purchaser.

Failure to follow the instructions, warnings, and notes in the owner's manual and those located on your Pride lift product can result in personal injury or product damage and will void Pride's product warranty.

There is no other express warranty.

IMPLIED WARRANTIES

Implied warranties, including those of merchantability and fitness for a particular purpose, are limited to one (1) year from the date of purchase and to the extent permitted by law. Any and all implied warranties are excluded. This is the exclusive remedy. Liabilities for consequential damages under any and all warranties are excluded.

Some states do not allow limitations on how long an implied warranty lasts or do not allow the exclusion or limitation of incidental or consequential damages. The above limitation or exclusion may not apply to you.

This warranty gives you specific rights, and you may also have other rights, which vary from state to state.

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B.V.