**Pride®**  
Lifts & Ramps

**Basic Operation Instructions**

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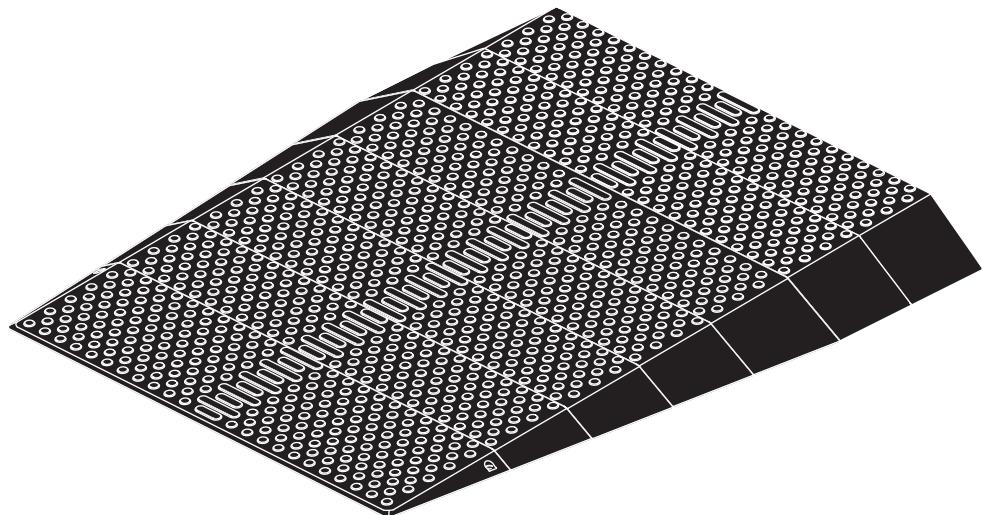
## SAFETY GUIDELINES

---

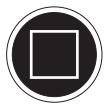
**WARNING!** A Pride Provider or a qualified technician must perform the initial setup of this product and must perform all of the instructions in this manual.

The symbols below are used throughout this owner's manual and on the product to identify warnings and important information. It is very important for you to read them and understand them completely.

**WARNING!** Indicates a potentially hazardous condition/situation. Failure to follow designated procedures can cause component damage or malfunction. On the product, this icon is represented as a black symbol on a yellow triangle with a black border.

**MANDATORY!** These actions should be performed as specified. Failure to perform mandatory actions can cause equipment damage. On the product, this icon is represented as a white symbol on a blue dot with a white border.

**PROHIBITED!** These actions are prohibited. These actions should not be performed at any time or in any circumstances. Performing a prohibited action can cause equipment damage. On the product, this icon is represented as a black symbol with a red circle and a red slash.

*NOTE: These instructions are compiled from the latest specifications and product information available at the time of publication. We reserve the right to make changes as they become necessary. Any changes to our products may cause slight variations between the illustrations and explanations in this manual and the product you have purchased. The latest/current version of this manual is available on our website.*

### INTENDED USE

Pride Ramps are intended for use as utility devices for the purpose of negotiating steps, thresholds, or other obstacles with scooters, power chairs, manual wheelchairs, hand trucks, and similar products. The Rubber Threshold Ramp has a maximum incline angle of 4° (7%), which meets the requirement established by the ADA.

## TABLE OF CONTENTS

---

3

### **ENGLISH**

<b>SAFETY</b> .....	4
<b>INSTALLATION AND ADJUSTMENT</b> .....	5
<b>CARE AND MAINTENANCE</b> .....	6
<b>WARRANTY</b> .....	7

### **APPENDIX**

<b>MULTI-LANGUAGE SPECIFICATIONS</b> .....	48
--	----

**PRODUCT SAFETY SYMBOLS**

The symbols below are used on the product to identify warnings, mandatory actions, and prohibited actions. It is very important for you to read and understand them completely.

**Read and follow the information in the owner's manual.**

**The maximum weight capacity is 850 lbs. (385.55 kg).**

**PRECAUTIONARY GUIDELINES**

**WARNING! Do not modify in any way not authorized by your provider.**

**WARNING! Be aware that exposure to prolonged sunlight can affect the surface temperature of the ramp and may cause skin irritation if touched. Use caution when your ramp has been used in an extremely hot, sunny environment.**

**WARNING! Flammable material. Do not expose to heat sources such as open flame or sparks.**

**WARNING! Use caution when walking on or negotiating the ramp as it is a sloped surface. Use extreme caution when using the ramp during inclement weather. Be aware that the ramp may become slippery when wet and may freeze faster than other surfaces.**

**WARNING! The ramp is heavy. See specifications table. If you are unable to lift that much weight, be sure to get help. Use proper lifting techniques and avoid lifting beyond your capacity.**

**WARNING! This product is made from recycled natural rubber which may contain latex, and which may cause an allergic reaction in some individuals. The use of personal protection equipment such as gloves is strongly recommended while lifting or moving the ramp.**

## INSTALLATION AND ADJUSTMENT

Read and follow all instructions, warnings, and notes in this manual before attempting to use your ramp for the first time. Your safety depends upon you, as well as your provider, caretaker, or healthcare professional in using good judgment.


**WARNING! Pride recommends installation on a flat, level surface such as cement. Avoid installing the ramp on gravel, sand, or any surface susceptible to erosion.**

### Single-piece Ramp Installation Instructions

Pride recommends that your Rubber Threshold Ramp be securely fastened to the surface upon which the Rubber Threshold Ramp is being installed prior to use. Use mounting screws or anchors (not supplied) to fasten the first section of the Rubber Threshold Ramp to the floor. Ensure that the ramp is free of dirt, debris, snow, and ice prior to use. **See figure 1.**

*NOTE: This product is recommended for outdoor use, but can also be used indoors.*

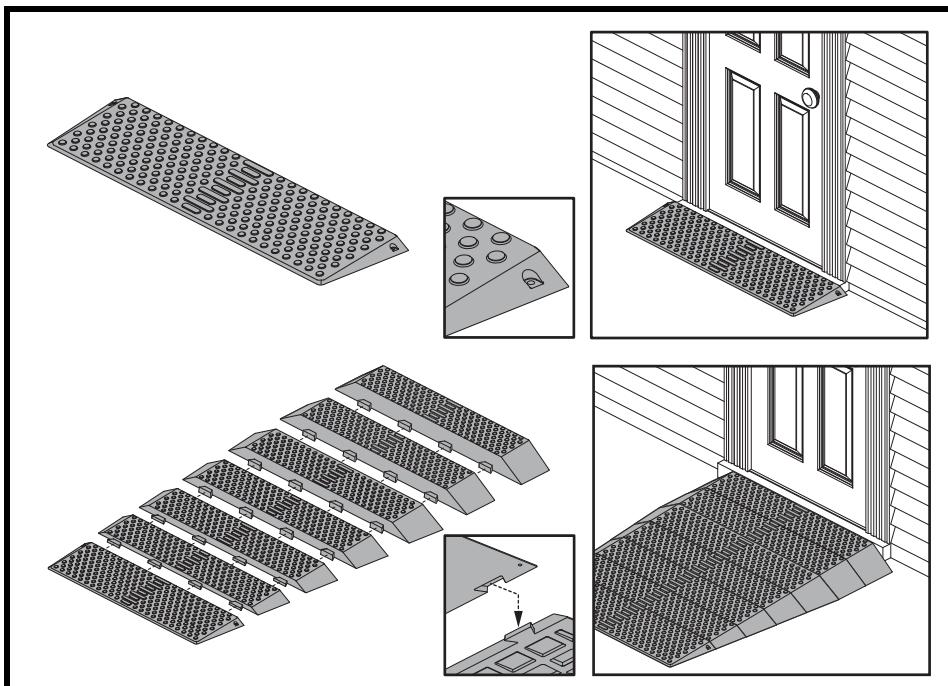**Figure 1. Rubber Ramp Threshold Installation**

**Multiple-piece Ramp Installation Instructions**

Rubber Threshold Ramps are shipped in multiple pieces and must be joined together prior to being fastened to the installation surface. Your ramp is equipped with a dove-tail interlock system, which allows the end of one ramp piece to be placed above the head of another ramp piece. Place the largest ramp piece into the desired position, then place the next piece over the first. Gently push the top piece down until it locks securely into the bottom piece. See **figure 1**. Once all pieces have been joined together, anchor the last ramp piece as described in the Single-piece Ramp installation instructions on the preceding page. See **figure 1**.

**Trimming the Ramp**

**WARNING! A Pride Provider or a qualified technician must perform any trimming necessary during the initial setup of this product.**

If during the installation of the Rubber Threshold Ramp you encounter fit issues that require the ramp to be trimmed or cut, **do not attempt to make such modifications yourself**. Contact your authorized Pride Provider for assistance. **Do not secure the ramp until it has been trimmed by the provider.**

---

**CARE AND MAINTENANCE****CARE AND MAINTENANCE**

- Clean the Rubber Threshold Ramp with a mild detergent and warm water only. Avoid using bleach and other harsh cleansers or shine agents that may cause the ramp to deteriorate or become slippery.
- Rinse well and use extra caution when the ramp surface is wet.

**THREE-YEAR TRANSFERABLE LIMITED WARRANTY**

For three (3) years from the date of purchase, Pride will repair or replace at our option, free of charge, any component found upon examination by an authorized representative of Pride to be defective in material and/or workmanship.

This warranty does not extend to those items which may require replacement due to normal wear and tear.

- Labor, service calls, shipping, and other charges incurred for repair of the product, unless specifically authorized by Pride Mobility Products IN ADVANCE, are excluded.

**Exclusions also include components with damage cause by:**

- Contamination
- Abuse, misuse, accident, or negligence
- Commercial use, or use other than normal
- Improper operation, maintenance, or storage
- Repairs and/or modifications made to any part without specific consent from Pride Mobility Products
- Circumstances beyond the control of Pride

**SERVICE CHECKS AND WARRANTY SERVICE**

An authorized Pride Provider must perform warranty service. Do not return faulty product to Pride without prior written authorization. All transportation costs and shipping damage incurred while submitting product for repair or replacement is the responsibility of the purchaser.

Failure to follow the instructions, warnings, and notes in the owner's manual and those located on your Pride ramp product can result in personal injury or product damage and will void Pride's product warranty.

There is no other express warranty.

**IMPLIED WARRANTIES**

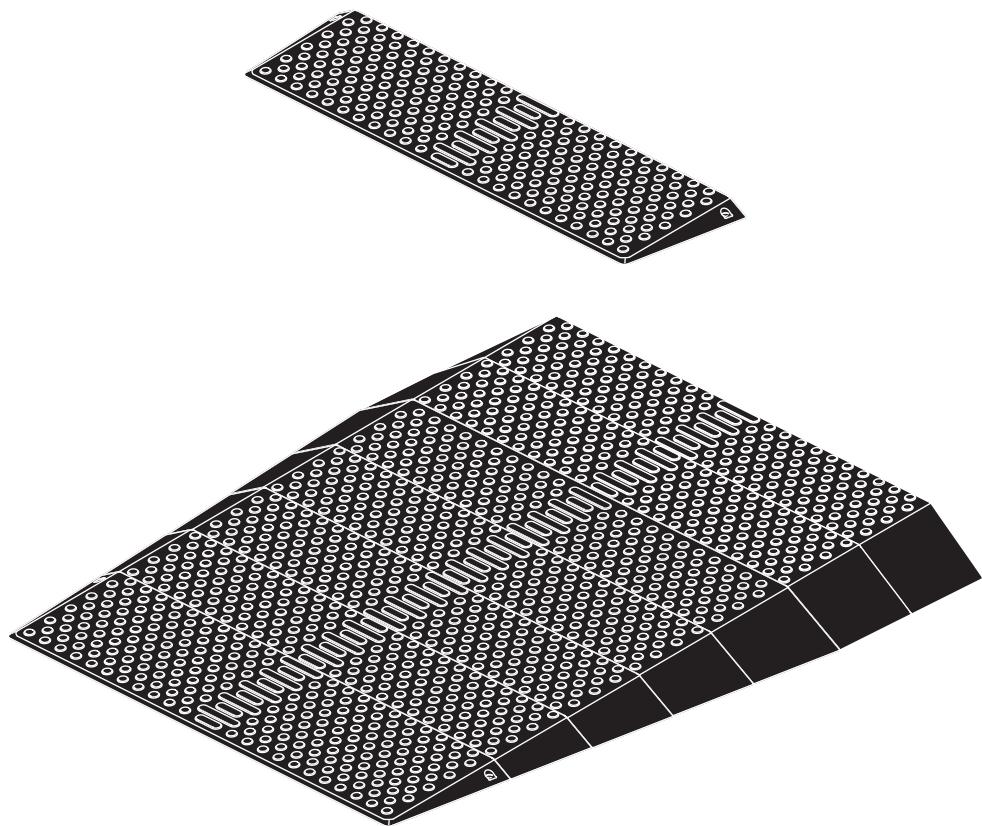
**Implied warranties, including those of merchantability and fitness for a particular purpose, are limited to one (1) year from the date of purchase and to the extent permitted by law. Any and all other implied warranties are excluded.** This is the exclusive remedy. Liabilities for consequential damages under any and all warranties are excluded.

Some states do not allow limitations on how long an implied warranty lasts or do not allow the exclusion or limitation of incidental or consequential damages. The above limitation or exclusion may not apply to you.

This warranty gives you specific rights, and you may also have other rights, which vary from state to state.

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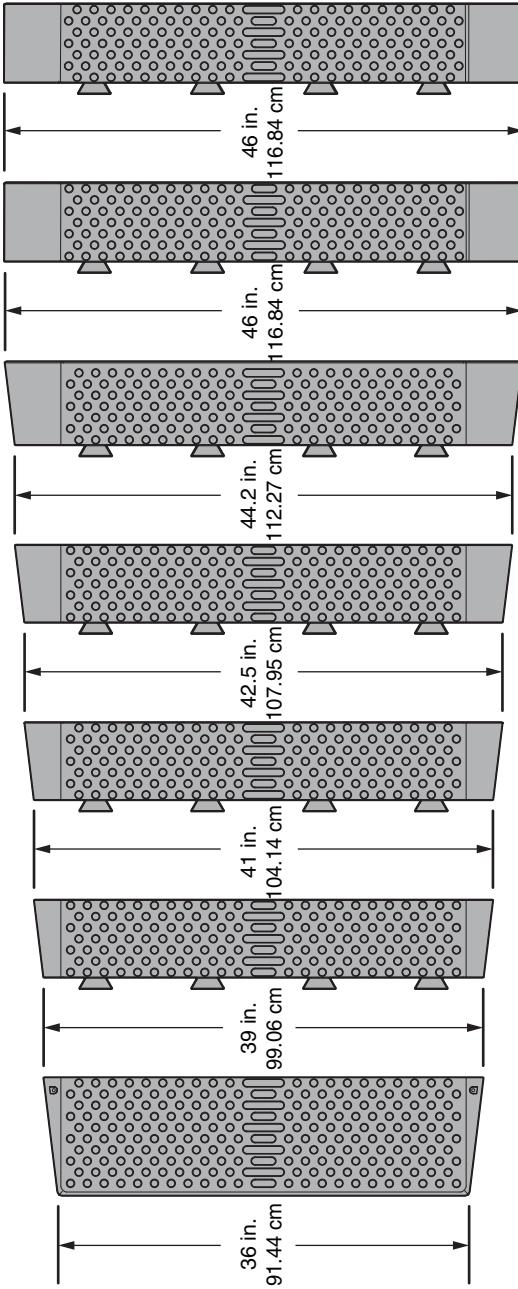
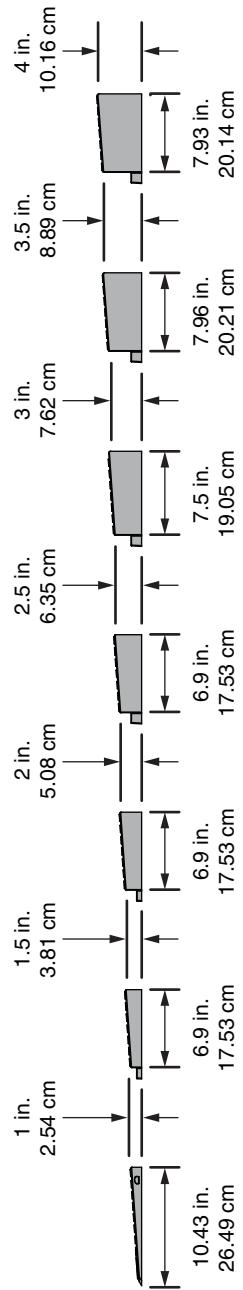
## SPECIFICATIONS

RUBBER THRESHOLD RAMP		RAMPE EN CAOUTCHOUC POUR SEUIL DE PORTE	
Length of Each Piece	6.9 in. (17.53 cm), 7.5 in. (19.05 cm), or 10.43 in. (26.49 cm)	Longueur de chaque pièce	17.53 cm (6.9 po), 19.05 cm (7.5 po), ou 26.49 cm (10.43 po)
Width of Each Piece	39 in. (99.06 cm), 41 in. (104.14 cm), 42.5 in. (107.95 cm), 44.2 in. (112.27 cm), or 46 in. (116.84 cm)	Largeur de chaque pièce	99.06 cm (39 po), 104.14 cm (41 po), 107.95 cm (42.5 po), 112.27 cm (44.2 po), or 116.84 cm (46 po)
Usable Surface of Each Piece	All pieces: 36" (91.44 cm)	Surface utile de chaque pièce	Toutes les pièces 91.44 cm (36 po)
Rise (Height) of Each Piece	1 in. (2.54 cm), 1.5 in. (3.81 cm), 2 in. (5.08 cm), 2.5 in. (6.35 cm), 3 in. (7.62 cm), 3.5 in. (8.89), or 4 in. (10.16)	Hauteur de chaque pièce	2.54 cm (1po), 3.81 cm (1.5 po), 5.08 cm (2 po), 6.35 cm (2.5 po), 7.62 cm (3 po), 8.89 cm (3.5 po) ou 10.16 cm (4 po)
Weight of Each Piece	1 in. (2.54 cm) Rise 9.5 lbs. (4.31 kg)  1.5 in. (3.81 cm) Rise: 12 lbs. (5.44 kg)  2 in. (5.08 cm) Rise: 15.5 lbs. (7.03 kg)  2.5 in. (6.35 cm) Rise: 18 lbs. (8.16 kg)  3 in. (7.62 cm) Rise: 23 lbs. (10.43 kg)  3.5 in. (7.62 cm) Rise: 28.81 lbs. (13.7 kg)  4 in. (10.16cm) Rise: 37.4 lbs (17kg)	Poids de chaque pièce	2.54 cm (1 po) Hauteur: 4.31 kg (9.5 lb) 1.5 in. (3.81 po) Hauteur: 5.44 kg (12 lb) 2 in. (5.08 po) Hauteur: 7.03 kg (15.5 lb) 2.5 in. (6.35 po) Hauteur: 8.16 kg (18 lb) 3 in. (7.62 po) Hauteur: 10.43 kg (23 lb) 3.5 in. (7.62 po) Hauteur: 13.7 kg (28.81 lb) 4 in. (10.16 po) Hauteur: 17 kg (37.4 lb)
Completely Assembled.	Total Length: 67.13 in. (170.5 cm) Total Weight: 166.8 lbs. (75.7 kg) Total Rise: 4 in. (10.16 cm)	Complètement assemblé	Longueur totale: 170.5 cm (67.13 po) Poids total: 75.7 kg (166.8 lb) Hauteur totale: 10.16 cm (4 po)

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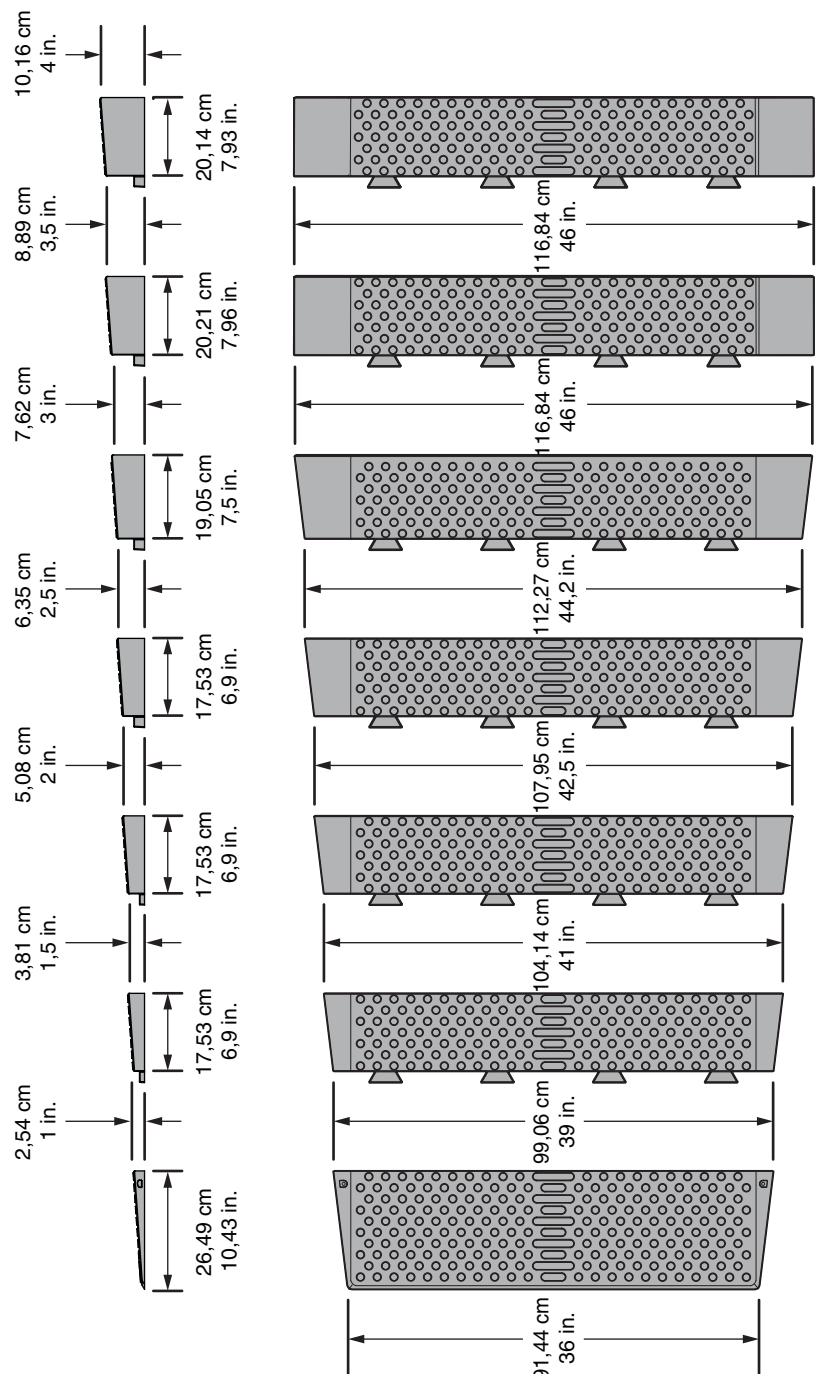
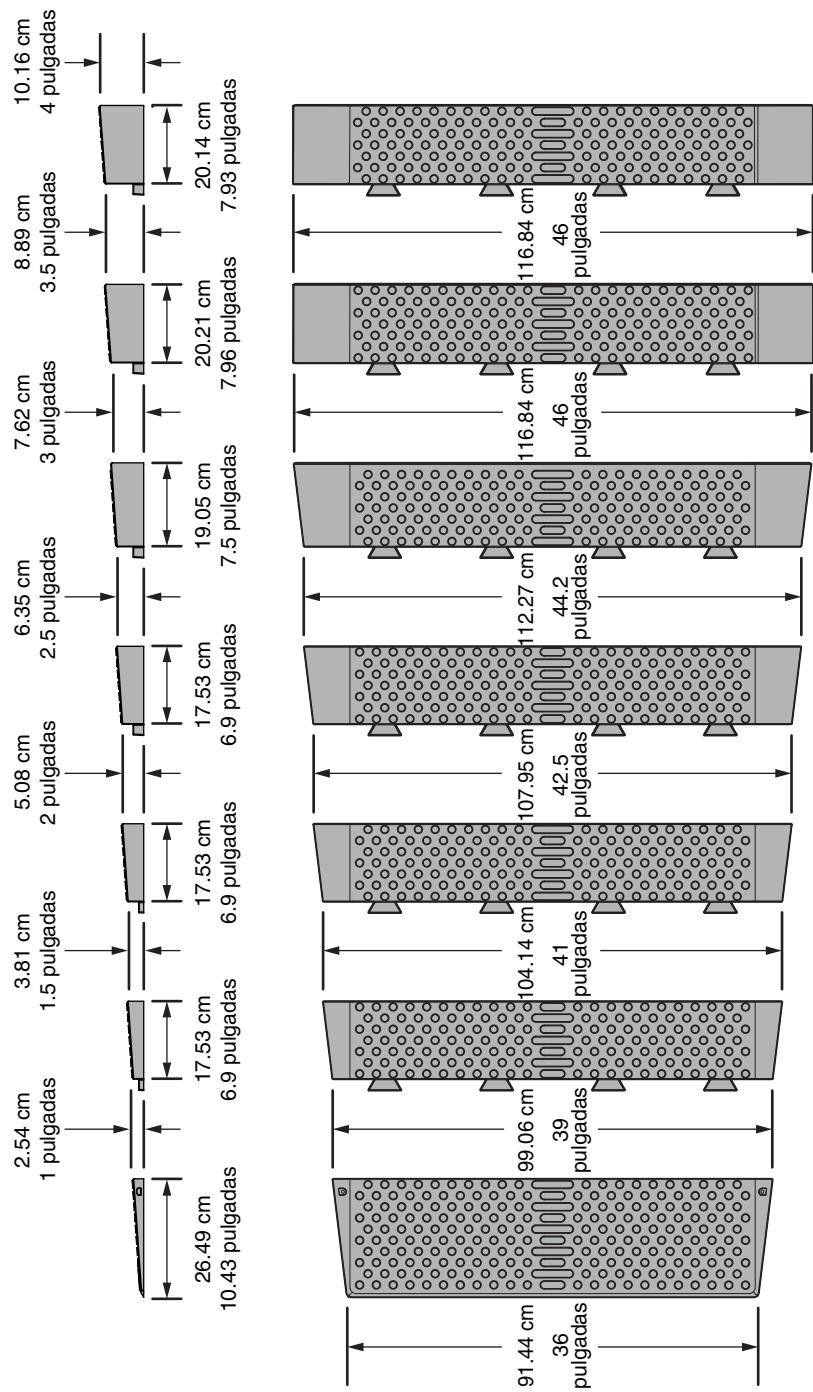
182 Susquehanna Avenue  
Exeter, PA 18643-2694

**Canada**

5096 South Service Road  
Beamsville, Ontario L0R 1B3

**UK**

32 Wedgwood Road  
Bicester, Oxfordshire OX26 4UL

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**B.V.**