


Appendix to MDA/2009/010


Appendix I – Examples of Transit Securement Packages (TSP)

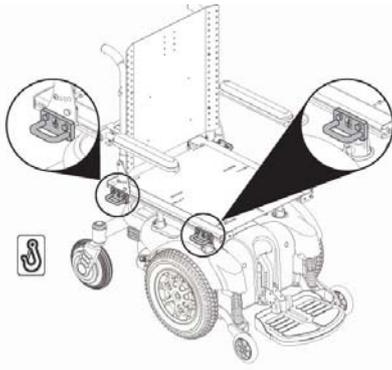


Jazzy 1121 with Synergy Seat




Jazzy 1121 with Modular Seat

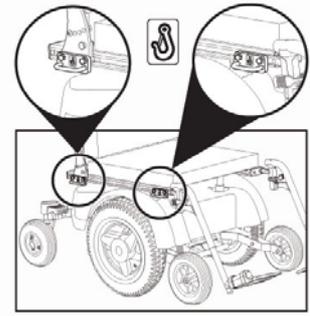


Quantum 1420 with Synergy Seat

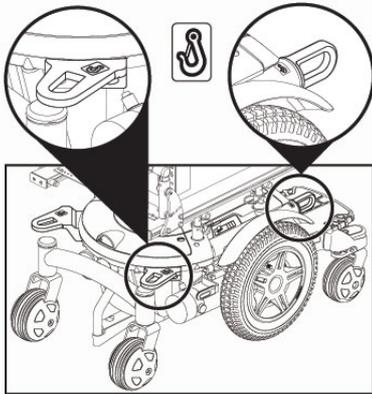


Quantum 600

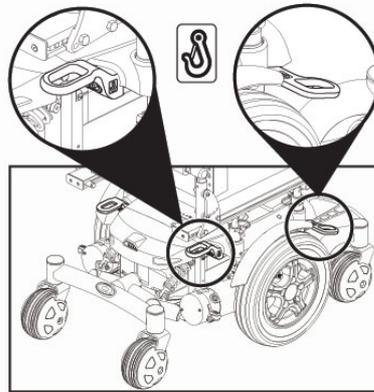


Quantum 6000 / 600 Sport

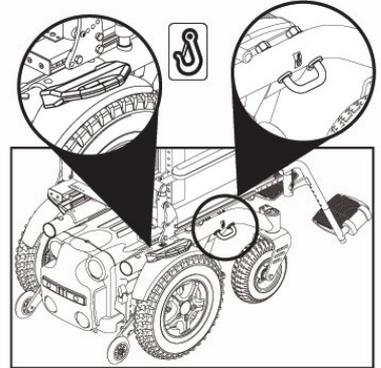

Quantum R4000

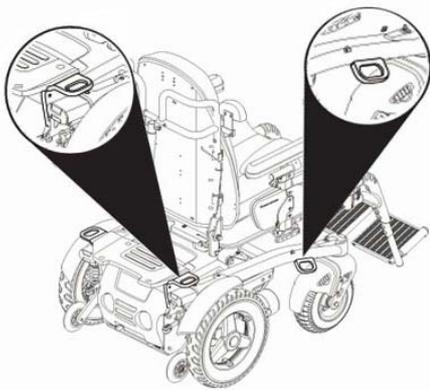

Quantum R400

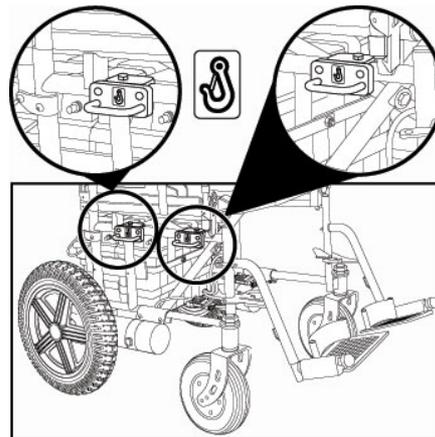

LX

Appendix 2 – Powerchair transportation compatibility chart

POWERCHAIR TRANSPORTATION									
COMPATIBILITY CHART									
	Economy Seat	Euro Seat	Euro Power	Modular Seat	Synergy Seat	Tru Comfort	Synergy Power	Vogue Seating	Vogue Power
Jazzy Range	Options			Options					
Jazzy Select									
Jazzy Select 8									
Jazzy 810									
Jazzy 1103 Ultra				When fitted with TSP	When fitted with TSP	When fitted with TSP			
Jazzy 1107									
Jazzy 1121		When fitted with TSP							
Quantum 8 Range									
Quantum 810				When fitted with TSP	When fitted with TSP	When fitted with TSP			
Quantum 800		When fitted with TSP							
Quantum 800 Sport		When fitted with TSP	When fitted with TSP		When fitted with TSP	When fitted with TSP			
Quantum 8000		When fitted with TSP							
Heavy Duty Range									
Quantum 1420		When fitted with TSP							
Quantum 1660									
Rear Wheel Range									
LX	When fitted with TSP								
Quantum R400					When fitted with TSP	When fitted with TSP		When fitted with TSP	When fitted with TSP
Quantum R4000					When fitted with TSP	When fitted with TSP		When fitted with TSP	When fitted with TSP

Key:

When fitted with TSP	When a Transit Securement Package has been factory-fitted by Pride Mobility before delivery, the wheelchair can be used as a seat in a motor vehicle providing the instructions for use for the wheelchair tiedown and occupant restraint system (WTORS) are followed. Note 1: Without a TSP they are not intended to be used as a seat in a motor vehicle and these wheelchairs should be clearly labelled as such. Note 2: Small metal loops on Jazzy and Quantum seat units are not part of a TSP.
Blank Space	Models not intended by Pride Mobility to be used as a seat in a motor vehicle as no TSP option is available. These wheelchairs should have a label which states that that the user 'should not remain seated in the mobility device while travelling in a motor vehicle'.

If you are unsure of the suitability of your powerchair please contact Pride Mobility on 01869 324600.