



182 Susquehanna Avenue  
Exeter, PA 18643  
570-655-5574  
FAX 570- 883-4139  
[www.pridemobility.com](http://www.pridemobility.com)

---

**To: Pride Mobility Products Ltd. Dealers and Users**  
**From: Research & Development – Pride Mobility Products Corp.**  
**Re: Power Chair Field Safety Notice**  
**Date: Monday, November 12, 2007**

The following Field Safety Notice (FSN) is being distributed to authorised Pride Dealers and users of Pride power chairs equipped with a Transit Securement Package. This FSN will pertain to dealers and users of the following products:

Quantum 6000  
Quantum 6000Z  
Quantum 600  
Quantum R-4000  
Quantum 1420  
Pride LX

Users and dealers of power chairs equipped with a Transit Securement Package should be aware that a supplemental document has been created pertaining to proper securement of an occupied power chair during motor vehicle transport. This document is called the Transit Securement Package Supplement and will be supplied with power chairs that have been properly configured, tested, labelled, and rated for occupied transport in a motor vehicle by the power chair manufacturer. This document will also be distributed by Pride Mobility Products Ltd. to those users who already own an occupied transport approved power chair.

Furthermore, users and dealers must be aware that if the total power chair mass is greater than 125 kg (275 lbs.), then transport of the power chair should be done in a motor vehicle with a gross vehicle weight greater than 4000 kg (8800 lbs.). Currently, this applies to Quantum 6000 and Quantum 1420 power chairs equipped with the Transit Securement Package.

This FSN will be made available on the MHRA web site (<http://www.mhra.gov.uk>). The Transit Securement Package Supplement will be made available on the Pride Mobility Products Ltd. web site (<http://www.pridemobility.com/international/uk/uk.asp>).