DAY ITES

Owner's Manual



How a Power Chair Should Feel!



Exeter, PA St. Catharines, ON

1-800-800-8586

SAFETY GUIDELINES

Please read and follow all instructions in this owner's manual before attempting to operate your power chair for the first time. If there is anything in this manual you do not understand, or if you require additional assistance for set-up, contact your authorized Pride provider.

Using your Pride product safely depends upon your diligence in following the warnings, cautions, and instructions in this owner's manual. Using your Pride product safely also depends upon your own good judgement and/or common sense, as well as that of your provider, caregiver, and/or healthcare professional. Pride is not responsible for injuries and/or damage resulting from any person's failure to follow the warnings, cautions, and instructions in this owner's manual. Pride is not responsible for injuries and/or damage resulting from any person's failure to exercise good judgement and/or common sense.

The symbols below are used throughout this owner's manual to identify warnings and cautions. It is very important for you to read and understand them completely.



WARNING! Failure to heed the warnings in this owner's manual may result in personal injury.



CAUTION! Failure to heed the cautions in this owner's manual may result in damage to your power chair.

Copyright © 2003
Pride Mobility Products Corp.
INFMANU2090

CONTENTS

I.	INTRODUCTION	4
II.	SAFETY	6
III.	EMI/RFI	12
IV.	THE JAZZY 1133	14
V.	DISASSEMBLY	19
VI.	COMFORT ADJUSTMENTS	22
VII.	BATTERIES AND CHARGING	27
VIII	.OPERATION	31
IX.	CARE AND MAINTENANCE	35
X.	OPTIONAL ACCESSORIES	41
XI.	WARRANTY	47

I. INTRODUCTION

INTRODUCTION

Welcome to Pride Mobility Products Corporation (Pride). Congratulations on the purchase of your new Jazzy Power Chair. The Jazzy design combines the most advanced state-of-the-art components with modern, attractive styling. We are certain that the design features and trouble-free operation of your new power chair will add convenience to your daily living.

At Pride, your safety is important to us. Please read and follow all of the instructions in this manual before you attempt to operate your power chair for the first time. These instructions were produced for your benefit. Your understanding of these instructions is essential for the safe operation of your new power chair.

Pride is not liable for damage to property or personal injury arising out of unsafe use of a power chair. Pride is also not liable for any property damage or personal injury arising out of the failure of any person and/or user to following the instructions and recommendations set forth in this manual or any other instructions or recommendations contained in other Jazzy related literature issued by Pride or contained on the power chair itself.

This owner's manual is compiled from the latest specifications and product information available at the time of publication. We reserve the right to make changes as they become necessary. Any changes to our products may cause slight variations between the illustrations and explanations in this manual and the product you have purchased.

If you experience any problems with your power chair that you are unable to solve, or if you do not feel capable of safely following any of the instructions and/or recommendations as contained in this manual, please contact your authorized Pride provider for assistance.

Once you understand how to operate and take care of your power chair, we are certain that it will give you years of trouble-free service and enjoyment.

Information Exchange

We want to hear your questions, comments, and suggestions about this manual. We would also like to hear about the safety and reliability of your new power chair, and about the service you received from your authorized Pride provider.

Please notify us of any change of address, so we can keep you apprised of important information about safety, new products, and new options that can increase your ability to use and enjoy your power chair. Please feel free to contact us at the address below:

Pride Mobility Products Corporation Attn: Customer Care Department 182 Susquehanna Avenue Exeter, PA 18643-2694 customercare@pridemobility.com 800-424-8205

I. INTRODUCTION

Pride Owners Club

My Authorized Pride Provider Is:

As an owner of a Pride product, you are invited to register your product's warranty and enroll in the Pride Owners Club. You may do so by filling out and returning your enclosed registration card or by visiting Pride's web site at www.pridemobility.com. As a registered member, each time you visit our site, you will have access to the most interactive and honest educational venue available today for people with mobility needs, their families, and friends.

From our home page, select "Owners Club" to enter a page dedicated to current and potential Pride product owners. You will gain access to interviews, stories, recreation ideas, daily living tips, product and funding information, and interactive message boards. These message boards invited you to communicate with other Pride customers as well as Pride representatives who are available to assist you with any questions or concerns you may have. You will receive a free gift simply for registering with the Pride Owners Club.

Name:				
Address:				
Phone Number:				
Quick Reference Information:				
Purchase Date:				

NOTE: If you ever lose or misplace your product registration card or your copy of this manual, contact us and we will be glad to send you a new one immediately.

SAFETY



WARNING! Do not operate your new power chair for the first time without completely reading and understanding this owner's manual.

Your power chair is a state-of-the-art life-enhancement device designed to increase mobility. Pride provides an extensive variety of products to best fit the individual needs of the power chair user. Please be aware that the final selection and purchasing decision regarding the type of power chair to be used is the responsibility of the power chair user, who is capable of making such a decision, and his/her healthcare professional (i.e., medical doctor, physical therapist, etc.).

The contents of this manual are based on the expectation that a mobility device expert has properly fitted the power chair to the user and has assisted the prescribing healthcare professional and/or the authorized Pride provider in the instruction process for the use of the product.

There are certain situations, including some medical conditions, where the power chair user will need to practice operating the power chair in the presence of a trained attendant. A trained attendant can be defined as a family member or care professional specially trained in assisting a power chair user in various daily living activities.

As you begin using your power chair during daily activities, you will probably encounter situations in which you will need some practice. Simply take your time and you will soon be in full and confident control as you maneuver through doorways, on and off of elevators, up and down ramps, and over moderate terrain.

Below are some precautions, tips, and other safety considerations that will help the user become accustomed to operating the power chair safely.

Modifications

Pride has designed and engineered your power chair to provide maximum mobility and utility. A wide range of accessories is available from your authorized Pride provider to further customize your power chair to better suit your needs and/or preferences. However, under no circumstances should you modify, add, remove, or disable any feature, part, or function of your power chair.



WARNING! Do not modify your power chair in any way not authorized by Pride. Unauthorized modifications may result in personal injury and/or damage to your power chair.

Pre-Ride Safety Check

Get to know the feel of your power chair and its capabilities. Pride recommends that you perform a safety check before each use to make sure your power chair operates smoothly and safely. See IX. "Care and Maintenance."

Perform the following inspections prior to using your power chair:

- Check for proper tire inflation. Maintain 30-35 psi in each tire (if equipped with pneumatic tires).
- Check all electrical connections. Make sure they are tight and not corroded.
- Check all controller connections to the electronics tray. Make sure they are secured properly.
- Check the brakes. See IX. "Care and Maintenance."
- Check battery charge. See VII. "Batteries and Charging."

NOTE: If you discover a problem, contact your authorized Pride provider for assistance.

Weight Limitations

Your power chair is rated for a maximum weight capacity. Please refer to the specifications table for this limit.



WARNING! Exceeding the weight capacity voids your warranty and may result in personal injury and/or damage to your power chair. Pride will not be held responsible for injuries and/or property damage resulting from failure to observe weight limitations.

WARNING! Do not carry passengers on your power chair. Carrying passengers on your power chair may result in personal injury and/or property damage.

Tire Inflation

If your power chair is equipped with pneumatic tires, you should check or have the air pressure checked at least once a week. Proper inflation pressures will prolong the life of your tires and help ensure the smooth operation of your power chair.



WARNING! It is important that 30-35 psi tire pressure be maintained in pneumatic tires at all times. Do not underinflate or overinflate your tires. Low pressure may result in loss of control, and overinflated tires may burst. Failure to maintain 30-35 psi tire pressure in pneumatic tires at all times may result in tire and/or wheel failure, causing serious personal injury and/or damage to your Jazzy.

WARNING! Inflate your power chair drive tires from a regulated air source with an available pressure gauge. Inflating your tires from an unregulated air source could overinflate them, resulting in a burst tire and/or personal injury.

Incline Information

More and more buildings have ramps with specified degrees of inclination, designed for easy and safe access. Some ramps may have turning switchbacks (180-degree turns) that require you to have good cornering skills on your power chair.

- Proceed with extreme caution as you approach the downgrade of a ramp or other incline.
- Take wide swings with your power chair's front wheels around any tight corners. If you do that, the power chair's rear wheels will follow a wide arc, not cut the corner short, and not bump into or get hung up on any railing corners.
- When driving down a ramp, keep the power chair's speed adjustment set to the slowest speed setting to ensure a safely controlled descent. See VIII. "Operation."
- Avoid sudden stops and starts.

When climbing an incline, try to keep your power chair moving. If you must stop, start up again slowly and then accelerate cautiously. When driving down an incline, set your power chair to the slowest setting and drive in the forward direction only. If your power chair starts to move down the incline faster than you anticipated or desired, allow it to come to a complete stop by releasing the joystick, then push the joystick forward slightly to ensure a safely controlled descent.

WARNING! When climbing an incline, do not zigzag or drive at an angle up the face of the incline. Drive your power chair straight up the incline. This greatly reduces the possibility of a tip or a fall. Always exercise extreme caution when negotiating an incline.



WARNING! You should not travel up or down a potentially hazardous incline (i.e., areas covered with snow, ice, cut grass, or wet leaves).

WARNING! When on any sort of an incline or decline, never place the power chair in freewheel mode while seated on it or standing next to it. Doing so may result in personal injury and/or damage to your power chair.

WARNING! Never travel down an incline backwards. This may result in personal injury.

WARNING! Even though your power chair is capable of climbing slopes greater than those illustrated in figure 1, do not, under any circumstances, exceed the incline guidelines or any other specifications presented in this manual. Doing so could cause instability in your power chair, resulting in personal injury and/or damage to your power chair.

In compliance with the Americans with Disabilities Act of 1990, all handicap public access ramps are required to have a maximum slope of 5°. Therefore, Pride recommends that the maximum slope of an incline you attempt to safely ascend or descend on your power chair does not exceed 5°. See figure 1.



WARNING! Any attempt to climb or descend a slope steeper than 5° may put your power chair in an unstable position and cause it to tip, resulting in personal injury.

Braking Information

Your power chair is equipped with two powerful brake systems:

- 1. Regenerative uses electricity to rapidly slow the vehicle when the joystick returns to the center/stop position.
- 2. Disc Park Brake activates mechanically after regenerative braking slows the vehicle to near stop, or when power is removed from the system for any reason.

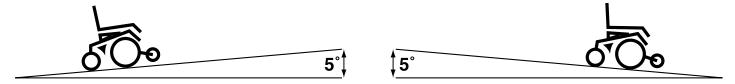


Figure 1. Maximum Safe Angle (Ascending and Descending)

Cornering Information

While your power chair is equipped with rear caster wheels in back and anti-tip wheels in front, excessively high cornering speeds can still create the possibility of tipping. Factors which affect the possibility of tipping include, but are not limited to: cornering speed, steering angle (how sharply you are turning), uneven road surfaces, inclined road surfaces, riding from an area of low traction to an area of high traction (such as passing from a grassy area to a paved area – especially at high speed while turning), and abrupt directional changes. High cornering speeds are not recommended. If you feel that you may tip over in a corner, reduce your speed and steering angle (i.e., lessen the sharpness of the turn) to prevent your power chair from tipping.



WARNING! When cornering sharply, reduce your speed. This greatly reduces the possibility of a tip or fall. To avoid personal injury and/or property damage, always exercise common sense when cornering.

Outdoor Driving Surfaces

Your power chair is designed to provide optimum stability under normal driving conditions—dry, level surfaces composed of concrete, blacktop, or asphalt. However, Pride recognizes that there will be times when you will encounter other surface types. For this reason, your power chair is designed to perform admirably on packed soil, grass, and gravel. Feel free to use your power chair safely on lawns and in park areas.

- Reduce your power chair's speed when driving on uneven terrain and/or soft surfaces.
- Avoid tall grass that can entangle the running gear.
- Avoid loosely packed gravel and sand.
- If you feel unsure about a driving surface, avoid that surface.

Freewheel Mode

Your power chair is equipped with a manual freewheel lever to allow for manual maneuverability by a trained attendant. For more information about how to place your power chair into and out of freewheel mode, see IV. "The Jazzy 1133."

WARNING! Do not use your power chair in freewheel mode without an attendant present. Personal injury may result.




WARNING! Do not attempt to personally place your power chair in freewheel mode while seated on it. Personal injury may result. Ask an attendant for assistance if necessary.

WARNING! Do not place your power chair in freewheel mode while on an incline. The chair could roll uncontrollably on its own, causing personal injury.

Stationary Obstacles (Steps, Curbs, etc.)

Proceed with extreme caution when driving near raised surfaces, unprotected ledges and/or drop-offs (curbs, porches, stairs, etc.). The correct method for approaching a curb is illustrated in figure 2.

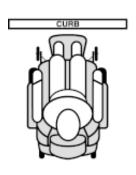







WARNING! Do not attempt to have your power chair climb or descend an obstacle that is higher than two inches unless you have the assistance of an attendant.

WARNING! Do not attempt to have your power chair proceed backward down any step, curb, or other obstacle. This may cause the power chair to tip and cause personal injury.

Public Streets and Roadways



WARNING! You should not operate your power chair on public streets and roadways. Be aware that it may be difficult for traffic to see you when you are seated on your power chair. Obey all local pedestrian traffic rules. Wait until your path is clear of traffic, and then proceed with extreme caution.

Stairs and Escalators

Power chairs are not designed to travel up or down stairs or escalators. Always use an elevator.



WARNING! Never use your power chair to negotiate steps or escalators. You may cause injury to yourself and to others and/or damage your power chair.

Doors

- Determine if the door opens toward or away from you.
- Drive your power chair gently and slowly forward to push the door open. Or drive your power chair gently and slowly backward to pull the door open.

Elevators

Modern elevators have a door edge safety mechanism that, when pushed, reopens the elevator door(s).

- If you are in the doorway of an elevator when the door(s) begin to close, push on the rubber door edge or allow the rubber door edge to contact the power chair and the door will reopen.
- Use care that pocketbooks, packages, or power chair accessories do not become caught in elevator doors.

EMI & RFI

Laboratory tests performed by the Food and Drug Administration (FDA) have shown that radio waves can cause unintended motion of electric mobility vehicles. Radio waves are a form of electromagnetic energy (EM). When EM adversely affects the operation of an electronic device, it is called Electromagnetic Interference (EMI) or Radio Frequency Interference (RFI). For more information, see III. "EMI/RFI."

Lift/Elevation Products

If you will be traveling with your power chair, you may find it necessary to use a lift/elevation product to aid in transportation. Pride recommends that you closely review the instructions, specifications, and safety information set forth by the manufacturer of the lift/elevation product before using that product.

Motor Vehicle Transport

Currently, there are no standards approved for tie-down systems in a moving vehicle of any type to transport a person while seated in a power chair.

Transfers

Transferring onto and off of your power chair requires a good sense of balance. Always have an attendant or healthcare professional present while learning to properly transfer yourself.

To eliminate the possibility of injury, Pride recommends that you or a trained attendant perform the following tasks before attempting a transfer:

- Turn the power off. See VIII. "Operation."
- Ensure your power chair is not in freewheel mode. See IV. "The Jazzy 1133."
- Turn both caster wheels toward the transfer destination to improve power chair stability during transfer.
- Make sure both armrests are flipped up or removed from your power chair.
- Flip the footrest up, or move the leg rests aside; this will help to keep your feet from getting caught on the footrest or the leg rests during the transfer.
- Reduce the distance between your power chair and the object you are transferring onto.

WARNING! Before transferring, position yourself as far back as possible in the power chair seat to prevent the power chair from tipping forward during transfer and causing injury.



WARNING! Avoid using your armrests for weight bearing purposes. Such use may cause the power chair to tip and cause personal injury.

WARNING! Avoid putting all of your weight on the footrest. Such use may cause the power chair to tip and cause personal injury.

Positioning Belts

Your authorized Pride provider, therapist(s), and other healthcare professionals are responsible for determining your requirement for a positioning belt in order to operate your power chair safely.

WARNING! If you require a positioning belt to safely operate your power chair, make sure it is fastened securely. Serious personal injury may result if you fall from the power chair.



WARNING! The positioning belt is not designed for use as a seat belt in a motor vehicle. Nor is your Jazzy suitable for use as a seat in any vehicle. Anyone traveling in a vehicle should be properly belted into seats approved by the vehicle manufacturer.

WARNING! Do not sit on your power chair while it is in a moving vehicle. Personal injury and/or property damage may result.

WARNING! Always be sure your power chair and its batteries are properly secured when it is being transported. Failure to do so may result in personal injury and/or damage to your power chair.

Inclement Weather Precautions



WARNING! Pride recommends that you do not operate your power chair in icy or slippery conditions or on salted surfaces (i.e., walks or roads). Such use may adversely affect the performance and safety of your power chair, resulting in an accident and personal injury.

WARNING! Do not expose your power chair to any type of moisture at any time (rain, snow, mist, or wash). Such exposure can damage your power chair. Never operate your power chair if it has been exposed to moisture until it has dried thoroughly.

Reaching and Bending

Never reach, lean, or bend while driving your power chair. If it is absolutely necessary to reach, lean, or bend while seated on your power chair, it is important to maintain a stable center of gravity and keep the power chair from tipping. Pride recommends that the power chair user determine his/her personal limitations and practice bending and reaching in the presence of a qualified healthcare professional.



WARNING! Do not bend, lean, or reach for objects if you have to pick them up from the floor by reaching down between your knees. Movements such as these may change your center of gravity and the weight distribution of the power chair. This may cause your power chair to tip, possibly resulting in personal injury. Keep your hands away from the tires when driving.

Batteries

In addition to following the warnings below, be sure to comply with all other battery handling information. For more information about your power chair's batteries, see VII. "Batteries and Charging."

WARNING! Power chair batteries are heavy. See specifications table. If you are unable to lift that much weight, be sure to get help. Lifting beyond your capacity can result in personal injury.



WARNING! Battery posts, terminals, and related accessories contain lead and lead compounds. Wash hands after handling.

WARNING! Always protect the batteries from freezing and never charge a frozen battery. Charging a frozen battery may result in personal injury and/or damage to the battery.

Preventing Unintended movement



WARNING! If you anticipate being seated in a stationary position for an extended period of time, turn off the power. This will prevent unexpected motion from inadvertent joystick contact. This will also eliminate the possibility of unintended chair movement from electromagnetic (EM) sources. Failure to do so may result in personal injury.

Prescription Drugs/Physical Limitations

Users must exercise care and common sense when operating a power chair. This includes awareness of safety issues when taking prescribed or over-the-counter drugs or when the user has specific physical limitations.



WARNING! Consult your physician if you are taking prescribed or over-the-counter medication or if you have certain physical limitations. Some medications and limitations may impair your ability to operate your power chair in a safe manner.

Alcohol

The power chair user must exercise care and common sense when operating his/her power chair. This includes awareness of safety issues while under the influence of alcohol.



WARNING! Do not operate your power chair while you are under the influence of alcohol, as this may impair your ability to operate your power chair in a safe manner.

Removable Parts



WARNING! Do not attempt to lift or move a power chair by any of its removable parts. Personal injury and/ or damage to the power chair may result.

III. EMI/RFI

EMI/RFI

Laboratory tests performed by the Food and Drug Administration (FDA) have shown that radio waves can cause unintended motion of power chairs. Radio waves are a form of electromagnetic energy. When this energy adversely affects the operation of an electronic device, it is called Electromagnetic Interference (EMI) or Radio Frequency Interference (RFI).



WARNING! Radio waves may interfere with the control of power chairs.

FREQUENTLY ASKED QUESTIONS (FAQS)

The following FAQs summarize what you should know about EMI/RFI. Use this information to minimize the risk that EMI/RFI will affect your power chair.

Where do radio waves come from?

Radio waves are emitted from the antennas of cellular phones, mobile two-way radios (such as walkie-talkies), radio stations, TV stations, amateur radio (HAM) transmitters, wireless computer links, microwave sources, and paging transmitters. Radio waves are a form of electromagnetic energy. Because electromagnetic energy is more intense closer to the transmitting antenna (source of emission), the electromagnetic energy fields from two-way radios are of special concern to power chair users.

If my power chair is affected by EMI/RFI, what kind of motion should I expect?

This is hard to predict. It would depend on a number of factors:

- Strength of the radio waves
- Construction of the power chair
- Position of the power chair (whether it is on level ground or on an incline)
- Whether or not the power chair is in motion

An affected power chair's movement can be erratic. It may move by itself or come to a sudden stop. Furthermore, it is possible for EMI/RFI to unexpectedly release the brakes on a power chair. Some intense sources of EMI/RFI can even damage the control system of the Jazzy.

Is there any way to know for sure whether radio waves are responsible for the unintended motion of my power chair?

Unfortunately, interference from radio wave sources may be difficult to recognize, since the signals from these sources are invisible and may be intermittent. However, the FDA recommends that you report all incidents of unintended motion or brake release to the power chair manufacturer and, if possible, note whether there was a radio wave source nearby at the time of the incident.

Has anyone been hurt from erratic, unintended motion of power chairs?

The FDA has reports of injuries that resulted from the apparent uncontrolled motion of power chairs. However, it is unclear how many of these incidents were actually caused by radio wave interference.

III. EMI/RFI

Are all power chairs susceptible to EMI/RFI?

Each make and model of chair differs in its ability to resist electromagnetic interference. That is, each has a particular level of "immunity" to interference, measured in volts per meter (V/m). A higher immunity level offers greater protection. In other words, a power chair with a high immunity level is less likely to be affected by a strong radio source than one with a low immunity level.

What is the FDA doing about the problem?

The FDA has written to the manufacturers of power chairs and requested that they test their new products to be sure that they provide a reasonable degree of immunity against EMI/RFI. The letter states that power chairs should have an immunity of at least 20 V/m. This provides a reasonable degree of protection against the common sources of EMI/RFI.

The FDA has also requested that these manufacturers clearly label their products with the immunity level, or state that the immunity level is not known. The labeling and informational material supplied with the power chair must explain what the immunity level means, and the labeling or informational material must warn users about the possibility of EMI/RFI and how to avoid it. In addition, the FDA has recommended that manufacturers establish an educational program to inform users of power chairs, and their caregivers, about the problems associated with EMI/RFI, and the actions they can take to minimize the risk.

What can I do to find out if my power chair is likely to be affected by EMI/RFI?

If you've had your power chair for a long time and haven't experienced any unintended motion, it is not likely that you will have problems in the future. But it is always possible that problems could arise if you are close to a source of radio waves. Therefore, it is very important to be alert to this possibility. The Jazzy meets or exceeds an immunity level of at least 20 V/m.

What can I do to reduce the risk that my power chair could be affected by EMI/RFI?

Here are some precautions you can take:

- Do not turn on or use hand-held personal communications devices, such as citizens band (CB) radios and cellular phones, while the power chair is on.
- Be aware of nearby transmitters, such as radio or TV stations and hand-held or mobile two-way radios, and try not to come close to them. For example, if you are in a power chair with an immunity level of 20 V/m, you should remain at least three feet from a hand-held two-way radio and ten feet from a mobile two-way radio.
- Be aware that adding accessories and/or components, or modifying the power chair may make it more susceptible to interference from radio wave sources. There is no easy way to evaluate their effect upon the overall immunity of the power chair.

What should I do if my power chair moves unexpectedly?

If unintended motion or brake release occurs, turn the power chair off as soon as it is safe to do so. Call Pride at 800-424-8205 to report the incident.

14

THE JAZZY 1133

The Jazzy 1133 has three main assemblies: the power base, the seat assembly, and the controller assembly. See figures 3, 4, 5, and 6. Information on the standard seat assembly is in VI. "Comfort Adjustments." Information on the controller assembly is in VIII. "Operation."



Figure 3. The Jazzy 1133

	JAZZY 1133 SPECIFICATIONS				
Suspension:	Full suspension with Active-Trac				
Drive Wheels:	14 in., pneumatic (solid tires are optional)				
Caster Wheels:	8 in., solid, rear articulating				
Anti-tip Wheels:	6 in., solid, front mounted				
Maximum Speed:	Programmable up to 5 mph				
Brakes:	"Intelligent Braking," electronic regenerative, disc park brake				
Ground Clearance:	3.5 in.				
Turning Radius:	21.25 in.				
Overall Size:	Length: 41 in.				
	Width: 24.5 in.				
Seating Options:	Medium Back (standard)				
	Synergy (optional)				
	Versa Tilt (optional)				
	High Back (optional)				
	Reclining (optional)				
Drivetrain:	Two motor, mid-wheel				
Batteries:	Two 12-volt, NF-22 batteries				
Range:	Up to 25 miles				
Battery Charger:	3-amp, onboard				
	Off-board (optional)				
Electronics:	50-amp P & G VSI Controller				
Weight Capacity:	300 lbs.				
Component Weights:	Right Power Base Frame: 46 lbs.				
	Left Power Base Frame: 49 lbs.				
	Front Battery Box: 42 lbs. (w/ NF-22 battery)				
	Back Battery Box: 49 lbs. (w/ NF-22 battery)				
	Total Power Base Weight: 181 lbs. (w/ batteries)				
	Medium-Back Seat: 33 lbs.				
	Batteries: 38 lbs. each				
Warranty:	Lifetime limited warranty on frame				
	Two-year warranty on electronics				
	Eighteen-month warranty on drive motors				

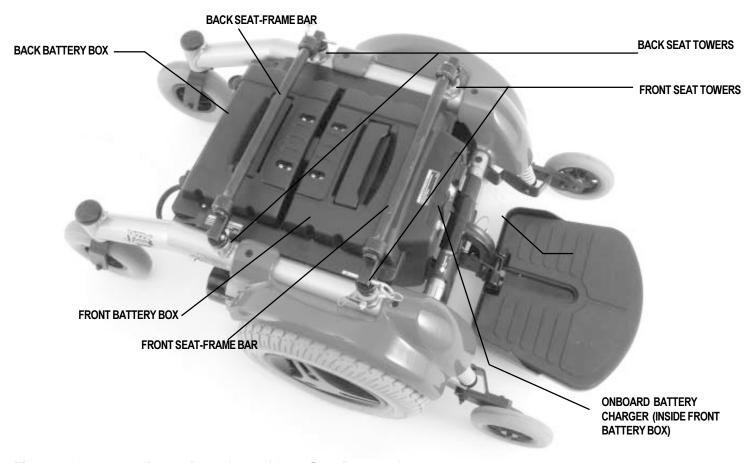


Figure 4. Jazzy 1133 Power Base Assembly — Seat Removed

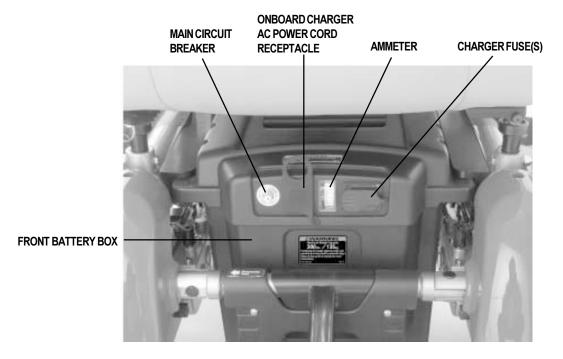


Figure 5. Jazzy 1133 — Front of Power Base

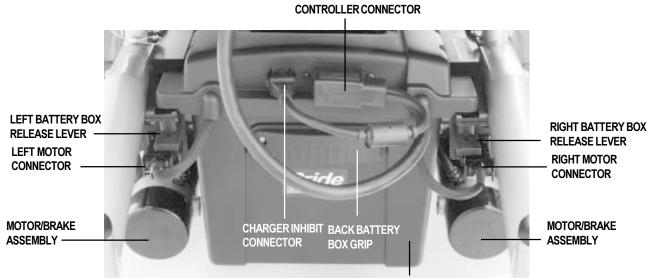


Figure 6. Jazzy 1133 — Back of Power Base

BACK BATTERY BOX

The Power Base Assembly

The power base assembly consists of the right and left frame assemblies, the front and back battery boxes, and the seat bars. See figures 4, 5, and 6. The battery boxes provide connectivity for the controller, the onboard battery charger, and the batteries. The front battery box contains the onboard charger, the ammeter, the onboard charger AC power cord receptacle, and the main circuit breaker. The back battery box contains the controller connector and the charger inhibit connector.

Ammeter: The ammeter displays the charger's current output in amps. See VII. "Batteries and Charging."

Onboard Charger AC Power Cord Receptacle: This is where the AC power cord plugs into the onboard charger. The AC power cord plugs into a standard wall outlet.

Charger Fuse(s): The charger fuse protects the ammeter from overload during charging. A spare fuse is included.

Main Circuit Breaker: The main circuit breaker is a safety feature built into your power chair. When the batteries and the motors are heavily strained (e.g., from excessive loads), the main circuit breaker trips to prevent damage to the motors and the electronics. If the circuit breaker trips, allow your power chair to "rest" for approximately one minute. Then push in the circuit breaker reset button, turn on the controller, and continue normal operation. If the main circuit breaker trips repeatedly, contact your authorized Pride provider.

Controller Connector: This is where the controller connects to the battery, the motors, and the motor brakes. The standard controller connector is shown.

Charger Inhibit Connector: The controller is equipped with a charger inhibit. The charger inhibit enables the onboard battery charger to disable the controller during charging. The charger inhibit connector is coded with colored dots. The dots are positioned so that you can align the flat side of the male connector with the flat side of the female connector before making the connection.



CAUTION! Failure to properly align the connectors can result in damage to the controller, the onboard charger, and the connectors.

Manual Freewheel Levers

For your convenience, your power chair is equipped with two manual freewheel levers - one for each motor/brake assembly. See figure 7. These levers allow you to disengage the drive motors and maneuver the chair manually.



WARNING! Do not use your power chair while the drive motors are disengaged unless you are in the presence of an attendant! Do not disengage the drive motors when your power chair is on an incline. The chair could roll down on its own, causing injury!

To engage or disengage the drive motor:

- 1. Pull the manual freewheel lever up to engage the drive motor.
- 2. Push the manual freewheel lever down to disengage the drive motor.



CAUTION! It is important to remember that when your power chair is in freewheel mode, the electronic braking system is disengaged.

NOTE: If a lever is difficult to move in either direction, rock your power chair back and forth slightly. The levers should then move to the desired position.

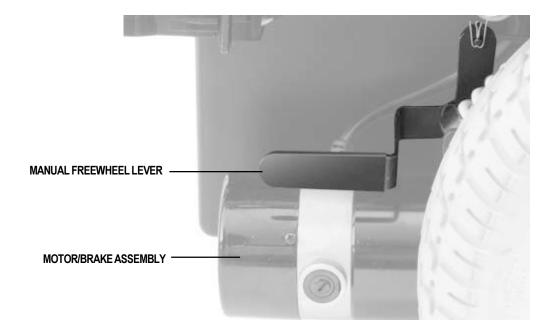


Figure 7. Jazzy 1133 Manual Freewheel Lever (right side shown)

V. DISASSEMBLY

DISASSEMBLY

You can disassemble your Jazzy 1133 into six pieces (seat not shown) for easy transport. See figure 8.

NOTE: Assembly is the reverse of disassembly.

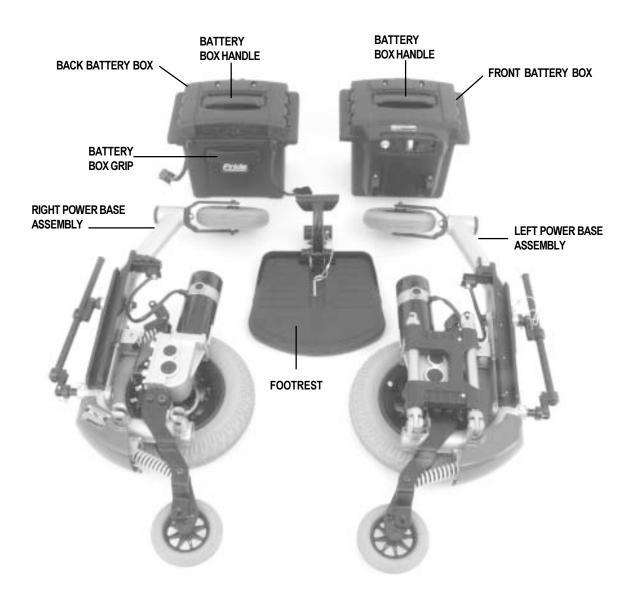


Figure 8. Jazzy 1133 — Power Base Components



WARNING! Some power chair components are heavy. You may need assistance to lift or carry them. Please refer to IV. "The Jazzy 1133" for specific component weights before you disassemble the power chair.

- 1. Make sure that the power chair is in drive mode. See IV. "The Jazzy 1133."
- 2. Unplug the controller and charger inhibit connectors from the back battery box. See figure 6.
- 3. Push the seatback release lever down and fold down the seatback. The seatback release lever is located on the right side of the seat. See figure 3.

V. DISASSEMBLY

4. Flip up the safety latch.



WARNING! Make sure latch is flipped down before using your power chair.

- 5. Squeeze together the two seat latches and lift the seat up enough to release it from the front seat bar. See figure 9.
- 6. Hold the seat at the base of the seatback and the front of the seat base.
- 7. Lift the seat off the power base and set it aside.



WARNING! Do not pick up the seat frame by the armrests. They are free to pivot, and you may lose control of the seat if they do so, resulting in personal injury and/or damage to the chair.

- 8. Unplug the left and the right motor connectors. See figure 6.
- 9. Push down both battery box release levers. See figure 10.



CAUTION! Make sure that both battery box release levers are in the up (locked) position during reassembly.

- 10. With one hand, hold the front battery box by the handle. See figure 8.
- 11. With the other hand, hold the back battery box by the grip on the back (see figure 6) and pull it rearwards until you disconnect it from the front battery box.
- 12. Slide the rear battery box off the frame and set aside.
- 13. Slide the front battery box rearward and lift it off the frame. Set it aside.
- 14. Remove the ball detent pin from the footrest bracket. See figure 11.

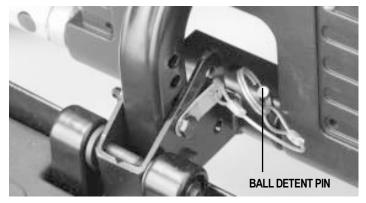


Figure 11. Footrest Bracket

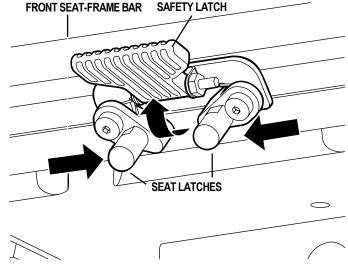


Figure 9. Seat Latches

BATTERY BOX RELEASE LEVER (IN DOWN POSITION)



Figure 10. Back Battery Box Release Lever

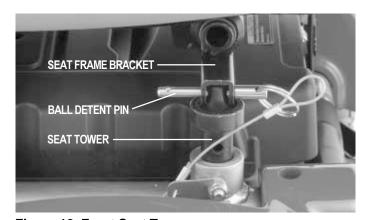


Figure 12. Front Seat Tower

V. DISASSEMBLY

- 15. Pull the footrest up and off the frame and set it aside.
- 16. Remove the ball detent pin from the right and left rear seat towers. See figure 12.
- 17. Rotate the front seat-frame bar back to the left rear seat tower until it is parallel to the power base frame and slide the front seat-frame bracket along the front seat-frame bar until it lines up with the left rear seat tower. See figure 13.
- 18. Reinstall the ball detent pin into the left rear seat tower.
- 19. Rotate the back seat-frame bar back to the right front seat tower until it is parallel to the power base frame and slide the back seat-frame bracket along the back seat-frame bar until it lines up with the right front seat tower. See figure 13.
- 20. Reinstall the ball detent pin into the right front seat tower.


NOTE: If necessary, rotate the seat towers until they line up with the seat frame brackets.

NOTE: The seat-frame bar now serves a handle to carry each power base frame.

- 21. Hold the right power base by the seat frame bar.
- 22. Hold the front crossbar with the other hand. See figure 14.
- 23. Rotate the right power base frame down and pull it away from the left power base frame. See figure 14.
- 24. Lift up the crossbar until it separates from the right power base assembly.
- 25. Press the crossbar release latch and rotate the crossbar toward the left power base frame until the cross bar latch clicks. See figures 15 and 16.



Figure 15. Crossbar Release Latch

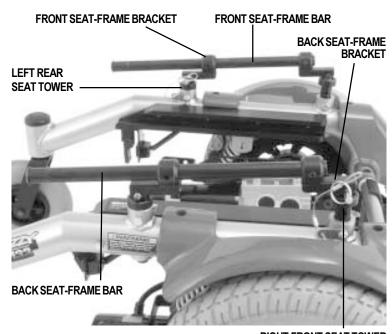

RIGHT FRONT SEAT TOWER





Figure 14. Crossbar



Figure 16. Crossbar in Folded Position

COMFORT ADJUSTMENTS

After becoming familiar with your power chair's operation, you may find the need to make some adjustments to increase your comfort, such as seat height and angle, armrest angle, footrest height and angle, and the controller's position. If your power chair is equipped with a Synergy Seat or the Versa Tilt, refer to the information provided in separate manuals. If your power chair is equipped with a medium back, a high back, or a reclining seat, refer to the following information.



WARNING! If your Jazzy was configured at your authorized Pride provider or service center, please consult your healthcare professional before changing the seat position or making any other adjustment. Some adjustments may degrade your Jazzy's performance and safety by changing its center of gravity.

You may need the following to make comfort adjustments:

- metric/standard hex key set
- metric/standard socket set and ratchet
- adjustable wrench

Seat Height and Seat Angle Adjustment

There are four steel towers that connect the seat to the power base. These seat towers are fastened to the base with nuts and bolts. See figure 17. You can change the seat height to one of three positions in 1-in. increments by removing these nuts and bolts and raising or lowering the seat towers. If you raise or lower only one set of seat towers (either front set or rear set), you can also change the seat base angle.

To change the seat height or seat angle:

- 1. Remove the nuts and bolts from each of the four seat towers.
- 2. Raise or lower each seat pin to the desired position. To change the angle, set either the front or rear seat towers higher or lower than the other.
- 3. Reinstall the ball detent pin into each seat tower.

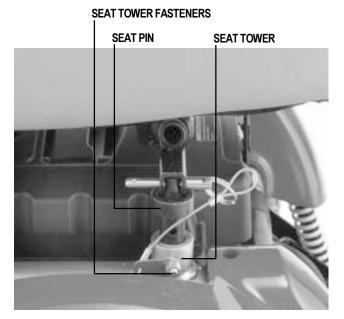


Figure 17. Seat Tower Detail

Seat Position

You can move the seat forward or rearward by changing the extrusion mounting position.

NOTE: To change the seat position, you must remove the seat.

To change the position:

- 1. Turn off the power to the controller.
- 2. Remove the seat. See V. "Disassembly."
- 3. Remove both extrusions from the bottom of the seat. See figure 18.
- 4. Reposition the extrusions on a different set of mounting holes. See figure 18. You must move both extrusions the same number of holes either forward or backward.
- 5. Reinstall the extrusions onto the bottom of the seat.
- 6. Reinstall the seat.

Reclining Seat

If your power chair is equipped with a reclining seat, you can adjust the seatback angle with the seatback release lever. The lever is located on the right side of the seat base. See figure 3.

To adjust the seatback angle:

- 1. Push down on the seatback release lever.
- 2. Move the seatback down or up to the desired position.
- 3. Release the lever.

Seatback Angle Adjustment (no seatback release lever)

If your Jazzy is equipped with an adjustable seatback, you can adjust it to four (4) different angles: 90°, 102°, 105°, or 107°.

To adjust the seatback angle:

- 1. Remove the adjusting screws on both seat hinges. See figure 19.
- 2. Set the seatback at desired angle.
- 3. Reinstall the screws on both seat hinges and tighten.

Armrest Width Adjustment

You can change each armrest's width independently of each other.

NOTE: Changing the armrest width may increase the overall width of your Jazzy.

To change the armrest width:

- 1. Locate the two armrest knobs on each side of the armrest receiver bracket. See figure 19.
- 2. Loosen the knobs.
- 3. Slide the armrests in or out to the desired width.
- 4. Tighten the knobs.

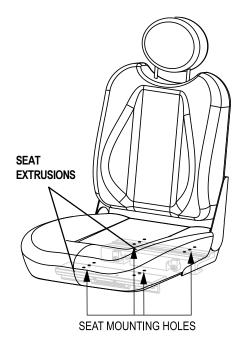


Figure 18. Seat Mounting Holes

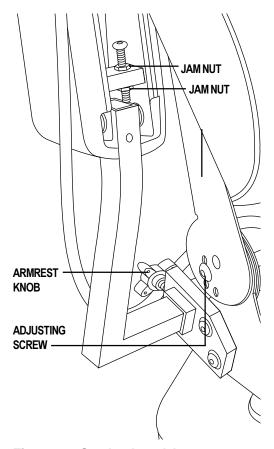


Figure 19. Seatback and Armrest Adjustments

Armrest Angle Adjustment

To change the armrest angle:

- 1. Lift the armrest straight up so that it is perpendicular to the floor.
- 2. Loosen the jam nuts. See figure 19.
- 3. Loosen the adjusting screw.
- 4. Turn the adjusting screw clockwise to raise the front of the armrest, or turn the adjusting screw counterclockwise to lower the front of the armrest. See figure 19.
- 5. Tighten the jam nuts to lock the adjusting screw into place.

Footrest Height

The footrest height is easily adjusted to one of four different heights in 1-in, increments.

To raise or lower the footrest:

- 1. Remove the fasteners from footrest bracket. See figure 20.
- 2. Raise or lower the footrest to the desired height.
- 3. Reinstall the fasteners into the footrest bracket and tighten.

Footrest Depth

To adjust the footrest depth:

- 1. Remove the fasteners from the footrest bracket. See figure 20.
- 2. Move the footrest in or out to the desired depth.
- 3. Reinstall the fasteners into the bracket and tighten.

Footrest Angle

You can adjust the angle of the footrest with a hex key. See figure 21.

To adjust the footrest angle:

- 1. Turn the setscrew counterclockwise to raise the front of the footrest.
- 2. Turn the setscrew clockwise to lower the front of the footrest.

Swing-away Footrests (Optional Equipment)

Swing-away Footrests (SFRs) enable you to rotate the leg rests to the side before you transfer on or off your Jazzy.

To move the SFRs:

- 1. Push in the release lever. See figure 22.
- 2. Rotate the SFRs.

To adjust the SFR length:

- 1. Remove the two screws from the side of each leg rest extension. See figure 24.
- 2. Slide the leg rest in or out to the desired length.
- 3. Tighten the bolt and reinstall the two screws.

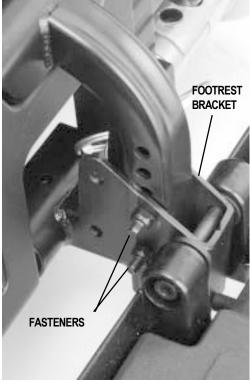


Figure 20. Footrest Height and Depth Adjustment

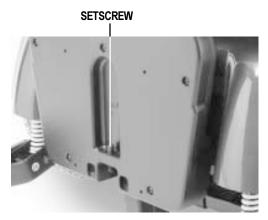


Figure 21. Bottom of Footrest



Figure 22. Swing-Away Footrests



Figure 23. Elevating Leg Rests

Elevating Leg Rests (Optional Equipment)


Elevating Leg Rests (ELRs) offer an infinite range of adjustment for the leg angle and a footrest adjustment range of 12–19 in.

To adjust the ELR angle:

- 1. Press the release lever. See figure 23.
- 2. Move the leg rest to the desired angle.

To adjust the ELR length:

- 1. Remove the two screws from the side of each leg rest extension. See figure 24.
- 2. Slide the leg rest in or out to the desired length.
- 3. Reinstall the two screws and tighten.

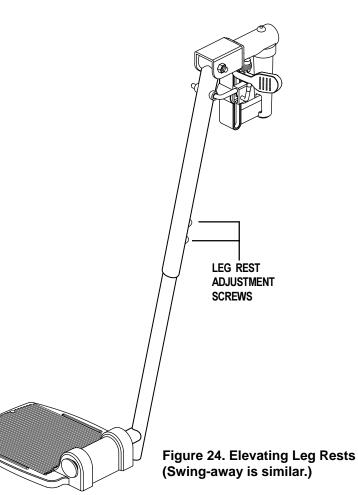

Controller Position

You can position the controller for either left-hand or right-hand use.

To change the controller position:

- 1. Unplug the controller and the charger inhibit connectors from the back battery box. See figure 6.
- 2. Cut the wire tie(s) that attach the controller cable to the seat.
- 3. Use a hex key to loosen the setscrew underneath the armrest. See figure 25.
- 4. Slide the controller out of the armrest.
- 5. Loosen the setscrew underneath the other armrest.
- 6. Place the controller in the other armrest.
- 7. Tighten the setscrew.
- 8. Plug in the controller and the charger inhibit connectors to the back battery box.
- 9. Use a wire tie to secure the controller cable to the armrest.

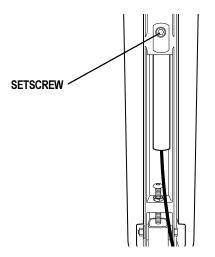


Figure 25. Underside of Armrest

Anti-Tip Wheels

The anti-tip wheels are designed to give your power chair increased stability on rough surfaces. The anti-tip wheels are preset at the factory for use on smooth surfaces or indoors. If you plan on using your power chair on rough surfaces, it may be necessary to adjust the anti-tip wheels to better suit your needs. The anti-tip wheels may need adjustment if either of the following occur:

- When coming to a stop, your power chair tips forward excessively.
- The anti-tip wheels constantly rub the ground.

NOTE: Make sure that each drive tire has at least 30 psi and the user is seated in the power chair.

WARNING! The higher you raise the anti-tip wheels, the more tendency your power chair has to tilt forward when coming to a stop. You can compensate for this by having your authorized Pride provider make a small adjustment to the pre-programmed deceleration setting in the controller or by moving the seat assembly further to the rear of your power chair.



WARNING! Consult your authorized Pride provider before attempting to change the anti-tip wheel height! Changing the anti-tip wheel height affects handling under deceleration!

To adjust the anti-tip wheels:

- 1. Remove the anti-tip wheel. See figure 26.
- 2. Remove the nut and bolt on the anti-tip wheel bracket. See figure 26.
- 3. Raise or lower the anti-tip wheel.
- 4. Reinstall the nut and bolt on the anti-tip wheel bracket.
- 5. Reinstall the wheel.
- 6. Repeat the same procedure for the other anti-tip wheel.

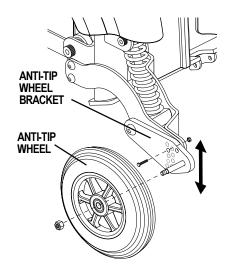


Figure 26. Anti-Tip Wheel Assembly

BATTERIES AND CHARGING

Your power chair uses two long-lasting, 12-volt, deep-cycle batteries. These batteries are sealed and maintenance free. Since they are sealed, there is no need to check the electrolyte (fluid) level. Deep-cycle batteries are designed to handle a longer and deeper discharge. Though they are similar in appearance to automotive batteries, they are not interchangeable. Automotive batteries are not designed to handle a long, deep discharge, and also are unsafe for use in power chairs.



WARNING! Battery posts, terminals, and related accessories contain lead and lead compounds. Wash hands after handling.

Charging the Batteries

The battery charger is essential in providing long life for your power chair's batteries. It is designed to optimize your power chair's performance by charging the batteries safely, quickly, and easily. The standard battery charger is the onboard charger. The onboard charging system consists of the charger, the ammeter, the AC power cord, and the charger fuse. These components are all located on or in the front battery box. See figure 5. The ammeter indicates the rate of charge necessary to fully recharge the batteries. It is also a good indication of whether or not the charger is working. The ammeter and the charger are only functional when the charger power cord is plugged into a wall outlet using the AC power cord. The charger fuse protects the ammeter from overload during charging. For battery replacement instructions, see IX. "Care and Maintenance."



WARNING! You must recharge your power chair's batteries with the supplied onboard or the optional off-board charging system. Do not use an automotive-type battery charger.

WARNING! Never use an extension cord to plug in your battery charger. Plug the charger directly into a properly wired standard wall outlet.

To charge the batteries using the onboard charger:

- 1. Position the rear of your power chair close to a standard wall outlet.
- 2. Be certain the controller power is turned off and the freewheel levers are in the engaged position. See IV. "The Jazzy 1133."
- 3. Extend the charger AC power cord and plug it into the wall outlet. If your power chair uses a separate AC power cord, plug it into the onboard AC power cord receptacle first. Your power incorporates an inhibit function that disables the power chair when the charger is plugged into a wall outlet. This should be indicated on the controller. See VIII. "Operation."
- 4. Wait about a minute for the charger to warm up and check the ammeter. The ammeter indicates how much charge is needed to fully charge the batteries. The ammeter may move as high as 5.5 amps, then gradually move back down to 0 amps as the batteries charge.
- 5. We recommend you charge the batteries for 8 to 14 hours. As the batteries charge, the ammeter needle slowly drops to 0. When the batteries are fully charged, the needle vibrates on or about the 0 mark on the ammeter.
- 6. When your power chair's batteries are fully charged, wind up the charger AC power cord using the hook and loop strap and place the cord back into the power chair's electronics tray. If your AC power cord plugs into the onboard AC power cord receptacle as well, then roll it up and store it in a safe place.

To charge the batteries using the off-board charger:

NOTE: There are two types of off-board chargers. One is the type that plugs into the controller. If your charger plugs into the controller, then follow the instructions on the charger. The other is the type that plugs into the front battery box. If your off-board charger plugs into the front battery box, follow the instructions below. If you are not sure how to use your off-board charger, contact your authorized Pride provider.

- 1. Position your power chair next to a standard wall outlet.
- 2. Be certain the controller power is turned off.
- 3. Remove the run plug from the charger inhibit connector on the front battery box.
- 4. Plug the off-board charger into the charger inhibit connector on the front battery box.
- 5. Plug the off-board charger into the wall outlet.
- 6. Wait about a minute for the charger to warm up and check the ammeter. The ammeter indicates how much charge is needed to fully charge the batteries. The ammeter may move as high as 5.5 amps, then gradually move back down to 0 amps as the batteries charge.
- 7. We recommend you charge the batteries for 8 to 14 hours. As the batteries charge, the ammeter needle slowly drops to 0. When the batteries are fully charged, the needle vibrates on or about the 0 mark on the ammeter.
- 8. When your power chair's batteries are fully charged, disconnect the charger AC power cord from the wall outlet.
- 9. Disconnect the charger from the front battery box.
- 10. Reinstall the run plug into the charger inhibit connector.

Battery Break-in

We work closely with our battery manufacturer to provide a battery that best suits your power chair's specific demands. Fresh batteries arrive regularly at Pride and are promptly shipped with a full charge. During shipping, the batteries encounter temperature extremes that may influence initial performance. Heat robs the charge from the battery, and cold slows the power available and extends the time needed to recharge the battery (just as with a car battery).

It might take a few days for the temperature of the battery to stabilize and adjust to its new ambient temperature. More importantly, it will take a few "charging cycles" (a partial drain—then a full recharge) to establish the critical chemical balance that is essential to the battery's peak performance and long life. It will be well worth it to take the time to break in your battery properly.

NOTE: The useful life of a battery is quite often a reflection of the care it receives.

To break in new batteries for maximum efficiency:

- 1. Fully recharge any new battery prior to its initial use. This brings the battery up to about 90% of its peak performance level.
- 2. Operate your power chair throughout the house and yard. Move slowly at first, and don't stray too far until you become accustomed to the controls and break in the batteries.
- 3. Give the batteries another full charge of 8 to 14 hours and operate your power chair again. The batteries will now perform at over 90% of their potential.
- 4. After four or five charging cycles, the batteries will top off at 100% charge and last for an extended period.

FREQUENTLY ASKED QUESTIONS (FAQS)

How does the charger work?

The battery charger takes the standard wall outlet voltage of 120 VAC (alternating current) and converts it to 24 VDC (direct current). The power chair batteries use direct current to run your power chair. When the battery voltage is low, the charger works harder to charge the battery. This is why the ammeter initially reads five or more amps. As the battery voltage approaches full charge, the charger doesn't work as hard to complete the charging cycle. This explains why the ammeter drops as it approaches a full charge. When the battery is fully charged, the amperage from the charger is nearly at zero. This is how the charger maintains a charge but does not overcharge the battery.

Can I use a different battery charger?

You should use the charger supplied with the power chair. It is the safest, most efficient tool to charge the batteries. We do not recommend using other types of chargers (e.g., an automotive battery charger). Your power chair's charger will not operate after the batteries have been discharged to nearly zero voltage. If this happens, call your authorized Pride provider for assistance.

How often must I charge the batteries?

Many factors come into play when deciding how often to charge the batteries. You may use your power chair all day on a daily basis or you may not use it for weeks at a time.

■ Daily Use

If you use your power chair on a daily basis, charge the batteries as soon as you are finished using your power chair. Your power chair will be ready each morning to give you a full day's service. It is recommended that you charge the batteries 8 to 14 hours after daily use. Do not charge the batteries for more than 24 hours at a charging cycle.

■ Infrequent Use

If you use your power chair infrequently (once a week or less), you should charge the batteries at least once per week for 12 to 14 hours.

NOTE: Keep your batteries fully charged and avoid deeply discharging your batteries. Do not charge the batteries for more than 24 hours at a charging cycle.

How can I get maximum range or distance per charge?

Rarely do you have an ideal driving situation such as smooth, flat, hard terrain with no wind, hills, or curves. More often you are presented with hills, sidewalk cracks, uneven and loosely packed surfaces, curves, and wind. All of these factors will affect the distance or running time per battery charge. Below are a few suggestions for obtaining the maximum range per charge:

- Always charge the batteries fully prior to your trip.
- Plan your trip in advance to avoid inclines if possible.
- Limit baggage weight to essential items.
- Try to maintain an even speed and avoid stop-and-go driving.

What type of batteries should I use?

We recommend deep-cycle batteries that are sealed and maintenance free. Both AGM and Gel-Cell are deep-cycle batteries that are similar in performance.

Use these specifications to reorder deep-cycle batteries:

BATTERY SPECIFICATIONS				
Type:	Deep-cycle (AGM or Gel-Cell)			
Size:	NF-22			
Voltage:	12V each			



WARNING! Corrosive chemicals contained in batteries. Use only AGM or Gel-Cell batteries to reduce the risk of leakage or explosive conditions.

Why do my new batteries seem weak?

Deep-cycle batteries employ a much different chemical technology than that used in car batteries, nickel-cadmium (nicads), or in other common battery types. Deep-cycle batteries are specifically designed to provide power, drain down their charge, and then accept a relatively quick recharge. Lead-acid batteries should be charged as often as possible. They do not have a "memory" like nickel-cadmium batteries.

How can I ensure maximum battery life?

A fully charged deep-cycle battery will provide reliable performance and extended battery life. Keep your power chair's batteries fully charged whenever possible. Batteries that are regularly and deeply discharged, infrequently charged, or stored without a full charge may be permanently damaged, causing unreliable power chair operation and limited battery life.

How should I store my power chair and its batteries?

If you do not use your power chair regularly, we recommend maintaining battery vitality by charging the batteries at least once per week.

If you do not plan on using your power chair for an extended period, fully charge the batteries prior to storage. Disconnect the battery harnesses and store the power chair in a warm, dry environment. Avoid temperature extremes, such as freezing and excessively hot conditions, and never attempt to charge a frozen battery. A cold or frozen battery should be warmed for several days prior to recharging.

What about public transportation?

Gel-Cell and AGM batteries are designed for application in power chairs and other mobility vehicles. These batteries are Federal Aviation Administration (FAA) approved, allowing safe transportation on aircraft, buses, and trains, as there is no danger of spillage or leakage. We suggest you contact the carrier's ticket counter in advance to determine that carrier's specific requirements.

What about shipping?

If you wish to use a freight company to ship your power chair to your final destination, repack your power chair in the original shipping container and ship the batteries in separate boxes.

VSI CONTROLLER

The electronic controller is what you use to operate your power chair. It takes the battery voltage and sends it to the appropriate system. The electronic controller enables you to move the power chair, as well as monitor battery charge, electronic controller functions, and the condition of your electrical system. Also, it may be used to control some optional systems such as power elevating seats and lights.

The VSI controller is an integral electronic controller. All of the electronics necessary to operate the power chair are contained in one module. See figure 27.

The VSI consists of:

- 1. joystick
- 2. keypad (see figure 28)
- 3. off-board charger/programming socket
- 4. actuator connector (for optional equipment on some models)
- 5. controller connector
- 6. 3-pin charger inhibit connector

Typically, the VSI is mounted to one of the armrests and is connected to the motors, batteries, and the onboard charger at the front battery box.

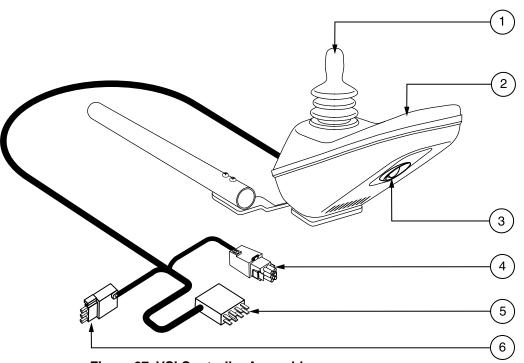


Figure 27. VSI Controller Assembly

Joystick

The joystick controls the direction and speed of your power chair. When you move the joystick from the neutral (center) position, the electromagnetic brakes release and allow your power chair to move. The further you push the joystick from its neutral position, the faster your power chair moves. When you release the joystick and allow it to return to the neutral position, you engage the electromagnetic brakes. This causes your power chair to decelerate and come to a complete stop.



WARNING! If your power chair begins to move in an unexpected manner, immediately release the joystick. Unless the joystick is damaged, this action should stop your power chair.

Keypad

The keypad is located in front of the joystick. It contains keys necessary to operate your power chair. See figure 28.

On/Off Key

The on/off key turns the VSI on and off.




WARNING! Unless faced with an emergency situation, do not use the on/off key to stop the chair. This will cause the power chair to stop abruptly.

WARNING! Always turn the power off when you are stationary to prevent unexpected movement.

Battery Condition Meter

The battery condition meter is located in front of the joystick. This is a 10-segment illuminated display that indicates that the VSI is turned on and also gives the battery status, the VSI status, and the electrical system status.

- Red, yellow, and green lights lit: Battery charged; VSI and electrical system OK.
- Red and yellow lights lit: Charge battery if possible; VSI and electrical system OK.
- Red lights only lit or slow flash: Charge battery as soon as possible; VSI and electrical system OK.
- Rapid flash of lights: Indicates a fault in the VSI or the electrical system. Refer to "VSI Error Codes."

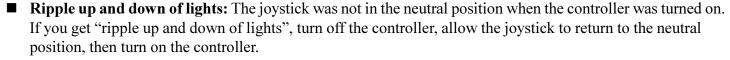


NOTE: If you still get "ripple up and down of lights", contact your authorized Pride provider.

NOTE: When the batteries approach a discharged state, the first red light will begin to slowly flash, reminding you the batteries need to be charged immediately!

Speed/Profile Keys

There are two keys that control either the speed or the profile. This depends on how your VSI was programmed. Press the speed/profile increase key to increase the speed or change the profile. Press the speed/profile decrease key to decrease the speed or change the profile. The speed/profile setting is displayed on the maximum speed/profile indicator. If your power chair was programmed with a drive profile, contact your authorized Pride provider for more information.

NOTE: We recommend that the first few times you operate your power chair, you set the speed to the slowest setting until you become familiar with your new power chair.

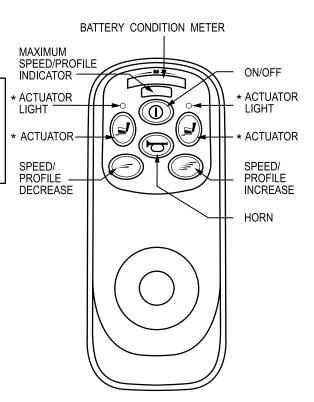

*For optional equipment on some models.

Figure 28. VSI Keypad

Actuator Key and Actuator Lights (for optional equipment on some models)

Actuator keys and actuator lights are used for optional equipment such as power elevating seats or power elevating leg rests. For specific operation of the actuator keys and actuator lights, contact your authorized Pride provider.

Horn Key

The horn key activates the horn.

Off-board Charger/Programming Socket

You may use an off-board charger to charge the power chair batteries through the 3-pin socket located on the front of the VSI. See figure 27. If you use an off-board charger, the charger current should not exceed 12 amps. Contact your authorized Pride provider for more information.



CAUTION! Only chargers with Neutrik NC3MX plugs should be connected to the onboard charger/programming socket. See your authorized Pride provider for more information.

NOTE: The socket may also be used for reprogramming the VSI. Contact your authorized Pride provider for more information.

Controller Connector

This connects the VSI to the power chair's batteries, motors, and motor brakes.

3-pin Charger Inhibit Connector

This connects the VSI to the onboard battery charger. This connection provides an inhibit that disables the VSI when the battery charger is on. The charger inhibit connector is coded with colored dots. The dots are positioned so that you align the flat side of the male connector with the flat side of the female connector before making the connection.



CAUTION! Failure to properly align the connectors can result in damage to the controller, the charger, and the connectors.

Thermal Rollback

The VSI controller is equipped with a thermal rollback circuit. The circuit monitors the temperature of the controller, which roughly translates to motor temperature. In the event that the VSI controller becomes excessively hot (above 140° F), motor current (amperage) is reduced. For every degree above 140°, the motor current limit is reduced by .55 amps until the VSI controller reaches 158° F, at which time the current output is reduced to zero. This reduces your power chair's "power," which could also reduce your power chair's speed, and allows the electrical components and motors to cool down. When the temperature returns to a safe level, your power chair resumes its normal operation.

VSI Error Codes

The VSI controller is designed with the user's safety as the prime consideration. It incorporates many sophisticated self-test features which search for potential problems at a rate of 100 times per second. If the VSI detects a problem either in its own circuits or in the power chair's electrical system, it may stop the power chair, depending on the severity of the problem. The VSI is designed to maximize the user's safety under all normal conditions. The table below identifies the individual error codes. Error codes are displayed as a rapid flashing of the lights. If you get one of these error codes, contact your authorized Pride provider.

FLASHING LIGHTS	DIAGNOSIS/SOLUTION
1	The battery needs charging or there is a bad connection to the battery. Check the
	connections to the battery. If the connections are good, try charging the battery.
2	The left motor has a bad connection. Check the left motor connection.
3	The left motor has a short circuit to a battery connection. Contact your authorized Pride provider.
4	The right motor has a bad connection. Check right motor connection.
5	The right motor has a short circuit to a battery connection. Contact your authorized Pride provider.
6	The power chair is being inhibited by the battery charger. Unplug the battery charger.
7	A joystick fault is indicated. Make sure that the joystick is in the neutral (center) position before
	turning on the controller.
8	A controller fault is indicated. Make sure that all connections are secure.
9	The parking brakes have a bad connection. Check the parking brake and motor connections.
	Make sure the controller system connections are secure.
10	An excessive voltage has been applied to the controller system. This is usually caused by a poor
	battery connection. Check the battery connections.

CARE AND MAINTENANCE

Your power chair is a sophisticated, motorized vehicle. Like any motorized vehicle, it requires routine maintenance checks. You can perform some of these checks yourself. But other maintenance checks may require assistance from an authorized Pride provider. Routine maintenance is very important. If you follow the routine maintenance checks in this section as scheduled, you can help ensure that your power chair gives you years of trouble-free operation. If you have any doubt as to your power chair's care or operation, contact your authorized Pride provider.

General Guidelines

- Avoid knocking or bumping the controller, especially the joystick.
- Avoid prolonged exposure of your power chair to extreme conditions, such as heat, cold, or moisture.
- Keep the controller clean.
- When the battery condition meter is completely lit, the batteries are fully charged, and the controller and the electrical system are OK.
- If one red light on the battery condition meter is blinking slowly, the batteries are low and need to be charged, but the controller and the electrical system are OK.
- If the battery condition meter is blinking rapidly, the controller has detected a fault in either its own circuits or in your power chair's circuits. See VIII. "Operation."
- Make sure the drive tires are inflated to 30-35 psi.



WARNING! Overinflating tires can cause them to explode and can result in personal injury.

WARNING! Do not use a high pressure hose to inflate your tires.

■ Use a rubber conditioner on the tire sidewalls to help preserve them.



WARNING! Never use a rubber conditioner on the tread area of the tires; doing so may make the tires slippery and cause your power chair to skid.

- The body shroud has been sprayed with a clear sealant coating. You can apply a light coat of car wax to help it retain its high-gloss appearance.
- Check all electrical connections. Make sure they are tight and are not corroded. Batteries must sit flat within the battery well, with the battery terminals facing inward, toward each other. Refer to the battery harness decal for the correct battery layout. See figure 31.
- All wheel bearings are prelubricated and sealed. They require no subsequent lubrication.

Daily Checks

- With the controller turned off, check the joystick. Make sure it is not bent or damaged and that it returns to the neutral position when you release it.
- Check the rubber boot around the base of the joystick for damage.
- Visually inspect the boot. Do not handle or try to repair it. See your authorized Pride provider if there is a problem.
- Visually inspect the controller harness. Make sure that it is not frayed, cut, or has any wires exposed. See your authorized Pride provider if there is a problem with any harnesses.

Weekly Checks

■ Disconnect and inspect the controller from the electronics tray. Look for corrosion. Contact your authorized Pride provider if necessary.

- Ensure that all parts of the controller system are securely fastened to your power chair. Do not overtighten any screws.
- Check for proper tire inflation. There should be 30 35 psi in each drive tire. If a tire does not hold air, see an authorized Pride provider for replacement.
- Calibrate the joystick if you detect a noticeable difference in performance or if the joystick does not operate properly. To calibrate the joystick, power off the unit, place the joystick in the neutral position, and power the unit back on. If a problem still exists with your joystick's performance, contact your authorized Pride provider.
- Check the brakes. This test should be carried out on a level surface with at least three feet of clearance around your power chair.

To check the brakes:

- 1. Turn on the controller and turn down the speed of your power chair to the lowest setting.
- 2. After one second, check the battery condition meter. Make sure that it remains lit.
- 3. Slowly push the joystick forward until you hear the electric brakes click. Immediately release the joystick. You must be able to hear each electrical brake operating within a few seconds of joystick movement. Repeat this test three times, pushing the joystick backwards, then left, and then right.

Monthly Checks

- Check that the anti-tip wheels do not rub the ground when you operate the power chair. Adjust them as
- necessary. See VI. "Comfort Adjustments."
- Check for extreme wear on the anti-tip wheels. Replace them as necessary.
- Check for drive tire wear. See an authorized Pride provider for repair.
- Check the caster wheels for wear. Replace them as necessary.
- Check the rear forks for damage or fluttering which indicates that they may need to be adjusted or have the bearings replaced. See an authorized Pride provider for repair.
- Keep your power chair clean and free of foreign material, such as mud, dirt, hair, food, and drink.

Yearly Checks

Take your power chair to an authorized Pride provider for yearly maintenance. This helps ensure that your power chair is functioning properly and helps prevent future complications.

Storage

Your power chair should be stored in a dry place, free from temperature extremes. When storing, disconnect the batteries from the power chair. See VII. "Batteries and Charging."



WARNING! If you fail to store the unit properly, the frame can rust and the electronics can be damaged.

Cleaning Precautions



CAUTION! Never hose off your power chair or place it in direct contact with water. Your power chair has a painted, **ABS** plastic body shroud that allows it to be easily wiped clean with a damp cloth.



WARNING! Never use any chemicals to clean a vinyl seat, as they may cause the seat to become slippery or dry out and crack. Use soapy water and dry the seat thoroughly.

Your power chair, like most electrical equipment, is susceptible to damage from the elements. Avoid damp areas of any kind.



CAUTION! Direct exposure to water or dampness could cause the power chair to malfunction electronically and mechanically. Water can cause electrical components to corrode and the chair's frame to rust.

Should your power chair come in contact with water:

- 1. Dry your power chair as thoroughly as possible with a dry towel.
- 2. Allow your power chair to sit in a warm, dry place for 12 hours to allow unseen water to evaporate.
- 3. Check the joystick operation and the brakes before using your power chair again.
- 4. If you find any inconsistencies, take your power chair to an authorized Pride provider.

Temperature

Some of the parts of your power chair are susceptible to extreme changes in temperature. Always keep your power chair between the temperatures of 18° F and 122° F.

- In extremely cold temperatures the batteries may freeze. The specific temperature at which they freeze depends on a number of factors, such as battery charge, usage, and composition of the batteries (e.g., AGM or Gel-Cell).
- Temperatures above 122° F may cause your power chair to operate at a reduced speed. This reduced speed is a safety feature built into the controller that helps prevent damage to the motor and other electrical components. See VIII. "Operation."

Corrective Maintenance

If the battery condition meter does not light up when you turn on the power:

- Check the harness connections. Make sure they are tight.
- Check the circuit breaker. Reset it if necessary.
- Check the battery connections.

If the above conditions prove normal, you can load test the batteries with a battery load tester. These testers are available at automotive parts stores. Disconnect both batteries before load testing and follow the directions that come with the load tester. If either one of the batteries fails the load test, replace both of them. If your power chair still does not power up, contact your authorized Pride provider.

Battery Replacement

The batteries on your power chair are located inside battery boxes, in the power base. See figure 4. For information on how to reconnect the batteries, refer to the battery harness decal which is located inside each battery box. See figure 31.



WARNING! Battery posts, terminals, and related accessories contain lead and lead compounds. Wear goggles and gloves when handling batteries and wash hands after handling.

WARNING! Power chair batteries are heavy. See specifications table. If you are unable to lift that much weight, be sure to get help. Lifting beyond your capacity can result in personal injury.

To replace the batteries:

- 1. Remove each battery box. See V. "Disassembly."
- 2. Remove the fasteners that attach the battery box lid to the battery box. See figures 29 and 30.
- 3. Disconnect the battery harness.
- 4. Remove the battery from the battery box.
- 5. Install the new battery into the battery box. Make sure that the terminals are facing the the correct direction. Refer to the battery layout label on the bottom of each battery box. See figure 31. The battery terminals in the front battery box face the the back of the box. The battery terminals in the rear battery box face the front of the box. When both batteries are installed correctly, the battery terminals face each other.
- 6. Install the battery harness onto the new battery. Connect the black lead to the negative (-) battery terminal and the red lead is connected to the positive (+) battery terminal. See figures 29 and 30.
- 7. Reinstall the battery box lid.



WARNING! Make sure you tighten the fasteners so that the connections are secure.

6. Reinstall the battery boxes onto the power base.

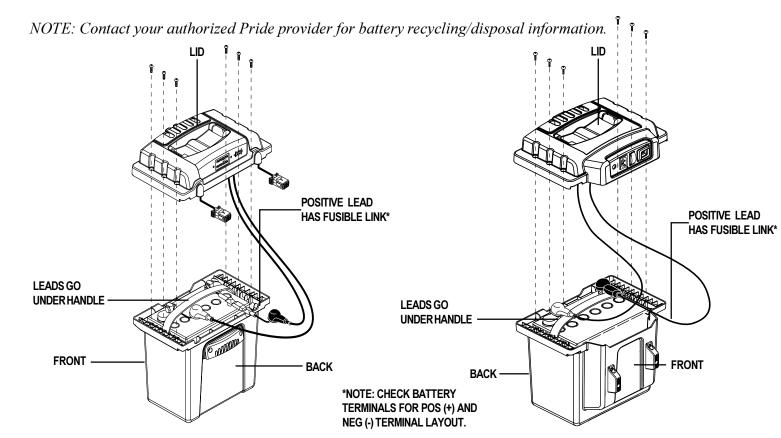


Figure 29. Jazzy 1133 Rear Battery Box

Figure 30. Jazzy 1133 Front Battery Box

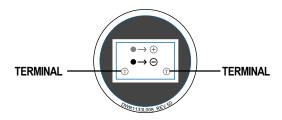


Figure 31. Battery Layout Label

TIRE/WHEEL REPLACEMENT

Your power chair may be equipped with either pneumatic or solid tires. If you have pneumatic tires and you have a flat tire, replace the tube. If your chair is equipped with a solid tire insert, then you must replace the whole wheel assembly. Replacement tires, tubes, and wheel assemblies are readily available through your authorized Pride provider.



WARNING! To avoid possible injury, be sure that the controller's power is turned off and the power chair is not in freewheel mode before performing this procedure.

WARNING! Completely deflate the tire before attempting repair.

Follow these easy steps for a quick and safe repair for both solid and pneumatic tires:

- 1. Turn off the power to the controller.
- 2. Make sure the power chair is in drive mode.
- 3. Set the power chair up on blocks.
- 4. If you are changing a pneumatic tire, make sure that the tire is completely deflated before removing the wheel.
- 5. Remove the fastener that attaches the drive wheel to the axle.
- 6. Pull the drive wheel off the axle.

NOTE: Your wheel may use one or more small, rectangular keys to connect it to the axle. Make sure you keep them for reassembly.

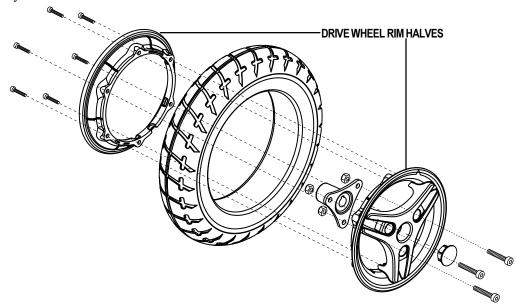


Figure 32. Jazzy 1133 Drive Wheel — Exploded View

- 7. Remove the fastens that connect the two drive wheel rim halves together. See figure 32.
- 8. Remove the old tube from the pneumatic tire and replace it with a new tube.
- 9. Reinstall the fasteners that connect the two drive wheel rim halves together.
- 10. Slide the wheel back onto the shaft. Make sure that the key is in the keyway.
- 11. Reinstall the drive wheel fastener on the center hub and tighten.
- 12. Inflate the pneumatic tire to 30 35 psi.

MOTOR BRUSHES

The electric motors that power your power chair use carbon brushes. These brushes may become susceptible to wear over a long period of time. The motor brushes are the two contacts located inside the motor assembly that supply power to the motor. They are designed to provide several thousand hours of operation. However, if the brushes become dirty with carbon deposits or wear out, the motor will run poorly or not at all.

To inspect or replace the motor brushes:

- 1. Unscrew the motor brush caps. See figure 33.
- 2. Remove the brushes.
- 3. Inspect for wear. See figure 34.
- 4. Replace brushes, if necessary. Contact your authorized Pride provider for replacement brushes.

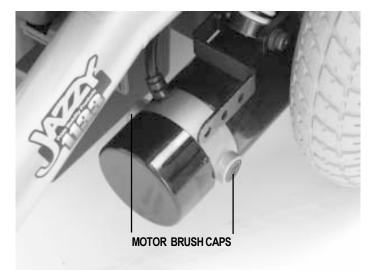


Figure 33. Motor Brush Location



CAUTION! If an inspection determines excessive wear on the brushes, they must be replaced or motor damage will result. Failure to maintain the brushes could void your power chair's warranty.

WHEN TO SEE YOUR AUTHORIZED PRIDE PROVIDER FOR SERVICE

The following symptoms could indicate a serious problem with your power chair. If necessary, contact your authorized Pride provider. When calling, have the model number, serial number, nature of the problem, and the trouble code if available.

- Motor noise
- Frayed harnesses
- Cracked or broken connectors
- Uneven wear on any of the tires
- Jerky motion
- Pulling to one side
- Bent or broken wheel assemblies
- Does not power up
- Powers up, but does not move

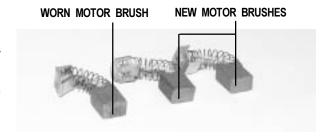


Figure 34. Motor Brushes

X. OPTIONAL ACCESSORIES

OPTIONAL ACCESSORIES

The following accessories are available from your authorized Pride provider.

Positioning Belt

The positioning belt is designed to support your torso so that you do not slide down or forward in the seat.

- The positioning belt is not designed for use as a restraining device.
- Make sure the belt is secure but does not cause discomfort.



WARNING! The positioning belt is not designed for use as a seat belt in a motor vehicle. Nor is your Jazzy suitable for use as a seat in any vehicle. Anyone traveling in a vehicle should be properly belted into seats approved by the vehicle manufacturer.

Cane and Crutch Holder

The removable cane and crutch holder is mounted to the back of the seat by means of the accessory bracket.

Walker Holder

The removable walker holder is mounted to the back of the seat by means of the accessory bracket. It can hold a standard size walker.

Cup Holder

The cup holder is mounted to the armrest.

Oxygen Tank Holder

The removable oxygen tank holder is mounted to the back of the seat by means of the accessory bracket.

Rear Basket

The rear basket is mounted to the accessory bracket. It has a handle that you can use when you remove the basket.

Swing-Away Joystick Bracket

This option enables you to move the joystick from the front of the armrest to the side of the armrest without disconnecting it.

Dust Cover

These sturdy canvas covers help protect your Jazzy when not in use.

XI. WARRANTY

LIFETIME LIMITED WARRANTY

For the lifetime of your Jazzy from the date of purchase, Pride will repair or replace at our option, to the original purchaser, free of charge, any of the following parts found upon examination by an authorized representative of Pride to be defective in material and/or workmanship:

Structural frame components

Two-Year Warranty

For two (2) years from the date of purchase, Pride will repair or replace at our option, to the original purchaser, free of charge, any of the following parts found upon examination by an authorized representative of Pride to be defective in material and/or workmanship:

■ Electronics

Eighteen-Month Warranty

For eighteen (18) months from the date of purchase, Pride will repair or replace, at our option to the original purchaser, free of charge, any of the following parts found upon examination by an authorized representative of Pride to be defective in material and/or workmanship:

■ Motor/gearbox assembly

Warranty Exceptions

Motor: If damage occurs to the commutator of a motor as a result of not replacing the motor brushes after heavy wear to the brushes. Motor brushes are wear items and are not warranted.

Brake: One (1) year warranty for the electrical function of the brake. Brake pads are a wear item and are not warranted.

Six-Month Limited Warranty

For six (6) months from date of purchase, Pride will repair or replace, at our option, to the original purchaser, free of charge, any of the following parts found upon examination by an authorized representative of Pride to be defective in material and/or workmanship:

■ Plastic components, except body

Rubber components (except tires and tubes)

Bearings and bushings

Casters and anti-tip wheels

Battery warranty is covered by the battery manufacturer. Battery warranty is not covered by Pride.

Warranty service can be performed by an authorized Pride provider or by Pride. Do not return faulty parts to Pride without prior consent. All transportation costs and shipping damage incurred while submitting parts for repair or replacement are the responsibility of the original purchaser.

Warranty Exclusions

- ABS plastic body shrouds and footrest covers (wear items)
- Batteries (battery manufacturer provides a six-month limited warranty)
- Tires and tubes
- Upholstery and seating
- Repairs and/or modifications made to any part without specific consent from Pride
- Circumstances beyond the control of Pride
- Labor, service calls, shipping, and other charges incurred for repair of the product, unless specifically authorized by Pride Mobility Products Corporation

Damage caused by:

- Battery fluid spillage or leakage
- Abuse, misuse, accident, or negligence

- Improper operation, maintenance, or storage
- Commercial use or use other than normal

There is no other express warranty.

Implied warranties, including those of merchantability and fitness for a particular purpose, are limited to one (1) year from the date of purchase and to the extent permitted by law. Any and all implied warranties are excluded. This is the exclusive remedy. Liabilities for consequential damages under any and all warranties are excluded.

Some states do not allow limitations on how long an implied warranty lasts or do not allow the exclusion of limitation of incidental or consequential damages, so the above limitation or exclusion may not apply to you.

This warranty gives you specific rights, and you may also have other rights which vary from state to state.

Please fill out and return the warranty card to Pride. This assists Pride in providing the best possible technical and customer service.

NOTES

NOTES

NOTES





Quality Control - 1133

Thank you for making the Pride Jazzy your choice in power chairs.

We have thoroughly inspected your Jazzy. The following checkmarks indicate that it has been test driven and inspected.



Pride keeps a more detailed report on file at the factory. Date Inspected	
Date Inspected	
Inspector	