

LIFT CHAIR

The Ultimate in Comfort & Style!



OWNER'S MANUAL

Pride


Mobility Products Corp.
www.pridemobility.com

1-800-800-8586 (US) 1-888-570-1113 (Canada)

The symbols below are used throughout this owner's manual and on the product to identify warnings and important information. It is very important for you to read them and understand them completely.



WARNING! Indicates a potentially hazardous condition/situation. Failure to follow designated procedures can cause either personal injury, component damage, or malfunction. On the product, this icon is represented as a black symbol on a yellow triangle with a black border.

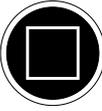

MANDATORY! These actions should be performed as specified. Failure to perform mandatory actions can cause personal injury and/or equipment damage. On the product, this icon is represented as a white symbol on a blue dot with a white border.



PROHIBITED! These actions are prohibited. These actions should not be performed at any time or in any circumstances. Performing a prohibited action can cause personal injury and/or equipment damage. On the product, this icon is represented as a black symbol with a red circle and red slash.

Please fill out the following information for quick reference:

Pride Provider: _____

Address: _____

Phone Number: _____

Purchase Date: _____ Serial Number: _____

NOTE: This owner's manual is compiled from the latest specifications and product information available at the time of publication. We reserve the right to make changes as they become necessary. Any changes to our products may cause slight variations between the illustrations and explanations in this manual and the product you have purchased. The latest/current version of this manual is available on our website.



PRODUCT SAFETY SYMBOLS

The symbols below represent labels used on the product to identify warnings, mandatory actions, and prohibited actions. It is very important for you to read and understand these symbols completely. Do not remove these labels from your product. Please note that not all of the symbols may be used on your lift chair model.



Pinch/Crush Points Hazard! Do not place objects or appendages in the path of moving parts.




Electrical Hazard



Read and follow the information in the owner's manual.



Maximum weight capacity

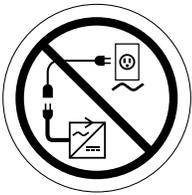

Do not connect an extension cord to the AC/DC converter or the battery charger.



Removal of grounding prong can create electrical hazard. If necessary, properly install an approved 3-pronged adapter to an electrical outlet having 2-pronged plug access.




Properly dispose of all electronic components, including the external transformer, hand control, batteries, actuator motors, and wiring. Contact your authorized Pride Provider for more information.



Do not place objects under lift chair. Make sure area is clear of obstructions including pets and small children during operation.




Red LED indicates low battery voltage. Replace with new batteries when lit. Not applicable to all models.

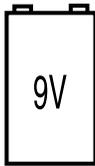


Battery Door Location



Indoor Use Only. Avoid exposure to rain, snow, ice, salt, or standing water. Maintain and store in a clean and dry environment.

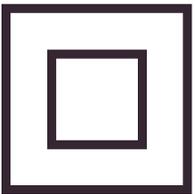

Class II Equipment

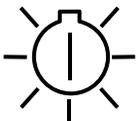

Green LED indicates power to the unit is on. Not applicable to all models.



TYPE B

Degree of protection against electric shock.



This product has been tested and complies to IEC 60601-1-2.

SAFETY GUIDELINES 2

LABEL INFORMATION 3

I. INTRODUCTION 6

SAFETY 6

PURCHASER’S AGREEMENT 6

INFORMATION EXCHANGE 6

PRIDE OWNER’S CLUB 6

II. GENERAL GUIDELINES 7

MODIFICATIONS 7

WEIGHT LIMITATIONS 7

PINCH/CRUSH HAZARDS 7

DEGREE OF PROTECTION/MODE OF OPERATION 7

STORAGE AND OPERATION TEMPERATURES 7

EMI/RFI 8

SHIPPING AND DELIVERY 8

MOTOR VEHICLE TRANSPORT 8

III. YOUR LIFT CHAIR 9

BODY COMPONENTS 9

ELECTRICAL COMPONENTS 11

IV. ASSEMBLY/DISASSEMBLY 12

LIFT CHAIR SETUP 12

LIFT CHAIR PLACEMENT 14

BATTERY INSTALLATION 14

LIFT CHAIR DISASSEMBLY 16

V. OPERATION 17

OPERATION PRECAUTIONS 17

HAND CONTROL OPERATION 17

VI. TROUBLESHOOTING 20

545/555 MOTOR TIMING 20

FREQUENTLY ASKED QUESTIONS 20

VII. CARE AND MAINTENANCE 22

FABRIC CARE 22

ELECTRONICS CARE 22

DISPOSAL AND RECYCLING 22

VIII. WARRANTY 23

APPENDIX 24



SAFETY

WELCOME to Pride Mobility Products Corporation (Pride). The product you have purchased combines state-of-the-art components with **safety**, comfort, and styling in mind. We are confident the design features will provide you with the conveniences you expect during your daily activities. Understanding how to **safely** operate and care for this product should bring you years of trouble free operation and service.

Read and follow all instructions, warnings, and notes in this manual and all other accompanying literature before attempting to operate this product for the first time. In addition, your **safety** depends upon you, as well as your provider, caretaker, or healthcare professional in using good judgement.

If there is any information in this manual which you do not understand, or if you require additional assistance for setup or operation, please contact your authorized Pride Provider. **Failure to follow the instructions, warnings, and notes in this manual and those located on your Pride product can result in personal injury or product damage and will void Pride's product warranty.**

PURCHASER'S AGREEMENT

By accepting delivery of this product, you promise that you will not change, alter, or modify this product or remove or render inoperable or unsafe any guards, shields, or other safety features of this product; fail, refuse, or neglect to install any retrofit kits from time to time provided by Pride to enhance or preserve the safe use of this product.

INFORMATION EXCHANGE

We want to hear your questions, comments, and suggestions about this manual. We would also like to hear about the safety and reliability of your new lift chair, and about the service you received from your authorized Pride Provider. Please notify us of any change of address, so we can keep you apprised of important information about safety, new products, and new options that can increase your ability to use and enjoy your lift chair. Please feel free to contact us at the address below:

USA:

Pride Mobility Products Corporation
Attn.: Customer Care Department
182 Susquehanna Ave.
Exeter, PA 18643-2694
customercare@pridemobility.com
1-800-800-8586

Canada:

Pride Mobility Products Company
380 Vansickle Road Unit 350
St. Catharines, Ontario L2R 6P7
1-888-570-1113

NOTE: If you ever lose or misplace your product registration card or your copy of this manual, contact us and we will be glad to send you a new one immediately.

PRIDE OWNERS CLUB

As an owner of a Pride product, you are encouraged to enroll in the Pride Owners Club. Complete and return your enclosed product registration card or visit Pride's web site at www.pridemobility.com.

From our home page, select "Owners Club" to enter a page dedicated to current and potential Pride product owners. You will gain access to interviews, stories, recreation ideas, daily living tips, product and funding information, and interactive message boards. These message boards invite you to communicate with other Pride customers as well as Pride representatives who are available to assist you with any questions or concerns.



Your lift chair is a state-of-the-art life-enhancement device designed to increase mobility. Pride provides an extensive variety of products to best fit your individual needs. Please be aware that the final selection and purchasing decision regarding the type of lift chair to be used is the responsibility of you, the lift chair user, if capable of making such a decision, and/or your healthcare professional (i.e., medical doctor, physical therapist, etc.).



MANDATORY! Read and follow the information provided in this owner's manual before attempting to operate your lift chair for the first time.

There are certain situations, including some medical conditions, where you will need to practice operating the lift chair in the presence of a trained attendant. A trained attendant can be defined as a family member or healthcare professional specially trained in assisting you with performing various daily living activities while safely operating a lift chair.

Below are some precautions, tips, and other safety considerations that will help you become accustomed to operating the lift chair in a safe manner.

MODIFICATIONS

Pride has designed and engineered your lift chair to provide maximum comfort and utility. However, to prevent personal injury and/or damage to your lift chair, you should not modify, add, remove, or disable any feature, part, or function of your lift chair. Unauthorized modifications may also void your product's warranty.

NOTE: Use Pride parts only for all repairs and replacements.

WEIGHT LIMITATIONS

Your lift chair is rated for a maximum weight capacity. Refer to the "Appendix A" for more information.



MANDATORY! Stay within the specified weight capacity of your lift chair. Pride will not be held responsible for injuries and/or product damage resulting from failure to observe weight limitations.

PINCH/CRUSH HAZARDS

The scissor and lift mechanisms are labeled as pinch/crush point hazards on your lift chair. Keep clear of these areas and make sure the path of motion is unobstructed. See figure 3.1 for pinch/crush point locations.



WARNING! Do not place objects or appendages in the path of moving parts.

DEGREE OF PROTECTION/MODE OF OPERATION

- Class II equipment/Type B protection against electric shock
- Degree of protection against the ingress of solid/liquids (splash proof)—IPX4
- Mode of operation—Maximum Duty Cycle: 2min ON/18min OFF

STORAGE AND OPERATION TEMPERATURES

- Transportation or storage: -104°F/-75°C to 158°F/70°C
- Operation: 50°F/10°C to 104°F/40°C

ELECTROMAGNETIC AND RADIO FREQUENCY INTERFERENCE (EMI/RFI)



WARNING! Laboratory tests have shown that electromagnetic and radio frequency waves can have an adverse affect on the performance of electrically-powered devices, such as lift chairs.

Electromagnetic and Radio Frequency Interference can come from sources such as cellular phones, mobile two-way radios (such as walkie-talkies), radio stations, TV stations, amateur radio (HAM) transmitters, wireless computer links, microwave signals, paging transmitters, and medium-range mobile transceivers used by emergency vehicles. In some cases, these waves can cause unintended movement or damage to the control system of electrically-powered devices. Every electrically-powered device has an immunity (or resistance) to EMI. The higher the immunity level, the greater the protection against EMI. Per EMC standards, this product has passed immunity testing and is suitable for use in all establishments, including domestic establishments.

WARNING! Be aware that cell phones, two-way radios, laptops, and other types of radio transmitters may cause unintended movement of your electrically-powered device due to EMI. Exercise caution when using any of these items while operating your lift chair.



WARNING! The addition of accessories or components to the lift chair can increase the susceptibility of the chair to EMI. Do not modify your lift chair in any way not authorized by Pride.

WARNING! Your lift chair itself can disturb the performance of other electrical devices located nearby, such as alarm systems.

NOTE: If unintended motion occurs, discontinue use of the lift chair. Call Pride to report the incident.

SHIPPING AND DELIVERY

Before using your lift chair, make sure your delivery is complete as some components may be individually packaged. If you do not receive a complete delivery, please contact your authorized Pride Provider immediately. Where damage has occurred during transport, either to the packaging or content, please contact the delivery company responsible.

MOTOR VEHICLE TRANSPORT

If you will be transporting your lift chair in a motor vehicle, individual components (external transformer, etc.) should be secured against slipping. The lift chair itself must also be secured against slipping (a possible hazard during vehicle braking).



BODY COMPONENTS

This section describes the features of your lift chair. Carefully review the function and location of each item described, and note that the illustrations and option locations shown in this manual may not reflect the type of lift chair that you own.

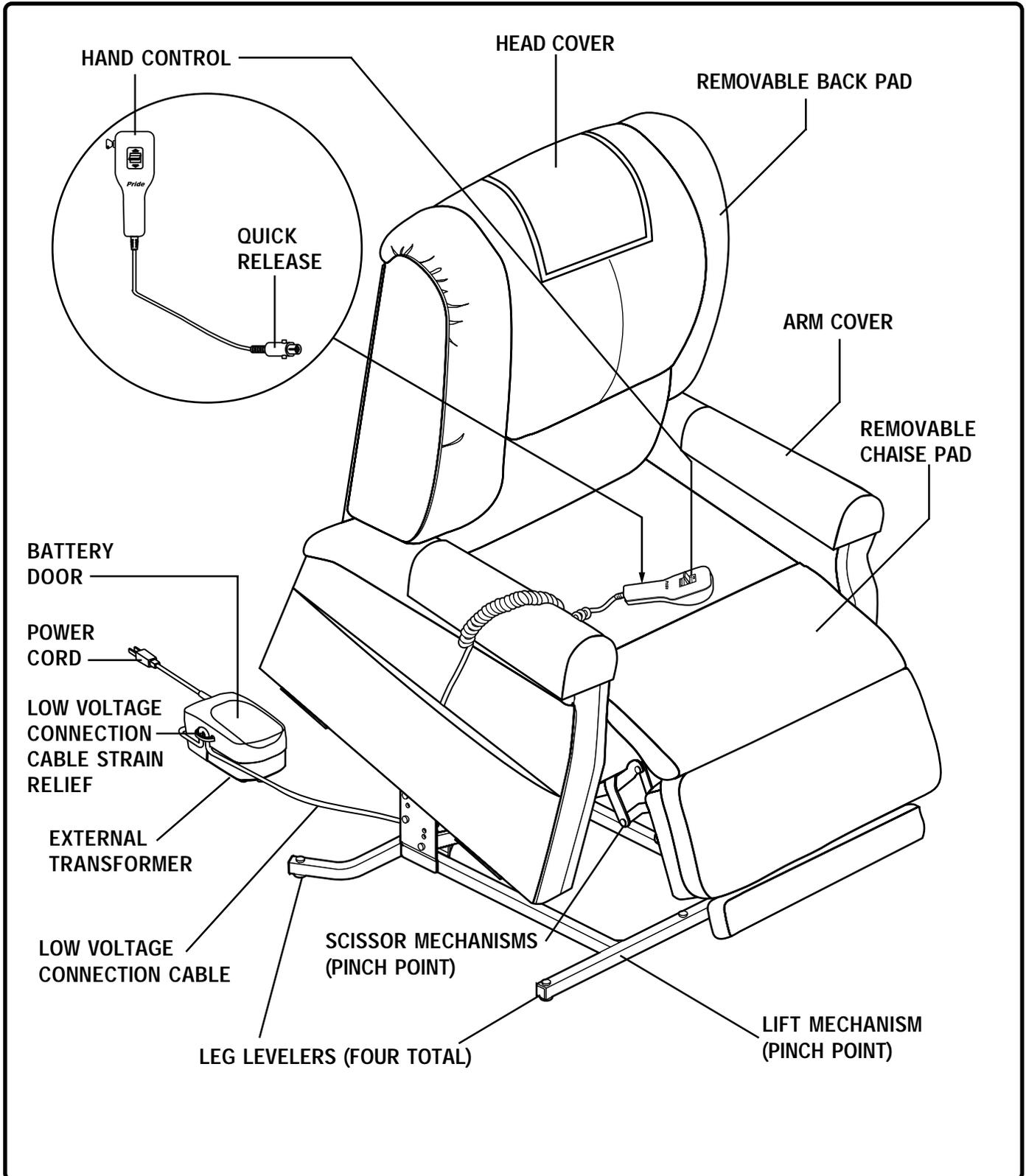


Figure 3.1 Pride Lift Chair Main Components

Removable Back Pad: Attaches to the lift chair with reusable fasteners to enable the use of the bed option on some models and to make servicing the chair more convenient.

Removable Chaise Pad: Attaches to the lift chair with reusable fasteners to make servicing the chair more convenient.

NOTE: The chair must be in the recline position in order to remove or attach the back pad, chaise pad, or bed option mattress.

NOTE: The DMR 670 bed option consists of a mattress, bedsheet, strap, and storage bag. After removing the mattress, it is recommended that you store it in the bag provided.

NOTE: If your lift chair is equipped with a removable back pad and/or chaise pad and has the options of heat, heat and massage, or deluxe heat and massage, the back pad and/or chaise pad can be removed without disconnecting the heating unit by detaching the white clip located on the heat and massage wire. The TMR 450 Lift Chair has a clip for both heating units.

Lift Mechanism: Responds to the hand control commands to position the chair in the sit, recline, and stand positions.

Leg Levelers: Manually rotate up or down in order to stabilize the chair before use (Fig. 3.2).

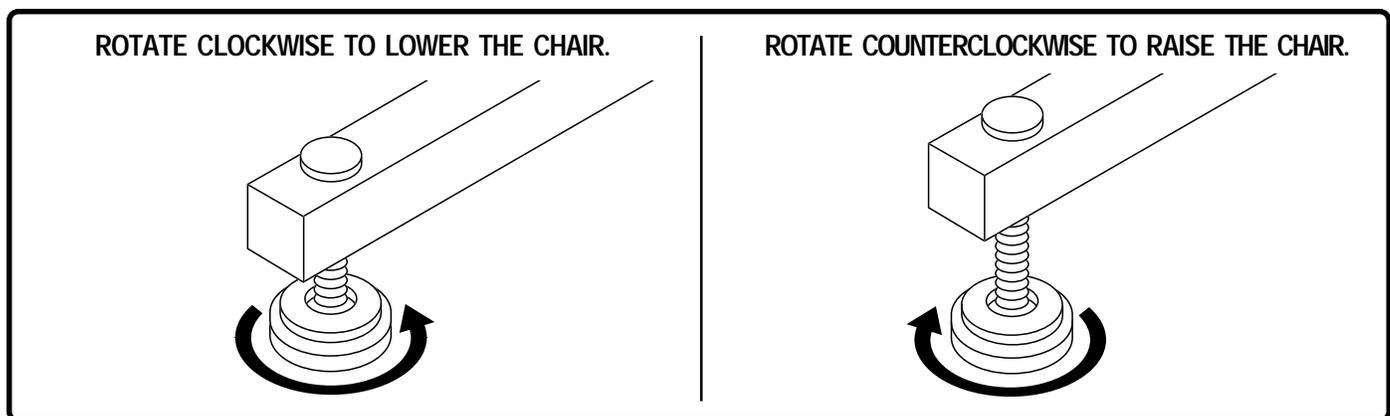


Figure 3.2 Leg Levelers

Scissor Mechanisms: Extend or retract to take the chair through the various stages of recline (Fig. 3.3).



WARNING! Be aware that the scissor mechanisms are a pinch point on the lift chair. Keep appendages and objects clear of these points when operating the footrest.

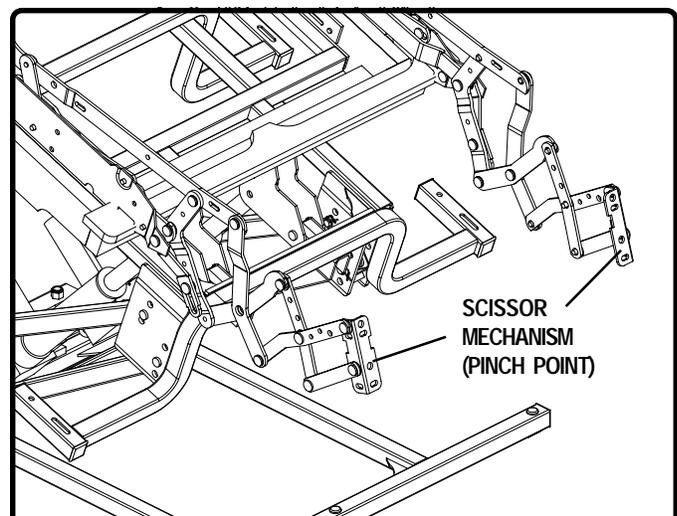


Figure 3.3 Scissor Mechanism

Footrest Extension: Provides 5 in. (13 cm) of extension to the footrest, increasing the leg support for taller individuals when the chair is in the recline position (Fig. 3.4). This feature is specific to the TMR 575 Lift Chair and optional to the TMR 570 Lift Chair.

NOTE: *The footrest extension folds up neatly and is not visible when the chair is in the seated and lift positions.*

ELECTRICAL COMPONENTS

Your Pride Lift Chair is equipped with a low voltage DC motor system that reduces the standard household alternating current of 110/115V AC (220/230/240 V AC) to direct current (24/39V DC). The electrical components include the external transformer, batteries, and hand control. Keep these areas free from moisture at all times.

External Transformer: Connects to a standard electrical outlet to provide power to the lift chair. Some transformers are equipped with colored LEDs—red indicates low battery power and green indicates that power to the transformer is on.

The external transformer may also come equipped with a molded portion that allows the transformer to be mounted to the wall directly under the standard outlet. The screws must be mounted to the wall stud for security. Refer to “Appendix B” for a mounting diagram.

NOTE: *Pride is not responsible for damage due to improper installation.*

Batteries: Provide the power needed to return the lift chair from the recline position to the sit and/or lift position in the event of a power failure. The batteries are located inside the external transformer.

Hand Control: Contains the controls needed to operate the various functions of the lift chair. Refer to V. “Operation” for more information. The hand control may be equipped with a quick-release connector (Fig. 3.5) which enables the user to detach the hand control from the chair, disabling all functions of the hand control.

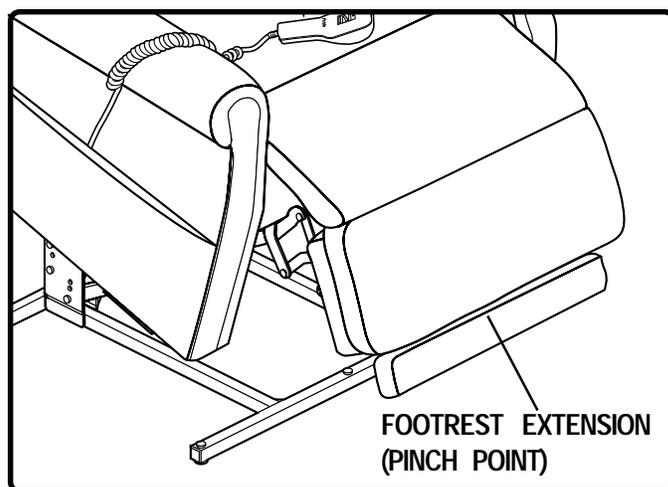


Figure 3.4 Footrest Extension

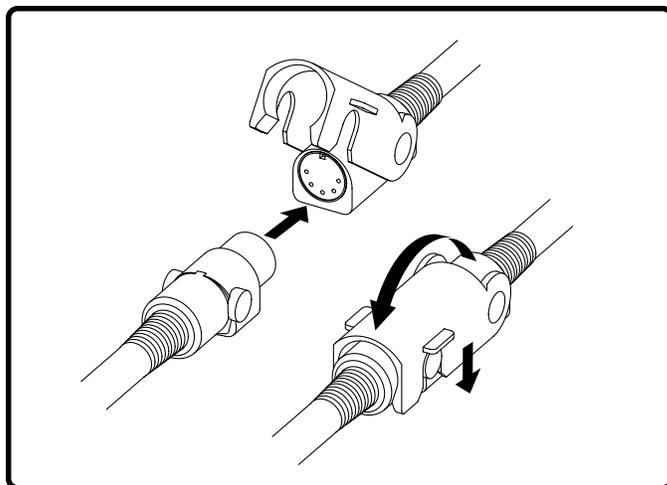


Figure 3.5 Quick-Release Connector



Your lift chair may require some assembly before initial use. It may also require disassembly to make servicing the chair more convenient. If your lift chair is a Knock-Down (KD) model, follow the instructions in “Lift Chair Setup.” If your lift chair is not a KD model, proceed to “Lift Chair Placement.”

LIFT CHAIR SETUP

KD models will arrive in the knock-down position (Fig. 4.1).



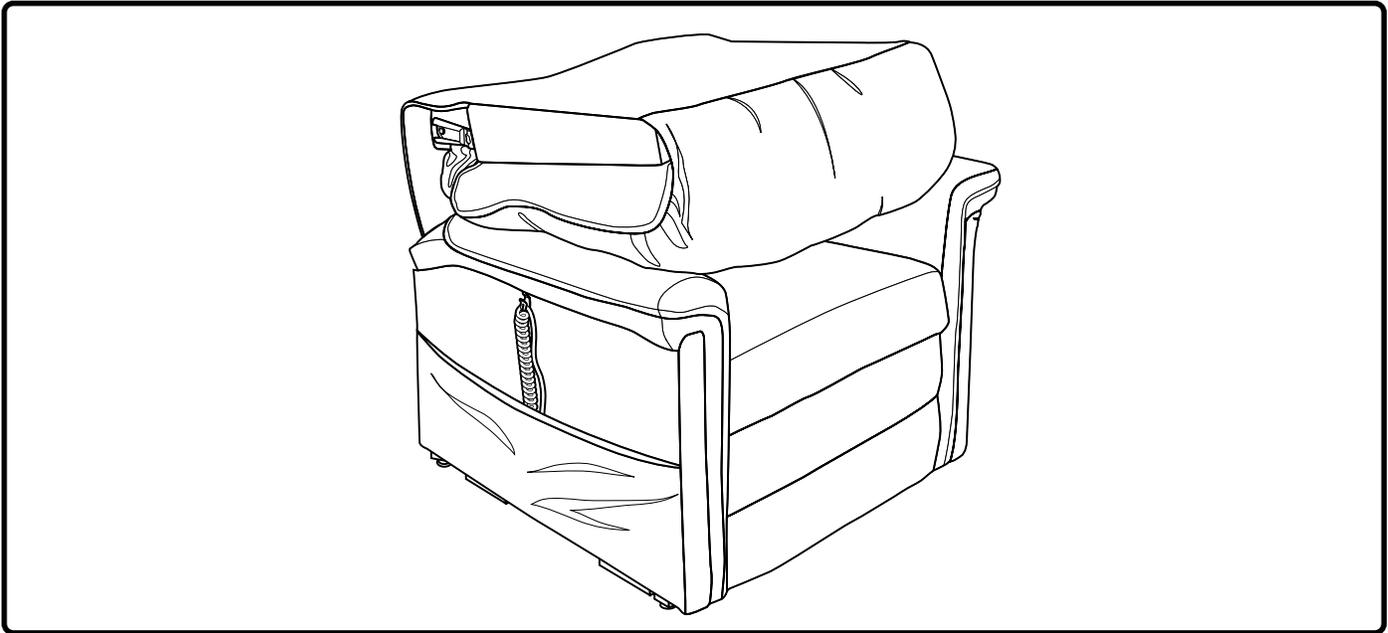


Figure 4.1 Knock-Down (KD) Position

Follow these steps to assemble the lift chair:

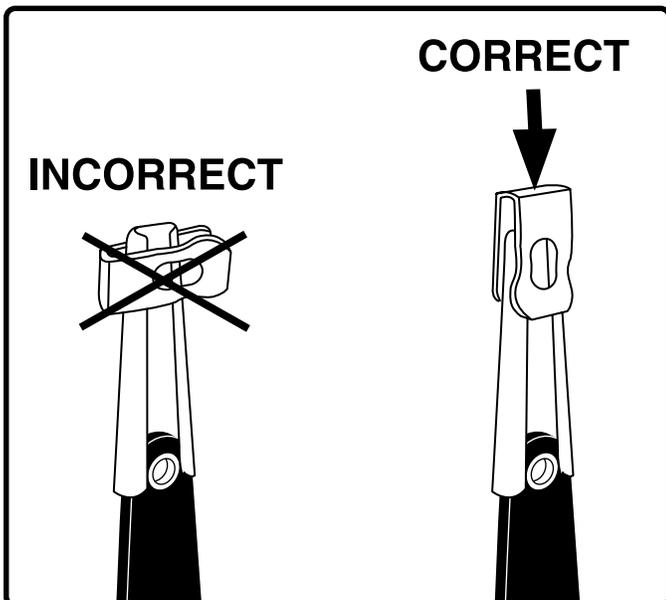


4.1.1. Remove the packaging from the back portion of the chair and check that the locking clip is in the vertical position. Reposition if necessary.

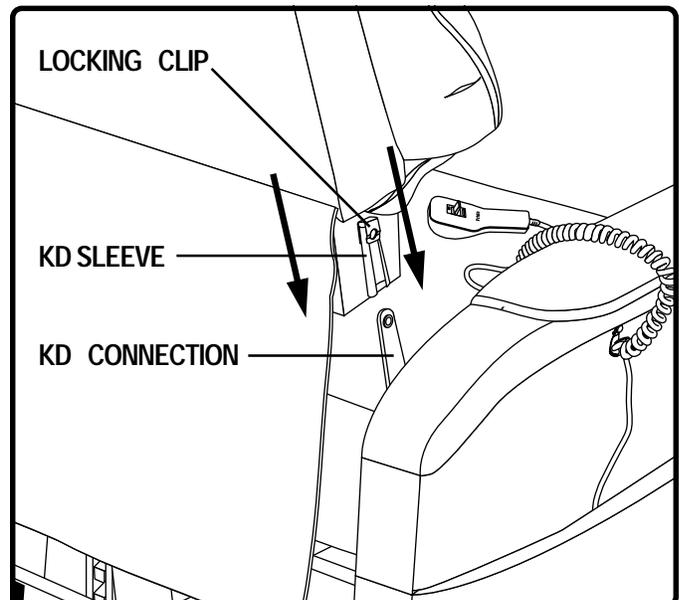


4.1.2. Slide the left and right KD sleeves of the chair-back onto the KD connections on the arms and seat of the chair.

NOTE: Depending on chair width, you may need to slide on one side of the chair-back at a time. If this is the case, level the chair-back before pressing down to lock into position.

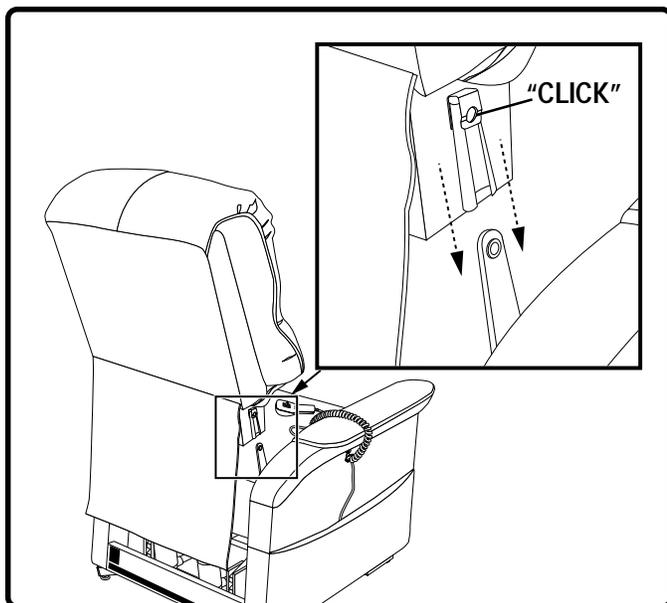


4.1.3. Push firmly on the top of the chair-back to lock into position. Listen for a click on each side to ensure the chair-back is locked in properly.

NOTE: Pull up on the chair-back to ensure that it is locked in place. If the back can be pulled off, then repeat steps 4.1.1 through 4.1.3.

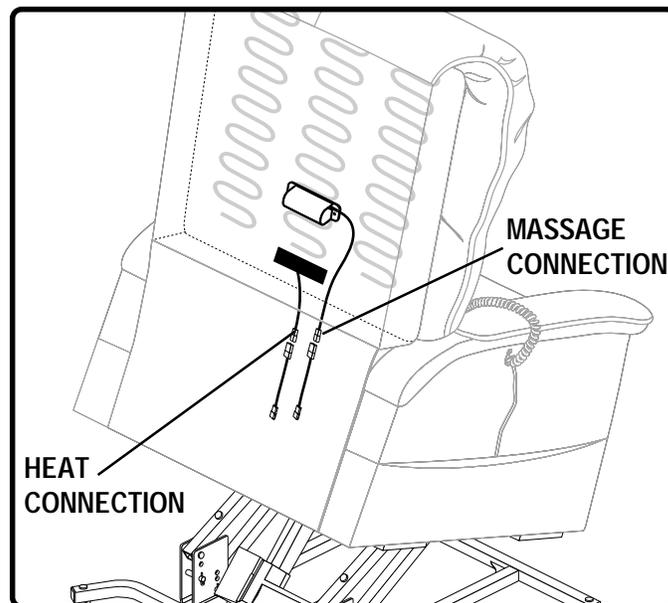

4.1.4. If your chair is equipped with heat and/or massage options, connect the quick-release connections on the chair-back at this time.

NOTE: The heat connection is labeled while the massage connection is not. The 450 and C30 model lift chairs come standard with two heat pads. The upper massage unit on KD deluxe heat and massage chairs is labeled with a red A.

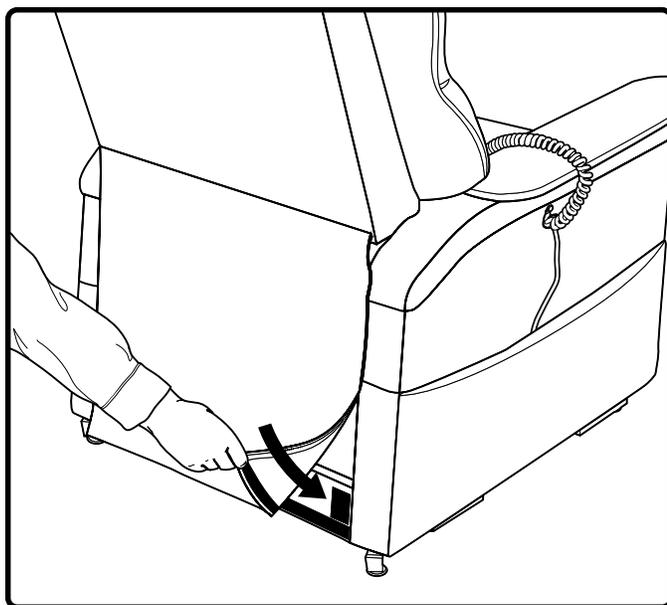

4.1.5. Attach the chair-back to the hook and loop flaps on the bottom of the chair base.

LIFT CHAIR PLACEMENT

Your lift chair should be placed near a standard electrical outlet on dry, level ground where there is ample room to allow for proper operation. Pride recommends that you have the assistance of an attendant when positioning the lift chair to avoid the possibility of injury when lifting.

Follow these steps to position the lift chair in a safe manner:

1. Place the back of the lift chair 30 in. (76 cm) from the nearest obstruction while the chair is in the seated position. This measurement may vary depending on model.
2. Adjust the leg levelers to stabilize the lift chair (Fig. 3.2).
3. Install the batteries into the external transformer. Refer to "Battery Installation" for more information.
4. Position the external transformer on the floor in an open, well-ventilated area where it will not be an obstruction, or if equipped, utilize the optional wall mount.
5. Position the low voltage connection cable where it will not be pinched between the frame and the lift mechanism.
6. Connect the low voltage connection cable to the external transformer if it is not already connected.
7. Plug the power cord directly into the electrical outlet. **Do not use an extension cord!**

NOTE: If you discover a problem at any point during the setup and positioning of your lift chair, stop and contact your authorized Pride Provider immediately. To avoid personal injury and/or product damage, do not plug the unit into the electrical outlet until the problem is corrected and do not attempt to fix electrical problems by yourself.

NOTE: For chairs with heat and massage options, the green light on the external transformer is lit to indicate proper operation. The blinking red light indicates that no batteries are installed or battery power is low.

BATTERY INSTALLATION

Your lift chair is equipped with a battery backup system that will activate during a power failure. The backup system is powered by two 9V batteries (not included) that need to be installed into the external transformer.

Follow these steps to install the batteries:

1. Unplug the external transformer power cord from the electrical outlet.
2. Open the marked battery door on the external transformer.
3. Install two 9V batteries into the external transformer (Fig. 4.2 and 4.3).
4. Replace the battery door.

NOTE: Always make sure the external transformer is equipped with two fresh 9V batteries, since the battery backup system does not recharge itself. Fresh batteries are defined as 9V alkaline batteries that are replaced every time the battery backup system is activated during a power failure, or once a year if the battery backup system has not been activated.

NOTE: If you unplug your lift chair for an extended period of time (more than one hour), remove the batteries from the external transformer. The lift chair will draw power from the batteries even when it is not in use.

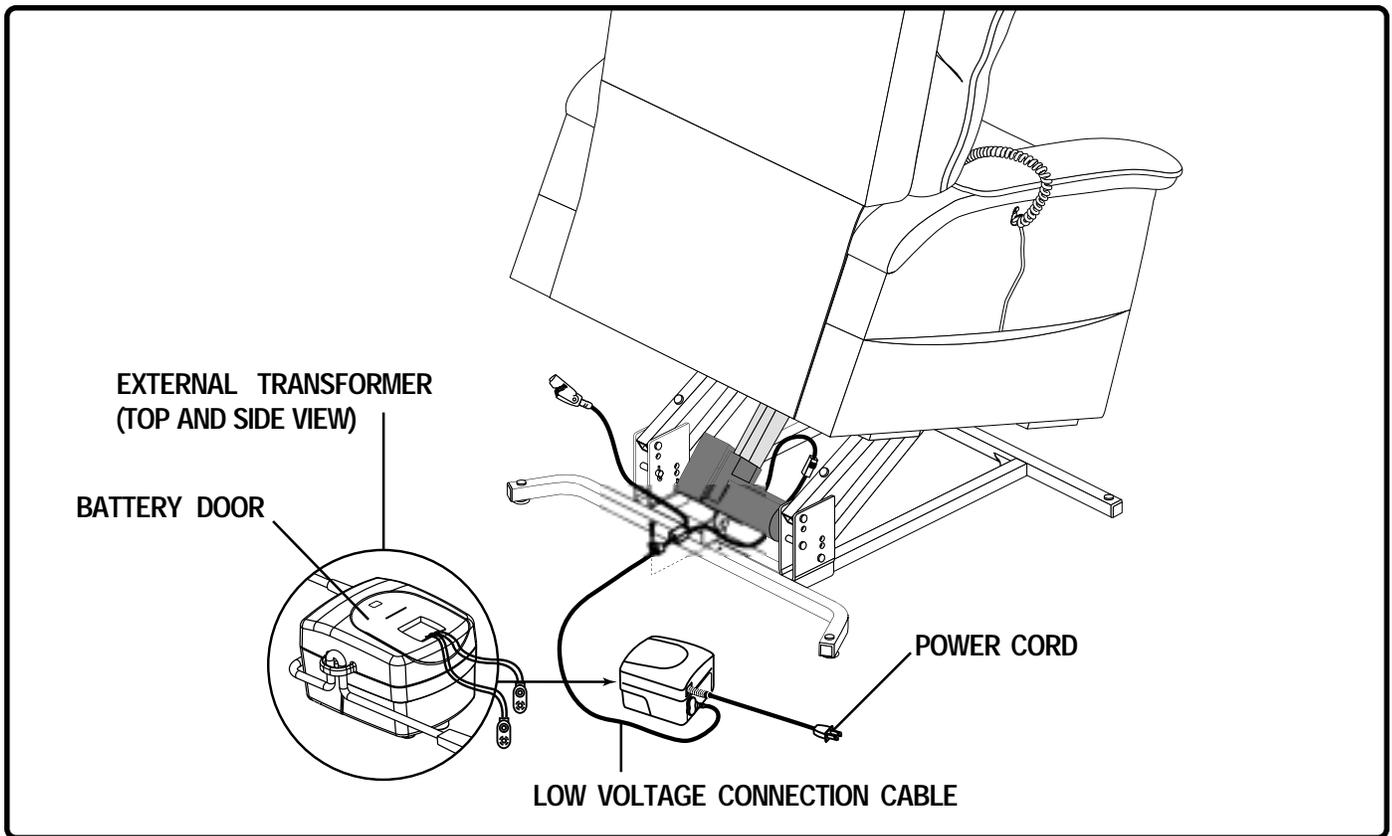


Figure 4.2 Standard Lift Chair Battery Location


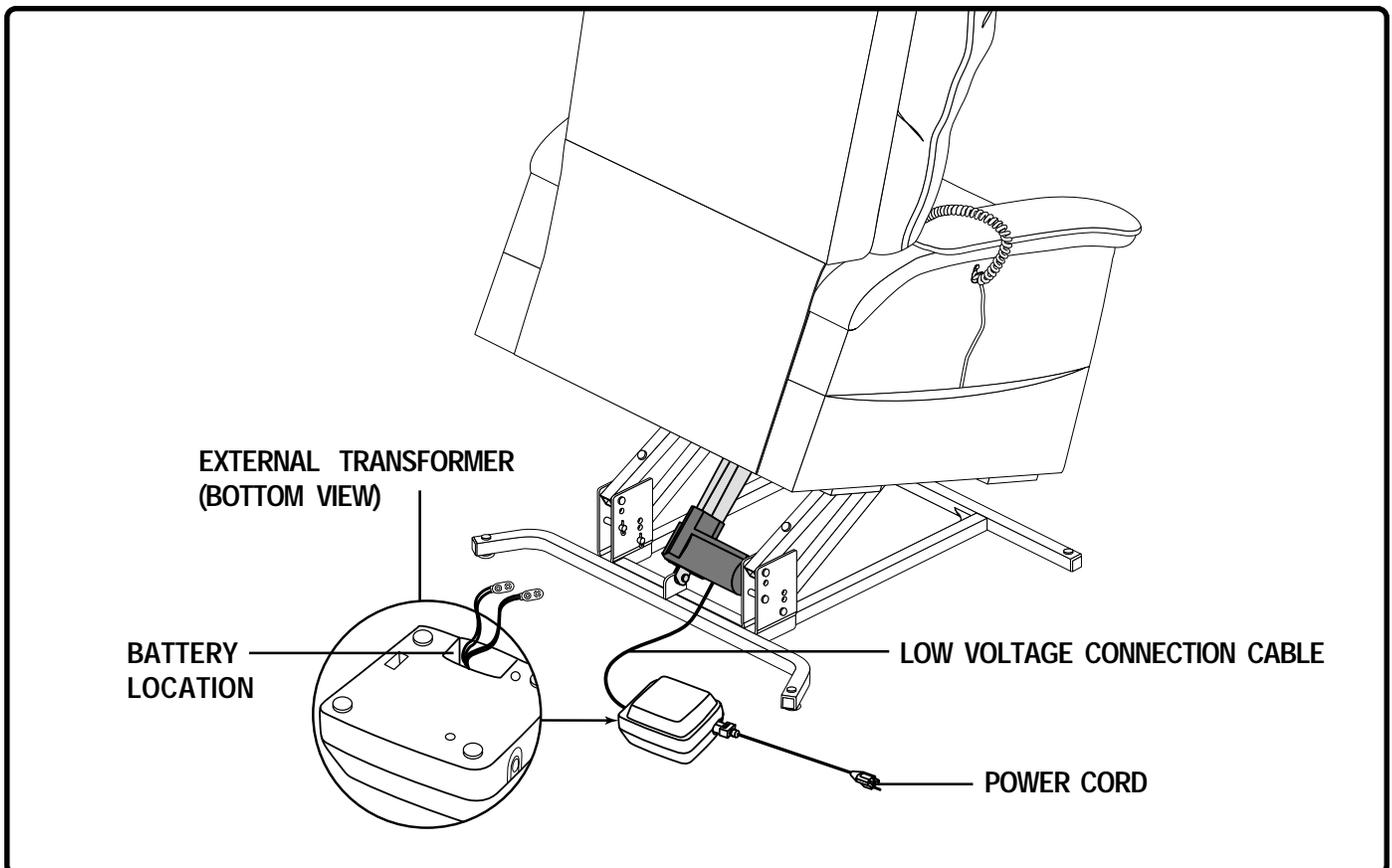


Figure 4.3 Heat and Massage Lift Chair Battery Location (Including 555 model)


LIFT CHAIR DISASSEMBLY

The back portion of KD Lift Chairs can be removed to make service and transport of the chair more convenient.


Follow these steps to disassemble the lift chair:

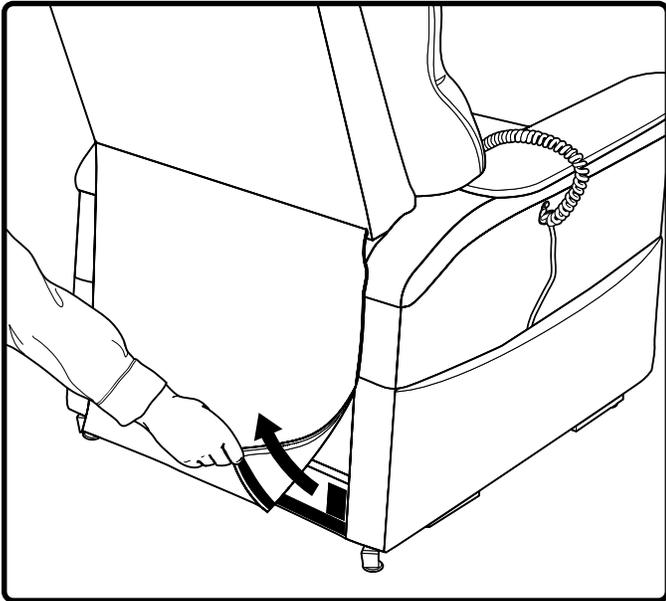


4.4.1. Remove the bottom of the back fabric from the hook and loop flaps on the chair base.

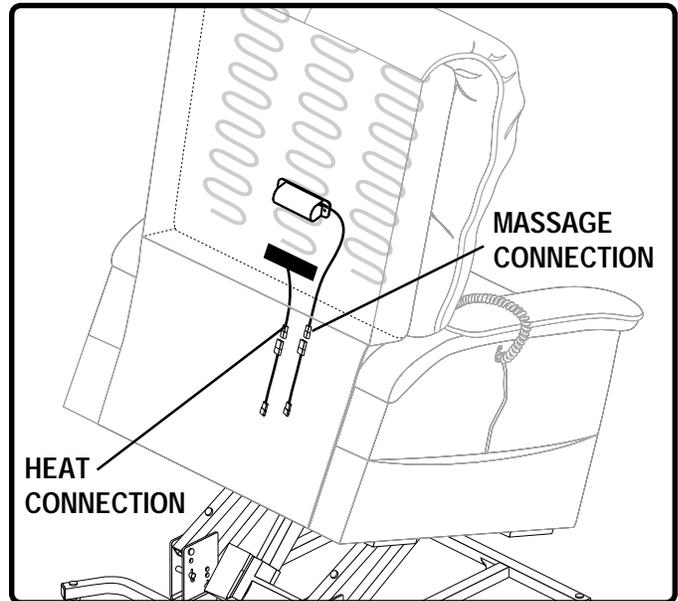

4.4.2. If your lift chair is equipped with heat and/or massage options, locate the quick-release connections, grip the plastic ends, and pull the connections apart. Avoid pulling on the wires when making the disconnection.

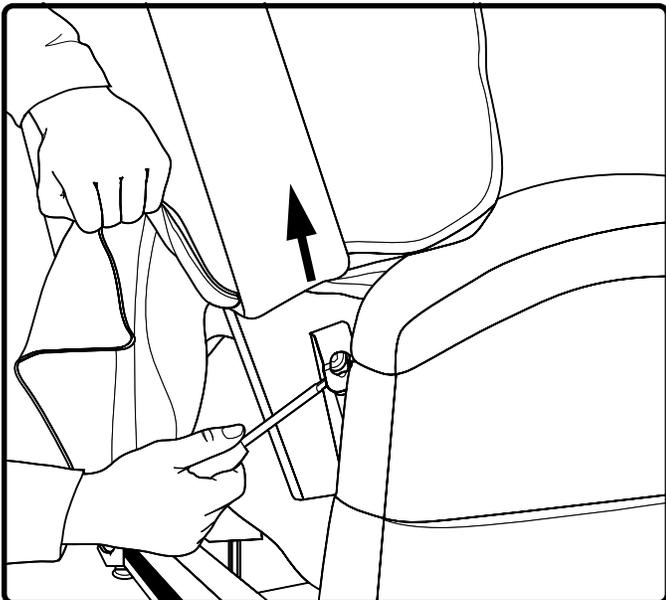

4.4.3. Using a large flat-head screwdriver, lift the left and right locking clips on the chair-back one side at a time to raise the back from the connections on the chair base. As you lift each locking clip, pull up on the chair-back to lift that side over the lock.

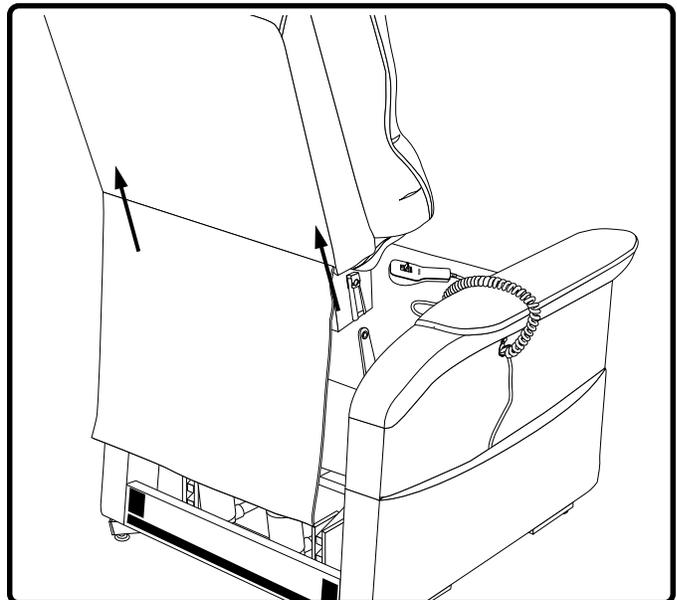

4.4.4. Lift the chair-back up and away from the chair base.



OPERATION PRECAUTIONS

There are certain precautions that should be taken during the operation of your lift chair. Read and follow these precautions carefully in order to ensure safe lift chair operation and to prevent injury and/or product damage.

- Plug the power cord directly into the electrical outlet. **Do not use an extension cord!**
- Do not place anything (for example, a drinking glass) on top of or near the external transformer.
- If the external transformer box or hand control requires cleaning, unplug the power cord from the electrical outlet and use a clean, dry cloth or lightly dampened cloth. Allow ample drying time before plugging the power cord back into the electrical outlet.
- Periodically check the hand control and all power cords for visible damage.
- Keep the hand control away from all heated surfaces.
- Ensure the hand control is out of the way before sitting in the chair.
- Keep children and pets away from all moving parts while operating the lift chair.
- Do not allow children to play on or operate the lift chair. Only the intended user should operate the lift chair.
- Avoid pinch points, such as the lift and scissor mechanisms. Keep hands and feet clear of these areas.
- Always leave the chair in an upright and closed position when not in use.
- Do not sit or stand on the footrest or the extended footrest.
- Do not “drop” into the lift chair when sitting if it is in a partially raised position.



PROHIBITED! Do not place objects under the lift chair. Make sure area is clear of obstructions including pets and small children during operation.



WARNING! Prevent the risk of electrical shock, fire, falls and/or being pinched. Follow all instructions provided.

HAND CONTROL OPERATION

Depending on lift chair model, the hand control may be equipped with switches that control the movement of the chair-back, chair base, and footrest, as well as the heat and massage options (Fig. 5.1 and 5.2).

NOTE: For the DMR/Infinite Series Lift Chairs, move the lift/recline switch (Fig. 5.3) to the “lift” position before raising your chair. On the deluxe heat and massage hand control, this is a push button located above the toggle switch. A green LED will indicate whether you are in lift or recline mode.

NOTE: Heat and massage functions will be temporarily shut down when using the hand control to change lift chair position. Normal function will return when the motion stops.

Follow these steps to sit in your lift chair (Fig. 5.1):

1. Engage and hold the up/down switch in the UP position to raise your lift chair to the standing position.
2. Back into the lift chair and sit in the center of the seat, using the armrests for support if needed.
3. Engage and hold the up/down switch in the DOWN position to lower the lift chair to a comfortable seated position.

Follow these steps to recline your lift chair from the seated position (Fig. 5.1):

1. Engage and hold the up/down switch in the DOWN position until comfortably reclined.
2. Engage and hold the up/down switch in the UP position to return to an upright, seated position. Release the switch when the lift chair reaches a comfortably seated position.

Follow these steps to stand up from your lift chair (Fig. 5.1):

1. Engage and hold the up/down switch in the UP position to raise your lift chair.
2. Release the switch when the lift chair reaches a height where you can stand up comfortably.

NOTE: To avoid personal injury, the DMR/Infinite Series models require that the lift chair be upright in the seated position before raising it to the standing position.

Follow these steps to activate/deactivate the massage option (Fig. 5.2):

1. Slide the white massage switch to the ON position to activate the massage option.
2. Slide the white massage switch to the OFF position to deactivate the massage option.

Follow these steps to activate/deactivate the heat option (Fig. 5.2):

1. Slide the red heat switch to the ON position to activate the heat option.
2. Slide the red heat switch to the OFF position to deactivate the heat option.

NOTE: Heat units are designed with an auto-shutoff mechanism that turns off the heat after 15 minutes of use. To reactivate the heat function, slide the red heat switch to the OFF position and then back to the ON position.

NOTE: Pride recommends that the heat and massage units not be used for more than 15 minutes at a time. However, you should consult your physician about how often you should use the heat and massage options on your lift chair.

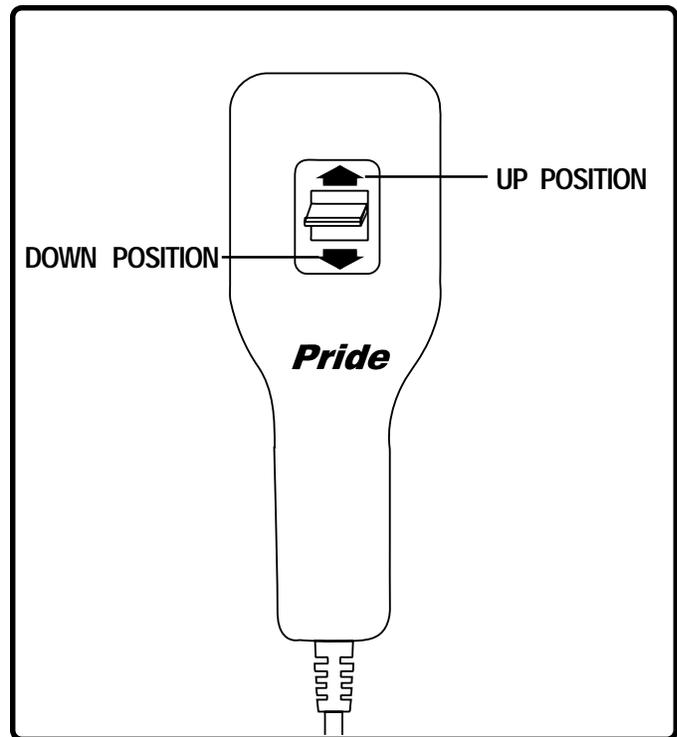


Figure 5.1 Pride Lift Chair Standard Hand Control

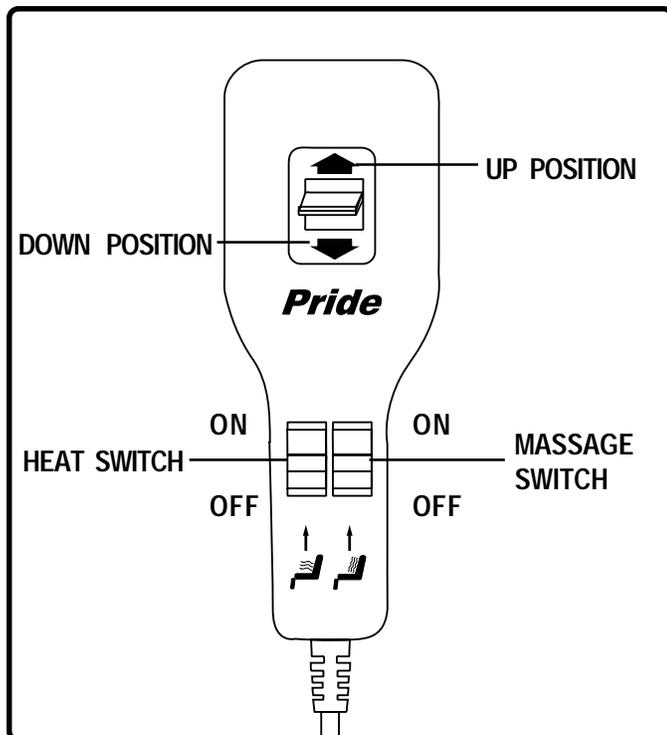


Figure 5.2 Heat and Massage Hand Control

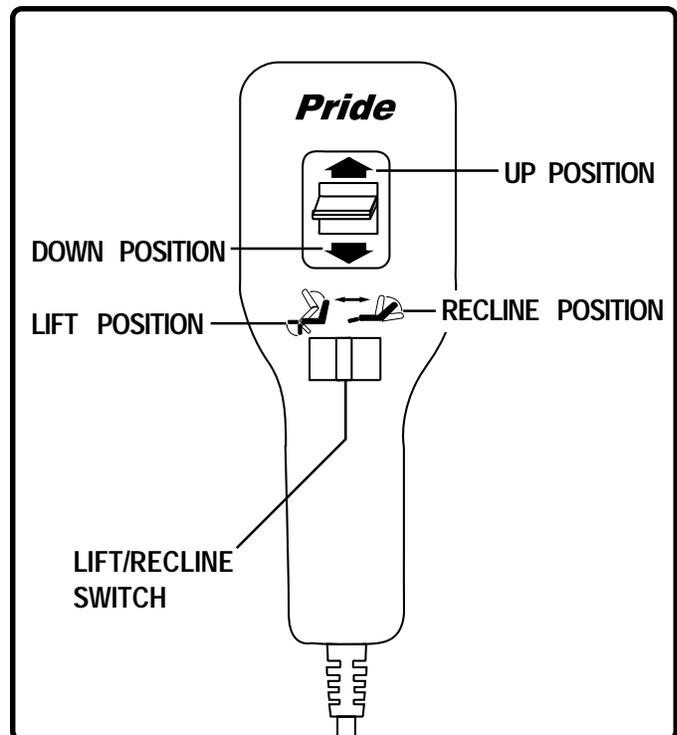


Figure 5.3 Infinite Series Hand Control

Follow these steps to activate the deluxe heat option:

1. Press the red heat on/off button to activate the deluxe heat option (Fig. 5.4). The heat will shut off after 12-15 minutes of continuous use.

Follow these steps to activate and adjust the deluxe massage option:

1. Press the blue massage on/off button to activate the deluxe massage option (Fig. 5.4).
2. Press the blue massage increase/decrease button until the desired massage intensity is selected (Fig. 5.4).
3. Press the blue massage mode button until the desired type of massage is selected (Fig. 5.4).

NOTE: The massage function will shut off after 12-15 minutes of continuous use.

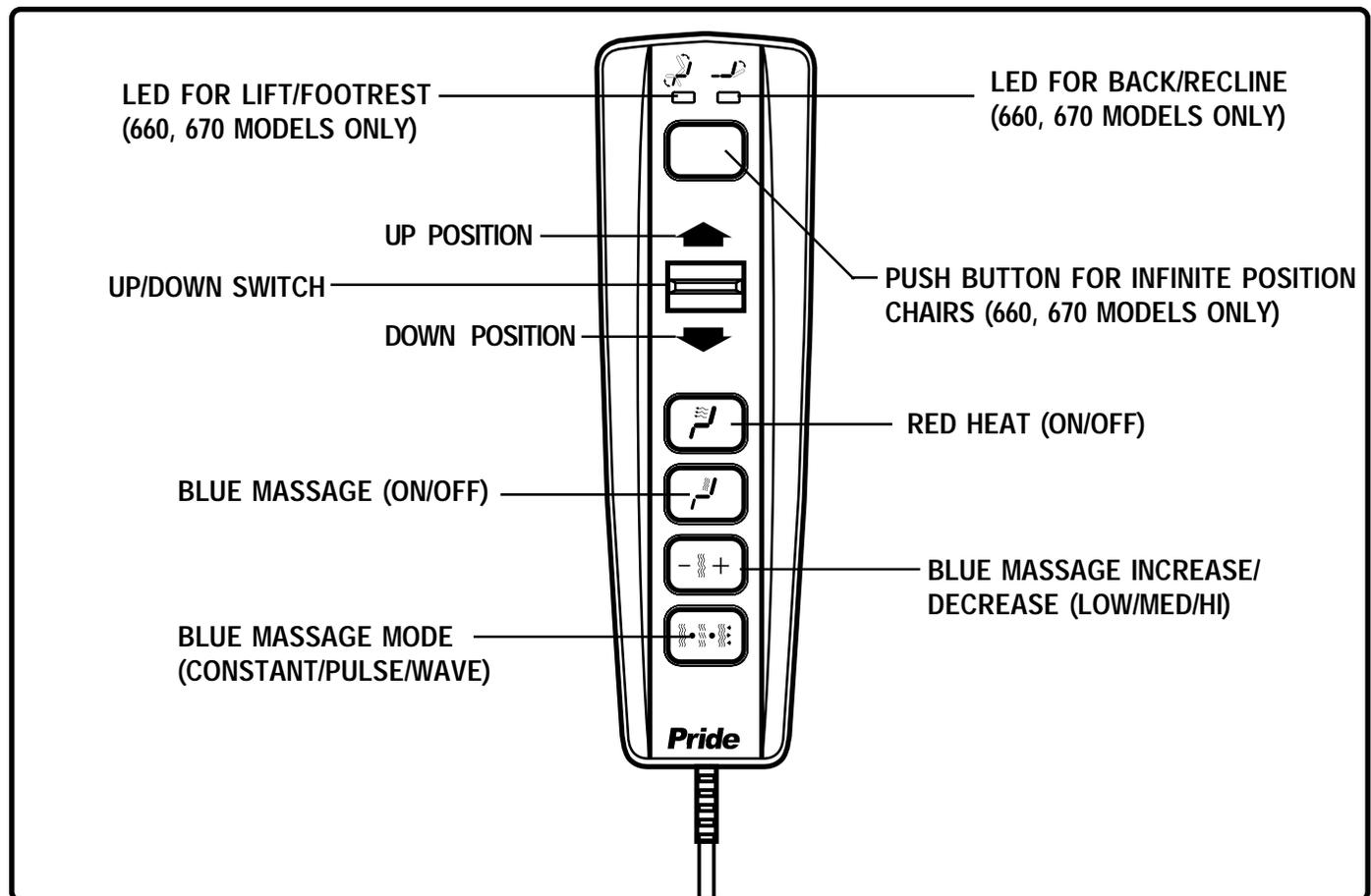


Figure 5.4 Deluxe Heat and Massage Hand Control



Your Pride Lift Chair is a state-of-the-art product designed to enhance your mobility. Your lift chair should bring you years of trouble-free service, however, it may require occasional troubleshooting. The following troubleshooting tips and FAQs should summarize what you need to know about your lift chair. If at any time you do not feel comfortable performing the troubleshooting steps listed in this manual, contact your authorized Pride Provider for service. Please have the model number, serial number, and nature of the problem when calling.

FREQUENTLY ASKED QUESTIONS

What if my lift chair does not operate at all?

- Ensure the external transformer is plugged into a properly wired electrical outlet.
- Check the circuit breaker box connected to the electrical outlet.
- Ensure the low voltage connection cable is plugged into the external transformer.
- Ensure the hand control and motor(s) are plugged in properly.
- Ensure the low voltage connection cable is plugged into the motor(s).
- Move the up/down switch on the hand control to the UP and DOWN positions and listen for a clicking noise from the external transformer. For heat and massage models, the clicking noise comes from the motor control box under the chair. If there is no clicking noise, then the motor is not receiving power from the hand control and the hand control may have to be replaced. If you do hear a clicking noise, then there may be a problem with the motor and it may need to be serviced or replaced. Contact your authorized Pride Provider for further assistance.

What if my lift chair operates in one direction only?

- Check the up/down switch on the hand control to make sure there is a smooth movement from the UP position to the DOWN position. If the switch is sticking in either position, the hand control may need to be replaced.
- Move the up/down switch on the hand control to the UP and DOWN positions and listen for a clicking noise from the external transformer. If there is no clicking noise, then the motor may not be receiving power from the hand control and the hand control may need to be replaced.
- Contact your authorized Pride Provider for further assistance.

What if my lift chair stops during a lifting cycle?

- Your lift chair is equipped with an internal thermal shutoff switch located inside the external transformer that prevents the motor control box from overheating. If the thermal shutoff activates, allow the lift chair to remain in a stationary position for 10 minutes to allow the motor to cool, then resume normal operation. If you notice the motors overheating frequently, contact your authorized Pride Provider.
- There may have been a power failure and/or there are no batteries in the external transformer or the batteries have no charge. Always make sure there are fresh 9V batteries in the external transformer.
- Check the circuit breaker box connected to the electrical outlet.

What if my lift chair is rocking from corner to corner after I position the chair?

The floor may be uneven or the carpet may be affecting chair position. Adjust the leg levelers in the area where the chair is rocking. See III. "Your Lift Chair" for instructions on leveling your chair.

What if the footrest and recline functions on my DMR/Infinite Series lift chair do not work properly?

Make sure the switches on your hand control are at the proper settings.

Where can I place the external transformer box?

- On the floor in an open, well-ventilated area where it will not be an obstruction.
- On the wall utilizing the optional external transformer wall mount.

Where do I find the serial number on my lift chair?

You can find the serial number in two locations—one is near the rear of the motor on the steel lift mechanism, the second is attached to the frame below the scissor mechanisms. The model number for your lift chair is printed below the serial number bar code.

Who do I call for service?

Contact your authorized Pride Provider for service.



Your Pride Lift Chair will require routine maintenance checks. You can perform some of these checks, but others may require assistance from an authorized Pride Provider. By following the maintenance checks in this section as scheduled, you can help ensure your lift chair gives you years of trouble-free operation. If you have any questions regarding your lift chair's care, contact your authorized Pride Provider.

FABRIC CARE

- Inspect the fabric on a regular basis for any pulls, tears, or gaps.
- Choose the right methods to clean the fabric on your lift chair. Before cleaning your lift chair, consult your authorized Pride Provider for cleaning codes.

WARNING! Fabric should not be allowed to come in direct contact with any type of heat source, e.g., a space heater or cigarette. Personal injury or fire damage may occur.



WARNING! Pride strongly recommends that you do not smoke cigarettes while seated in or using your lift chair, although the lift chair has passed the necessary testing requirements for cigarette smoking. You must adhere to the following safety guidelines if you decide to smoke cigarettes while seated in or using your lift chair.

- Do not leave lit cigarettes unattended.
- Keep ashtrays a safe distance from the lift chair.
- Always make sure cigarettes are completely extinguished before disposal.

NOTE: Do not use solvent-type cleaners to spot clean. Do not saturate the fabric. Prevent damage to the fabric; do not expose the lift chair to direct sunlight.

ELECTRONICS CARE

- Inspect all wiring harnesses to make sure they are not damaged or frayed.
- If damage is present, unplug the lift chair and contact your authorized Pride Provider for service.
- Keep all electronics free from moisture and temperature extremes. **Pride Lift Chairs are intended for indoor use only!**

WARNING! Even though the lift chair has passed the necessary testing requirements for ingress of liquids, you should keep electrical connections away from sources of dampness, including direct exposure to water or bodily fluids and incontinence. Check electrical components frequently for signs of corrosion and replace as necessary.



WARNING! Prevent the risk of electrical shock. Ensure that any wires leading from the external transformer to the electrical outlet are kept clear of lift chair pinch points and mechanical linkages.

DISPOSAL AND RECYCLING

Your lift chair must be disposed of according to applicable local and national statutory regulations. Contact your local waste disposal agency or authorized Pride Provider for information on proper disposal of lift chair packaging, metal frame components, fabric, electronic components, and batteries.



WARNING! Plastic bags are a suffocation hazard. Dispose of plastic bags properly and do not allow children to play with them.



LIFETIME WARRANTY

Pride offers a lifetime warranty on the steel lift mechanism. Pride warrants that the steel lift mechanism on your Pride Lift Chair will be free from defects under normal use and service for as long as the chair is in use by the original owner. This lifetime warranty covers the steel lift mechanism against metal fatigue, defective welds, and bushings.

NOTE: Pride reserves the right to replace only the part of the steel lift mechanism that may be defective.

SEVEN-YEAR PRO-RATED WARRANTY (LUXURY LINE, INFINITE SERIES)

For the first four (4) years from the date of purchase, Pride will repair or replace at our option to the original purchaser any of the following parts found upon examination by an authorized representative of Pride to be defective in material and/or workmanship:

- Wood Frame
- Motor
- Hand Control
- Scissor Mechanisms

For three (3) additional years, a pro-rated warranty will be made available for the following parts:

- Motor
- Hand Control

Pride will warrant these components with the following pro-rated schedule:

- **Fifth year:** 25% off suggested retail price
- **Sixth year:** 15% off suggested retail price
- **Seventh year:** 5% off suggested retail price

SEVEN-YEAR PRO-RATED WARRANTY (GENTLE LINE)

For the first three (3) years from the date of purchase, Pride will repair or replace at our option to the original purchaser any of the following parts found upon examination by an authorized representative of Pride to be defective in material and/or workmanship:

- Wood Frame
- Motor
- Hand Control
- Scissor Mechanisms

For four (4) additional years, a pro-rated warranty will be made available for the following parts:

- Motor
- Hand Control

Pride will warrant these components with the following pro-rated schedule:

- **Fourth year:** 35% off suggested retail price
- **Fifth year:** 25% off suggested retail price
- **Sixth year:** 15% off suggested retail price
- **Seventh year:** 5% off suggested retail price

TWO-YEAR LIMITED WARRANTY (CASUAL LINE/PHC LINE)

For two (2) years from the date of purchase, Pride will repair or replace at our option to the original purchaser any of the following parts found upon examination by an authorized representative of Pride to be defective in material and/or workmanship:

- Wood Frame
- Motor
- Hand Control
- Scissor Mechanisms

WARRANTY EXCLUSIONS

This warranty does not extend to fabric. The Wallhugger (805) Lift Chair is excluded from the pro-rated portion of the warranty schedule above.

Pride is not responsible or obligated to pay for labor, service calls, shipping, and/or any other charges incurred for repair of the product. All transportation costs and shipping damage incurred while submitting parts for repair or replacement are the responsibility of the purchaser. Please contact your authorized Pride Provider for information on the current cost associated with a service visit.


LIFT CHAIR WEIGHT CAPACITIES					
Model Number	Wallhugger (805)	C10, C15	545	555	All other models
Rated Weight Capacity	300 lbs. (136 kg)	325 lbs. (147 kg)	500 lbs. (227 kg)	600 lbs. (272 kg)	375 lbs. (170 kg)



MANDATORY! Stay within the specified weight capacity of your lift chair. Exceeding the weight capacity voids your Pride Lift Chair warranty.

EXTERNAL TRANSFORMER OPTIONAL WALL MOUNT DIAGRAM

Cut out this diagram (1:1 scale) and use to mark area of placement for the external transformer.

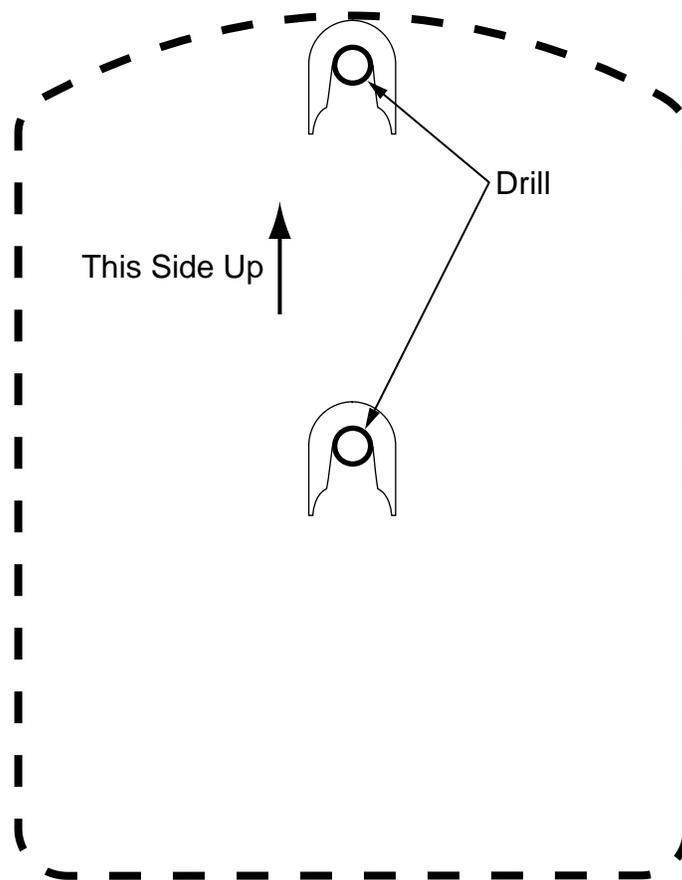



Pride

Mobility Products Corp.

182 Susquehanna Avenue
Exeter, PA 18643-2694
www.pridemobility.com