



FOLDABLE POWER WHEELCHAIR

LIGHTWEIGHT AND **FOLDABLE** FOR TRAVEL

The new **i-Go™** from Pride® features advanced folding technology, enabling it to be quickly transported with ease. The i-Go features front suspension, a durable seating system, under seat storage and much more. Designed to fit inside any small space and weighing only 19.8 kg (43.7 lbs.), the i-Go is the perfect choice for the active individual.



Ultra compact
when folded



Convenient for travel

PRIDE® **MOBILITY** SCOOTERS
LIVE YOUR BEST®

www.pridemobility.com.au



SPECIFICATIONS

I-GO™ FOLDABLE POWER WHEELCHAIR

Max. Weight capacity:	120 kg (265 lbs.)
Maximum speed ¹ :	4,5 km/h (2.8 mph)
Overall dimensions ² :	90 cm x 59 cm x 88 cm (35.4" x 23.2" x 34.6")
Seat height:	50 cm (19.7")
Seat width:	47 cm (18.5")
Arm height:	69 cm (27.2")
Seatback height:	86 cm (33.9")
Folded dimensions ² :	58 cm x 31 cm x 75,5 cm (22.8" x 12.2" x 29.7")
Battery weight ³ :	1,35 kg (3 lbs.)
Total weight without batteries:	19,8 kg (43.7 lbs.)
Braking distance (on level surface):	100 cm (39.4")
Minimum turning radius ² :	90 cm (35.4")
Theoretical travel ranging:	10 km (6.2 miles) per battery
Static stability:	≥6°
Dynamic stability:	≥6°
Motor type:	(2) 24 volt, 200W
Batteries:	(2) 24 volt, 6.6AH (Lithium-ion)
Contoller type:	PG VR2
Max. output current of controller:	35A
Max. output current of charger:	2A
Castors:	14,5 cm (5.7") (solid)
Rear wheels:	21,6 cm (8.5") (pneumatic)

Colour:



Silver

FEATURES

- Quickly folds in 5 easy steps
- Unique seating for added comfort
- Front suspension
- Large, secure storage compartment
- Lightweight Lithium-ion batteries
- Easy-to-use VR2 controller

HOW TO FOLD:



1 Switch off and remove joystick



2 Place joystick in backrest pocket and remove cushion



3 Fold backrest down, then pull both arm rests back



4 Pull the 2 black toggles together (located on the back)



5 Pull down on the seat frame until unit fully flattens



6 The i-Go is now ready to go!

- ¹ Varies with user weight, terrain type, battery amp-hour (AH), battery charge, battery condition and tyre condition. These specifications can be subject to a variance of (+/- 10%).
- ² Due to manufacturing tolerances and continual product improvement, this specification can be subject to a variance of (+ or - 3%).
- ³ Battery weight may vary based on manufacturer.

NOTE: All specifications subject to change without notice.

The information contained herein is correct at the time of publication; due to Pride's commitment to constant improvement and development, we reserve the right to alter specifications without prior notice. Theoretical range is calculated under test conditions in accordance with European standards. Actual range in normal use will depend on many factors, including the condition of the vehicle and its batteries, the weight of the driver, correct tyre pressures, ambient temperature, and the gradient and surface of the road or pavement. Dimensions listed are for power base only. Overall measurements will vary based on seating and accessory selections.

Pride
Mobility Products
Australia Pty. Ltd.
— LIVE YOUR BEST —

PRIDE® MOBILITY SCOOTERS
LIVE YOUR BEST®



(Tel) 03 8770 9600 • (Fax) 03 9703 2720
20-24 Apollo Drive Hallam, Victoria 3803

ACN 088 609 661 ABN 66 088 609 661

www.pridemobility.com.au

