SHELL REMOVAL AND INSTALLATION (CANADIAN UNITS ONLY)


	<p>WARNING: Always remove the child from the chair prior to removal of the shell assembly from the mobility base. Removal of the shell assembly is only intended for lifting and transportation of the chair. The child should never be placed in the shell assembly if it is not properly attached to the mobility base. The mobility base should never be used without the shell assembly properly installed.</p>
--	--

1. Shell Removal (Reference Images Page 22 of User Manual)

- a. Engage the wheel locks.
- b. Turn the ACTIVE handle to the OFF position.
- c. Remove the child from the chair.
- d. Pull outward on pull pin (shown).
- e. Pull up the hip angle adjustment lever [D] (**Fig. 40**).
- f. Lift shell assembly upward and out of mobility base.
- g. Release pull pin.

2. Shell Installation (Reference Images Pages 22-23 of User Manual)

- a. Lower shell assembly into mobility base so that pivot bushings [E] enter slot [F] (**Fig. 42**).
- b. Pull outward on pull pin (shown).
- c. Pull up the hip angle adjustment lever [D] and adjust the shell to the desired hip angle position (**Fig. 40**). Engage lever. Check for proper pin engagement and lever position. - refer to page 17, section C.
- d. Turn ACTIVE lever to the ON position and cycle ACTIVE motion to ensure proper installation.
- e. Apply forward pressure to the handlebar and turn the ACTIVE lever to the off position.

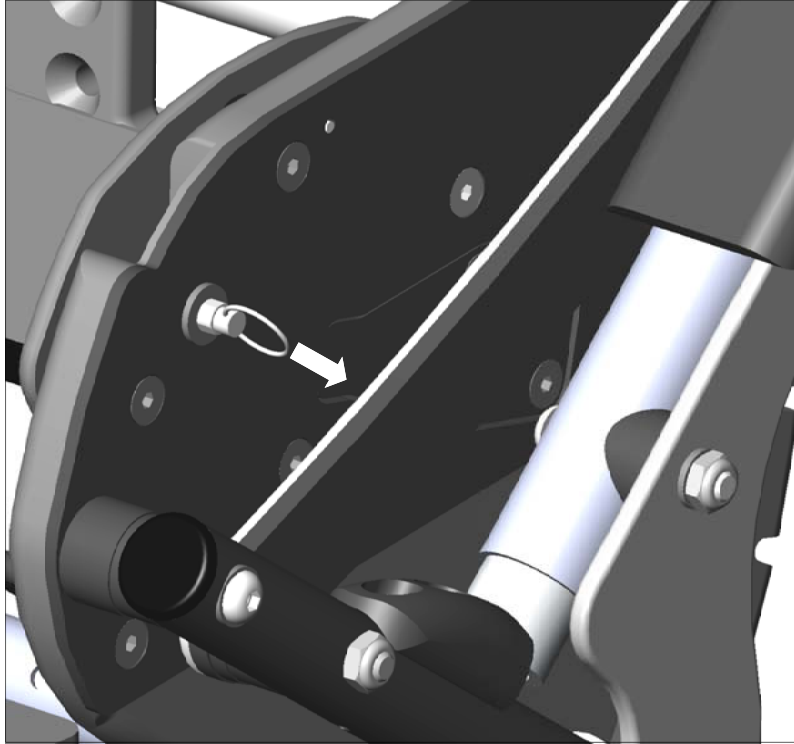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