NOTE: The following information is in addition to the owner's manual supplied with your power chair. It explains how to safely push your Jazzy® Passport/iGo+® in both the folded and unfolded positions. You must read the power chair owner's manual and all supplemental information before use. Please retain this information for your records, and contact your authorized Pride Provider with any questions.

To push your Jazzy® Passport/iGo+® in the unfolded position:

1. Ensure the power chair is turned off.
2. Standing behind the power chair, grasp the top of the seatback and tilt the power chair backward to balance it onto the anti-tip wheels. **See figure 1.** Push your chair forward.

WARNING! Do not tilt and manually transport your power chair while it is occupied. Doing so can cause personal injury and/or property damage.

To push your Jazzy® Passport/iGo+® in the folded position:

1. Ensure the power chair is turned off.
2. Grasp the edge of the seat base with one hand while supporting the seatback with your other hand, and pull your power chair up onto the four (4) wheels in a standing position. **See figure 2.**
3. Grasp the top of the seat base and push your power chair forward. **See figure 2.**

Figure 1. Unfolded Manual Transport

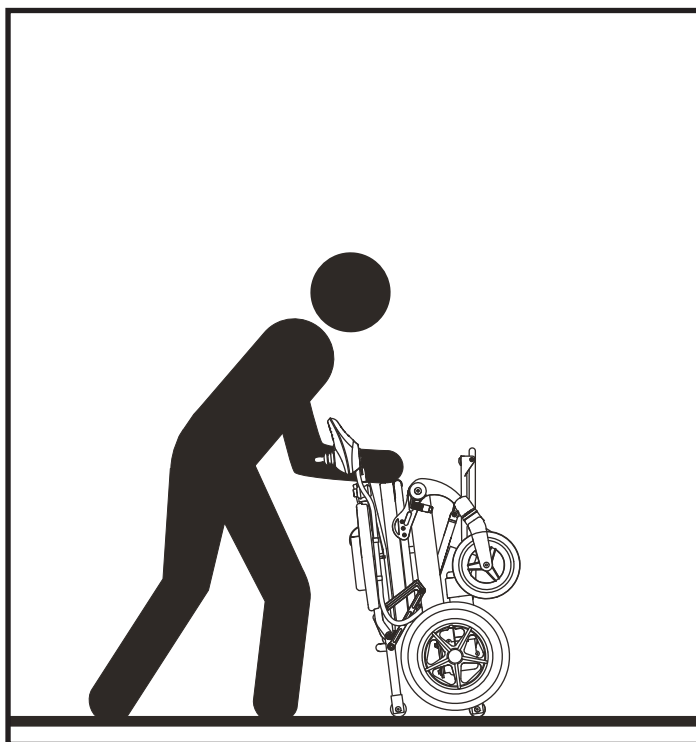
Figure 2. Folded Manual Transport

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