MANUAL RECLINE

LL460 CHAIR

OWNER'S MANUAL

The symbols below are used throughout this owner's manual and on the product to identify warnings and important information. It is very important for you to read them and understand them completely.

WARNING! Indicates a potentially hazardous condition/situation. Failure to follow designated procedures can cause either personal injury, component damage, or malfunction. On the product, this icon is represented as a black symbol on a yellow triangle with a black border.

MANDATORY! These actions should be performed as specified. Failure to perform mandatory actions can cause personal injury and/or equipment damage. On the product, this icon is represented as a white symbol on a blue dot with a white border.

PROHIBITED! These actions are prohibited. These actions should not be performed at any time or in any circumstances. Performing a prohibited action can cause personal injury and/or equipment damage. On the product, this icon is represented as a black symbol with a red circle and red slash.

Please fill out the following information for quick reference:


Pride Provider: _____

Address: _____

Phone Number: _____

Purchase Date: _____ Serial Number: _____

NOTE: This owner's manual is compiled from the latest specifications and product information available at the time of publication. We reserve the right to make changes as they become necessary. Any changes to our products may cause slight variations between the illustrations and explanations in this manual and the product you have purchased. The latest/current version of this manual is available on our website.

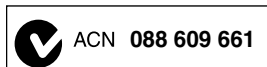Copyright © 2010

Pride Mobility Products Europe B.V.

INFCOVR1001/Rev A/September 2010

INFMANU4078/Rev A/September 2010

PRODUCT SAFETY SYMBOLS

The symbols below represent labels used on the product to identify warnings, mandatory actions, and prohibited actions. It is very important for you to read and understand these symbols completely. Do not remove these labels from your product. Please note that not all of the symbols may be used on your chair model.

Pinch/Crush Points Hazard! Do not place objects or appendages in the path of moving parts.

Read and follow the information in the owner's manual.

Maximum weight capacity

Do not place objects under the footrest when chair is in fully reclined position. Make sure area is clear of obstructions, including pets and small children during operation.

Indoor Use Only. Avoid exposure to rain, snow, ice, salt, or standing water. Maintain and store in a clean and dry environment.

SAFETY GUIDELINES	2
LABEL INFORMATION	3
I. INTRODUCTION.....	5
SAFETY	5
PURCHASER'S AGREEMENT	5
INFORMATION EXCHANGE.....	5
II. GENERAL GUIDELINES	6
MODIFICATIONS	6
WEIGHT LIMITATIONS	6
PINCH/CRUSH HAZARDS.....	6
SHIPPING AND DELIVERY	6
MOTOR VEHICLE TRANSPORT	6
CHAIR PLACEMENT	6
III. SETUP AND OPERATION	7
SETUP	7
CHAIR OPERATION.....	9
OPERATION PRECAUTIONS.....	10
IV. CARE AND MAINTENANCE	11
FABRIC CARE.....	11
DISPOSAL AND RECYCLING	11
V. WARRANTY	12

SAFETY

The product you have purchased combines state-of-the-art components with **safety**, comfort, and styling in mind. We are confident the design features will provide you with the conveniences you expect during your daily activities. Understanding how to **safely** use and care for this product should bring you years of trouble-free operation and service.

Read and follow all instructions, warnings, and notes in this manual and all other accompanying literature before attempting to use this product for the first time.

If there is any information in this manual which you do not understand, or if you require additional assistance for setup or use, please contact your authorized provider. **Failure to follow the instructions, warnings, and notes in this manual and those located on your product can result in personal injury or product damage and will void product warranty.**

PURCHASER'S AGREEMENT

By accepting delivery of this product, you promise that you will not change, alter, or modify this product or remove or render inoperable or unsafe any guards, shields, or other safety features of this product; fail, refuse, or neglect to install any retrofit kits from time to time provided to enhance or preserve the safe use of this product.

INFORMATION EXCHANGE

We want to hear your questions, comments, and suggestions about this manual. We would also like to hear about the safety and reliability of your new chair, and about the service you received from your authorized provider. Please notify us of any change of address, so we can keep you apprised of important information about safety, new products, and new options that can increase your ability to use and enjoy your chair.

NOTE: If you ever lose or misplace your product registration card or your copy of this manual, contact us and we will be glad to send you a new one immediately.

Below are some precautions, tips, and other safety considerations that will help you become accustomed to operating your product in a safe manner.

MANDATORY! Read and follow the information provided in this owner's manual before attempting to use your chair for the first time.

MODIFICATIONS

Your chair has been designed and engineered to provide maximum comfort and utility. However, to prevent personal injury and/or damage to your chair, you should not modify, add, remove, or disable any feature, part, or function of your chair. Unauthorized modifications may also void your product's warranty.

WEIGHT LIMITATIONS

Your chair is rated for a maximum weight capacity of 275 lbs. (113 kg).

MANDATORY! Stay within the specified weight capacity of your chair. The manufacturer and/or provider will not be held responsible for injuries and/or product damage resulting from failure to observe weight limitations.

PINCH/CRUSH HAZARDS

The scissor mechanism is labeled as a pinch/crush point hazard on your chair. Keep clear of these areas and make sure the path of motion is unobstructed.

WARNING! Do not place objects or appendages in the path of moving parts.

SHIPPING AND DELIVERY

Before using your chair, make sure your delivery is complete as some components may be individually packaged. If you do not receive a complete delivery, please contact your authorized provider immediately. Where damage has occurred during transport, either to the packaging or content, please contact the delivery company responsible.

MOTOR VEHICLE TRANSPORT

If you will be transporting your chair in a motor vehicle, individual components should be secured against slipping. The chair itself must also be secured against slipping (a possible hazard during vehicle braking).

CHAIR PLACEMENT

Place the back of the chair 30 in. (76 cm) from the nearest obstruction while the chair is in the seated position.

SETUP

Your chair will arrive in the knock-down (KD) position (**Fig. 3.1**) and will require some assembly before initial use. It may also require disassembly to make servicing more convenient. Refer to the following instructions and contact your authorized provider with any questions.




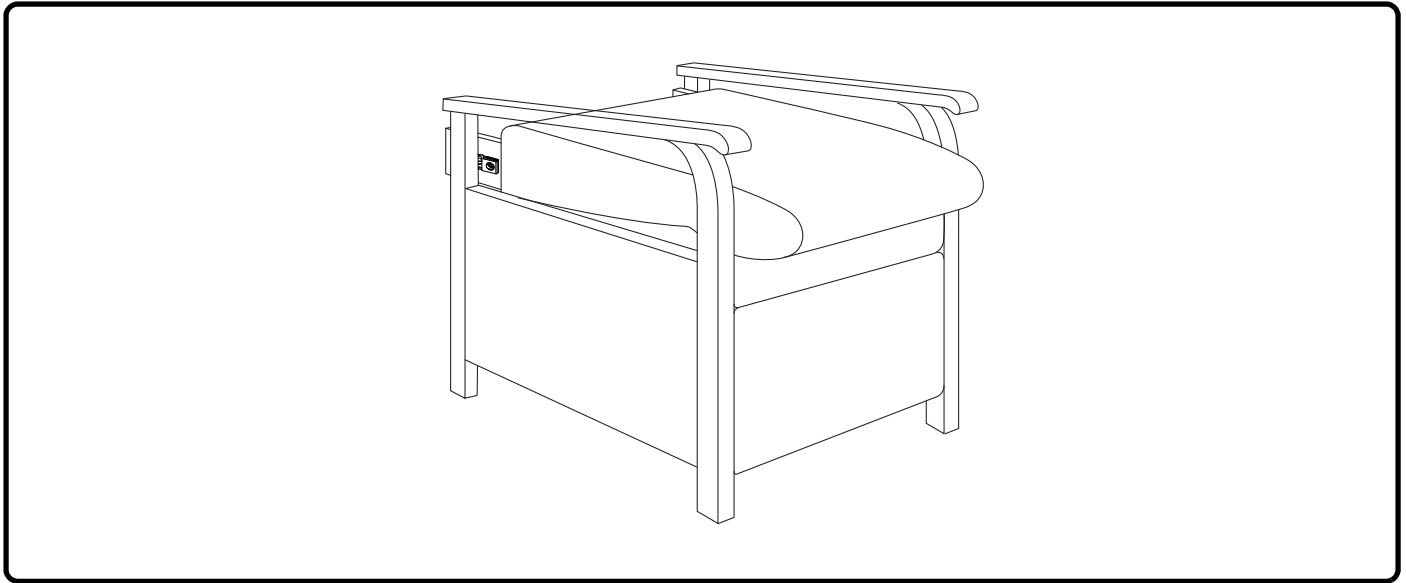
Figure 3.1 Knock-Down (KD) Position

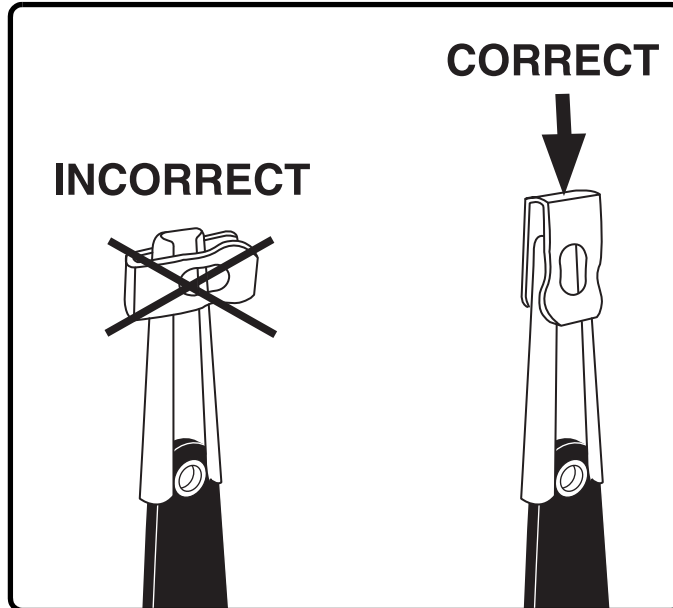3.1.1 Remove the packaging from the back portion of the chair and check that the locking clip is in the vertical position. Reposition if necessary.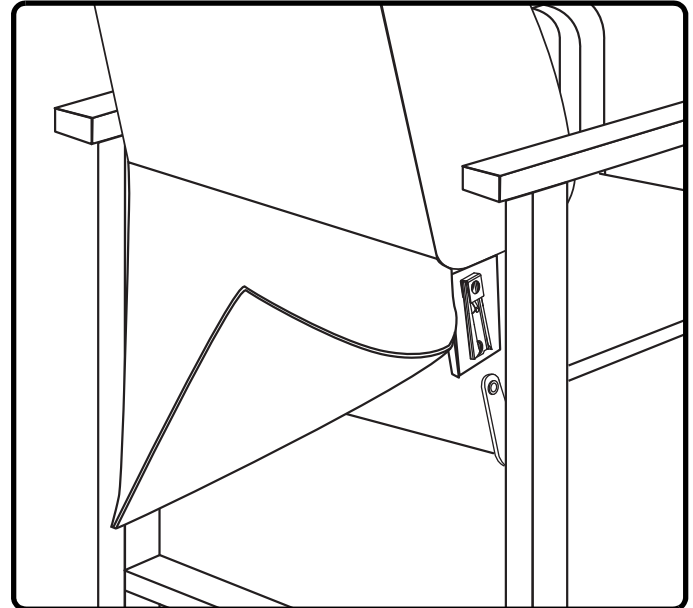


3.1.2 Slide the left and right KD sleeves of the chair-back onto the KD connections on the arms and seat of the chair.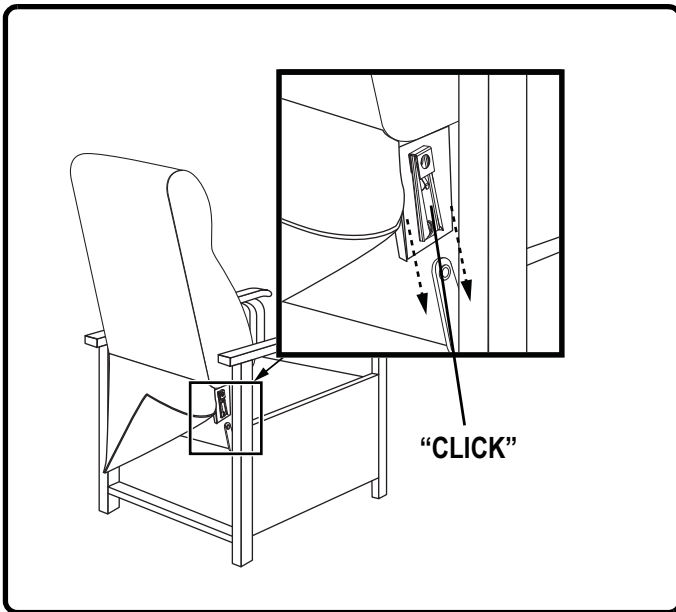


NOTE: Pull up on the chair-back to ensure that it is locked in place. If the back can be pulled off, then repeat steps 3.1.1 and 3.1.3. You must hear a click from each side of the chair.

- 3.1.3 Push firmly on the top of the chair-back to lock into position. Listen for a click on each side to ensure the chair-back is locked in properly.

Follow these steps to disassemble the chair: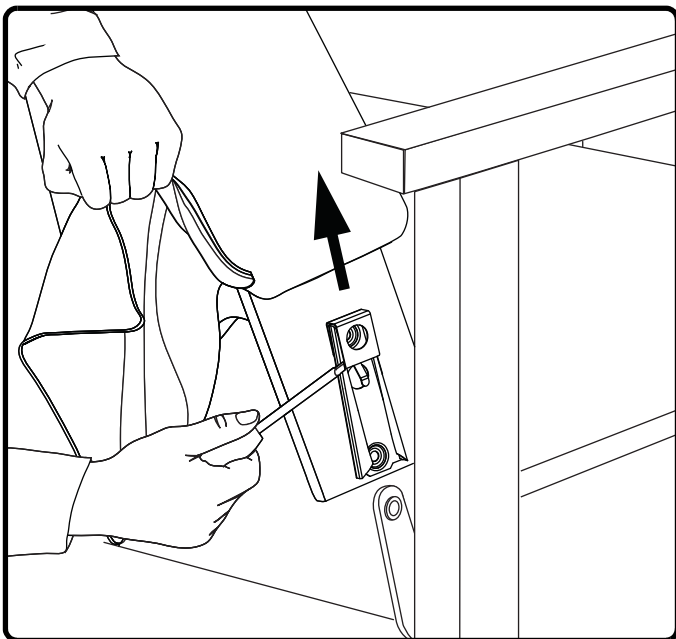

- 3.1.4 Using a large flat-head screwdriver, lift the left and right locking clips on the chair-back one side at a time to raise the back from the connections on the chair base. As you lift each locking clip, pull up on the chair-back to lift that side over the lock.

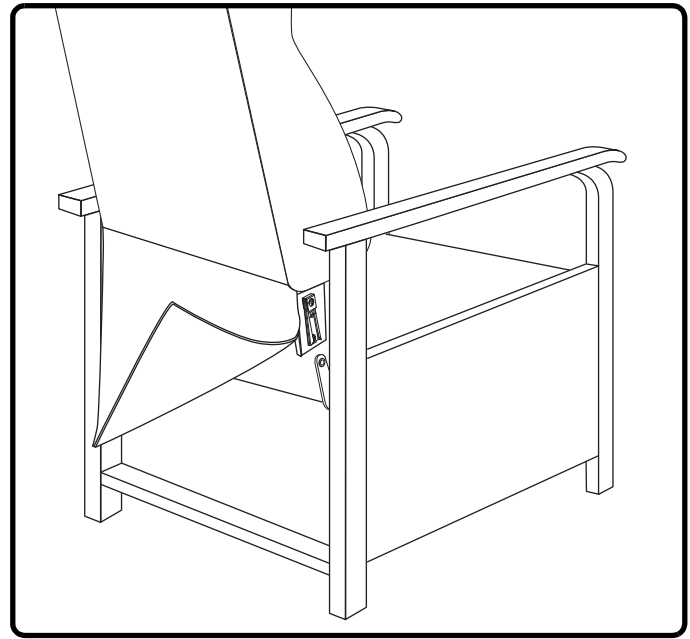- 3.1.5 Lift the chair-back up and away from the chair base.

CHAIR OPERATION

Scissor Mechanisms: Extend or retract to take the chair through the various stages of movement.

WARNING! Pinch/Crush Points Hazard! Be aware that the scissor mechanisms are a pinch point on the chair. Keep the following items and similar objects clear of these points when operating the chair:

- appendages
- small children
- pets
- wires
- oxygen tubes
- loose items, including but not limited to clothing and blankets
- electrical power cords

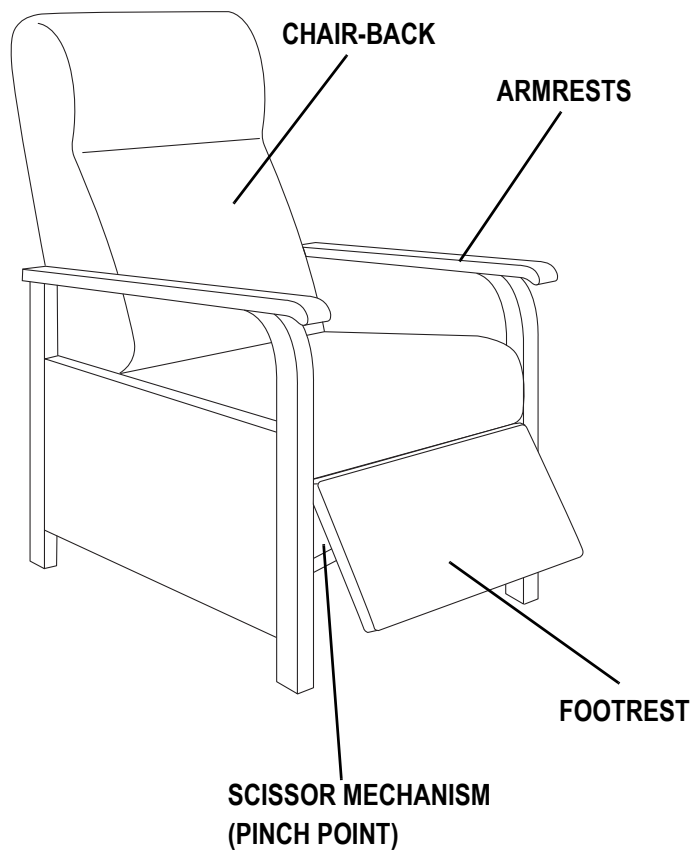
Figure 3.2 Body Components

Follow these steps to recline your chair from the seated position:

1. Back into the chair and sit in the center of the seat, using the armrests for support if needed.
2. Grip the armrests firmly and lean backward until the chair reaches a comfortably reclined position.

Follow these steps to bring your chair from the recline position to the seated position:

1. Grip the armrests firmly and move your torso forward while using your legs to apply pressure to the footrest until the chair reaches the seated position.

OPERATION PRECAUTIONS

There are certain precautions that should be taken during the operation of your chair. Read and follow these precautions carefully in order to ensure safe chair operation and to prevent injury and/or product damage.

- Keep children and pets away from all moving parts while operating the chair.
- Do not allow children to play on or operate the chair. Only the intended user should operate the chair.
- Avoid pinch points, such as the scissor mechanisms. Keep hands and feet clear of these areas.
- Always leave the chair in an upright and closed position when not in use.
- Do not sit or stand on the footrest.
- Do not “drop” into the chair when sitting in it.

WARNING! Do not place objects under the chair. Make sure area is clear of obstructions, including pets and small children during operation.

Your chair will require routine maintenance checks. You can perform some of these checks, but others may require assistance from an authorized provider. By following the maintenance checks in this section as scheduled, you can help ensure your chair gives you years of trouble-free operation. If you have any questions regarding your chair's care, contact your authorized provider.

FABRIC CARE

- Inspect the fabric on a regular basis for any pulls, tears, or gaps.
- Choose the right methods to clean the fabric on your chair. Before cleaning your chair, consult your authorized provider for cleaning codes.

WARNING! Fabric should not be allowed to come in direct contact with any type of heat source (e.g., a space heater or cigarette). Personal injury or fire damage may occur.

WARNING! It is strongly recommended that you do not smoke cigarettes while seated in or using your chair, although the chair has passed the necessary testing requirements for cigarette smoking. You must adhere to the following safety guidelines if you decide to smoke cigarettes while seated in or using your chair.

- Do not leave lit cigarettes unattended.
- Keep ashtrays a safe distance from the chair.
- Always make sure cigarettes are completely extinguished before disposal.

NOTE: Do not use solvent-type cleaners to spot clean. Do not saturate the fabric. Prevent damage to the fabric; do not expose the chair to direct sunlight.

DISPOSAL AND RECYCLING

Your chair must be disposed of according to applicable local and national statutory regulations. Contact your local waste disposal agency or authorized provider for information on proper disposal of chair packaging, metal frame components, and fabric.

WARNING! Plastic bags are a suffocation hazard. Dispose of plastic bags properly and do not allow children to play with them.

TWO-YEAR LIMITED WARRANTY

For two (2) years from the date of purchase, your provider will repair or replace at our option to the original purchaser any of the following parts found upon examination by an authorized representative to be defective in material and/or workmanship:

- Wood Frame
- Scissor Mechanisms

WARRANTY EXCLUSIONS

- Fabric
- Damage resulting from abuse, misuse, unauthorized repairs, negligence, accident(s), alteration, or modification of the product.
- Damage caused by common carrier abuse or mishandling.
- Any special, indirect, incidental, and/or consequential damages.

After the warranty period, there are no obligations under any warranties, expressed or implied (including implied warranties of merchantability and fitness for a particular purpose).

In the event that concealed damage is discovered after the freight is accepted, the consignee/individual who signed for the merchandise must call the carrier within 48 hours and request an inspection. The consignee/individual must retain the shipping carton. Please leave the merchandise in question in the area where it was first received. All claims are negotiated between the consumer and freight company.

The following parts are classed as wear items, which may under normal wear and tear require replacing. These items are not therefore covered under warranty: all fabric. Warranty will also be refused if damage is deemed to have been caused through misuse or accident for which the provider cannot be deemed responsible.

NOTE: Parts are provided only under warranty. Your provider is responsible for labor and service. Please contact your provider for information about these services and for any applicable charges.

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