STAND-ASSIST CHAIR

LL440

OWNER'S MANUAL

The symbols below are used throughout this owner's manual and on the product to identify warnings and important information. It is very important for you to read them and understand them completely.

WARNING! Indicates a potentially hazardous condition/situation. Failure to follow designated procedures can cause either personal injury, component damage, or malfunction. On the product, this icon is represented as a black symbol on a yellow triangle with a black border.

MANDATORY! These actions should be performed as specified. Failure to perform mandatory actions can cause personal injury and/or equipment damage. On the product, this icon is represented as a white symbol on a blue dot with a white border.

PROHIBITED! These actions are prohibited. These actions should not be performed at any time or in any circumstances. Performing a prohibited action can cause personal injury and/or equipment damage. On the product, this icon is represented as a black symbol with a red circle and red slash.

Please fill out the following information for quick reference:

Pride Provider: _____

Address: _____

Phone Number: _____

Purchase Date: _____ Serial Number: _____

NOTE: This owner's manual is compiled from the latest specifications and product information available at the time of publication. We reserve the right to make changes as they become necessary. Any changes to our products may cause slight variations between the illustrations and explanations in this manual and the product you have purchased. The latest/current version of this manual is available on our website.

NOTE: This product is compliant with WEEE, RoHS, and REACH directives and requirements.

NOTE: This product meets IPX4 classification (IEC 60529).

NOTE: This product and its components are not made with natural rubber latex. Consult with the manufacturer regarding any after-market accessories.

PRODUCT SAFETY SYMBOLS

The symbols below represent labels used on the product to identify warnings, mandatory actions, and prohibited actions. It is very important for you to read and understand these symbols completely. Do not remove these labels from your product. Please note that not all of the symbols may be used on your chair model.

Pinch/Crush Points Hazard! Do not place objects or appendages in the path of moving parts.


Electrical Hazard

Read and follow the information in the owner's manual.

Maximum weight capacity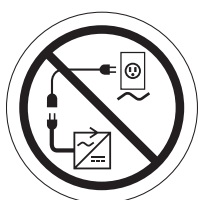


Do not connect an extension cord to the AC/DC converter or the battery charger.


Properly dispose of all electronic components, including the external transformer, hand control, batteries, actuator motors, and wiring. Contact your authorized provider for more information.

Battery Door Location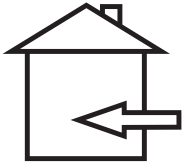

Indoor Use Only. Avoid exposure to rain, snow, ice, salt, or standing water. Maintain and store in a clean and dry environment.

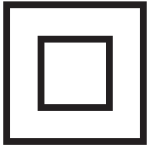Class II Equipment

TYPE B

Degree of protection against electric shock.

This product has been tested and complies to IEC 60601-1-2.

SAFETY GUIDELINES	2
LABEL INFORMATION	3
I. INTRODUCTION.....	6
SAFETY	6
PURCHASER'S AGREEMENT.....	6
INFORMATION EXCHANGE.....	6
II. GENERAL GUIDELINES	7
MODIFICATIONS	7
WEIGHT LIMITATIONS	7
PINCH/CRUSH HAZARDS	7
DEGREE OF PROTECTION/MODE OF OPERATION	7
STORAGE AND OPERATION TEMPERATURES	7
SHIPPING AND DELIVERY	7
MOTOR VEHICLE TRANSPORT	7
EMI/RFI	8
III. SETUP AND OPERATION	9
SETUP	9
PLACEMENT	11
BODY COMPONENTS	11
ELECTRICAL COMPONENTS	11
BATTERY INSTALLATION	12
OPERATION PRECAUTIONS	13
HAND CONTROL OPERATION	13
IV. TROUBLESHOOTING	14
FREQUENTLY ASKED QUESTIONS.....	14
V. CARE AND MAINTENANCE	15
FABRIC CARE	15
ELECTRONICS CARE	15
DISPOSAL AND RECYCLING	15
VI. WARRANTY	16
APPENDIX.....	48

SAFETY

The product you have purchased combines state-of-the-art components with **safety**, comfort, and styling in mind. We are confident the design features will provide you with the conveniences you expect during your daily activities. Understanding how to **safely** operate and care for this product should bring you years of trouble-free operation and service.

Read and follow all instructions, warnings, and notes in this manual and all other accompanying literature before attempting to operate this product for the first time. In addition, your **safety** depends upon you, as well as your provider, in using good judgement.

If there is any information in this manual which you do not understand, or if you require additional assistance for setup or operation, please contact your authorized provider. **Failure to follow the instructions, warnings, and notes in this manual and those located on your product can result in personal injury or product damage and will void the product warranty.**

PURCHASER'S AGREEMENT

By accepting delivery of this product, you promise that you will not change, alter, or modify this product or remove or render inoperable or unsafe any guards, shields, or other safety features of this product; fail, refuse, or neglect to install any retrofit kits from time to time provided to enhance or preserve the safe use of this product.

INFORMATION EXCHANGE

We want to hear your questions, comments, and suggestions about this manual. We would also like to hear about the safety and reliability of your new chair, and about the service you received from your authorized provider. Please notify us of any change of address, so we can keep you apprised of important information about safety, new products, and new options that can increase your ability to use and enjoy your chair.

NOTE: If you ever lose or misplace your copy of this manual, contact us and we will be glad to send you a new one immediately.

Below are some precautions, tips, and other safety considerations that will help you become accustomed to operating your product in a safe manner.

MANDATORY! Read and follow the information provided in this owner's manual before attempting to operate your chair for the first time.

MODIFICATIONS

Your chair has been designed and engineered to provide maximum comfort and utility. However, to prevent personal injury and/or damage to your chair, you should not modify, add, remove, or disable any feature, part, or function of your chair. Unauthorized modifications may also void your product's warranty.

WEIGHT LIMITATIONS

Your chair is rated for a maximum weight capacity of 275 lbs. (125 kg).

MANDATORY! Stay within the specified weight capacity of your chair. The manufacturer and/or provider will not be held responsible for injuries and/or product damage resulting from failure to observe weight limitations.

PINCH/CRUSH HAZARDS

The lift mechanism is labeled as pinch/crush point hazards on your chair. Keep clear of these areas and make sure the path of motion is unobstructed.

WARNING! Do not place objects or appendages in the path of moving parts.

DEGREE OF PROTECTION/MODE OF OPERATION

- Class II equipment/Type B protection against electric shock
- Degree of protection against the ingress of solid/liquids—IPX0
- Mode of operation—Maximum Duty Cycle: 2 min. ON/18 min. OFF

STORAGE AND OPERATION TEMPERATURES

- Transportation or storage: -104°F/-75°C to 158°F/70°C
- Operation: 50°F/10°C to 104°F/40°C

SHIPPING AND DELIVERY

Before using your chair, make sure your delivery is complete as some components may be individually packaged. If you do not receive a complete delivery, please contact your authorized provider immediately. Where damage has occurred during transport, either to the packaging or content, please contact the delivery company responsible.

MOTOR VEHICLE TRANSPORT

If you will be transporting your chair in a motor vehicle, individual components (external transformer, etc.) should be secured against slipping. The chair itself must also be secured against slipping (a possible hazard during vehicle braking).

ELECTROMAGNETIC AND RADIO FREQUENCY INTERFERENCE (EMI/RFI)

WARNING! Laboratory tests have shown that electromagnetic and radio frequency waves can have an adverse effect on the performance of electrically-powered devices.

Electromagnetic and Radio Frequency Interference can come from sources such as cellular phones, mobile two-way radios (such as walkie-talkies), radio stations, TV stations, amateur radio (HAM) transmitters, wireless computer links, microwave signals, paging transmitters, and medium-range mobile transceivers used by emergency vehicles. In some cases, these waves can cause unintended movement or damage to the control system of electrically-powered devices. The chair user can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment. It is recommended that at least 9 feet (3 meters) of distance be maintained between the chair and any handheld equipment emitting 10 W or more of output power. Refer to the manufacturer's literature for the handheld device to determine the maximum output of that device.

Every electrically-powered device has an immunity (or resistance) to EMI. The higher the immunity level, the greater the protection against EMI. Per EMC standards, this product has passed immunity testing and is rated as a Group 1, Class B product, meaning the chair uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment, making the chair suitable for use in all establishments, including domestic establishments and hospitals.

WARNING! Be aware that cell phones, two-way radios, laptops, electricity generators or high power sources, and other types of radio transmitters may cause unintended movement of your electrically-powered device due to EMI. Do not turn ON hand-held personal communication devices, such as citizens band (CB) radios and cellular phones, while operating your chair. Be aware of nearby transmitters, such as radio or TV stations, and avoid coming into close proximity to them. Exercise caution when using any of these items while operating your chair.

WARNING! The addition of accessories or components to the chair can increase the susceptibility of the chair to EMI. Do not modify your chair in any way.

WARNING! Your chair itself can disturb the performance of other electrical devices located nearby, such as alarm systems.

NOTE: If unintended motion occurs, discontinue use of the chair. Contact your authorized provider to report the incident.

SETUP

Your chair will arrive in the knock-down (KD) position (**Fig. 3.1**) and will require some assembly before initial use. It may also require disassembly to make servicing more convenient. Refer to the following instructions and contact your authorized provider with any questions.



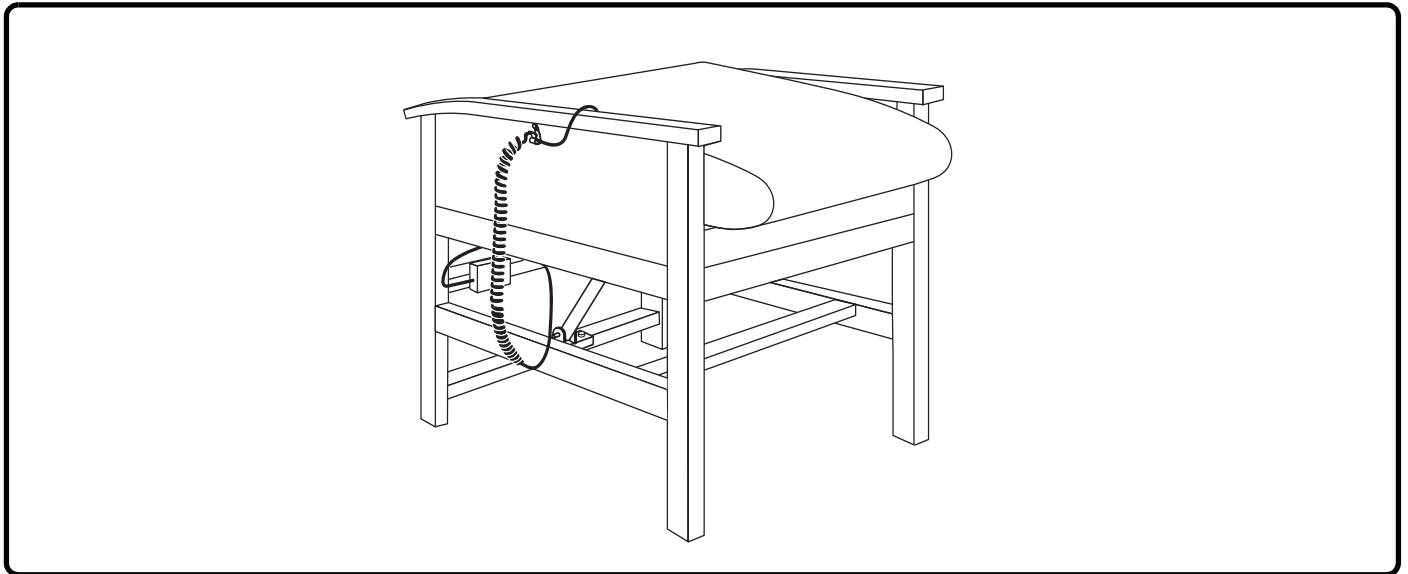
Figure 3.1 Knock-Down (KD) Position

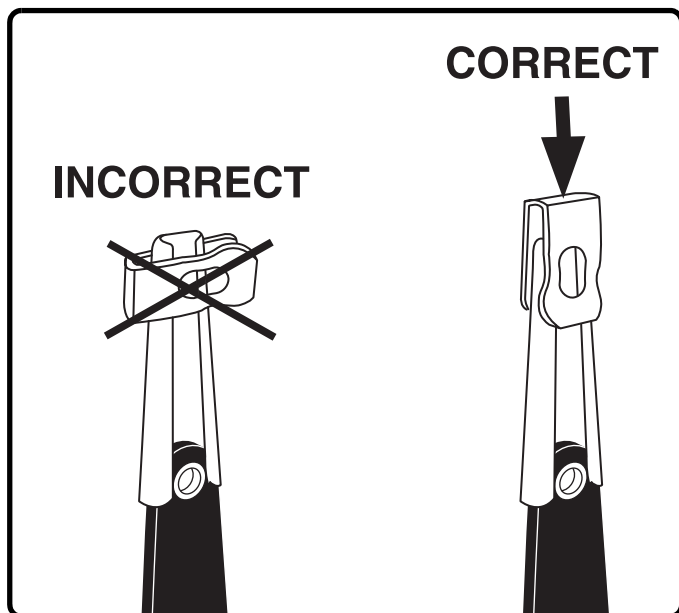
3.1.1 Remove the packaging from the back portion of the chair and check that the locking clip is in the vertical position. Reposition if necessary.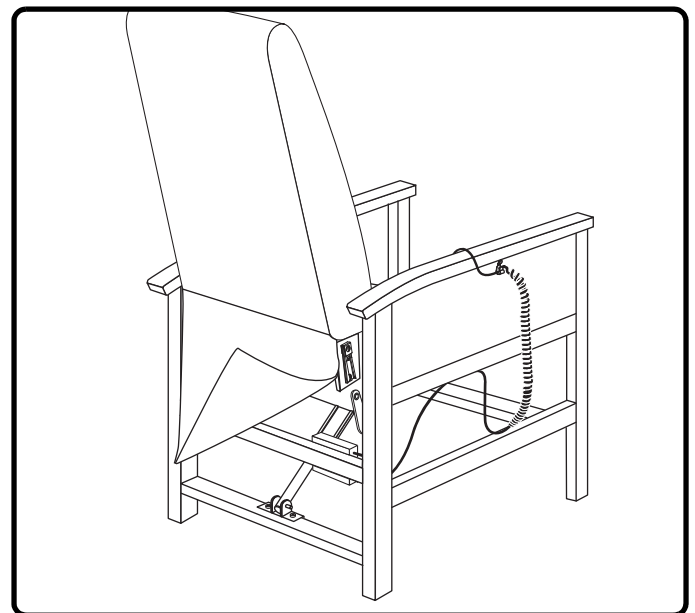


3.1.2 Drape the hand control and cable over the chair base, then slide the left and right KD sleeves of the chair-back onto the KD connections on the arms and seat of the chair.

NOTE: Pull up on the chair-back to ensure that it is locked in place. If the back can be pulled off, then repeat steps 3.1.1 and 3.1.3. You must hear a click from each side of the chair.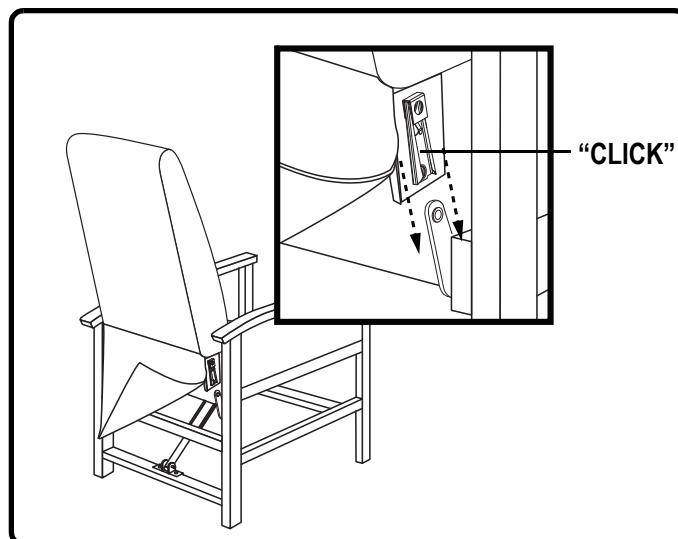


- 3.1.3 Push firmly on the top of the chair-back to lock into position. Listen for a click on each side to ensure the chair-back is locked in properly.

Follow these steps to disassemble the chair: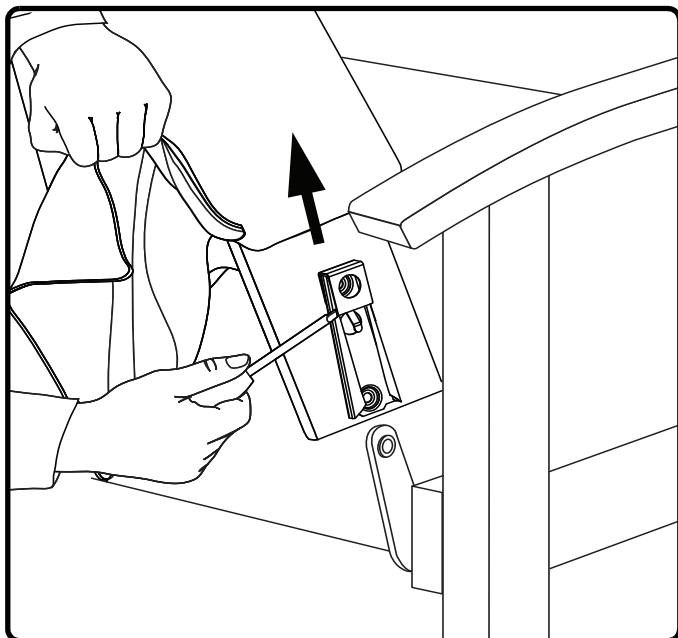

- 3.1.4 Using a large flat-head screwdriver, lift the left and right locking clips on the chair-back one side at a time to raise the back from the connections on the chair base. As you lift each locking clip, pull up on the chair-back to lift that side over the lock.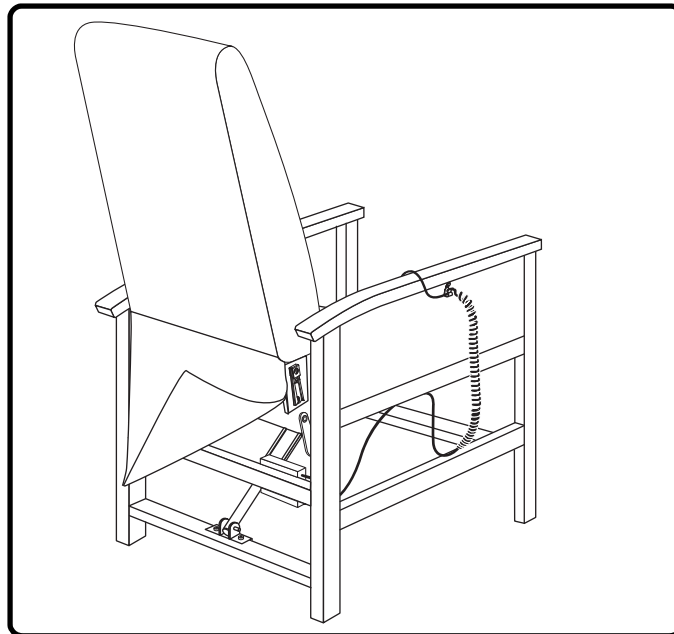

- 3.1.5 Lift the chair-back up and away from the chair base.

PLACEMENT

Follow these steps to position the chair in a safe manner:

1. Install the batteries into the external transformer. Refer to “Battery Installation” for more information.
2. Position the external transformer on the floor in an open, well-ventilated area where it will not be an obstruction.
3. Connect the low voltage connection cable to the external transformer if it is not already connected.
4. Plug the power cord directly into the electrical outlet. **Do not use an extension cord!**

NOTE: If you discover a problem at any point during the setup and positioning of your chair, stop and contact your authorized provider immediately. To avoid personal injury and/or product damage, do not plug the unit into the electrical outlet until the problem is corrected and do not attempt to fix electrical problems by yourself.

BODY COMPONENTS

This section describes the features of your chair. Carefully review the function and location of each item described, and note that the illustrations and option locations shown in this manual may not reflect the type of chair that you own.

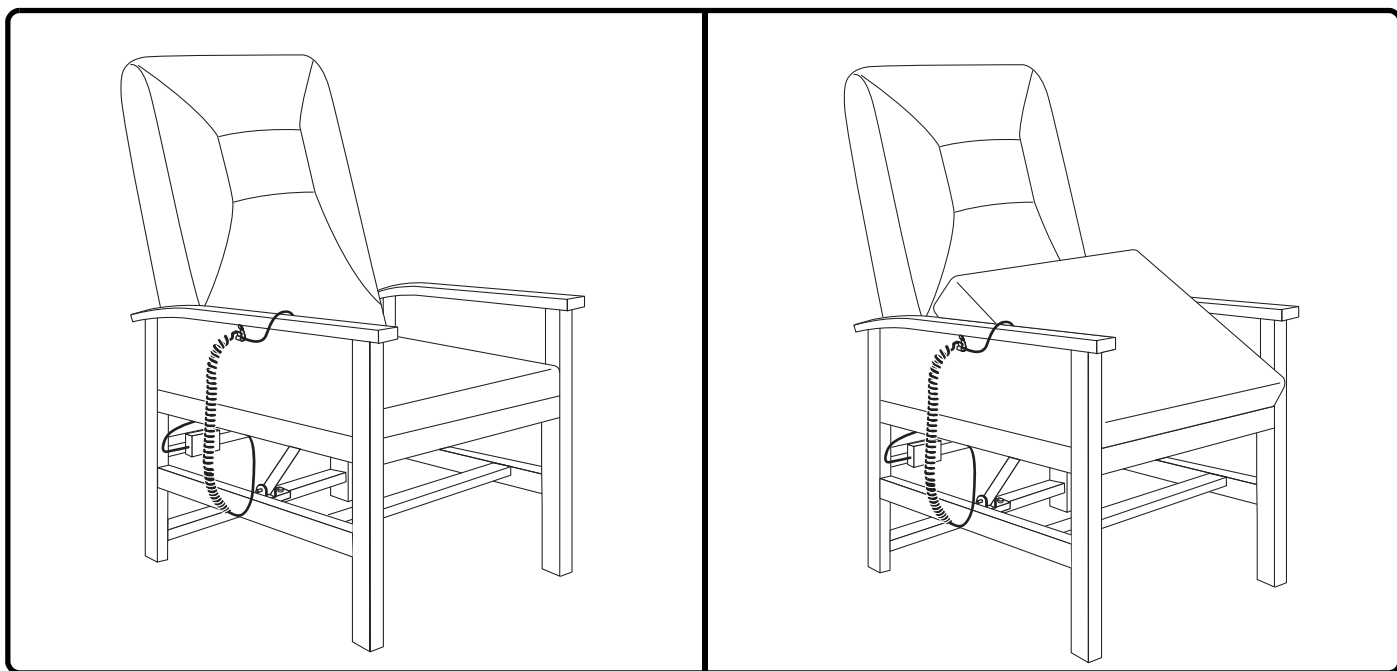
Figure 3.2 Stand-Assist Chair

ELECTRICAL COMPONENTS

Your chair is equipped with a low voltage DC electrical system that reduces the standard household alternating current of 100V AC - 240V AC to direct current (24/39V DC). The electrical components include the external transformer, batteries (not included), and hand control. Keep these areas free from moisture at all times.

WARNING! Keep all electrical components free from moisture at all times to prevent shock and/or electrical hazard.

External Transformer: Connects to a standard electrical outlet to provide power to the chair. The transformer is equipped with an LED indicator, which shows green when power to the transformer is on.

The external transformer may also come equipped with mounting brackets that allow the transformer to be mounted to the wall directly under the standard electrical outlet. The screws must be mounted to the wall stud for security. Refer to "Appendix" for a mounting diagram.

Batteries: The batteries (not included) must be installed into the external transformer.

NOTE: *The batteries will provide power for only a short period of time after power is removed from the system, so it is important to act quickly in the event of a power failure. Engage and hold the up/down switch in the up position as soon as possible. Maintain a steady pressure on the switch to avoid excessive draw from the battery and achieve the most amount of power. Keep in mind that the amount of power the batteries will be able to provide is affected by user weight and battery condition.*

Hand Control: Contains the controls needed to operate the various functions of the chair. The hand control may be equipped with a quick-release connector, which enables the user to detach the hand control from the chair, disabling all functions of the hand control.

BATTERY INSTALLATION

Your chair is equipped with a battery backup system that will activate during a power failure. The backup system is powered by two 9V batteries (not included) that need to be installed into the external transformer.

Follow these steps to install the batteries:

1. Unplug the external transformer power cord from the electrical outlet.
2. Open the marked battery door on the external transformer.
3. Install two 9V batteries into the external transformer (**Fig. 3.3**).
4. Replace the battery door.

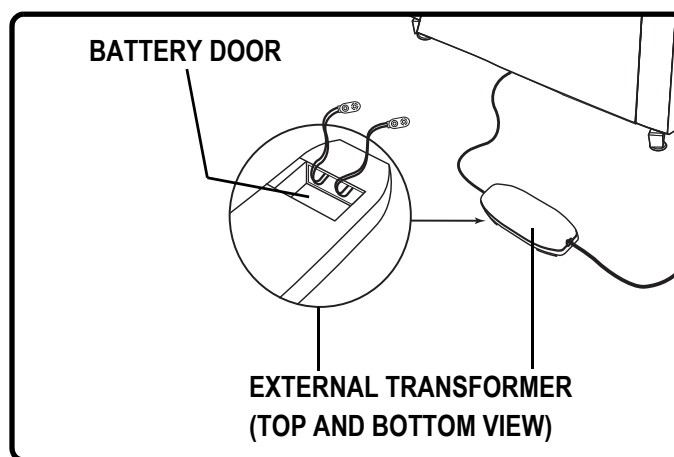
Figure 3.3 Battery Backup Location

NOTE: *Always make sure the external transformer is equipped with two fresh 9V batteries, since the battery backup system does not recharge itself. Fresh batteries are defined as 9V alkaline batteries that are replaced every time the battery backup system is activated during a power failure, or once a year if the battery backup system has not been activated.*

NOTE: *If you unplug your chair or there is a loss of power for an extended period of time (more than one hour), remove the batteries from the external transformer. The chair will draw power from the batteries even when it is not in use.*

OPERATION PRECAUTIONS

There are certain precautions that should be taken during the operation of your chair. Read and follow these precautions carefully in order to ensure safe chair operation and to prevent injury and/or product damage.

- Plug the power cord directly into the electrical outlet. **Do not use an extension cord!**
- Do not place anything (for example, a drinking glass) on top of or near the external transformer.
- If the external transformer box or hand control requires cleaning, unplug the power cord from the electrical outlet and use a clean, dry cloth or lightly dampened cloth. Allow ample drying time before plugging the power cord back into the electrical outlet.
- Periodically check the hand control and all power cords for visible damage.
- Keep the hand control away from all heated surfaces.
- Ensure the hand control is out of the way before sitting in the chair.
- Keep children and pets away from all moving parts while operating the chair.
- Do not allow children to play on or operate the chair. Only the intended user should operate the chair.
- Utilize the quick-disconnect feature on the standard hand control when the chair is not in use to prevent unintended operation of the chair.
- Avoid pinch points, such as the lift mechanism. Keep hands clear of these areas.
- Always leave the chair in a closed position when not in use.
- Do not “drop” into the chair when sitting in it.

WARNING! Do not place objects under the chair. Make sure area is clear of obstructions, including pets and small children during operation.

WARNING! Prevent the risk of electrical shock, fire, falls, and/or being pinched. Follow all instructions and precautions provided.

WARNING! Keep in an open, well-ventilated area free from foreign material and away from possible pinch points.

HAND CONTROL OPERATION

WARNING! Be sure to lock or disconnect the hand control when the chair is not in use.

Follow these steps to raise/lower your seat (Fig. 3.4):

1. Engage and hold the up/down switch in the UP position to raise the seat.
2. Engage and hold the up/down switch in the DOWN position to lower the seat.

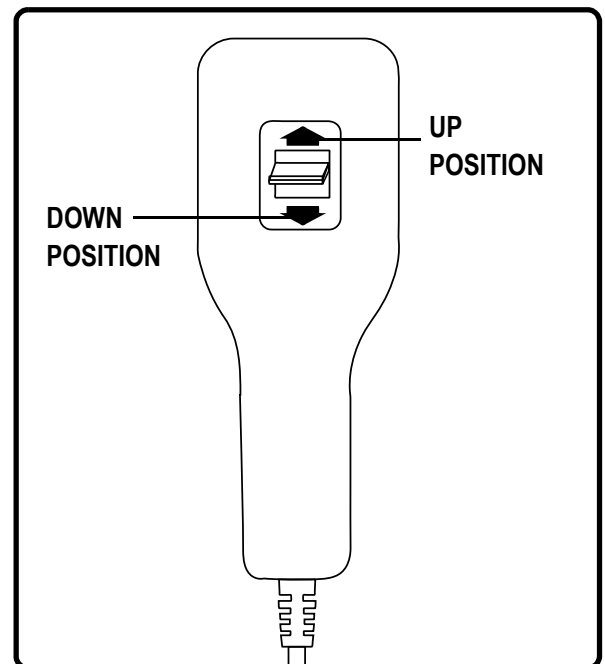
Figure 3.4 Hand Control

Your chair is a state-of-the-art product designed to enhance your comfort. Your chair should bring you years of trouble-free service, however, it may require occasional troubleshooting. The following troubleshooting tips and FAQs should summarize what you need to know about your chair. If at any time you do not feel comfortable performing the troubleshooting steps listed in this manual, contact your authorized provider for service. Please have the model number, serial number, and nature of the problem when calling.

FREQUENTLY ASKED QUESTIONS

What if my chair does not operate at all?

- Ensure the external transformer is plugged into a properly wired electrical outlet.
- Check the circuit breaker box connected to the electrical outlet to ensure the outlet is receiving power.
- Ensure the low voltage connection cable is plugged into the external transformer.
- Ensure that all cables are connected properly and the green power light is lit.

What if my chair operates in one direction only?

- Check the toggle switch on the hand control to make sure it does not stick. If the switch is sticking in either position, the hand control may need to be replaced.
- Contact your authorized provider for further assistance.

What if my chair stops during a cycle?

- Your chair is equipped with an internal thermal shutoff switch located inside the external transformer that prevents the motor control box from overheating. If the thermal shutoff activates, allow the chair to remain in a stationary position for 10 minutes to allow the motor to cool, then resume normal operation. If you notice the motors overheating frequently, contact your authorized provider.
- There may have been a power failure and/or there are no batteries in the external transformer or the batteries have no charge. Always make sure there are fresh 9V batteries in the external transformer if equipped with battery backup.
- Check the circuit breaker box connected to the electrical outlet.

Where can I place the external transformer box?

- On the floor away from heat sources in an open, well-ventilated area where it will not be an obstruction.
- On the wall away from heat sources utilizing the optional external transformer wall mount. Refer to "Appendix."

Where do I find the serial number on my chair?

You can find the serial number near the rear of the motor. The model number for your lift chair is printed below the serial number bar code.

Who do I call for service?

Contact your authorized provider for service.

Your chair will require routine maintenance checks. You can perform some of these checks, but others may require assistance from an authorized provider. By following the maintenance checks in this section as scheduled, you can help ensure your chair gives you years of trouble-free operation. If you have any questions regarding your lift chair's care, contact your authorized provider.

FABRIC CARE

- Inspect the fabric on a regular basis for any pulls, tears, or gaps.
- Choose the right methods to clean the fabric on your chair. Before cleaning your chair, consult your authorized provider for cleaning codes.

WARNING! Fabric should not be allowed to come in direct contact with any type of heat source (e.g., a space heater or cigarette). Personal injury or fire damage may occur.

WARNING! It is strongly recommended that you do not smoke cigarettes while seated in or using your chair, although the chair has passed the necessary testing requirements for cigarette smoking. You must adhere to the following safety guidelines if you decide to smoke cigarettes while seated in or using your chair.

- Do not leave lit cigarettes unattended.
- Keep ashtrays a safe distance from the chair.
- Always make sure cigarettes are completely extinguished before disposal.

NOTE: Do not use solvent-type cleaners to spot clean. Do not saturate the fabric. Prevent damage to the fabric; do not expose the chair to direct sunlight.

ELECTRONICS CARE

- Inspect all wiring harnesses to make sure they are not damaged or frayed.
- If damage is present, unplug the chair and contact your authorized provider for service.
- Keep all electronics free from moisture and temperature extremes. **Chairs are intended for indoor use only!**

WARNING! Even though the chair has passed the necessary testing requirements for ingress of liquids, you should keep electrical connections away from sources of dampness, including direct exposure to water or bodily fluids and incontinence. Check electrical components frequently for signs of corrosion and replace as necessary.

DISPOSAL AND RECYCLING

Your chair must be disposed of according to applicable local and national statutory regulations. Contact your local waste disposal agency or authorized provider for information on proper disposal of lift chair packaging, metal frame components, fabric, electronic components, and batteries.

WARNING! Plastic bags are a suffocation hazard. Dispose of plastic bags properly and do not allow children to play with them.

TWO-YEAR LIMITED WARRANTY

For two (2) years from the date of purchase, your provider will repair or replace at our option to the original purchaser any of the following parts found upon examination by an authorized representative to be defective in material and/or workmanship:

- Wood Frame
- Hand Control

WARRANTY EXCLUSIONS

- Fabric
- Damage resulting from abuse, misuse, unauthorized repairs, negligence, accident(s), alteration, or modification of the product.
- Damage caused by common carrier abuse or mishandling.
- Any special, indirect, incidental, and/or consequential damages.

After the warranty period, there is no obligations under any warranties, expressed or implied (including implied warranties of merchantability and fitness for a particular purpose).

In the event that concealed damage is discovered after the freight is accepted, the consignee/individual who signed for the merchandise must call the carrier within 48 hours and request an inspection. The consignee/individual must retain the shipping carton. Please leave the merchandise in question in the area where it was first received. All claims are negotiated between the consumer and freight company.

The following parts are classed as wear items, which may under normal wear and tear require replacing. These items are not therefore covered under warranty: all fabric. Warranty will also be refused if damage is deemed to have been caused through misuse or accident for which the provider cannot be deemed responsible.

NOTE: Parts are provided only under warranty. Your provider is responsible for labor and service. Please contact your provider for information about these services and for any applicable charges.

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Copyright © 2014
Pride Mobility Products Europe B.V.
Pride Mobility Products Australia Pty. Ltd.
Pride Mobility Products Italia S.r.l.