



Quick Reference

JSP Fabric Cover with Flap

Replacement Instructions

Stealth Products strives for 100% customer satisfaction. Your complete satisfaction is important. Please contact us with feedback or suggested changes that will help improve the quality and usability of these products. You may reach us at:

104 John Kelly Drive, Burnet, TX 78611

Phone: (512) 715-9995 Toll Free: 1 (800) 965-9229

Fax: (512) 715-9954 Toll Free: 1 (800) 806-1225

info@stealthproducts.com www.stealthproducts.com

General

Read and understand all instructions prior to the use of the product. Failure to adhere to instructions and warnings in this document may result in property damage, injury, or death. Product misuse due to failure of the following instructions will void the warranty.

Immediately discontinue use if any function is compromised, parts are missing, loose, or shows signs of excessive wear. Consult with you supplier for repair, adjustment, or replacement.

If this document contains information you do not understand, or there are concerns about safety or operation, contact your supplier.

Important Information!

All persons responsible for fitting, adjustment, and daily use of the devices discussed in these instructions must be familiar with and understand all safety aspects of the devices mentioned. In order for our products to be used successfully, you must:

- Read and understand all instructions and warnings.
- Maintain our products according to our instructions on care and maintenance.
- Devices should be installed and adjusted by a trained technician.

Ordering Documentation

You can download additional copies of this User & Installation Manual on the Stealth website: www.stealthproducts.com

REF

P116D238R1
Document #238 Revision 1
Monday, November 23, 2015

Copyright © 2015 Stealth Products, LLC
All rights reserved.
Published by Stealth Products, LLC

Installation Date: _____

Dealer: _____

Dealer Stamp:

Serial Number:

For UK warranty claims, please contact:



MDSS GmbH
Schiffgraben 41
30175 Hannover, Germany



The ADI Backs are designed and produced to the highest of standards. Our ADI Back is guaranteed to be free of defects in material and workmanship for a period of 24 months. If any defect in material or workmanship is found, Stealth Products will repair or replace, at our discretion, the ADI Back. This warranty does not apply to backs abused or misused by the user and deemed such by Stealth Products, LLC.

Claims and repairs should be processed through the nearest Stealth Products Provider.

Warranty Exclusions

4.1

This warranty does not extend to those items which may require replacement due to normal wear and tear:

- Outer covers
- Inner covers
- Circumstances beyond the control of Stealth.
- Labor, service calls, shipping, and other charges incurred for repair of the product, unless specifically authorized, IN ADVANCE, by Stealth Products, LLC.
- Repairs and/or modifications made to any part without specific consent from Stealth.

Exclusions also include components with damage caused by:

- Exposure to moisture

NOTICE

Any substitutions of supplied hardware void the warranty. Misuse of items and "acts of nature" (ex. flood, tornado, etc.) are not covered by the warranty.

1.0 Customer Satisfaction	i
2.0 Important Information	ii
3.0 Introduction	iii
4.0 Warranty	iv
4.1 Warranty Exclusions.....	iv
5.0 Table of Contents	v
6.0 Warning Labels	vi
6.1 Warning Labels.....	vi
6.2 Limited Liability.....	vi
6.3 Testing.....	vi
7.0 Installation Instructions	1
8.0 First Time Use	4
8.1 Dealer Assistance.....	4
8.2 User Testing.....	4
8.3 Conditions of Use.....	4
9.0 Maintenance	5
9.1 Cleaning.....	5
9.2 Disinfection.....	5

Warning Labels

6.1

Warnings are included for the safety of the user, client, operator and property. Please read and understand what the signal words **SAFETY**, **NOTICE**, **CAUTION**, **WARNING** and **DANGER** mean, how they could affect the user, those around the user, and property.

⚠ DANGER	Identifies an imminent situation which (if not avoided) will result in severe injury, death, and property damage .
⚠ WARNING	Identifies a potential situation which (if not avoided) will result in severe injury, death, and property damage .
⚠ CAUTION	Identifies a potential situation which (if not avoided) will result in minor to moderate injury, and property damage .
NOTICE	Identifies important information not related to injury, but possible property damage .
SAFETY	Indicates steps or instructions for safe practices, reminders of safe procedures, or important safety equipment that may be necessary.

Limited Liability

6.2

Stealth Products, LLC accepts no liability for personal injury or damage to property that may arise from the failure of the user or other persons to follow the recommendations, warnings, and instructions in this manual.

Testing

6.3

Initial setup and driving should be done in an open area free of obstacles until the user is fully capable of driving safely.

The product should always be tested without any person sitting in the wheelchair until every alteration of the physical installation or adjustment is complete.

Step 1



Step 2



Step 3



Step 4



1. Unzip the existing fabric cushion cover.
2. Remove the foam cushion from the existing fabric cover.
3. Insert the foam into the new cover, inserting the bottom first.

NOTICE

Keep the original orientation of the foam. It can help to fold the foam in half vertically.

4. Pull the top of the new cover over the top edge of the foam.

NOTICE

Some positioning may be required to align foam inside the new cover; do this by stretching the edges of the cover over the foam until it is aligned.

Step 5



Step 6



Step 7



Step 8



5. Remove existing JSP decal from the carbon fiber shell.
6. Clean the affected area of all adhesive residue with the provided alcohol wipe.
7. Remove all four Velcro strips from the inside of the carbon fiber shell. Clean the affected area of all adhesive residue with the provided alcohol wipe.
8. Cut 2in. Velcro to appropriate length, then remove the plastic adhesive cover and apply Velcro to the center of the inside of the carbon shell. Press down firmly to secure in place.

NOTICE

It is imperative that the original ROHO barcode label remains on the inside of the carbon shell. Please remove barcode label and place it to one side of the center Velcro strip.

Step 9



Step 10



Example 1



Step 11



9. Remove the plastic adhesive cover from the one inch wide Velcro. Place the Velcro strip along the exterior top edge of the carbon shell, horizontally. Place the strip as far down as necessary to avoid interference from the edge guard. (See example 1 above)
10. Stretch the flap of the cover down over the top edge of the carbon shell and secure it to the 1in. strip of Velcro.
11. Adhere the new JSP sticker to the exterior of the carbon fiber shell.

Dealer Assistance

8.1

During first time use by the client, it is advised that the dealer or service technician assists and explains the set-up to the customer (the user and/or the attendant). If needed, the dealer can make final adjustments.

 **CAUTION** Do not attach the back with the user in the chair.

User Testing

8.2

It is important that the customer is fully aware of the installation, how to use it, and what can be adjusted in order to gain as much mobility as possible. As a dealer, proceed as follows:

- Explain and show the customer how you have executed the installation, and explain the function of the hardware.
- Explain to the customer possible problems and how to address them.

Conditions of Use

8.3

The ADI hardware is intended for use as installed by the dealer, in accordance to the installation instructions in this manual.

 **CAUTION** Do not use backs during vehicle transportation.

- The foreseen conditions of use are communicated by the dealer or service technician to the user and/or attendant during the first time use.
- If the usage conditions change significantly, please contact your dealer or a qualified service technician to avoid excessive wear and tear or unintended damage.
- The CF back may not fit every chair. Please consult your clinician or supplier providing your equipment.

 **CAUTION**

Protect the back from blunt force shocks that may cause damage and/or breakage.


Cleaning


9.1

- Use a soft cloth to clean the back shell.
- Hand wash the foam pad cover with mild, soapy water or detergent and let it dry.
- Foam pad should NOT get wet.

CAUTION

Institution MUST clean and disinfect product between multiple patient uses.


 **WARNING** Do not wash or machine dry.

 **WARNING** Do not hand wash in water temperatures over 160°F (70°C)

Disinfection


9.2

- Wipe gently with a cloth dampened with a household disinfectant.
- Allow foam to air dry thoroughly.

 **WARNING** Do not immerse the foam in water

SAFETY

Always check all mounting hardware, making sure each fastener is properly tightened before using the back.

 **WARNING** 250lbs. (113.40kg) user weight limit.



Stealth Products, LLC. • info@stealthproducts.com • www.stealthproducts.com
+1(800) 965-9229 | +1(512) 715-9995 | 104 John Kelly Drive, Burnet TX 78611

Revision Date 2015-11-24