

Stealth Products



USER MANUAL

Head Support Hardware



HMO500

Stealth's User Manual and Maintenance Guide for
the HMO500 Headrest Hardware

Stealth Products strives for 100% customer satisfaction. Your complete satisfaction is important. Please contact us with feedback or suggested changes that will help improve the quality and usability of our products. You may reach us at:



Stealth Products, LLC

104 John Kelly Drive, Burnet, TX 78611

Phone: (512) 715-9995

Toll Free: 1(800) 965-9229

Fax: (512) 715-9954

Toll Free: 1(800) 806-1225

info@stealthproducts.com

www.stealthproducts.com



MDSS GmbH

Schiffgraben 41

30175 Hannover, Germany



For international orders, please contact:

Pride Mobility Products

5096 South Service Road

Beamsville, Ontario L0R 1B3 Canada

Phone: 888-570-1113 info@pridemobility.com

General

Read and understand all instructions prior to the use of the product. Failure to adhere to instructions and warnings in this document may result in property damage, injury, or death. Product misuse due to failure of the following instructions will void the warranty.

Immediately discontinue use if any function is compromised, parts are missing, loose, or show signs of excessive wear. Consult with your supplier for repair, adjustment, or replacement.

If this document contains information you do not understand, or there are concerns about safety or operation, contact your supplier.

Important Information!

All persons responsible for fitting, adjustment, and daily use of the devices discussed in these instructions must be familiar with and understand all safety aspects of the devices mentioned. In order for our products to be used successfully, you must:

- Read and understand all instructions and warnings.
- Maintain our products according to our instructions on care and maintenance.
- Devices should be installed and adjusted by a trained technician.

Supplier Reference

Supplier: _____

Telephone: _____

Address: _____

Purchase Date: _____

Model: _____

Before you install or begin using this product, it is important that you read and understand the content of these operating instructions. These instructions will guide you through the options and possibilities with the HMO500 Headrest Hardware.

Instructions are written with the expressed intent of use with configurations. They also contain important safety and maintenance information, as well as describe possible problems that can arise during use. For further assistance, or more advanced applications, please contact Pride Mobility Products at 888-570-1113.

Always keep the operating instructions in a safe place so they may be referenced as necessary.

All information, pictures, illustrations, and specifications are based on the product information that was available at the time of printing. Pictures and illustrations shown in these instructions are representative examples and are not intended to be exact depictions of the various parts of the product.

Ordering Documentation

You can download additional copies of this user manual on the Pride website:

<http://www.pridemobility.com/international/canada/canadaownersmanuals.asp>

The pads/cushions are designed and produced to the highest of standards. The cushions are guaranteed to be free of defects in material and workmanship for a period of 24 months. If any defect in material or workmanship is found, Pride Mobility Products will repair or replace, at our discretion, the pads. This warranty does not apply to pads abused or misused by the user and deemed such by Pride Mobility Products.

Claims and repairs should be processed through the nearest Pride Mobility provider.

- Modification of product without written approval from Pride Mobility Products.

No person is authorized to alter, extend, or waive the warranties of Stealth Products, LLC. Stealth Products warrants against failure due to defective materials or workmanship:

Hardware: 5 years

Electronics: 3 years

This warranty does not extend to those items which may require replacement due to normal wear and tear.

- Outer covers
- Inner covers
- Circumstances beyond the control of Pride
- Labor, service calls, shipping, and other charges incurred for repair of the product, unless specifically authorized, IN ADVANCE, by Pride Mobility Products
- Repairs and/or modifications made to any part without specific consent from Pride

Exclusions also include components with damage caused by:

- Exposure to moisture

1.0 Customer Satisfaction	i
2.0 Important Information	ii
3.0 Introduction	iii
4.0 Warranty	iv
5.0 Table of Contents	v
6.0 Warning Labels	vi
6.1 Warning Labels	vi
6.2 Limited Liability	vi
6.3 Testing	vi
7.0 Design and Function	1
7.1 Intended Use	1
7.2 Features	1
8.0 Features	2
8.1 Adjustment	2
Height Adjustment	2
Depth Adjustment	3
Angle Adjustment	3
9.0 Installation Instructions	4
9.1 Tools	4
9.2 Installation Plan	4
9.3 Hardware Installation	4
9.4 Installation	5
10.0 First Time Use	7
10.1 Dealer Assistance	7
10.2 User Testing	7
10.3 Conditions of Use	7
11.0 Maintenance	8
11.1 Care and Maintenance	8
11.2 Cleaning	8

Warning Labels

6.1

Warnings are included for the safety of the user, client, operator and property. Please read and understand what the signal words **SAFETY**, **NOTICE**, **CAUTION**, **WARNING** and **DANGER** mean, how they could affect the user, those around the user, and property.

 **DANGER**

Identifies an **imminent** situation which (if not avoided) will result in **severe injury, death, and property damage**.

 **WARNING**

Identifies a **potential** situation which (if not avoided) will result in **severe injury, death, and property damage**.

 **CAUTION**

Identifies a **potential** situation which (if not avoided) will result in **minor to moderate injury, and property damage**.

NOTICE

Identifies important information not related to injury, but possible **property damage**.

SAFETY

Indicates steps or instructions for safe practices, reminders of safe procedures, or important safety equipment that may be necessary.

Limited Liability

6.2

Stealth Products, LLC accepts no liability for personal injury or damage to property that may arise from the failure of the user or other persons to follow the recommendations, warnings, and instructions in this manual.

Testing

6.3

Initial setup and driving should be done in an open area free of obstacles until the user is fully capable of driving safely.

The HMO500 Headrest Hardware should always be tested without any person sitting in the wheelchair until every alteration of the physical installation or adjustment is complete.

Intended Use

7.1

The HMO500 is a lightweight, multi-mount mounting bracket for use with Stealth's headrests. This hardware is designed to provide multi-axis positioning of the head support, increasing the ability to properly accommodate asymmetrical head positioning.

Features

7.2

The HMO500 hardware:

- Is lightweight, multi-axis hardware.
- Offers height, depth, and angle adjustments through the loosening/tightening of a screw.
- Compatible with all Stealth headrests.
- Multi-hole pattern to accommodate existing headrests.
- Compatible with after-market headrests.

WARNING

Stealth does not recommend the HMO500 for client's with tone.

Adjustment

8.1

The HMO500 has height, depth, and angle adjustments that can easily be changed by simply loosening one of the four available points.

Height Adjustment

- The placement of the seat back mount on the vertical bar can be adjusted by loosening the screw on the stop collar, allowing the mount to be adjusted up and down along the vertical bar.
- The height of the HMO500 can be adjusted by turning the handle on the vertical bar counter clockwise.
- A total of 6" of height adjustment is available on the vertical bar.
- Once the vertical bar is placed at the desired height, turn the handle clockwise to tighten and secure.





Depth Adjustment

- The depth of the horizontal bar can be adjusted up to 4”.
- Loosen the M5 screw on the side of the bar clamp, which holds the horizontal bar in place. By loosening the screw, depth adjustment can be made.
- When the horizontal bar has reached the desired depth on the hardware, tighten the M5 screw on the bar clamp until it is secure.



Angle Adjustment

- Two points of angle adjustment is available on the hardware.
- Adjustment points include horizontal bar angle and the angle of the adapter plate.

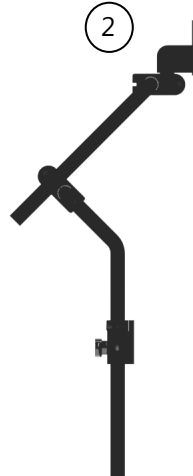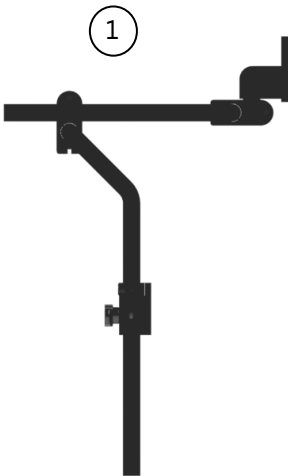

 **WARNING**

Only a qualified service technician may install the HMO500 hardware.

Tools

9.1

- 3mm T-Handle/Allen Wrench
- 4mm T-Handle/Allen Wrench
- 5/32 T-Handle/Allen Wrench
- 3/16 T-Handle/Allen Wrench
- Torque Wrench

 **CAUTION**

Use the proper tools to install and adjust the HMO500 hardware. The use of improper tools may cause damage to the device.

Installation Plan

9.2

Set up an installation plan before installing or making any adjustments to the hardware. Based on the users' needs, this plan should specify:

- What height is required for the headrest?
- At what angle does the mounting bracket and horizontal bar need to be placed in order to promote the best fit of the headrest to the client.

Hardware Installation

9.3

For more detailed instructions on how to install the HMO500, refer to the instructions in the HMO500 Hardware Installation Manual.

Installation

9.4

- Install the hardware onto the corresponding headrest for the client.
- Determine the correct placement of the hardware on the chair. If the height needs to be changed, follow the instructions in the user manual.
- If the angles of the hardware need to be adjusted, loosen the desired screws and make the adjustments necessary until the headrest is in the correct placement. Secure the screws.
- Make any final adjustments necessary and ensure all screws are secure.

CAUTION

Tighten the screws firmly, but not excessively. Excessive force may damage the unit.

 **CAUTION** Only use screws provided in the package with the hardware.

WARNING

Discontinue use if any product's function becomes compromised, loose, shows signs of excessive wear, missing parts, or does not function as intended.

NOTICE

Refer to the HMO500 installation manual for torque specifications regarding the hardware.

The pictures below feature the HMO hardware installed on headrests using the three and four-point hole patterns.



*Headrests not included.

Dealer Assistance

10.1

During first time use by the client, it is advised that the dealer or service technician assists and explains the different configurations to the client. If needed, the dealer can make final adjustments.

User Testing

10.2

It is important that the customer is fully aware of the installation and how to adjust the hardware in order for the headrest to be placed in the best position for the user. As a dealer, proceed as follows:

- Is the HMO500 at the proper height for the headrest?
- Is the HMO500 placed at the correct angle?

If needed, adjust the hardware to the proper position.

Explain to the client the possible problems and how to address them.

Conditions of Use

10.3

The HMO500 headrest hardware is intended for use as installed by the dealer, in accordance to the installation and instructions in this manual.

- The foreseen conditions of use are communicated by the dealer or service technician to the user and/or attendant during the first time use.
- If the usage conditions change significantly, please contact your dealer or a qualified service technician to avoid excessive wear and tear or unintended damage.

Care and Maintenance

11.1

We recommend that the hardware be checked for wear and function on a periodic basis and replaced, if necessary, by a wheelchair professional.

- Discontinue use after identifying loose or broken components.
- Check and tighten all screws according to the installation instructions.
- Repairs should be done with Stealth parts, fasteners, or items appropriate for the service needed.

Cleaning

11.2

- Surfaces may be wiped clean with a damp cloth and mild, nonabrasive cleaner.
- Ensure all cleaners are approved for finished steel, aluminum, plastic and upholstered surfaces.

 **DANGER**

Failure to comply with these warnings could result in serious bodily harm.



Stealth Products, LLC. • info@stealthproducts.com • www.stealthproducts.com
+1(800) 965-9229 | +1(512) 715-9995 | 104 John Kelly Drive, Burnet TX 78611