# Silver Star™ Ramps

OWNER'S MANUAL

**Pride**  
**SilverStar** ★  
Lifts and Ramps  
[www.silverstarlifts.com](http://www.silverstarlifts.com)

**Including: Single Fold Ramps and Solid Ramps**

[www.silverstarlifts.com](http://www.silverstarlifts.com) [www.pridemobility.com](http://www.pridemobility.com)

# SAFETY GUIDELINES

The symbols below are used throughout this owner's manual and on the product to identify warnings and important information. It is very important for you to read them and understand them completely.

**WARNING!** Indicates a potentially hazardous condition/situation. Failure to follow designated procedures can cause either personal injury, component damage, or malfunction. On the product, this icon is represented as a black symbol on a yellow triangle with a black border.

**MANDATORY!** These actions should be performed as specified. Failure to perform mandatory actions can cause personal injury and/or equipment damage. On the product, this icon is represented as a white symbol on a blue dot with a white border.

**PROHIBITED!** These actions are prohibited. These actions should not be performed at any time or in any circumstances. Performing a prohibited action can cause personal injury and/or equipment damage. On the product, this icon is represented as a black symbol with a red circle and red slash.

**Please fill out the following information for quick reference:**

<b>Pride Provider:</b> _____
<b>Address:</b> _____
<b>Phone Number:</b> _____
<b>Purchase Date:</b> _____ <b>Serial Number:</b> _____

***NOTE: This owner's manual is compiled from the latest specifications and product information available at the time of publication. We reserve the right to make changes as they become necessary. Any changes to our products may cause slight variations between the illustrations and explanations in this manual and the product you have purchased. The latest/current version of this manual is available on our website.***

(This page intentionally left blank.)

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# I . I N T R O D U C T I O N

Welcome to Silver Star, a division of Pride Mobility Products Corporation (Pride). Congratulations on the purchase of your new Silver Star Ramp. The Silver Star Ramp design combines the most advanced state-of-the-art components with modern, attractive styling. We are certain that the design features and trouble-free operation will add convenience to your daily living and ensure complete satisfaction.

At Pride, your safety is important to us. **Please read and follow all instructions in this manual before using your product for the first time.** These instructions were produced for your benefit. Your understanding of these instructions is essential for the safe operation of your new Silver Star Ramp.

Pride is not liable for damage to property or personal injury arising out of the unsafe use of the Silver Star Ramp. Pride is also not liable for any property damage or personal injury arising out of the failure of any person and/or user to follow the instructions and recommendations set forth in this manual or any other instructions or recommendations contained in other ramp related literature issued by Pride or contained on the Silver Star Ramps themselves.

## INTERNET AND PRIVATE PURCHASES

If you purchased your product over the internet or from a previous owner and you have any questions about the safe use and/or maintenance of the product, please visit Silver Star's web site [www.silverstarlifts.com](http://www.silverstarlifts.com) or contact your authorized Pride Provider.

## PURCHASER'S AGREEMENT

By accepting delivery of this product, you promise that you will not change, alter, or modify this product or remove or render inoperable or unsafe any guards, shields, or other safety features of this product; fail, refuse, or neglect to install any retrofit kits from time to time provided by Pride to enhance or preserve the safe use of this product.

## INFORMATION EXCHANGE

We want to hear your questions, comments, and suggestions about this manual. We would also like to hear about the safety and reliability of your new Silver Star Ramp, and about the service you received from your authorized Pride Provider.

Please notify us of any change of address, so we can keep you apprised of important information about safety, new products, and new options that can increase your ability to use and enjoy your new Silver Star Ramp.

***NOTE: If you ever lose or misplace your product registration card or your copy of this manual, contact us and we will be glad to send you a new one immediately.***

## PRIDE OWNERS CLUB

As an owner of a Pride product, you are encouraged to enroll in the Pride Owners Club. Complete and return your enclosed product registration card or visit Pride's web site at [www.pridemobility.com](http://www.pridemobility.com).

From our home page, select "Owners Club" to enter a page dedicated to current and potential Pride product owners. You will gain access to interviews, stories, recreation ideas, daily living tips, product and funding information, and interactive message boards. These message boards invite you to communicate with other Pride customers as well as Pride representatives who are available to assist you with any questions or concerns you may have.

***NOTE: On-line product registration and enrollment in the Pride Owner's Club valid for residents of the United States, Canada, Latin America, and the Caribbean only.***

# II. SAFETY

## PRODUCT SAFETY

Silver Star Ramps are intended for use as utility devices for the purpose of negotiating steps or other obstacles with scooters, power chairs, and manual wheelchairs or hand trucks and similar products. Check the ramp periodically for wear and loose or damaged parts. If your ramp is exhibiting wear that you feel is questionable in any way, contact your authorized Pride Provider.

The symbols below are used on the ramps to identify warnings, mandatory actions, and prohibited actions. It is very important for you to read and understand them completely.

Read and follow the information in the owner's manual before operating the product.

Maximum total ramp weight capacity is 600 lbs./272 kg. Do not exceed the recommended weight capacity for the ramp. When using the ramp, weight should be distributed evenly per side.

Ramp surfaces may become slippery when wet. Keep the ramp dry and free of foreign materials that may prevent safe operation. Exercise extreme caution when using the ramp during inclement weather conditions.

Do not exceed the maximum recommended incline angle of an occupied mobility device as determined by the manufacturer of the mobility device. Refer to the mobility device owner's manual, as well as the degree of inclination chart in Appendix A for more information.

Pinch point exists in Single Fold Ramp hinge. Keep fingers away from the hinge while folding or unfolding the ramp.

**WARNING!** Ramps are heavy. Refer to the specifications table for individual ramp weights. Use proper lifting techniques and avoid lifting weights beyond your physical capability. If necessary, get someone physically able to lift and maneuver the ramp for you.

**WARNING!** Do not modify the ramp in any way not authorized by Pride. Unauthorized modifications will void your warranty and may result in personal injury.

**WARNING!** Make sure that the ramp is securely positioned on a flat, level surface. The bottom of the ramp must be set firmly on the ground and the top of the ramp must be set flush to the step or vertical lip. Pride strongly recommends that you lock the ramp in place using the supplied detent pins. If necessary, drill holes in support surfaces to facilitate the use of the detent pins.

**WARNING!** Do not allow children to play on or around the ramp. The ramp should not be used as a ramp for skateboarding or biking jumps.

# III . WARRANTY

## THREE-YEAR TRANSFERABLE LIMITED WARRANTY

For three (3) years from the date of purchase, Pride Mobility Products will repair or replace at our option, free of charge, any mechanical or electrical component found upon examination by an authorized representative of Pride Mobility Products to be defective in material and/or workmanship.

This warranty does not extend to those items which may require replacement due to normal wear and tear.

- Labor, service calls, shipping, and other charges incurred for repair of the product, unless specifically authorized by Pride Mobility Products Corporation IN ADVANCE, are excluded.

### **Exclusions also include components with damage caused by:**

- Contamination
- Abuse, misuse, accident, or negligence
- Commercial use, or use other than normal
- Improper operation, maintenance, or storage
- Repairs and/or modifications made to any part without specific consent from Pride Mobility Products Corporation.
- Circumstances beyond the control of Pride

## SERVICE CHECKS AND WARRANTY SERVICE

An authorized Pride Mobility Products Provider must perform warranty service. Do not return faulty product to Pride Mobility Products without prior written authorization. All transportation costs and shipping damage incurred while submitting product for repair or replacement is the responsibility of the purchaser.

Failure to follow the instructions, warnings, and notes in the owner's manual and those located on your Pride ramp product can result in personal injury or product damage and will void Pride's product warranty.

There is no other express warranty.

## IMPLIED WARRANTIES

Implied warranties, including those of merchantability and fitness for a particular purpose, are limited to one (1) year from the date of purchase and to the extent permitted by law. Any and all implied warranties are excluded. This is the exclusive remedy. Liabilities for consequential damages under any and all warranties are excluded.

Some states do not allow limitations on how long an implied warranty lasts or do not allow the exclusion of limitation incidental or consequential damages. The above limitation or exclusion may not apply to you.

This warranty gives you specific rights, and you may also have other rights, which vary from state to state.

Please fill out and return the product registration card to Pride Mobility Products. This will aid Pride in providing the best possible technical and customer service.

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Copyright © 2007  
Pride Mobility Products Corp.  
INFMANU3325/Rev F/July 2007/French

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# APPENDIX A: INCLINE GUIDELINES

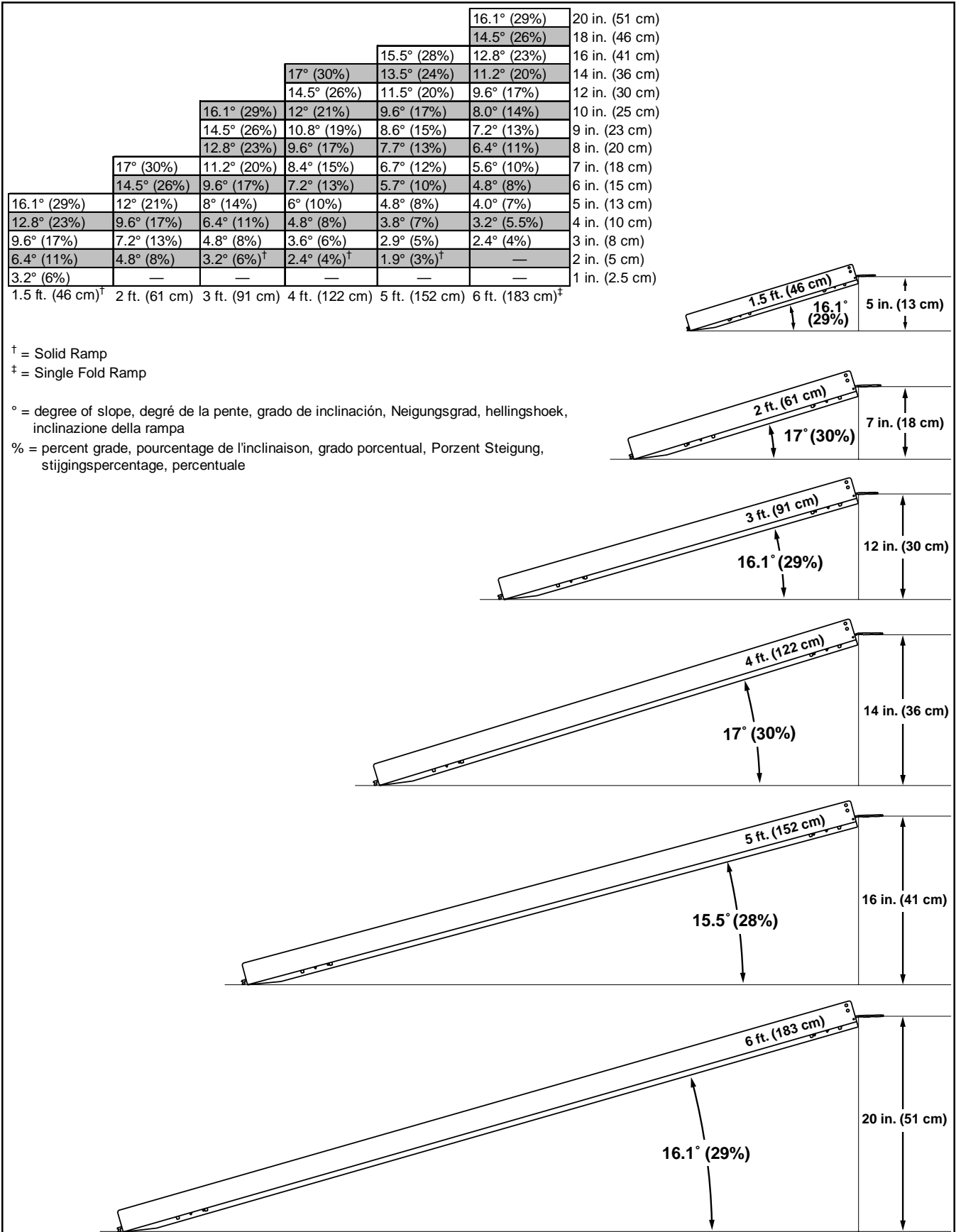
Figure 1. Maximum Ramp Height and Incline Angle

# APPENDIX B: SPECIFICATIONS

SPECIFICATIONS	
Length, Longueur, Longitud, Gesamtlänge, Lengte, Lunghezza	6 ft. (183 cm) <sup>†</sup> , 5 ft. (152 cm), 4 ft. (122 cm), 3 ft. (91 cm), 2 ft. (61 cm), 1.5 ft. (45.72 cm) <sup>†</sup>
Width (Folded), Largeur (Repliée), Anchura (Plegada), Gesamtbreite (Zusammengeklappt), Breedte (Opgevouwen), Larghezza (Piegata)	14.75 in. (37.5 cm)
Width (Unfolded), Largeur (Dépliée), Anchura (Desplegada), Gesamtbreite (Aufgeklappt), Breedte (Uitgevouwen), Larghezza (Aperta)	29.5 in. (75 cm)
Width, Largeur, Anchura, Gesamtbreite, Breedte, Larghezza	30 in. (76 cm) <sup>†</sup>
Weights, Poids, Pesos, Gewicht, Gewichten, Peso	1.5 ft. (7.7 lbs./3.5 kg) <sup>†</sup> , 2 ft. (12.0 lbs./5.4 kg), 3 ft. (17.5 lbs./8.0 kg), 4 ft. (23.5 lbs./10.7 kg), 5 ft. (29.5 lbs./13.4 kg), 6 ft. (35.0 lbs./16.0 kg) <sup>‡</sup>
Weight Capacity, Charge Utile, Capacidad de carga, Zuladung, Gewichtscapaciteit, Capacità di carico massima	600 lbs. (272 kg)

† Solid Ramp

‡ Single Fold Ramp

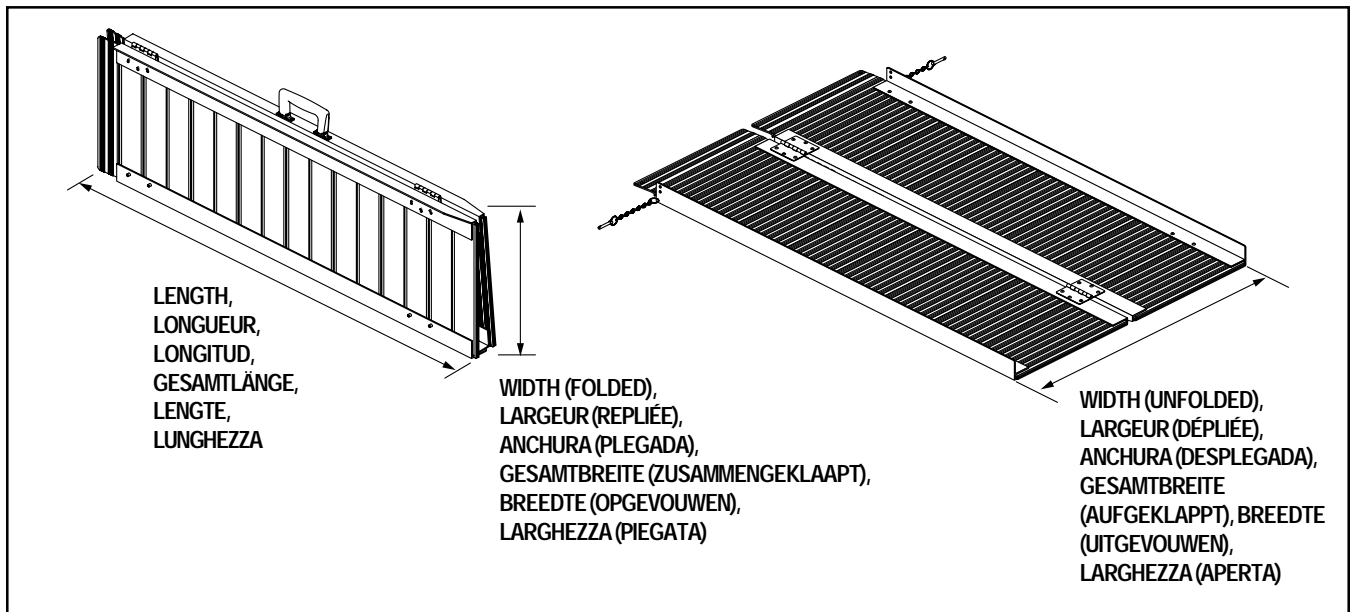
Figure 2. Specifications

# APPENDIX C : SETUP

1

**Place the ramp against a flat, level surface and remove the detent pins.**

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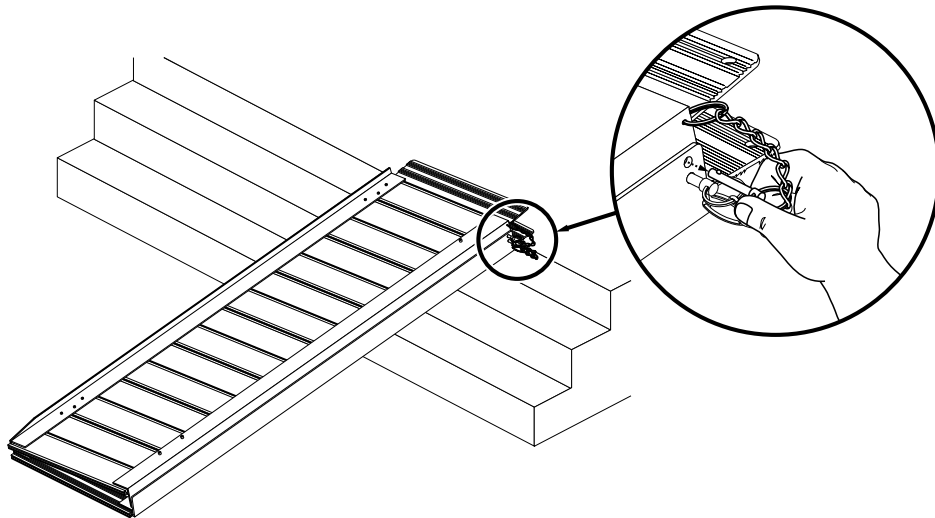
**NOTE: Make sure ramp is set securely against a flat, level surface. Refer to Figure 1 for maximum ramp height and incline angle.**

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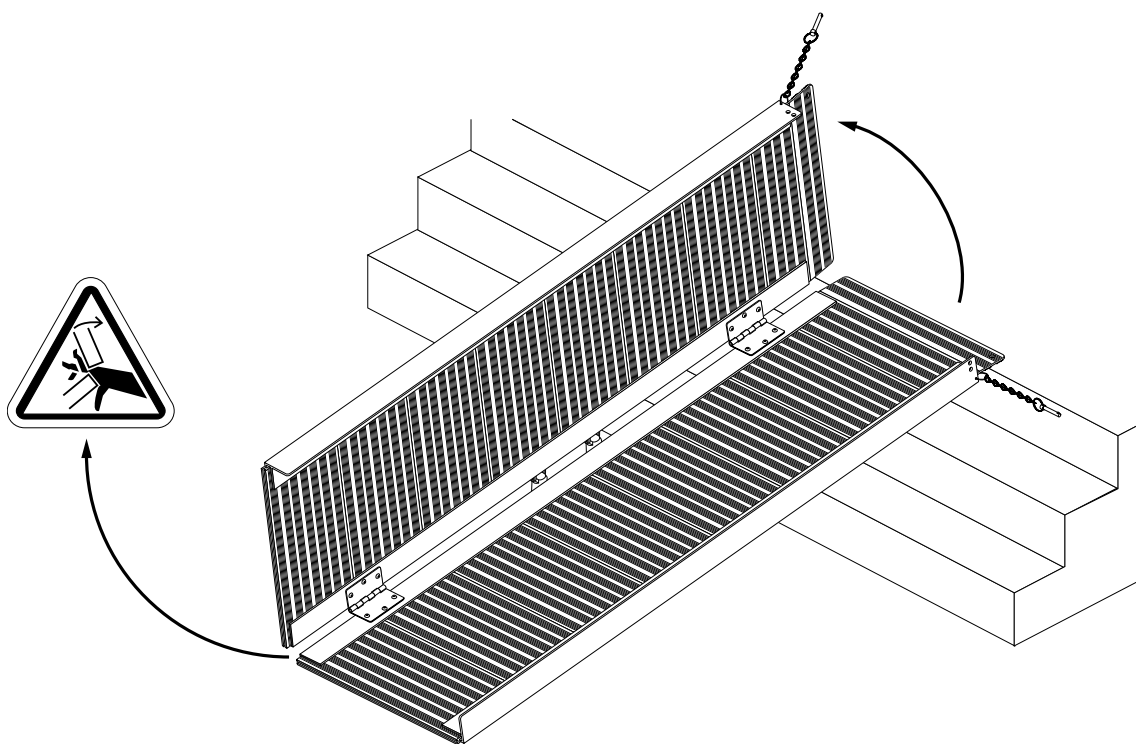
## APPENDIX C : SETUP

### 3

Insert detent pins into the support surface to secure the ramp.

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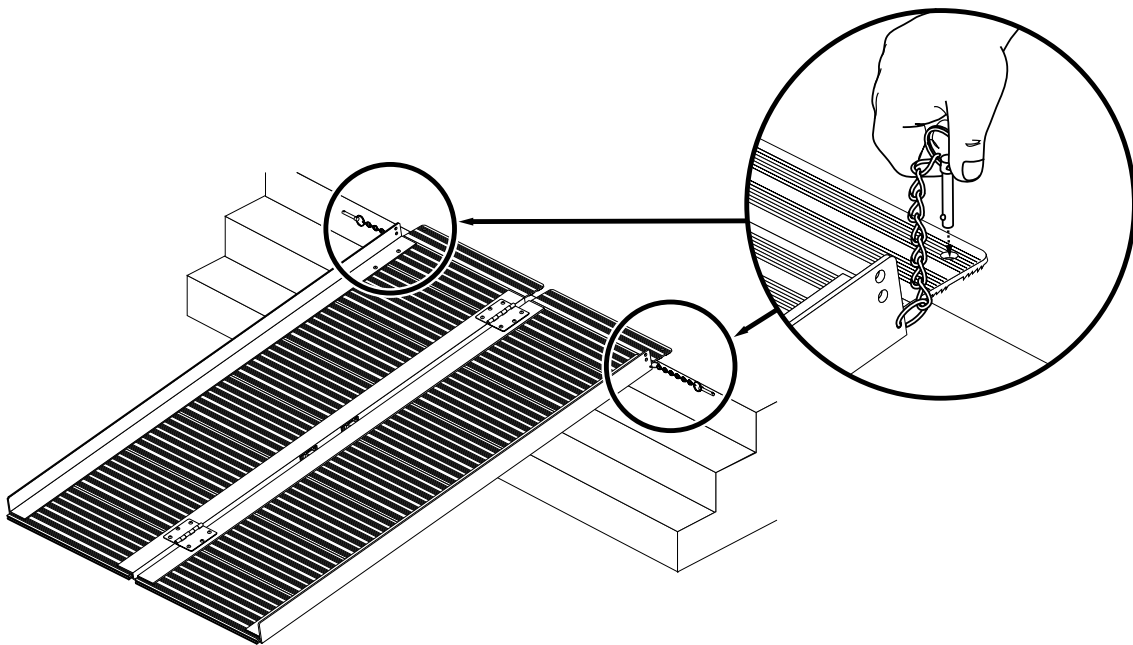
*NOTE: You may need to drill holes in the support surface for the detent pins.*

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Pride Mobility Products Company  
380 Vansickle Road Unit 350  
St. Catharines, Ontario L2R 6P7  
Canada

Pride Mobility Products Ltd.  
Unit 106, Heyford Park Camp Road  
Upper Heyford, Oxfordshire OX25 5HA

Pride Mobility Products Australia Pty. Ltd.  
21 Healey Road  
Dandenong, 3175  
Victoria, Australia

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Pride Mobility Products Italia S.r.l.
Pride Mobility Products Europe B.V.