

PRODUCT INSERT **LITESTREAM CASTER STEM EXTENSIONS**

NOTE: The following information shows the seat-to-floor heights when using the caster fork stem extensions and is in addition to information contained in the *Litestream XF or Litestream XF LTD Owner's Manual* provided with your manual wheelchair. Please retain this information with the owner's manual and contact your *Quantum Rehab Provider* with any questions.

LITESTREAM SEAT-TO-FLOOR HEIGHTS WITH STEM EXTENSIONS

The front and rear seat-to-floor (STF) heights will change according to the stem extensions for the bolt-on caster forks. Maximum seat dump is 5.08 cm (2 in.). Refer to the following charts for the changes to STF and required rear wheel size options.

Front Seat-to-Floor (STF) Height Matrix Front stem extension option						
No stem extension				0.75-in.	1.5-in.	
Selected front STF	Caster (inches)				New Seat-to-Floor Height with stem extension	
	4	5	6	8		
Litestream hemi frame						
12.5 in.	X				13.25 in.	14 in.
13 in.	X	X			13.75 in.	14.5 in.
13.5 in.	X	X			14.25 in.	15 in.
14 in.	X	X	X		14.75 in.	15.5 in.
14.5 in.	X	X	X		15.25 in.	16 in.
15 in.		X	X		15.75 in.	16.5 in.
15.5 in.			X		16.25 in.	17 in.
16 in.				X	16.75 in.	17.5 in.
16.5 in.				X	17.25 in.	18 in.
Litestream XF frame						
15 in.	X				15.75 in.	16.5 in.
15.5 in.	X				16.25 in.	17 in.
16 in.	X	X			16.75 in.	17.5 in.
16.5 in.	X	X			17.25 in.	18 in.
17 in.	X	X	X		17.75 in.	18.5 in.
17.5 in.	X	X	X		18.25 in.	19 in.
18 in.		X	X		18.75 in.	19.5 in.
18.5 in.			X		19.25 in.	20 in.
19 in.				X	19.75 in.	20.5 in.
19.5 in.				X	20.25 in.	21 in.

Rear Seat-to-Floor (STF) Height Matrix Front stem extension option					
No stem extension					
Selected rear STF height	Required rear wheel option (inches)				
	20	22	24	25	26
Litestream hemi frame					
12.5 in.	X				
13 in.	X				
13.5 in.	X	X			
14 in.	X	X			
14.5 in.	X	X	X		
15 in.	X	X	X	X	
15.5 in.	X	X	X	X	X
16 in.	X	X	X	X	X
16.5 in.		X	X	X	X
Litestream XF frame					
15 in.	X	X	X	X	X
15.5 in.	X	X	X	X	X
16 in.	X	X	X	X	X
16.5 in.		X	X	X	X
17 in.		X	X	X	X
17.5 in.			X	X	X
18 in.			X	X	X
18.5 in.				X	X
19 in.					X



ADDENDA AU PRODUIT

RALLONGES DE TIGES DE FOURCHES DE LITESTREAM

REMARQUE: Les informations suivantes indiquent les hauteurs sol-siège lorsque vous utilisez les rallonges de tiges de fourches et sont des informations supplémentaires aux informations contenues dans le manuel du propriétaire du Litestream XF ou du Litestream XF LTD fourni avec votre fauteuil roulant manuel. Veuillez garder ces informations avec le manuel du propriétaire et pour toute question contacter votre fournisseur Quantum Rehab.

HAUTEURS SOL-SIEGE AVEC LES RALLONGES DE TIGES DE FOURCHES DU LIFESTREAM

Les hauteurs sol-siège (HSS) avant et arrière changeront selon les rallonges de tiges des fourches. L'affaissement maximum du siège est de 5.08 cm (2 po). Reportez-vous aux tableaux suivants pour les modifications apportées à la HSS et pour les options de diamètres des roues arrière requises.

Tableau des Hauteurs Sol-Siège (HSS) avant avec l'option de rallonge de tige de fourche						
Sans rallonge de tige				0.75-po	1.5-po	
HSS avant sélectionnée	Roue avant (po)				Nouvelle HSS avant avec rallonge de tige	
	4	5	6	8		
Cadre Litestream hemi						
12.5 po	X				13.25 po	14 po
13 po	X	X			13.75 po	14.5 po
13.5 po	X	X			14.25 po	15 po
14 po	X	X	X		14.75 po	15.5 po
14.5 po	X	X	X		15.25 po	16 po
15 po		X	X		15.75 po	16.5 po
15.5 po			X		16.25 po	17 po
16 po				X	16.75 po	17.5 po
16.5 po				X	17.25 po	18 po
Cadre Litestream XF						
15 po	X				15.75 po	16.5 po
15.5 po	X				16.25 po	17 po
16 po	X	X			16.75 po	17.5 po
16.5 po	X	X			17.25 po	18 po
17 po	X	X	X		17.75 po	18.5 po
17.5 po	X	X	X		18.25 po	19 po
18 po		X	X		18.75 po	19.5 po
18.5 po			X		19.25 po	20 po
19 po				X	19.75 po	20.5 po
19.5 po				X	20.25 po	21 po

Tableau des Hauteurs Sol-Siège (HSS) arrière avec l'option de rallonge de tige de fourche					
Sans rallonge de tige					
HSS arrière sélectionnée	Option de roue arrière requise (pouces)				
	20	22	24	25	26
Cadre Litestream hemi					
12.5 po	X				
13 po	X				
13.5 po	X	X			
14 po	X	X			
14.5 po	X	X	X		
15 po	X	X	X	X	
15.5 po	X	X	X	X	X
16 po	X	X	X	X	X
16.5 po		X	X	X	X
Cadre Litestream XF					
15 po	X	X	X	X	X
15.5 po	X	X	X	X	X
16 po	X	X	X	X	X
16.5 po		X	X	X	X
17 po		X	X	X	X
17.5 po			X	X	X
18 po			X	X	X
18.5 po				X	X
19 po					X

