*Product Insert  
Addenda au Produit  
Productbijsluiter  
Beipackzettel  
Allegato al Prodotto  
Nota Informativa*

**Q-LOGIC 2**

Copyright @ 2015  
INFINF3364/Rev B/June 2015

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**WARNING! The iLevel® feature should be used only on flat, level, indoor surface, such as in malls, offices, and/or medical facilities. Never raise the seat from its lowest position on an inclined surface. Failure to heed this warning can result in the power chair tipping over.**

**WARNING! The iLevel® feature is intended for limited outdoor use on only flat, level, hardscape surfaces. If you are unsure of your environment, do not use iLevel® outdoors.**

**PROHIBITED! Never use the iLevel® outdoors on uneven terrain (gravel, hills or other uneven, non-compact terrain) or in fields.**

**WARNING! Never raise the seat into iLevel® when operating your power chair on bumpy or uneven surfaces. Failure to heed this warning can result in the power chair tipping over.**

**WARNING! Always fasten the positioning belt when operating the power chair.**

**WARNING! Do not operate the iLevel® or change the seat height in any way while your power chair is positioned under a fixed object, such as a table or a desk. Keep area clear prior to and during movement.**

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**Q-LOGIC 2**

**NOTE:** *The following information is an addition to the information contained in the Basic Operating Instructions provided with your Q-Logic 2 controller. It explains how to operate your iLevel® power elevating seat. Please retain this information for your records and contact your Quantum Rehab Provider with any questions.*

### **iLevel® Feature (Optional)**

Your power chair may be equipped with the iLevel® feature that will be used in conjunction with your TRU-Balance® 3 Power Positioning System. The iLevel® feature is mounted on a Quantum power base for enhanced stability when the seating system is elevated.

There are certain situations, including some medical conditions, where the power chair user will need to practice operating the power chair equipped with iLevel® in the presence of a trained attendant. A trained attendant can be defined as a family member or care professional specially trained in assisting a power chair user in various daily living activities.

As you begin using iLevel® during daily activities, you will probably encounter situations in which you will need some practice. Simply take your time and you will soon be in full and confident control.

### **Elevation (Seat Screen)**

The iLevel® feature can be activated by selecting the elevating seat actuator.

#### **To select and adjust the seat elevation using the Q-Logic 2 Controller:**

1. Push the On/Off and Mode Select Lever up once to power on the chair and the controller.
2. Push the On/Off and Mode Select Lever up several times until the seat screen is displayed on the LCD or push the key 2 select button once to go directly to seat screen. **See figure 1.**
3. Push the joystick to the left or right to cycle through to the elevating seat mode, this is illuminated on the actuator indicator. **See figure 1.**

**NOTE:** *If your Q-Logic 2 settings are different than those described, contact your Quantum Rehab Provider. Your Quantum Rehab Provider may have changed the programming of your Q-Logic 2 Controller.*

4. When the elevating seat mode is selected, give a forward command to the joystick to adjust position in one direction or give a reverse command to the joystick to adjust position in the opposite direction.
5. If you want to return to another profile, push up and release the On/Off and Mode Select Lever until you return to the desired drive profile.

**NOTE:** *The following information is an addition to the information contained in the Basic Operating Instructions provided with your Q-Logic 2 controller. It explains how to operate your iLevel® power elevating seat. Please retain this information for your records and contact your Quantum Rehab Provider with any questions.*

### **iLevel® Feature (Optional)**

Your power chair may be equipped with the iLevel® feature that will be used in conjunction with your TRU-Balance® 3 Power Positioning System. The iLevel® feature is mounted on a Quantum power base for enhanced stability when the seating system is elevated.

There are certain situations, including some medical conditions, where the power chair user will need to practice operating the power chair equipped with iLevel® in the presence of a trained attendant. A trained attendant can be defined as a family member or care professional specially trained in assisting a power chair user in various daily living activities.

As you begin using iLevel® during daily activities, you will probably encounter situations in which you will need some practice. Simply take your time and you will soon be in full and confident control.

### **Elevation (Seat Screen)**

The iLevel® feature can be activated by selecting the elevating seat actuator.

#### **To select and adjust the seat elevation using the Q-Logic 2 Controller:**

1. Push the On/Off and Mode Select Lever up once to power on the chair and the controller.
2. Push the On/Off and Mode Select Lever up several times until the seat screen is displayed on the LCD or push the key 2 select button once to go directly to seat screen. **See figure 1.**
3. Push the joystick to the left or right to cycle through to the elevating seat mode, this is illuminated on the actuator indicator. **See figure 1.**

**NOTE:** *If your Q-Logic 2 settings are different than those described, contact your Quantum Rehab Provider. Your Quantum Rehab Provider may have changed the programming of your Q-Logic 2 Controller.*

4. When the elevating seat mode is selected, give a forward command to the joystick to adjust position in one direction or give a reverse command to the joystick to adjust position in the opposite direction.
5. If you want to return to another profile, push up and release the On/Off and Mode Select Lever until you return to the desired drive profile.

**To activate the iLevel® using a single switch:**

1. Press once and hold the single switch once to raise the seat and activate iLevel®.
2. Press and hold the single switch to lower the seat.

***NOTE: The iLevel® feature is automatically engaged when the power elevating seat is activated and the iLevel® indicator is displayed on your controller. See figure 2.***

The power chair should be used on an indoor, level surface, such as malls, offices, and/or medical facilities to activate iLevel®. Once the seating system actuator is started, the iLevel® boots limit movement of the front caster wheels and restrict the power chair to 1 in. - 1.5 in. (2.54 cm - 3.81 cm) of climbing ability. The seating system can be reclined or tilted up to 20° while the iLevel® feature is activated.

**WARNING!** The iLevel® feature should be used only on flat, level, indoor surface, such as in malls, offices, and/or medical facilities. Never raise the seat from its lowest position on an inclined surface. Failure to heed this warning can result in the power chair tipping over.

**WARNING!** The iLevel® feature is intended for limited outdoor use on only flat, level, hardscape surfaces. If you are unsure of your environment, do not use iLevel® outdoors.

**PROHIBITED!** Never use the iLevel® outdoors on uneven terrain (gravel, hills or other uneven, non-compact terrain) or in fields.

**WARNING!** Never raise the seat into iLevel® when operating your power chair on bumpy or uneven surfaces. Failure to heed this warning can result in the power chair tipping over.

**WARNING!** Always fasten the positioning belt when operating the power chair.


**WARNING!** Do not operate the iLevel® or change the seat height in any way while your power chair is positioned under a fixed object, such as a table or a desk. Keep area clear prior to and during movement.

**NOTE: The iLevel® feature is equipped with a system that allows the power chair to drive at a predetermined, safe speed when the seat is elevated. Always check to be sure this system is operating properly before using your power chair.**


**NOTE: If a warning appears on the Q-Logic 2 controller LCD stating “iLevel® is not engaged,” the power chair’s speed will be reduced to 25% when elevated. If this warning appears while on level ground, please discontinue use of iLevel® feature and consult your Quantum Rehab Provider for assistance.**

**Figure 1. Elevating Seat Mode**

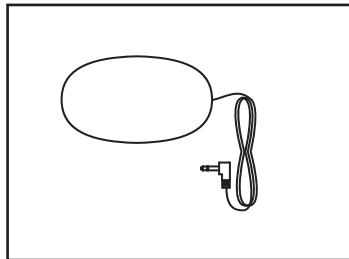**Figure 3. Single Switch**

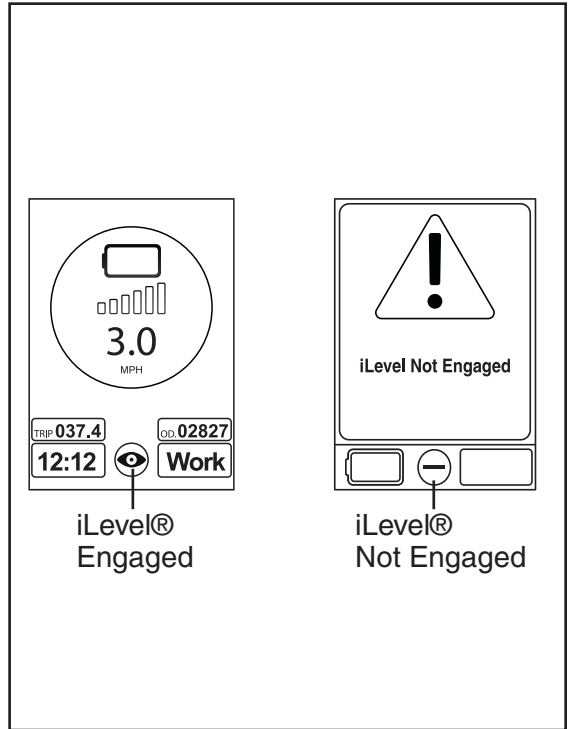**Figure 2. Q-Logic 2 Controller Indicators**

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**B.V.**