
INSTALLATION AND CARE INSTRUCTIONS

Synergy[®] Solution[™]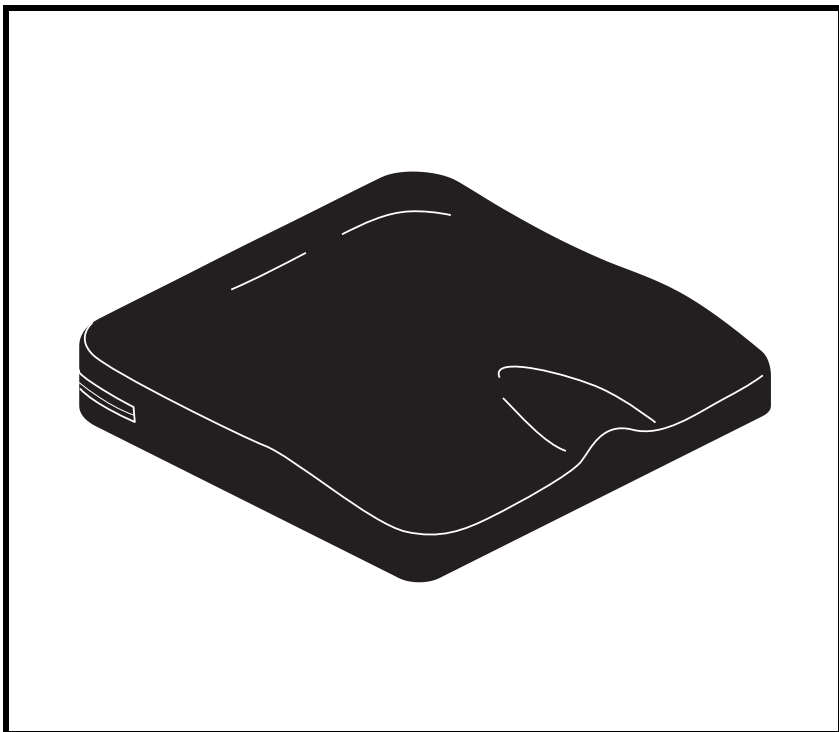

QUANTUM[®]
REHAB

A Division of Pride Mobility Products[®] Corp.

I N F M A N U 2 0 7 0

SAFETY GUIDELINES

WARNING! A Quantum Rehab Provider or a qualified technician must perform the initial setup of this product and must perform all of the instructions in this manual.

The symbols below are used throughout this owner's manual and on the product to identify warnings and important information. It is very important for you to read them and understand them completely.

WARNING! Indicates a potentially hazardous condition/situation. Failure to follow designated procedures can cause either personal injury, component damage, or malfunction. On the product, this icon is represented as a black symbol on a yellow triangle with a black border.


MANDATORY! These actions should be performed as specified. Failure to perform mandatory actions can cause personal injury and/or equipment damage. On the product, this icon is represented as a white symbol on a blue dot with a white border.

PROHIBITED! These actions are prohibited. These actions should not be performed at any time or in any circumstances. Performing a prohibited action can cause personal injury and/or equipment damage. On the product, this icon is represented as a black symbol with a red circle and a red slash.

NOTE: These instructions are compiled from the latest specifications and product information available at the time of publication. We reserve the right to make changes as they become necessary. Any changes to our products may cause slight variations between the illustrations and explanations in this manual and the product you have purchased. The latest/current version of this manual is available on our website.

NOTE: This product is compliant with WEEE, RoHS, and REACH directives and requirements.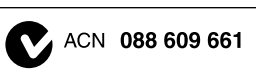

Copyright © 2010

Pride Mobility Products Corporation

INFMANU2070/Rev E/November 2010

TABLE OF CONTENTS

LABEL INFORMATION	4
INTRODUCTION	5
THE SYNERGY SOLUTION	7
INSTALLATION AND USE	7
CLEANING	8
CLEANING THE OPTIONAL SINGLE COVER	9
STORAGE	10
CARE AND MAINTENANCE	10
WARRANTY	10

LABEL INFORMATION

PRODUCT SAFETY SYMBOLS

The symbols below are used on the product to identify warnings, mandatory actions, and prohibited actions. It is very important for you to read and understand them completely.

Read and follow the information in the owner's manual.

Do not expose to heat sources such as open flame or sparks.

No smoking.

Cover only — Maximum washing temperature 140°F (60°C)

Do not tumble dry.

Do not iron.

INTRODUCTION

WELCOME to Quantum Rehab, a division of Pride Mobility Products (Pride). The product you have purchased combines state-of-the-art components with **safety**, comfort, and styling in mind. We are confident that the design features will provide you with the conveniences you expect during your daily activities. Understanding how to **safely** operate and care for this product should bring you years of trouble free operations and service.

Read and follow all instructions, warnings, and notes in this manual before attempting to operate your product for the first time. You must also read all instructions, warnings, and notes contained in any supplemental instructional booklets for the controller, front riggings, and/or seating system that accompanied your mobility product before initial operation. Your **safety** depends upon you, as well as your provider, caretaker, or healthcare professional in using good judgement.

This manual is to be used in addition to the owner's manual that came with your chair. If there is any information in this manual which you do not understand, or if you require additional assistance for setup or operation, please contact your Quantum Rehab Provider. **Failure to follow the instructions, warnings, and notes in this manual and those located on your Pride product can result in personal injury and/or product damage and will void Pride's product warranty.**

PURCHASER'S AGREEMENT

By accepting delivery of this product, you promise that you will not change, alter, or modify this product or remove or render inoperable or unsafe any guards, shields, or other safety features of this product; fail, refuse, or neglect to install any retrofit kits from time to time provided by Pride to enhance or preserve the safe use of this product.

INFORMATION EXCHANGE

We want to hear your questions, comments, and suggestions about this manual. We would also like to hear about the safety and reliability of your new Pride product, and about the service you received from your Quantum Rehab Provider. Please notify us of any change of address, so we can keep you apprised of important information about safety, new products, and new options that can increase your ability to use and enjoy your Pride product.

Contact Information

Pride Mobility Products Corporation
182 Susquehanna Avenue
Exeter, PA 18643-2694
USA

Pride Mobility Products Company
380 Vansickle Road Unit 350
St. Catharines, Ontario L2R 6P7
Canada

Pride Mobility Products Ltd.
32 Wedgwood Road
Bicester, Oxon OX26 4UL
UK

Pride Mobility Products Australia Pty. Ltd.
21 Healey Road
Dandenong, 3175
Victoria, Australia

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NOTE: If you ever lose or misplace your product registration card or your copy of this manual, contact us and we will be glad to send you a new one immediately.

THE SYNERGY SOLUTION

The Synergy Solution is a seatbase cushion designed for user comfort and positioning needs. **Figure 1** provides information on the Synergy Solution components and placement.

INSTALLATION AND USE

The Synergy Solution is labeled to assure proper placement on the chair.

To place the Synergy Solution on the chair:

1. Locate the “Back Bottom” label on the underside of the cushion.
2. Place the cushion on the seat so that the “Back Bottom” label is located on the back of the seat, label side facing down. The zipper on the cushion will face the rear side of the seat. See **figure 1**.

WARNING! Extreme temperatures can affect the temperature of your cushion components and may cause skin irritation. Use caution when your power chair has been used or stored in an extremely hot or cold environment.

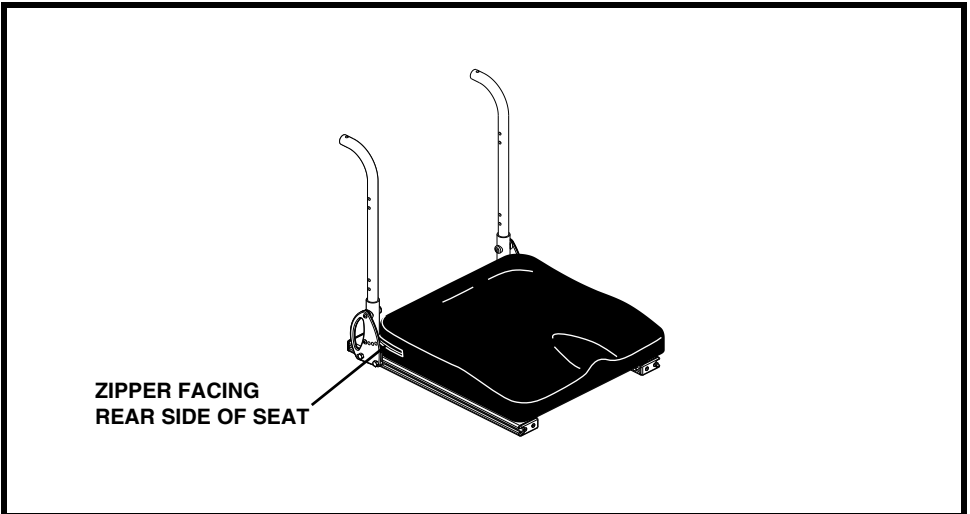
Figure 1. Cushion Placement

CLEANING

The Synergy Solution can be easily cleaned. For information on cleaning the Synergy Solution Single Cover refer to “Cleaning the Optional Single Cover.”

To clean the Synergy Solution:

1. Locate the zipper on the rear side of the outer cover. **See figure 2.**
2. Unzip the outer cover and pull out the inner cushion, which is wrapped in a fluid-resistant cover.
3. Wash the outer cover in a washing machine on the COLD setting.

WARNING! Do not use bleach when washing the outer cover. Doing so may cause damage to the cover.

4. Spray the inner cover with a disinfectant, then wipe it clean.

WARNING! Never submerge the inner cushion of your Synergy Solution in water. Doing so may cause damage to the cushion.

5. Allow both the outer and inner covers to air dry.

WARNING! Do not attempt to put the outer cover in a dryer. This may cause shrinking.

6. Once both covers are dry, put the inner cushion back inside the outer cover. Zip the outer cover closed, and place the Synergy Solution back onto the seat of your chair.

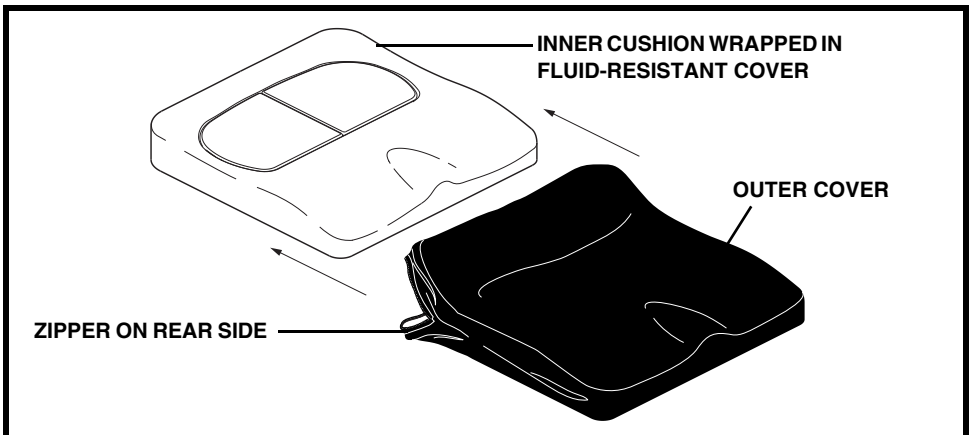
Figure 2. Synergy Solution

CLEANING THE OPTIONAL SINGLE COVER

The Synergy Solution Single Cover can be easily cleaned.

To clean the Synergy Solution Single Cover:

1. Locate the zipper on the rear side of the outer cover. See **figure 3**.
2. Unzip the cover and pull out the inner cushion.
3. Wash the outer cover in a washing machine using soap powder (not detergent) on the COLD setting.

WARNING! Do not use bleach when washing the outer cover. Doing so may cause damage to the cover.

WARNING! Never submerge the inner cushion of your Synergy Solution in water. Doing so may cause damage to the cushion.

4. Allow the cover to air dry.

WARNING! Do not attempt to put the outer cover in a dryer. This may cause shrinking.

5. Once the cover is dry, put the inner cushion back inside the cover. Zip the cover closed, and place the Synergy Solution back onto the seat of your chair.

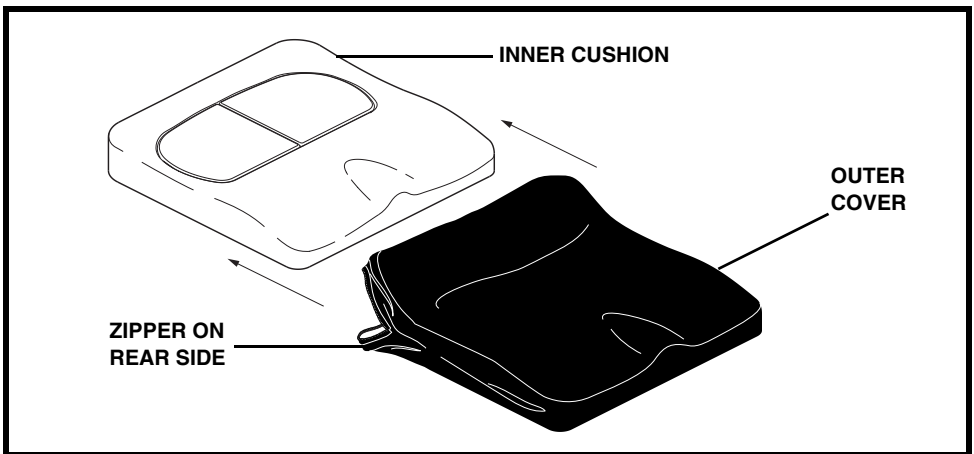
Figure 3. Synergy Solution With Optional Single Cover

STORAGE

The Synergy Solution should not be stored in extreme heat or cold temperatures. It should be stored at room temperature in a clean and dry location.

CARE AND MAINTENANCE

Your mobility product must be disposed of according to applicable local and national statutory regulations. Contact your local waste disposal agency or Quantum Rehab Provider for information on proper disposal of mobility product packaging, metal frame components, plastic components, electronics, batteries, neoprene, silicone, and polyurethane materials.

Be aware of safety issues while smoking.

WARNING! Pride strongly recommends that you do not smoke cigarettes while seated on your seat cushion although the seat cushion has passed the necessary testing requirements for cigarette smoking. You must adhere to the following safety guidelines if you decide to smoke cigarettes while seated on your seat cushion.

- **Do not leave lit cigarettes unattended.**
- **Keep ashtrays a safe distance from the seating system.**
- **Always make sure cigarettes are completely extinguished before disposal.**

WARRANTY

The Synergy Solution cushion is designed and produced to the highest of standards. Our Synergy cushion is guaranteed to be free of defects in material and workmanship for a period of 24 months. If any defect in material or workmanship is found, Pride Mobility Products will repair or replace, at our discretion, the Synergy cushion. This warranty does not apply to cushions abused or misused by the user and deemed such by Pride Mobility Products.

Claims and repairs should be processed through the nearest authorised Pride Provider or Quantum Rehab Provider.

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A Division of Pride Mobility Products® Corp.