
INSTALLATION AND CARE INSTRUCTIONS

Synergy[®] TRU-Comfort[™]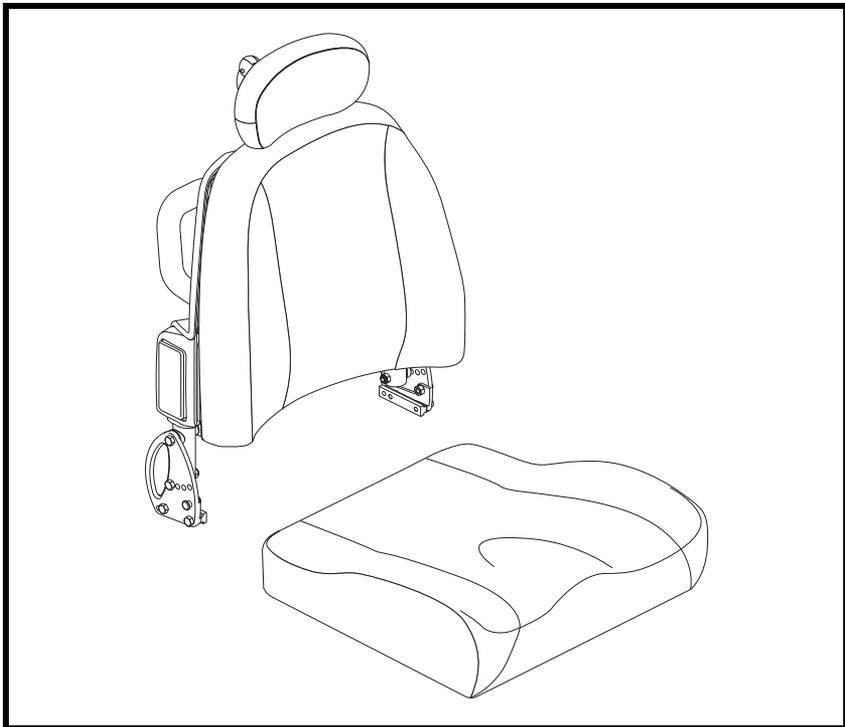

QUANTUM[®]
REHAB

A Division of Pride Mobility Products[®] Corp.

SAFETY GUIDELINES

WARNING! A Quantum Rehab Provider or a qualified technician must perform the initial setup of this product and must perform all of the instructions in this manual.

The symbols below are used throughout this owner's manual and on the product to identify warnings and important information. It is very important for you to read them and understand them completely.

WARNING! Indicates a potentially hazardous condition/situation. Failure to follow designated procedures can cause either personal injury, component damage, or malfunction. On the product, this icon is represented as a black symbol on a yellow triangle with a black border.

MANDATORY! These actions should be performed as specified. Failure to perform mandatory actions can cause personal injury and/or equipment damage. On the product, this icon is represented as a white symbol on a blue dot with a white border.

PROHIBITED! These actions are prohibited. These actions should not be performed at any time or in any circumstances. Performing a prohibited action can cause personal injury and/or equipment damage. On the product, this icon is represented as a black symbol with a red circle and a red slash.

NOTE: These instructions are compiled from the latest specifications and product information available at the time of publication. We reserve the right to make changes as they become necessary. Any changes to our products may cause slight variations between the illustrations and explanations in this manual and the product you have purchased. The latest/current version of this manual is available on our website.

NOTE: This product is compliant with WEEE, RoHS, and REACH directives and requirements.

Copyright © 2011
Pride Mobility Products Corporation
INFMANU3357/Rev G/December 2011

TABLE OF CONTENTS

LABEL INFORMATION	4
INTRODUCTION	5
THE SYNERGY TRU-COMFORT	7
INSTALLATION AND USE	10
FLIP-FORWARD BACK CANES	10
CANTILEVER ARMRESTS	11
CLEANING	13
LATERAL ADJUSTMENT (OPTIONAL)	15
HEADREST ADJUSTMENT	16
STORAGE	16
CARE AND MAINTENANCE	16
WARRANTY	18

LABEL INFORMATION

PRODUCT SAFETY SYMBOLS

The symbols below are used on the product to identify warnings, mandatory actions, and prohibited actions. It is very important for you to read and understand them completely.

Read and follow the information in the owner's manual.

Do not expose to heat sources such as open flame or sparks.

No smoking.

Cover only — Maximum washing temperature 140°F (60°C)

Do not tumble dry.

Do not iron.

INTRODUCTION

WELCOME to Quantum Rehab, a division of Pride Mobility Products (Pride). The product you have purchased combines state-of-the-art components with **safety**, comfort, and styling in mind. We are confident that the design features will provide you with the conveniences you expect during your daily activities. Understanding how to **safely** use and care for this product should bring you years of trouble free operations and service.

Read and follow all instructions, warnings, and notes in this manual before attempting to operate your product for the first time. You must also read all instructions, warnings, and notes contained in any supplemental instructional booklets for the controller, front riggings, and/or seating system that accompanied your power chair before initial operation. Your **safety** depends upon you, as well as your provider, caretaker, or healthcare professional in using good judgment.

This manual is to be used in addition to the owner's manual that came with your chair. If there is any information in this manual which you do not understand, or if you require additional assistance for setup or operation, please contact your Quantum Rehab Provider. **Failure to follow the instructions, warnings, and notes in this manual and those located on your Pride product can result in personal injury and/or product damage and will void Pride's product warranty.**

PURCHASER'S AGREEMENT

By accepting delivery of this product, you promise that you will not change, alter, or modify this product or remove or render unusable or unsafe any guards, shields, or other safety features of this product; fail, refuse, or neglect to install any retrofit kits from time to time provided by Pride to enhance or preserve the safe use of this product.

INFORMATION EXCHANGE

We want to hear your questions, comments, and suggestions about this manual. We would also like to hear about the safety and reliability of your new Pride product, and about the service you received from your Quantum Rehab Provider. Please notify us of any change of address, so we can keep you apprised of important information about safety, new products, and new options that can increase your ability to use and enjoy your Pride product.

Contact Information

Pride Mobility Products Corporation
182 Susquehanna Avenue
Exeter, PA 18643-2694
USA

Pride Mobility Products Company
380 Vansickle Road Unit 350
St. Catharines, Ontario L2R 6P7
Canada

Pride Mobility Products Ltd.
32 Wedgwood Road
Bicester, Oxon OX26 4UL
UK

Pride Mobility Products Australia Pty. Ltd.
20-24 Apollo Drive
Hallam, Victoria 3803
Australia

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NOTE: If you ever lose or misplace your product registration card or your copy of this manual, contact us and we will be glad to send you a new one immediately.

THE SYNERGY TRU-COMFORT

The Synergy TRU-Comfort is a unique seat cushion that is available in non-recline, power recline, and manual recline versions, and is adjustable for user comfort and positioning needs. **Figure 1** provides information on the non-recline version of the Synergy TRU-Comfort components. Use this diagram to familiarize yourself with the function and location of each component before using the Synergy TRU-Comfort.

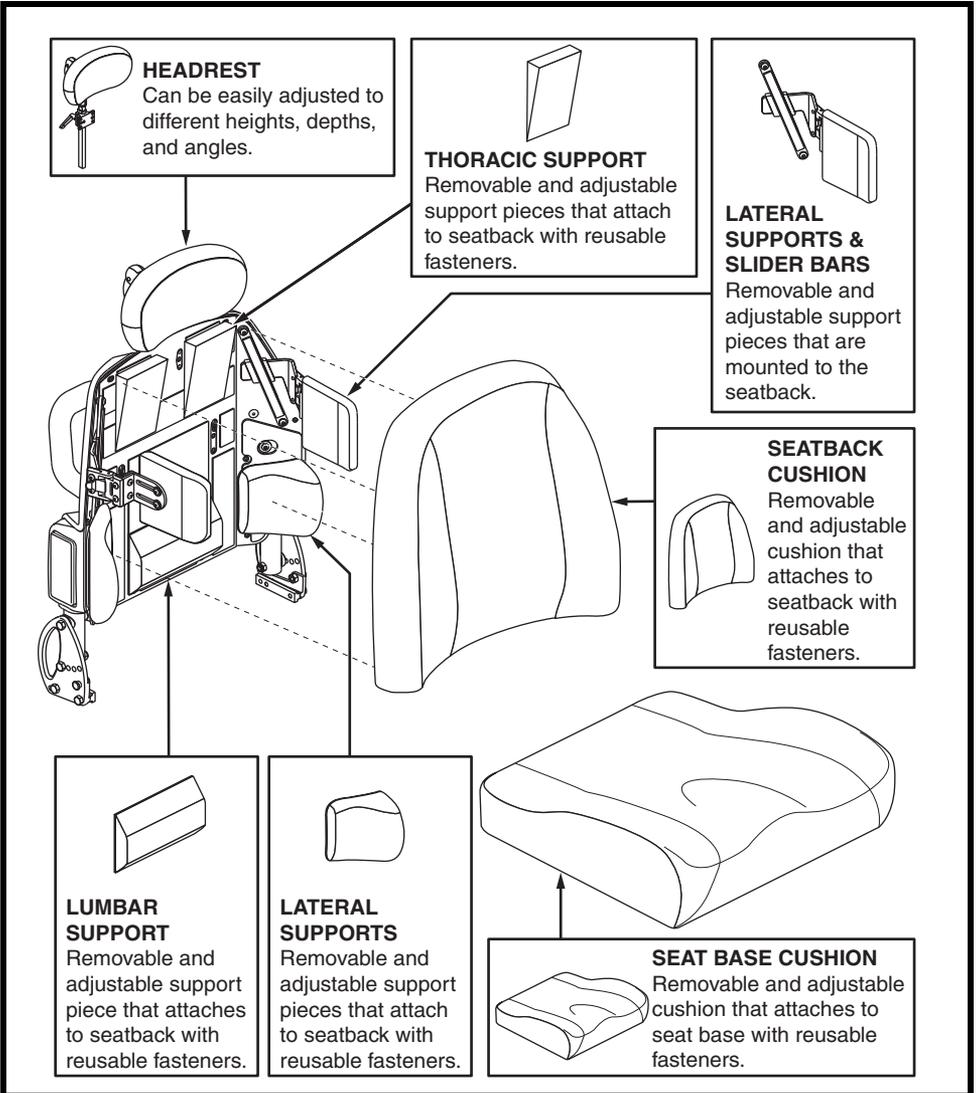
Figure 1. Synergy TRU-Comfort Components (Non-Recline)

Figure 2 provides information on the power recline version of the Synergy TRU-Comfort components. Use this diagram to familiarize yourself with the function and location of each component before using the Synergy TRU-Comfort.

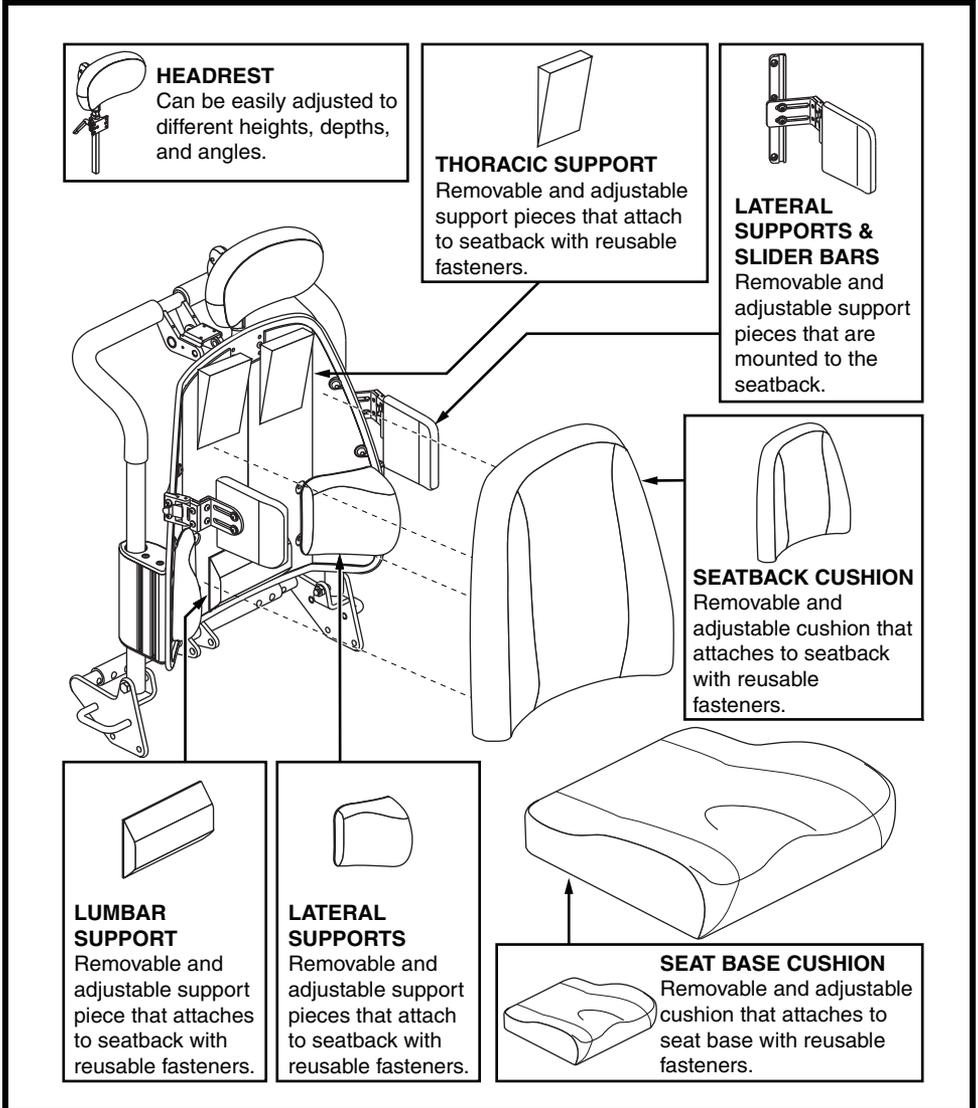
Figure 2. Synergy TRU-Comfort Components (Power Recline)

Figure 3 provides information on the manual recline version of the Synergy TRU-Comfort components. Use this diagram to familiarize yourself with the function and location of each component before using the Synergy TRU-Comfort.

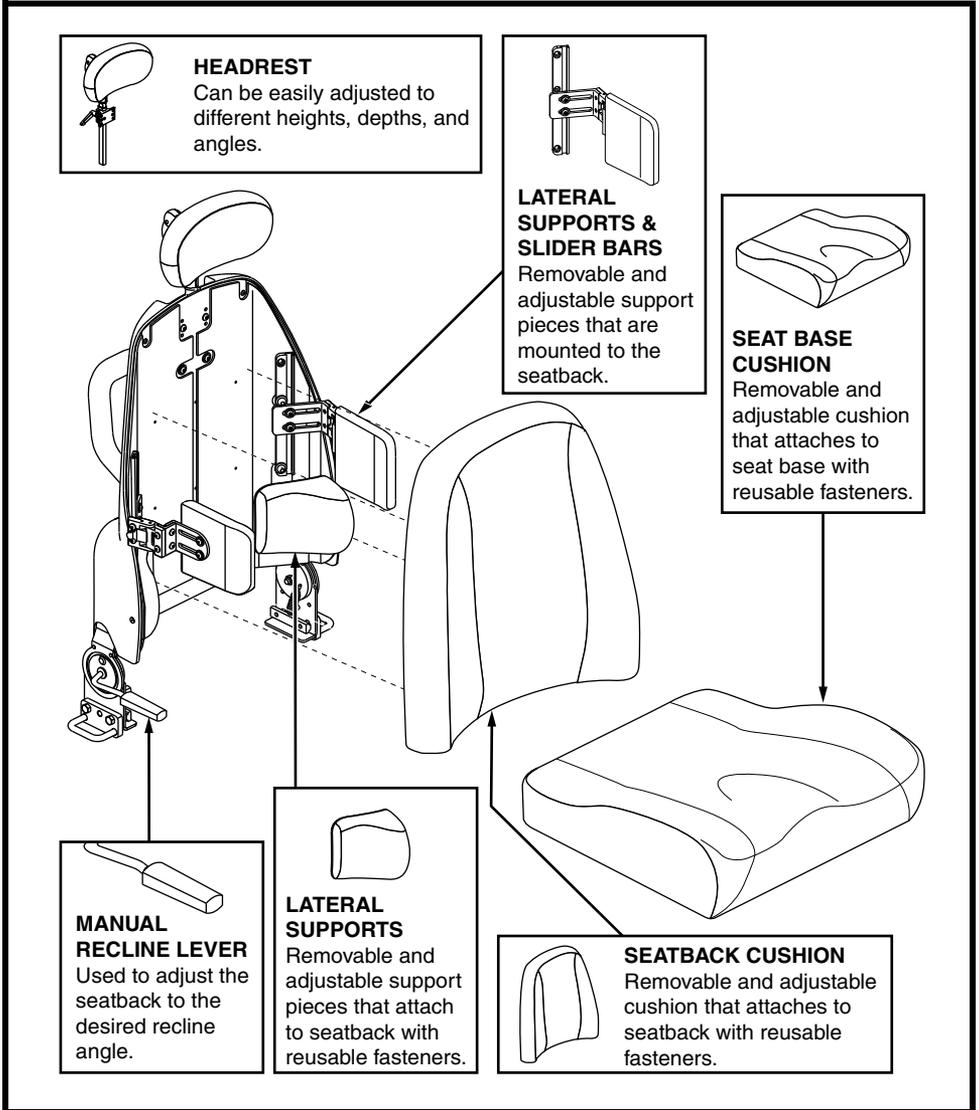
Figure 3. Synergy TRU-Comfort Components (Manual Recline)

INSTALLATION AND USE

The Synergy TRU-Comfort consists of a seatback cushion, lumbar support, lateral supports, and/or thoracic supports that can be placed as needed for user comfort and positioning needs. They are attached to the seatback with reusable hook and loop fasteners.

To place the Synergy TRU-Comfort on the chair:

1. Position the lumbar, lateral, and thoracic support(s) as needed for user comfort and positioning needs flat side down and push onto the seatback. See figure 1 or 2.
2. Position the seatback cushion upholstered side up over any lumbar, lateral, and thoracic support(s) and push the cushion into the seatback to secure. See figure 1, 2, or 3.

WARNING! Extreme temperatures can affect the temperature of your cushion components and may cause skin irritation. Use caution when your power chair has been used or stored in an extremely hot or cold environment.

FLIP-FORWARD BACK CANES

The flip-forward back canes allow the user to flip the seatback forward for easier transportation.

To flip the seatback forward:

1. Rotate the release lever on both back canes toward the rear of the seat to unlatch the back canes. See figure 4.
2. Fold the seatback down.

To return the seatback to an upright position:

1. Lift the seatback up and place in the upright position.
2. Rotate the release lever on both back canes toward the front of the seat to latch the back canes. See figure 5. Ensure a red sticker on the release lever is not visible.

WARNING! If a red sticker is visible, the release lever is not fully latched. Ensure both release levers are fully latched before sitting in your power chair.

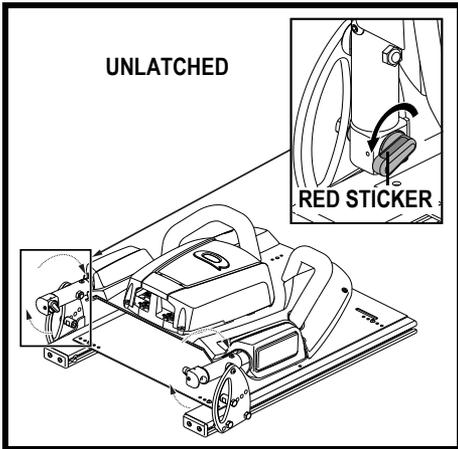
Figure 4. Flip-forward Back Canes (Unlatched)

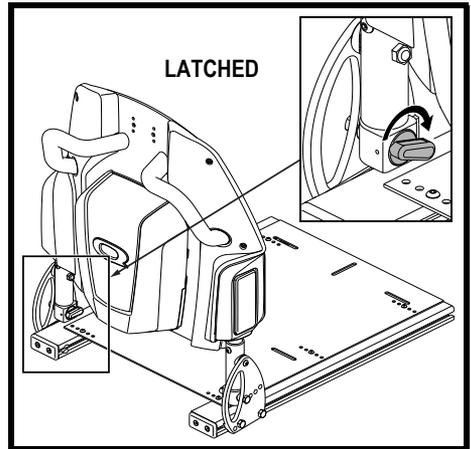
Figure 5. Flip-forward Back Canes (Latched)

CANTILEVER ARMRESTS

The Cantilever Armrests are attached to the back canes and are designed to lift up and out of the way for easier transfer onto and off of the power chair. Both the angle and height can be adjusted on the Cantilever Armrests.

To adjust the armrest angle:

1. Raise the armrest perpendicular to the floor.

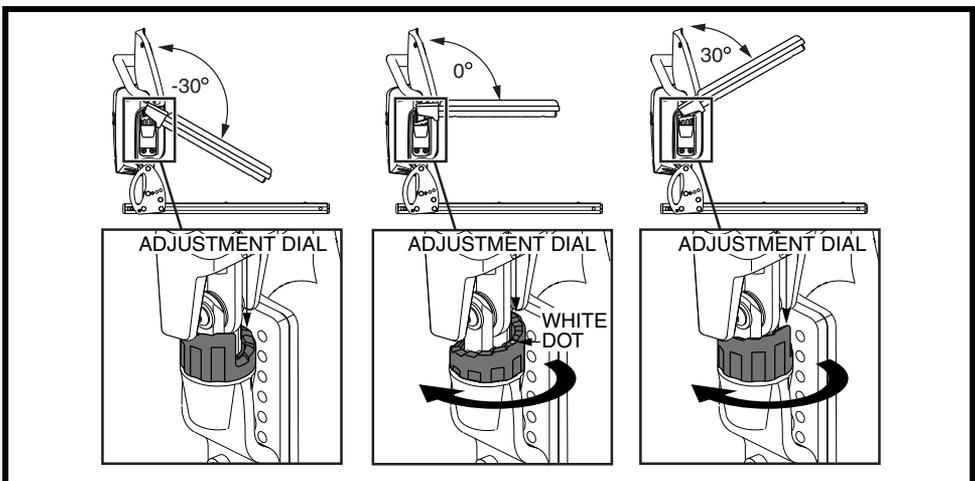
Figure 6. Cantilever Armrests Angle Adjustment

- Turn the adjustable dial clockwise to change the angle of the armrest. **See figure 6.** The armrest is at 0° when it sits on the white dot of the adjustable dial. From 0° , the angle can be decreased to -30° or increased to 30° in 5° increments. The adjustable dial moves only in the clockwise direction.
- Move the armrest down into position.
- Repeat for the other armrest, if necessary.

To adjust the armrest height:

- Raise the armrest perpendicular to the floor.
- Remove and retain the two hex screws from the armrest bracket. **See figure 7.**
- Raise or lower the armrest to the desired height by adjusting the mounting bracket position.

NOTE: The mounting bracket permits an armrest height range from the seat pan to the top of the arm tube of 11–17 in. (27.94–43.18 cm) in 0.5-in. (1.27-cm) increments. To achieve an armrest height of 11 in. (27.94 cm) or 17 in. (43.18 cm), only three of the four mounting screws will be used. All other adjustments will require all four mounting screws.

WARNING! Ensure all hardware is used when reinstalling the mounting bracket. Note that when the armrest is adjusted to the highest or lowest position that only three mounting screws can be used.

- Reinstall the armrest bracket using the retained hex screws and make sure they are properly tightened.

NOTE: To achieve full angle adjustment at 11 in. (27.94 cm), install the two hex screws in the second set of holes from the bottom of the mounting bracket.

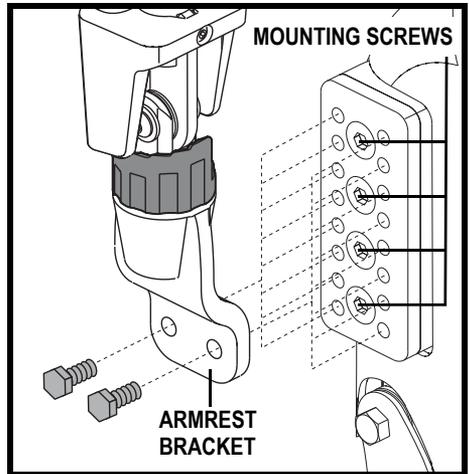
Figure 7. Cantilever Armrest Height Adjustment

5. Move the armrest down into position.
6. Repeat for the other armrest, if necessary.

To adjust the armrest height on a power recline:

1. Raise the armrest perpendicular to the floor.
2. Loosen the three bolts on the armrest bracket.
3. Slide the armrest up or down to the desired height. **See figure 8.**
4. Tighten the three bolts.
5. Move the armrest down into position.
6. Repeat for the other armrest, if necessary.

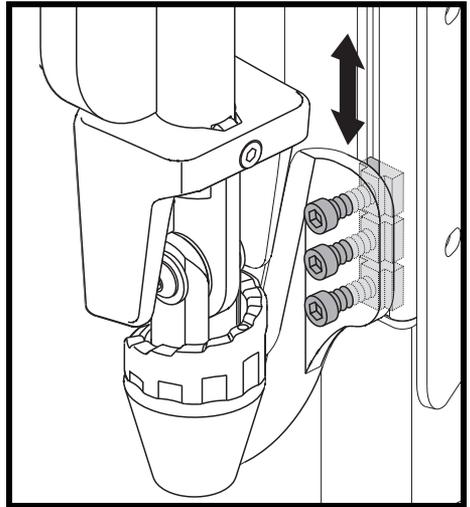
Figure 8. Cantilever Armrest Height Adjustment on power recline

WARNING! Ensure all hardware is installed and tightened properly.

CLEANING

The Synergy TRU-Comfort can be easily cleaned.

NOTE: *The Synergy TRU-Comfort and optional Synergy TRU-Comfort Plus have different cleaning requirements. Make sure you follow the proper cleaning instructions for your cushion.*

To clean the Synergy TRU-Comfort seatback cushion:

1. If necessary, remove the seatback cushion from the seatback by pulling the cushion toward you. **See figure 1, 2, or 3.**
2. Spray the seatback cushion cover with disinfectant, then wipe it clean.
3. Allow the seatback cushion cover to air dry before placing the seatback cushion on the seatback.
4. Once the cover is dry, place the seatback cushion on the seatback and push to secure.

WARNING! Do not use bleach when washing the outer cover. Doing so may cause damage to the cover.

WARNING! Follow all safety instructions for the proper use of the disinfectant and/or cleaning agent before applying it to your product. Failure to comply may result in skin irritation or premature deterioration of upholstery and/or power chair finishes.

To clean the optional Synergy TRU-Comfort Plus seatback cushion:

1. Remove the seatback cushion from the seatback by pulling the cushion toward you. See figure 1, 2, or 3.
2. Locate the zipper on the rear of the cover.
3. Unzip the cover and pull out the cushion.
4. Wash the cover in a washing machine at or below a temperature of 158°F (70°C).

WARNING! Do not use bleach when washing the outer cover. Doing so may cause damage to the cover.

WARNING! Never submerge the cushion of your Synergy TRU-Comfort Plus in water. Doing so may cause damage to the cushion.

5. Allow the cover to air dry.

WARNING! Do not attempt to put the cover in a dryer. This may cause shrinking.

6. Once the cover is dry, re-cover the seatback cushion, place it on the seatback, and push to secure.

WARNING! Follow all safety instructions for the proper use of the disinfectant and/or cleaning agent before applying it to your product. Failure to comply may result in skin irritation or premature deterioration of upholstery and/or power chair finishes.

LATERAL ADJUSTMENT (OPTIONAL)

The laterals can be easily installed and adjusted to different heights and widths.

NOTE: *Laterals should only be installed and adjusted by your Quantum Rehab Provider.*

To install laterals:

1. Loosen the attachment screws in the lateral track.
2. Slide the lateral support through the channel guide as shown in **figure 9**.
3. Line up the holes in the lateral bracket and the lateral support.
4. Attach using provided screws from bracket.
5. Repeat for other side, if necessary.

NOTE: *On a TRU-Comfort Power Recline, the slider bar is installed from the rear of the seatback. See **figure 2**.*

To adjust the laterals:

1. Slide the lateral support to the desired height and width. See **figure 9**.
2. Tighten the screws inserted into the lateral track.
3. Repeat for other side, if necessary.

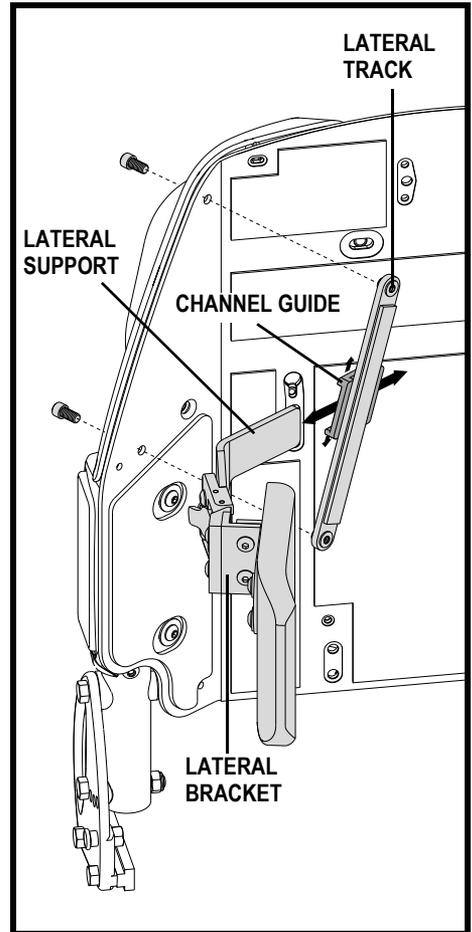
Figure 9. Laterals

HEADREST ADJUSTMENT

The Synergy TRU-Comfort headrest can be easily adjusted to different heights, depths, and angles.

To adjust the height, depth, or angle of the headrest:

1. Loosen the adjustment screws that correspond to the desired adjustments. See figure 10.
2. Adjust the headrest to the desired position.
3. Tighten the adjustment screws.

NOTE: *Once adjustments have been made and the headrest is in the desired position, make sure that all of the hardware is tight.*

WARNING! Do not extend the overall headrest height more than 7 in. (17.78 cm) above the headrest bracket.

STORAGE

The Synergy TRU-Comfort should not be stored in extreme hot or cold temperatures. It should be stored at room temperature in a clean and dry location.

CARE AND MAINTENANCE

Your mobility product must be disposed of according to applicable local and national statutory regulations. Contact your local waste disposal agency or Quantum Rehab Provider for information on proper disposal of mobility product packaging, metal frame components, plastic components, electronics, batteries, neoprene, silicone, and polyurethane materials.

Be aware of safety issues while smoking.

WARNING! Pride strongly recommends that you do not smoke cigarettes while seated in your seating system although the seating system has passed the necessary testing requirements for cigarette smoking. You must adhere to the following safety guidelines if you decide to smoke cigarettes while seated in your seating system.

- Do not leave lit cigarettes unattended.
- Keep ashtrays a safe distance from the seating system.
- Always make sure cigarettes are completely extinguished before disposal.

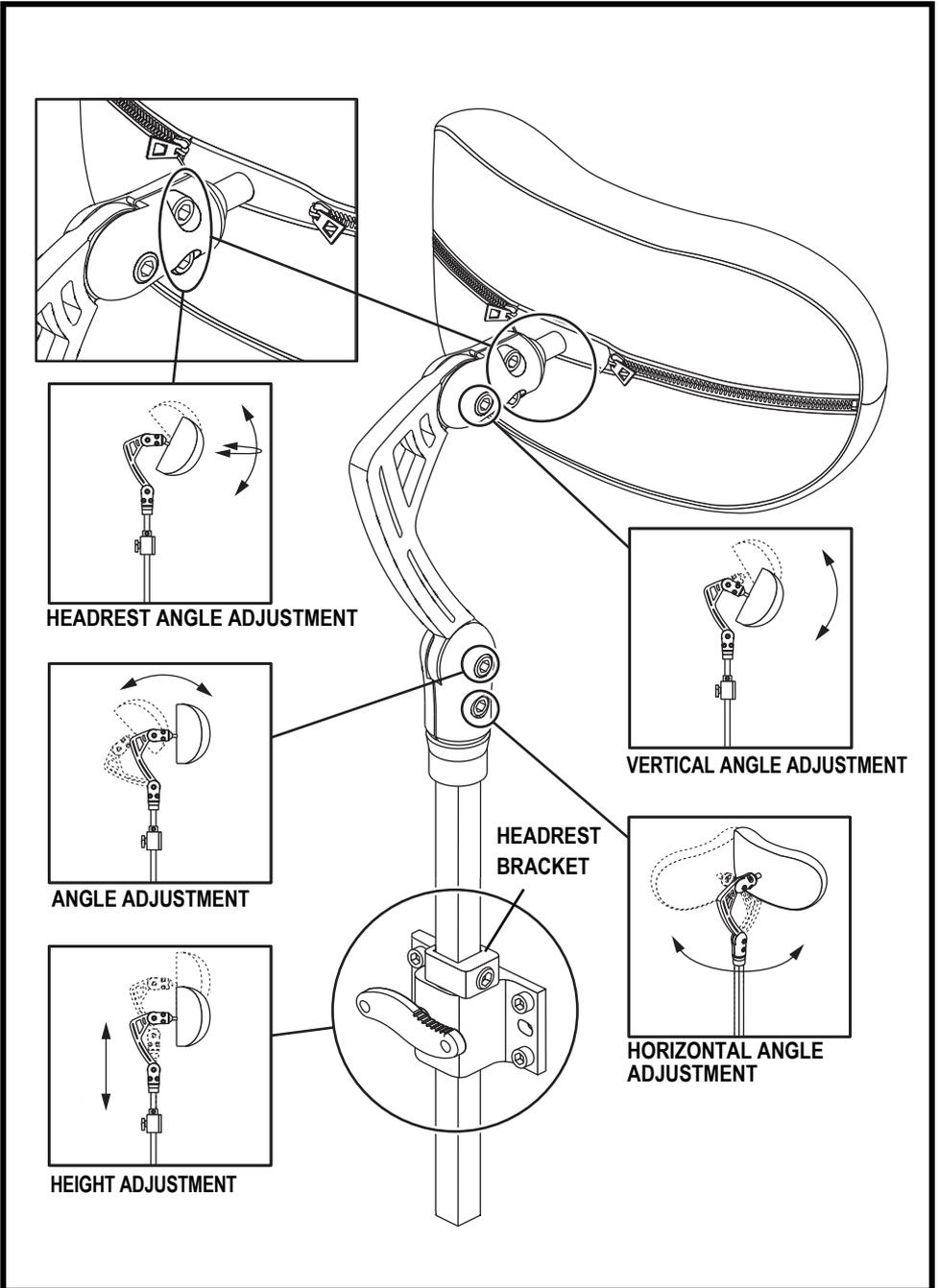
Figure 10. Headrest Adjustment Components

WARRANTY

The Synergy TRU-Comfort cushion is designed and produced to the highest of standards. Our Synergy cushion is guaranteed to be free of defects in material and workmanship for a period of 24 months. If any defect in material or workmanship is found, Pride Mobility Products will repair or replace, at our discretion, the Synergy cushion. This warranty does not apply to cushions abused or misused by the user and deemed such by Pride Mobility Products.

Claims and repairs should be processed through the nearest authorized Pride or Quantum Rehab Provider.

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A Division of Pride Mobility Products® Corp.