


# Synergy TRU-Balance<sup>TM</sup> Power Tilt and Recline Basic Operation Instructions

ACN# 088 609 661

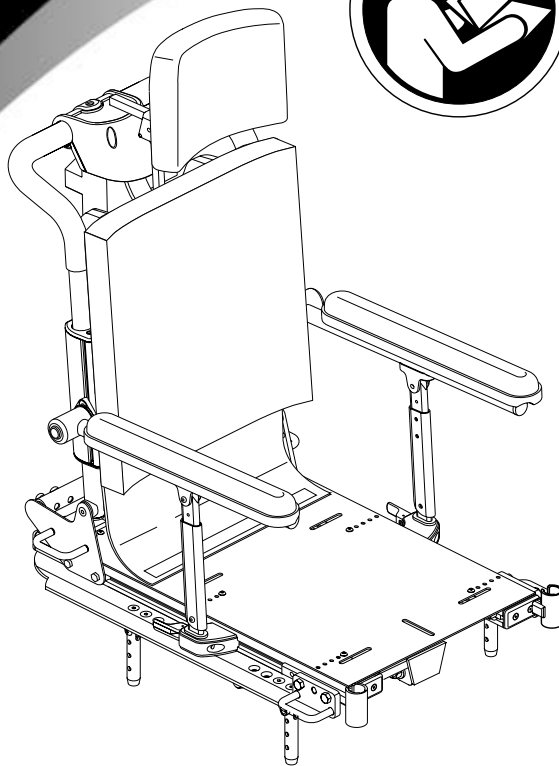

**QUANTUM<sup>®</sup>**  
**REHAB**  
**INNOVATIVE REHAB SOLUTIONS**

A Division of Pride Mobility Products<sup>®</sup> Corp.

## INFORMATION LABELING

The symbols below are used throughout this owner's manual and on the power chair to identify warnings and important information. It is very important for you to read them and understand them completely.



**WARNING!** Failure to follow designated procedures can cause either personal injury, component damage, or malfunction (black symbol on yellow triangle with black border).



**MANDATORY!** These actions should be performed as specified. Failure to perform mandatory actions can cause injury to personnel and/or damage to equipment (white symbol on blue dot with white border).



**PROHIBITED!** These actions are prohibited. These actions should not be performed at any time or in any circumstances. Performing a prohibited action can cause injury to personnel and/or damage to equipment (black symbol with red circle and red slash).



Read and follow the information in the owner's manual.



Avoid exposure to rain, snow, ice, salt, or standing water whenever possible. Maintain and store in a clean and dry condition.



Pinch/Crush points created during assembly.



EMI-RFI - Passed EMI-RFI 20 V/m

## TABLE OF CONTENTS

<b>SAFETY .....</b>	<b>4</b>
<b>THE SYNERGY TRU-BALANCE POWER TILT AND RECLINE..</b>	<b>6</b>
<b>PRECAUTIONARY GUIDELINES .....</b>	<b>6</b>
<b>OPERATING THE SYNERGY TRU-BALANCE POWER TILT AND RECLINE .....</b>	<b>9</b>
<b>POWER SEAT ELEVATOR OPTION .....</b>	<b>17</b>
<b>HIGH-PIVOT POWER ELEVATING LEG RESTS OPTION .....</b>	<b>18</b>
<b>POWER ARTICULATING LEG RESTS OPTION .....</b>	<b>19</b>
<b>SWING-AWAY POWER ARTICULATING LEG RESTS OPTION</b>	<b>20</b>
<b>CONTROLLER POSITION .....</b>	<b>21</b>
<b>REMOVING THE ARMRESTS .....</b>	<b>22</b>
<b>CARE AND MAINTENANCE .....</b>	<b>23</b>
<b>WARRANTY .....</b>	<b>23</b>

## INTRODUCTION

WELCOME to Quantum Rehab, a division of Pride Mobility Products Corporation (Pride). The product you have purchased combines state-of-the-art components with **safety**, comfort, and styling in mind. We are confident that the design features will provide you with the conveniences you expect during your daily activities. Understanding how to **safely** operate and care for this product should bring you years of trouble free operations and service.

**Read and follow** all instructions, warnings, and notes in this manual and all other accompanying literature before attempting to operate this product for the first time. In addition, your **safety** depends upon you, as well as your provider, caretaker, or healthcare professional in using good judgement.

This manual is to be used in addition to the power base owner's manual that came with your power chair. If there is any information in this manual which you do not understand, or if you require additional assistance for setup or operation, please contact your Quantum Rehab Specialist. **Failure to follow the instructions, warnings, and notes in this manual and those located on your Pride product can result in personal injury and/or product damage and will void Pride's product warranty.**

## PURCHASER'S AGREEMENT

By accepting delivery of this product, you promise that you will not change, alter, or modify this product or remove or render inoperable or unsafe any guards, shields, or other safety features of this product; fail, refuse, or neglect to install any retrofit kits from time to time provided by Pride to enhance or preserve the safe use of this product.

## INFORMATION EXCHANGE

We want to hear your questions, comments, and suggestions about this manual. We would also like to hear about the safety and reliability of your new Pride product, and about the service you received from your Quantum Rehab Specialist. Please notify us of any change of address, so we can keep you apprised of important information about safety, new products, and new options that can increase your ability to use and enjoy your Pride product. Please feel free to contact us at the following address:

Pride Mobility Products Corporation  
Attn: Customer Care Department  
182 Susquehanna Avenue  
Exeter, PA 18643-2694

***NOTE: If you ever lose or misplace your product registration card or your copy of this manual, contact us and we will be glad to send you a new one immediately.***

### **PRIDE OWNERS CLUB**

As an owner of a Pride product, you are encouraged to enroll in the Pride Owners Club. Complete and return your enclosed product registration card or visit Pride's web site at [www.pridemobility.com](http://www.pridemobility.com).

From our home page, select “Owners Club” to enter a page dedicated to current and potential Pride product owners. You will gain access to interviews, stories, recreation ideas, daily living tips, product and funding information, and interactive message boards. These message boards invite you to communicate with other Pride customers as well as Pride representatives who are available to assist you with any questions or concerns.

### **My Quantum Rehab Specialist Is:**

Name: \_\_\_\_\_

Address: \_\_\_\_\_

Phone Number: \_\_\_\_\_

Purchase Date: \_\_\_\_\_

## THE TRU-BALANCE POWER TILT AND RECLINE

The TRU-Balance Power Tilt and Recline is a unique seating system designed specifically for the Pride Power Chair. It is fully adjustable to meet the individual needs of the user, and it is mounted to a Pride power base to provide maximum maneuverability.

Reusable hook and loop fasteners are included in your owner's package for use in attaching the seat cushion to the seat base. These fasteners are not intended for use on cushions with anti-skid material, as they may damage the seat cushion.

Figure 1 provides information on the TRU-Balance Power Tilt and Recline components. Use this diagram to familiarize yourself with the function and location of each component before using the TRU-Balance Power Tilt and Recline.

## PRECAUTIONARY GUIDELINES

Before operating the TRU-Balance Power Tilt and Recline, please read the following. These guidelines are provided for your benefit and will aid you in the safe operation of the seating system.

- Turn off the power before you are seated in the seating system.
- Always have assistance when you are being seated in the seating system.
- Follow all of the procedures and heed the warnings as explained in your power chair owner's manual.

WARNING! Your Quantum Rehab Specialist configured your TRU-Balance Seating System to suit your specific requirements. Do not change the original configuration without first contacting your Quantum Rehab Specialist.

WARNING! Should the fittings on your TRU-Balance Seating System become loose, report the problem immediately to your Quantum Rehab Specialist.



WARNING! Your TRU-Balance Seating System is not approved for use as a seat in any vehicle. Use the seats and occupant restraints provided by the manufacturer of the vehicle.

WARNING! Do not exceed the weight capacity listed in your power chair owner's manual or 300 lbs. (136 kg), whichever is less.



WARNING! Traveling to the full recline position could create a pinch point. Keep all body parts away from the lower back cane brackets when reclining.

**Failure to heed the above warnings can result in personal injury and/or equipment damage.**

## Electromagnetic and Radio Frequency Interference (EMI/RFI)



WARNING! Laboratory tests have shown that electromagnetic and radio frequency waves can have an adverse affect on the performance of electrically-powered mobility vehicles.

Electromagnetic and Radio Frequency Interference can come from sources such as cellular phones, mobile two-way radios (such as walkie-talkies), radio stations, TV stations, amateur radio (HAM) transmitters, wireless computer links, microwave signals, paging transmitters, and medium-range mobile transceivers used by emergency vehicles. In some cases, these waves can cause unintended movement or damage to the control system. Every electrically-powered mobility vehicle has an immunity (or resistance) to EMI. The higher the immunity level, the greater the protection against EMI. This product has been tested and has passed at an immunity level of 20 V/m.




WARNING! Be aware that cell phones, two-way radios, laptops, and other types of radio transmitters may cause unintended movement of your electrically-powered mobility vehicle due to EMI. Exercise caution when using any of these items while operating your mobility vehicle and avoid coming into close proximity of radio and TV stations.

WARNING! The addition of accessories or components to the electrically-powered mobility vehicle can increase the susceptibility of the vehicle to EMI. Do not modify your power chair in any way not authorized by Pride.

WARNING! The electrically-powered mobility vehicle itself can disturb the performance of other electrical devices located nearby, such as alarm systems.

***NOTE: For further information on EMI/RFI, go to the Resource Center on [www.pridemobility.com](http://www.pridemobility.com). If unintended motion or brake release occurs, turn your power chair off as soon as it is safe to do so. Call Pride at 800-424-8205 to report the incident.***

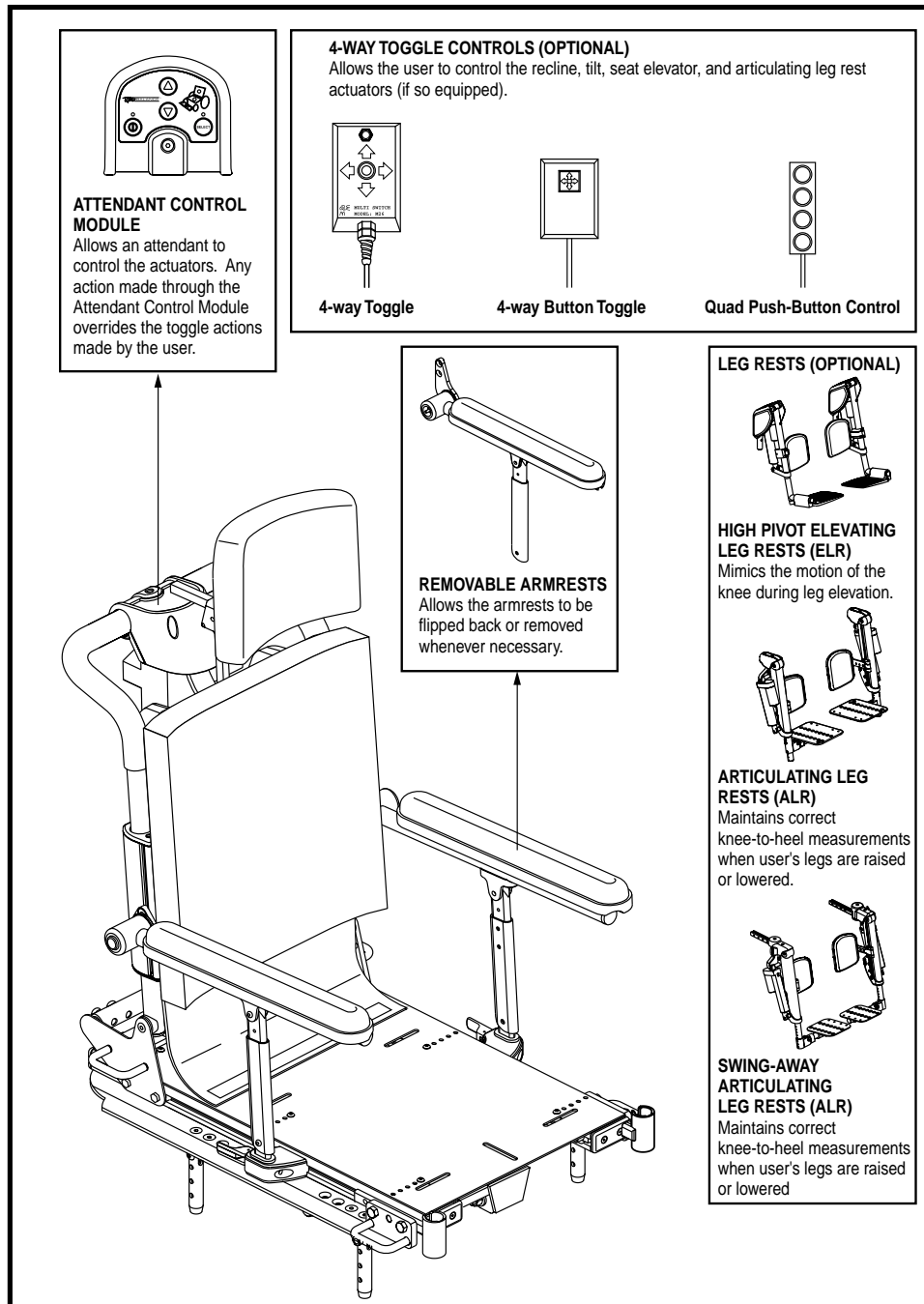

**Figure 1. Components of the Synergy TRU-Balance Power Tilt and Recline**



## OPERATING THE SYNERGY TRU-BALANCE POWER TILT AND RECLINE

The Synergy TRU-Balance Power Tilt and Recline can be operated through a PG Remote Plus controller, a specialty controller, the Attendant Control Module, or a toggle (see figure 4, 5, or 6). Your Synergy System can recline 95° to 168°, can tilt 55°, and can tilt and recline up to 180°. See figure 3. There are inhibits on your Synergy Seating System which will not allow you to drive your power chair or operate certain options when reclined or elevated. See figure 2.




**WARNING!** Check the immediate area before operating the tilt and recline features. Keep clear prior to and during movement. Failure to heed can result in personal injury and /or equipment damage.

**NOTE:** *This manual will discuss toggle, joystick, and Attendant Control Module operations only. If your Synergy Seating System is equipped with a different operating device, please refer to the operation manual supplied with that device and/or your power chair owner's manual.*

	Recline Lock Out	Tilt Lock Out	Elevate Lock Out	1/4 Speed	Full Drive Lock Out
Elevated	x	x		Active	
Tilted more than 5°				Active	
Tilted more than 20°			x		Active
Reclined more than 20°			x		Active
Tilted/reclined more than 170°	x	x	x		Active

**Figure 2. Inhibit Matrix**

**NOTE:** *Your Synergy TRU-Balance Power Tilt and Recline is equipped with safety pressure switches. If any obstruction is detected near the tilt mechanism while you are attempting to tilt or return to a sitting position, the tilt mechanism will automatically stop.*

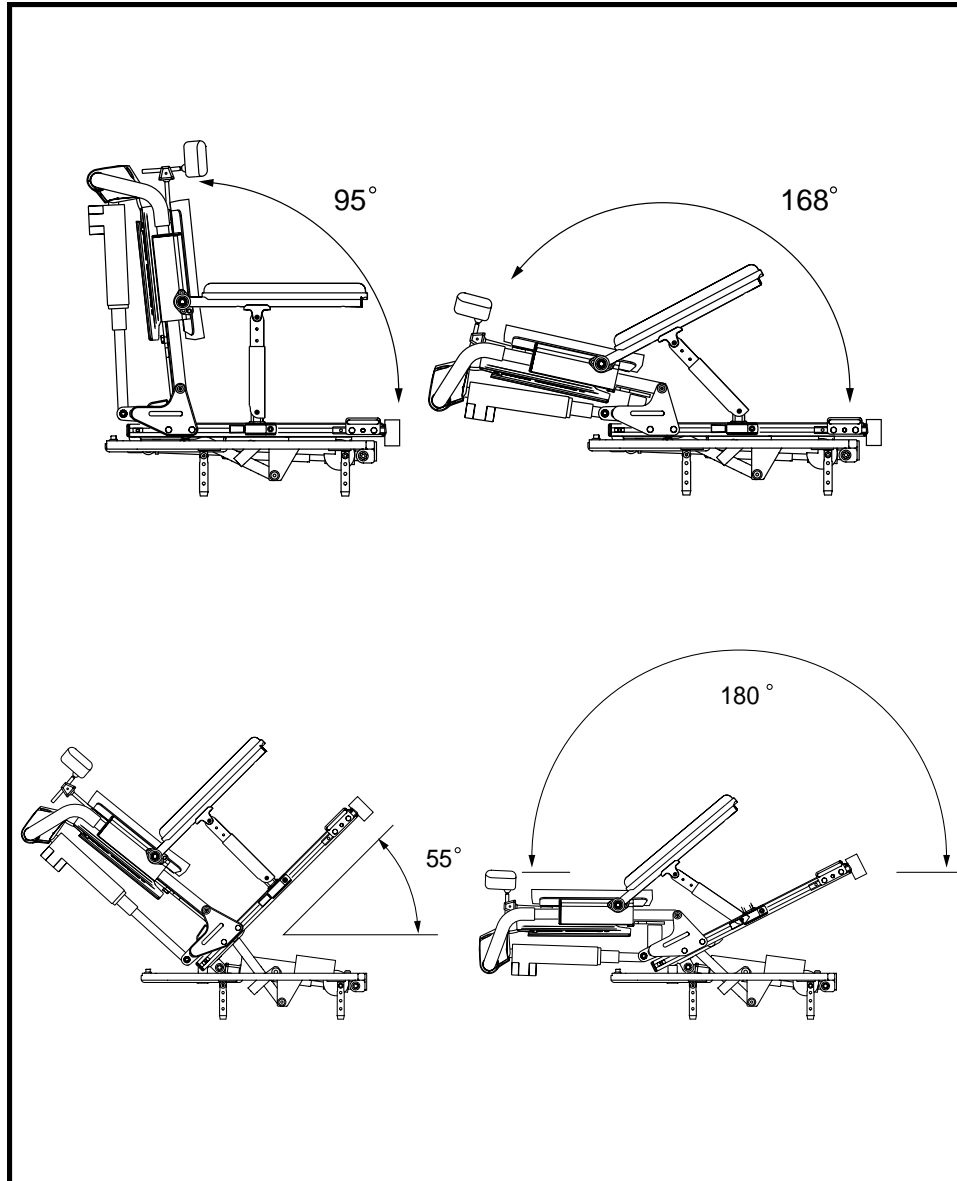

**Figure 3. The Synergy TRU-Balance Power Tilt and Recline Maximum Tilt and Recline Capabilities**

## Toggle Operation

Your Synergy TRU-Balance Power Tilt and Recline may be equipped with one of three available 4-way toggles, as well as an Attendant Control Module. The actuators can be adjusted either by the user through the toggle, or by an attendant through the Attendant Control Module. Any action made through the Attendant Control Module will override toggle actions.

**NOTE:** *Your TRU-Balance seating system may be equipped with any number of these functions.*

### 4-way Toggle

#### ■ Tilt Function

1. To tilt backward, pull the toggle switch backward.
2. To return to a sitting position or to a 0° tilt position, pull the toggle switch backward, release, then pull backward again.

#### ■ Recline Function

1. To recline, push the toggle switch forward.
2. To return to an upright position or a 95° recline position, push the toggle switch forward, release, then push forward again.

#### ■ Seat Elevator Function (Optional)


1. To elevate the seat, push the toggle switch to the left.
2. To lower the seat, push the toggle switch to the left, release, then push to the left again.

#### ■ Combined Leg Rest Function (Optional)

1. To elevate both leg rests, push the toggle switch to the right.
2. To lower both leg rests, push the toggle switch to the right, release, then push to the right again.

#### ■ Independent Leg Rest Function (Optional)

1. To elevate the left leg rest, push the toggle switch to the left.
2. To lower the left leg rest, push the toggle switch to the left, release, then push to the left again.
3. To elevate the right leg rest, push the toggle switch to the right.

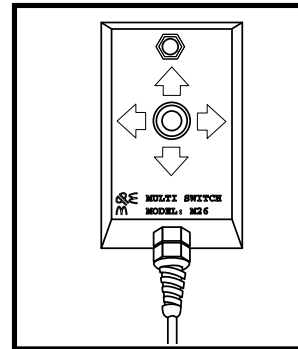

**Figure 4. 4-way Toggle Operation**

4. To lower the right leg rest, push the toggle switch to the right, release, then push to the right again.

**NOTE:** *The power elevator option is not available when the independent leg rest option is selected.*

#### 4-way Button Toggle

##### ■ Tilt Function

1. To tilt backward, pull the toggle button backward.
2. To return to a sitting position or to a 0° tilt position, pull the toggle button backward, release, then pull backward again.

##### ■ Recline Function

1. To recline, push the toggle button forward.
2. To return to an upright position or a 95° recline position, push the toggle button forward, release, then push forward again.

##### ■ Seat Elevator Function (Optional)


1. To elevate the seat, push the toggle button to the left.
2. To lower the seat, push the toggle button to the left, release, then push to the left again.

##### ■ Combined Leg Rest Function (Optional)

1. To elevate both leg rests, push the toggle button to the right.
2. To lower both leg rests, push the toggle button to the right, release, then push to the right again.

##### ■ Independent Leg Rest Function (Optional)

1. To elevate the left leg rest, push the toggle button to the left.
2. To lower the left leg rest, push the toggle button to the left, release, then push to the left again.
3. To elevate the right leg rest, push the toggle button to the right.
4. To lower the right leg rest, push the toggle button to the right, release, then push to the right again.

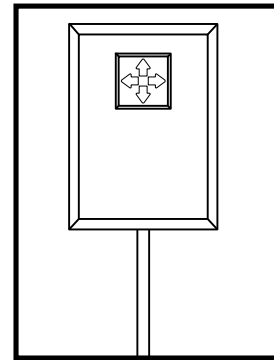

**Figure 5. 4-way Button Toggle Operation**

**NOTE:** *The power elevator option is not available when the independent leg rest option is selected.*

**Quad Push-Button Control Toggle****■ Tilt Function**

1. To tilt backward, push the first button.
2. To return to a sitting position or to a 0° tilt position, push the button again.

**■ Recline Function**

1. To recline, push the second button.
2. To return to an upright position or a 95° recline position, push the second button again.

**■ Seat Elevator Function (Optional)**

1. To elevate the seat, push the third button.
2. To lower the seat, push the third button again.


**■ Combined Leg Rest Function (Optional)**

1. To elevate both leg rests, push the fourth button.
2. To lower both leg rests, push the fourth button again.

**■ Independent Leg Rest Function (Optional)**

1. To elevate the left leg rest, push the third button.
2. To lower the left leg rest, push the third button again.
3. To elevate the right leg rest, push the fourth button.
4. To lower the right leg rest, push the fourth button again.


***NOTE: The power elevator option is not available when the independent leg rest option is selected.***

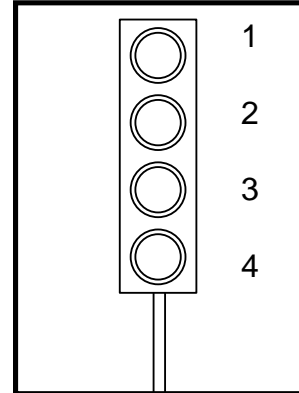

**Figure 6. Quad Push-Button Control Toggle Operation**

### Attendant Control Module

Your Synergy TRU-Balance Power Tilt and Recline is equipped with an Attendant Control Module (see figure 7). The actuators can be adjusted either by the user through the toggle or joystick, and by an attendant through the Attendant Control Module. Any action made through the Attendant Control Module will override toggle or joystick actions made by the user. To adjust any of the actuators, press Select to cycle until the actuator to be adjusted is illuminated on the actuator indicator, then press the up arrow to increase the lift or tilt or the down arrow to decrease the lift or tilt.

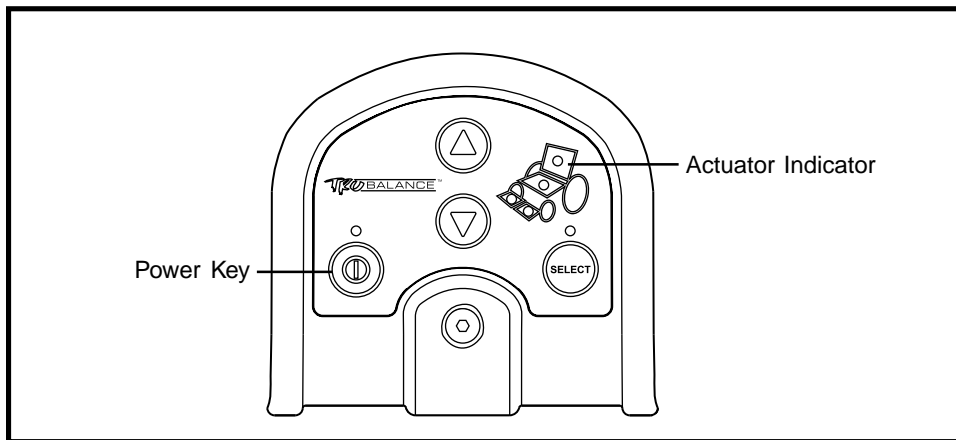

**Figure 7. Attendant Control Module Functions**

**NOTE:** *Your TRU-Balance seating system may be equipped with some or all of these functions.*

#### ■ Tilt Function

1. Press the power key to power on the module.
2. Press Select until the actuators in both the back and seat are illuminated on the actuator indicator on the module. Use the up and down arrows to increase or decrease the degree of tilt.
3. To return to a sitting position or to a 0° tilt position, press Select until the actuators in both the back and seat are illuminated on the actuator indicator on the module, and then press the down arrow until the seat is in a sitting position.

**■ Recline Function**

1. Press the power key to power on the module.
2. Press Select until the actuator on the back is illuminated on the actuator indicator on the module. Use the up and down arrows to increase or decrease the angle of recline.
3. To return to an upright position or a 95° recline position, press Select until the actuator on the back is illuminated on the actuator indicator on the module, and then press the down arrow until the seatback is in the upright position.

**■ Seat Elevator Function (Optional)**

1. Press the power key to power on the module.
2. Press Select until the actuator on the seat is illuminated on the actuator indicator on the module. Use the up and down arrows to increase or decrease the height of the seat.
3. To lower the seat to its original position, press Select until the actuator on the seat is illuminated on the actuator indicator on the module, and then press the down arrow until the seat is lowered to its original position.

***NOTE: The power elevator option is not available when the independent leg rest option is selected.***

**■ Leg Rest Function (Optional)**

1. Press the power key to power on the module.
2. Press Select until the desired leg rest actuator is illuminated on the actuator indicator on the module. Use the up and down arrows to increase or decrease the degree of elevation.
3. To return the leg rest to a lowered position, press Select until the desired leg rest actuator is illuminated on the actuator indicator on the module, and then press the down arrow until the leg rest is in a lowered position.

**■ Independent/Combined Leg Rest Function (Optional)**


1. Press the power key to power on the module.
2. Press Select to cycle through the left, right, and combined leg rests until the desired leg rest actuator is illuminated on the actuator indicator on the module. Use the up and down arrows to increase or decrease the degree of elevation.

3. To return the leg rest to a lowered position, press Select to cycle through the left, right, and combined leg rests until the desired leg rest actuator is illuminated on the actuator indicator on the module, and then press the down arrow until the leg rest is in a lowered position.

**NOTE:** *The combined leg rests functionality is available through the Attendant Control Module.*

### Joystick Operation

1. Press the On/Off key to power on the chair and the controller.
2. Press the Mode key twice to select actuator adjustment mode
3. Push the joystick to the right to cycle through the available actuators until the seatback and/or seat light is illuminated on the actuator indicator. See figure 8.
4. Push the joystick forward to raise the seatback or backward to recline/tilt the seatback.
5. Press the Mode key again to return to the drive mode.

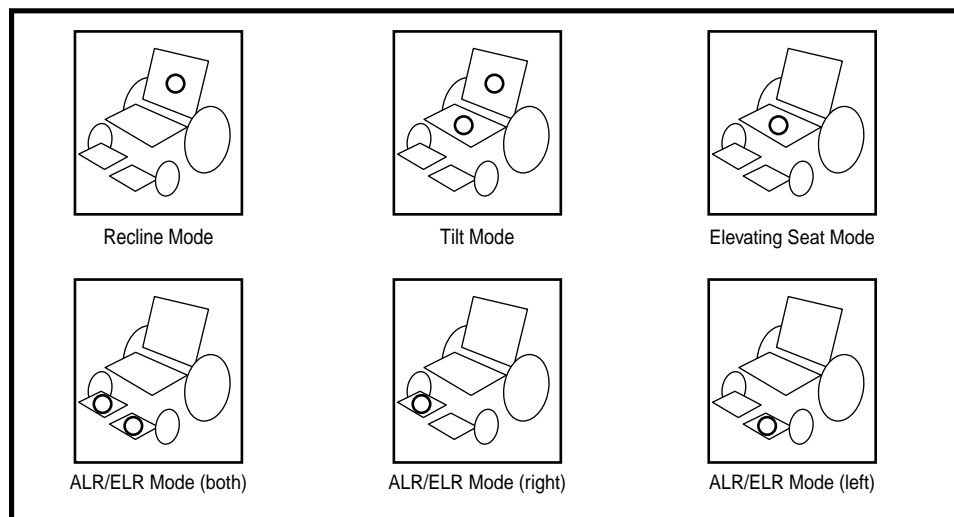

**Figure 8. Actuator Modes**



## POWER SEAT ELEVATOR OPTION


The power seat elevator provides lift of up to 7 in. (18 cm) from the power base using a single post plate and actuator. See figure 9. The maximum weight capacity of the power seat elevator is 300 lbs. (136 kg) and is available on some models. Contact your Quantum Rehab Specialist for specific information. The power seat elevator operates through the power chair controller, toggle switch, or Attendant Control Module.

**NOTE:** When the seat is elevated, a 1/4-speed drive inhibit is engaged and you are no longer able to tilt the seat. See figure 2.

**NOTE:** Operation instructions using the joystick 4-way toggles are discussed below. If you are using a different toggle, the Attendant Control Module, or a different operating device, please refer to the appropriate section of this manual or the operation manual supplied with that device and/or your power chair owner's manual.

### To operate the power seat elevator:

1. Push the toggle switch to the left to elevate the seat.
2. Push the toggle switch to the left, release, then push to the left again to lower the seat.

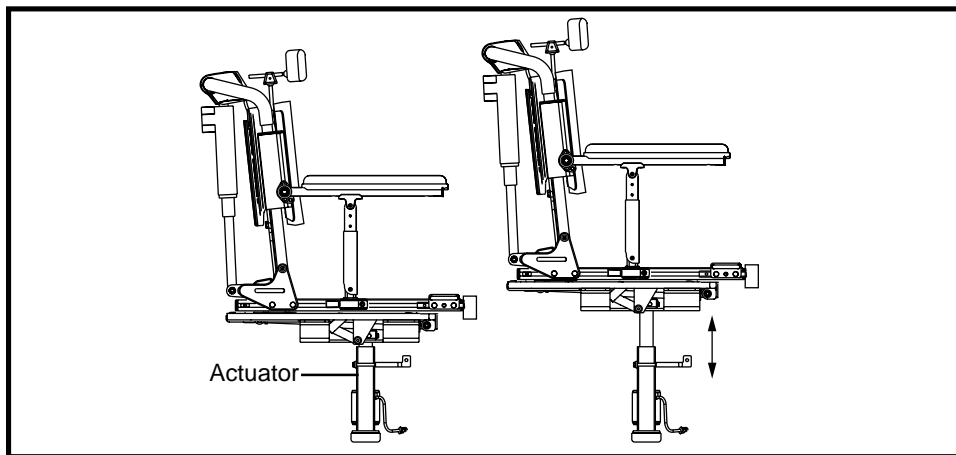

**Figure 9. The Power Seat Elevator Option on a Synergy TRU-Balance Power Tilt and Recline**

## HIGH-PIVOT POWER ELEVATING LEG RESTS OPTION


The high-pivot power elevating leg rests utilize a high pivot point to more closely mimic the motion of the knee during leg rest elevation. They can be adjusted either as a pair or individually. See figure 10. The high-pivot power elevating leg rests provide 70° of movement, have standard swing-away calf pads that are height adjustable, accept all foot plates, and have a maximum weight capacity of 300 lbs. (136 kg). They are available with all power bases that support upgraded electronics and that are available with Synergy Seating Systems. The high-pivot power elevating leg rests operate through the power chair controller or toggle switch.

**NOTE:** *Operation instructions using the joystick 4-way toggles are discussed below. If you are using a different toggle, the Attendant Control Module, or a different operating device, please refer to the appropriate section of this manual or the operation manual supplied with that device and/or your power chair owner's manual.*

**NOTE:** *For information on operating individual power elevating leg rests, please refer to the appropriate section of this manual that refers to the controller you are using or contact your Quantum Rehab Specialist.*

### To operate the combined leg rests:

1. Push the toggle switch to the right to elevate both leg rests.
2. Push the toggle switch to the right, release, then push to the right again to lower both leg rests.

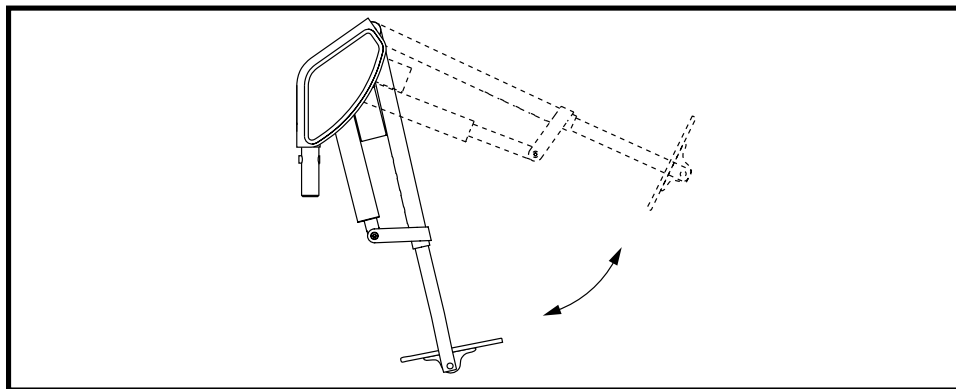

**Figure 10. The High-Pivot Power Elevating Leg Rests Option**

## POWER ARTICULATING LEG RESTS OPTION

The power articulating leg rests maintain the correct knee-to-heel measurements when the user's legs are being elevated and can be adjusted either as a pair or individually. See figure 11. The power articulating leg rests provide 70° or 80° of movement, have standard swing-away calf pads that are depth and height adjustable, accept all foot plates, and have a maximum weight capacity of 300 lbs. (136 kg). They are available with all power bases that support upgraded electronics and that are available with Synergy Seating Systems. The power articulating leg rests operate through the power chair controller, toggle switch, or Attendant Control Module.

**NOTE:** Operation instructions using the joystick 4-way toggles are discussed below. If you are using a different toggle, the Attendant Control Module, or a different operating device, please refer to the appropriate section of this manual or the operation manual supplied with that device and/or your power chair owner's manual.

**NOTE:** For information on operating individual power articulating leg rests, please refer to the appropriate section of this manual that refers to the controller you are using, or contact your Quantum Rehab Specialist.

### To operate the combined leg rests:

1. Push the toggle switch to the right to elevate both leg rests.
2. Push the toggle switch to the right, release, then push to the right again to lower both leg rests.

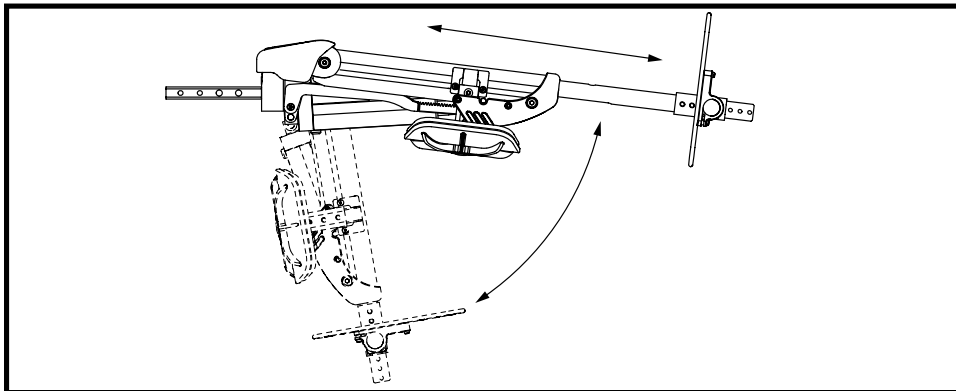


Figure 11. Power Articulating Leg Rests Option

### SWING-AWAY POWER ARTICULATING LEG RESTS OPTION

The swing-away power articulating leg rests are operated the same way and have the same features as the power articulating leg rests but have an added swing-away feature which enables you to rotate the leg rests to the side before you transfer onto or off of your power chair.

#### To rotate the swing-away power articulating leg rests:

1. Push in the release lever. See figure 12.
2. Rotate the leg rest.

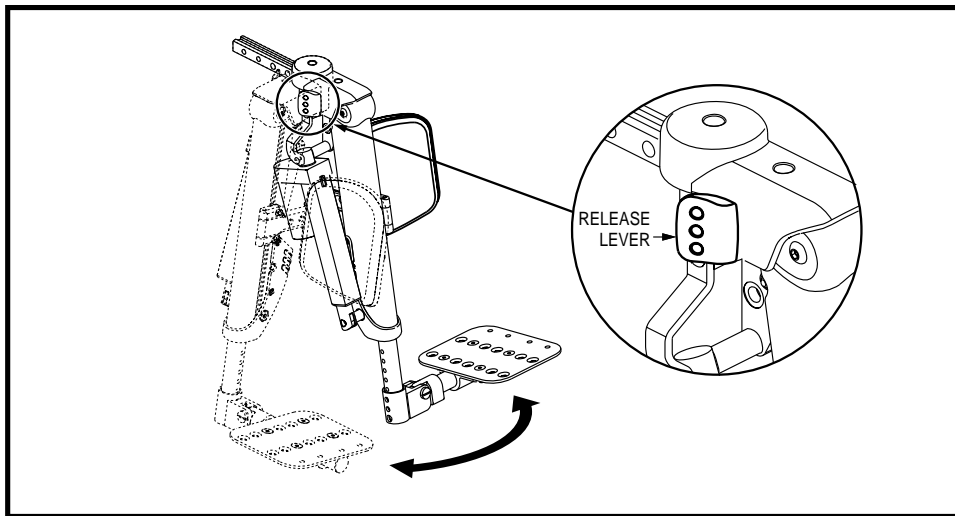


Figure 12. The Swing-away Power Articulating Leg Rests Option

## CONTROLLER POSITION

You can position the controller for either left-hand or right-hand use. You can also adjust the extension of the controller from the armrest.




**WARNING!** Do not place the controller cable so that it can be pinched in the seat frame or the power base frame.

### To change the controller position:


1. Turn off the power to the controller.
2. Unplug the controller connector from the power base. Refer to the power base owner's manual.
3. Remove any wire ties securing the controller cable to the armrest.
4. Loosen the mounting screws in the controller mounting block. See figure 13.
5. Move the controller mounting block and controller to the other armrest and tighten the mounting screws.
6. Route the controller cable to the back of the power base and reconnect the controller. See figure 14.

### To change the controller extension:

1. Turn off the power to the controller.
2. Loosen the setscrews in the controller mounting block. See figure 13.
3. Slide the controller into or out of the armrest to the desired position.
4. Tighten the setscrews to secure the controller.

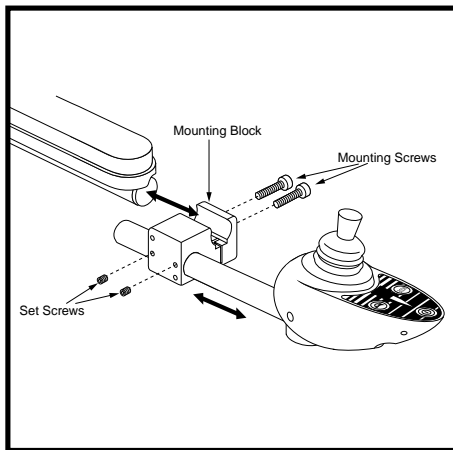

**Figure 13. Controller Position - Mounting Block**

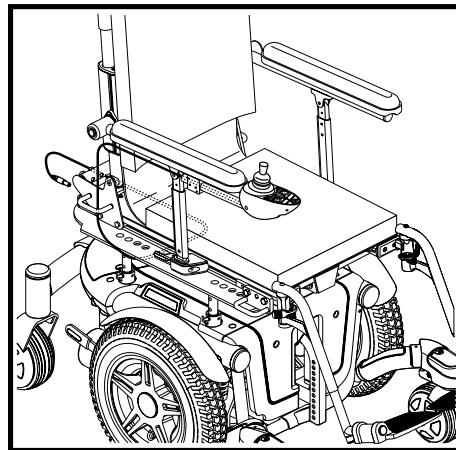

**Figure 14. Cable Routing**

## REMOVING THE ARMRESTS

The armrests on your Synergy TRU-Balance Power Tilt and Recline are unique to this model and can only be adjusted by your Quantum Rehab Specialist. However, you can remove the armrests if necessary.

### To remove the armrests:

1. Pull the armrest lock lever on the seat base toward you. See figure 15.
2. Press the button on the side tube. See figure 15.
3. Pull up and out on the armrest to remove it from the seat. See figure 16.

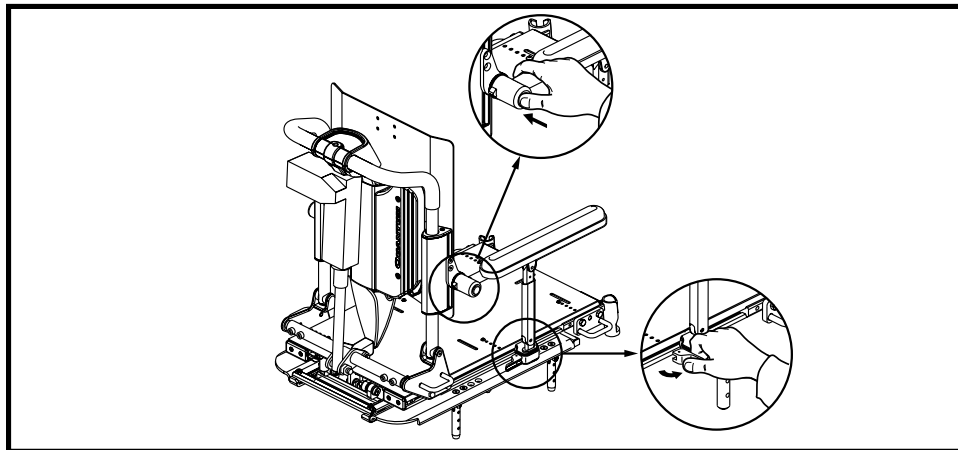


Figure 15. Armrest Unlock

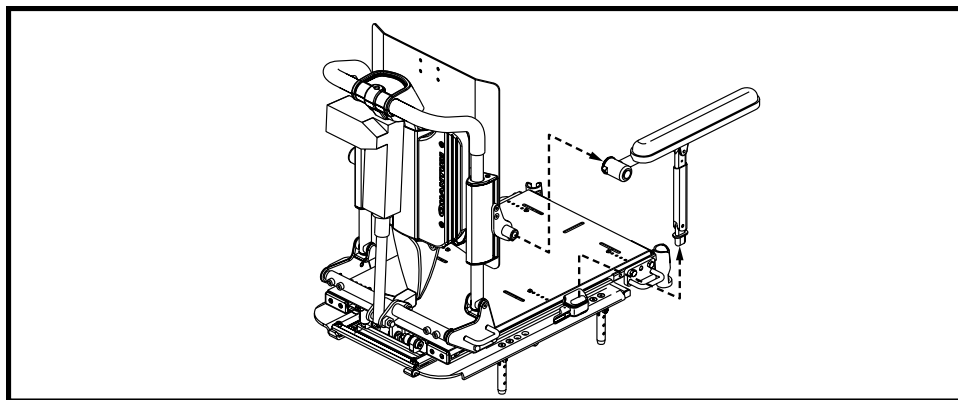


Figure 16. Armrest Removal

## CARE AND MAINTENANCE

- Make sure all hardware is secured properly, but do not overtighten any hardware.
- To clean the seating system, wipe it with a cloth dampened with mild soap and water. Thoroughly dry the unit before using.



**MANDATORY!** Avoid exposure to rain, snow, ice, salt, or standing water whenever possible. Maintain and store in a clean and dry condition.

**Failure to heed can cause personal injury and/or equipment damage.**

## WARRANTY

### Three-Year Limited Warranty

For three (3) years from the date of purchase, Pride will repair or replace at our option to the original purchaser, free of charge, any of the following parts found upon examination by an authorized representative of Pride to be defective in material and/or workmanship:

- Structural frame components

### Two-Year Limited Warranty

For two (2) years from the date of purchase, Pride will repair or replace at our option to the original purchaser, free of charge, any of the following parts found upon examination by an authorized representative of Pride to be defective in material and/or workmanship:

- Electronics

### Eighteen-Month Warranty

For eighteen (18) months from the date of purchase, Pride will repair or replace at our option to the original purchaser, free of charge, any of the following parts found upon examination by an authorized representative of Pride to be defective in material and/or workmanship:

- Actuator

Warranty service can be performed by an authorized Pride Provider or by Pride. Do not return faulty parts to Pride without prior consent. All transportation costs and shipping damage incurred while submitting parts for repair or replacement are the responsibility of the original purchaser.

**Warranty Exclusions**

- Upholstery and seating
- Repairs and/or modifications made to any part without specific consent from Pride
- Circumstances beyond the control of Pride
- Labor, service calls, shipping, and other charges incurred for repair of the product, unless specifically authorized by Pride.
  
- Damage caused by:
  - Battery fluid spillage or leakage
  - Abuse, misuse, accident, or negligence
  - Improper operation, maintenance, or storage
  - Commercial use or use other than normal

There is no other express warranty.

Implied warranties, including those of merchantability and fitness for a particular purpose, are limited to one (1) year from the date of purchase and to the extent permitted by law. Any and all implied warranties are excluded. This is the exclusive remedy. Liabilities for consequential damages under any and all warranties are excluded.

Some states do not allow limitations on how long an implied warranty lasts or do not allow the exclusion of limitation of incidental or consequential damages, so the above limitation or exclusion may not apply to you.

This warranty gives you specific rights, and you may also have other rights which vary from state to state.

Please fill out and return the product registration card to Pride Mobility Products Corporation. This will aid Pride in providing the best possible technical and customer service.

***NOTE: If you ever lose or misplace your product registration card or this manual, contact us and we will be glad to send you a new one immediately.***



## NOTES

## NOTES

## NOTES

# **QUANTUM®**

---

## **REHAB**

---

### **INNOVATIVE REHAB SOLUTIONS**

A Division of Pride Mobility Products® Corp.

[www.pridemobility.com](http://www.pridemobility.com)  
[www.prideservice.com](http://www.prideservice.com)  
[www.quantumrehab.com](http://www.quantumrehab.com)

**Pride Mobility Products Corporation**  
182 Susquehanna Avenue  
Exeter, PA 18643-2694  
USA

**Pride Mobility Products Company**  
380 Vansickle Road Unit 350  
St. Catharines, Ontario L2R 6P7  
Canada

**Pride Mobility Products Ltd.**  
Unit 106, Heyford Park Camp Road  
Upper Heyford, Oxfordshire OX25 5HA

**Pride Mobility Products Australia Pty. Ltd.**  
21 Healey Road  
Dandenong, 3175  
Victoria, Australia

**Pride Mobility Products Italia S.r.l.**  
Via del Progresso - ang. Via del Lavoro  
Loc. Prato della Corte  
00065-Fiano Romano (RM)

**Pride Mobility Products Europe B.V.**  
Tijnmuiden 28  
1046 AL Amsterdam  
The Netherlands

\* I NFMANU2397 \*

**Pride**