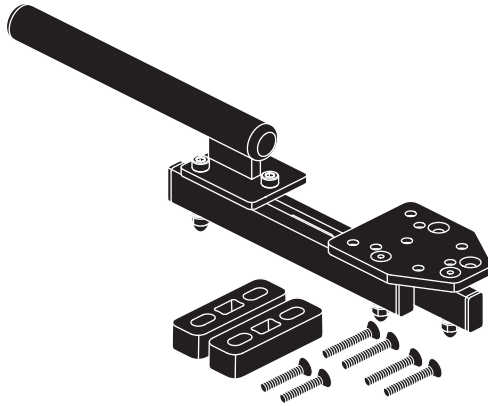
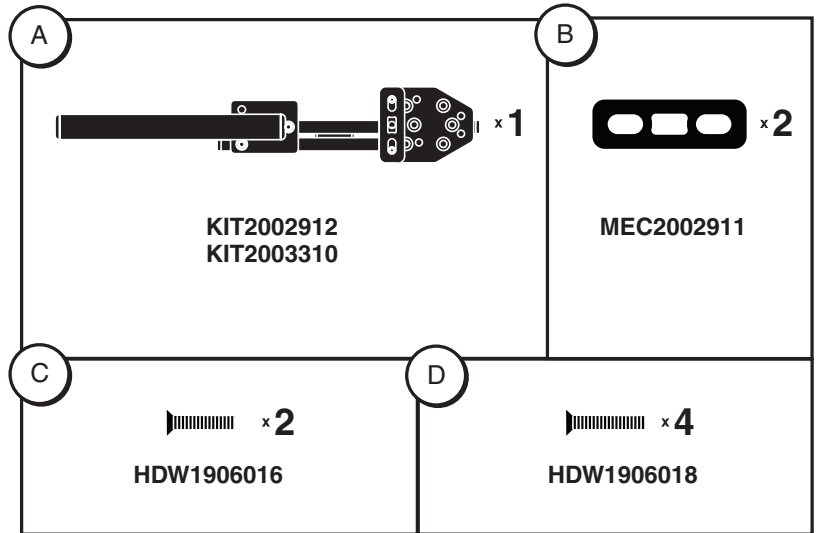
**Swing-away Joystick Bracket • Support de manette pivotant
Draaibare joystickbeugel • Swing-away Joystickhalterung
Staffa joystick swing-away • Soporte de palanca de mando abatible**

Contents • Contenu • Inhoud
Inhalt • Contenido • Contenido

KIT2002912
KIT2003310

Tools needed:

- 3 mm hex key
- Phillips screwdriver

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- Clés hexagonale de 3 mm
- Tournevis cruciforme

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- Llave hexagonal de 3 mm
- Destornillador Phillips

If you require any assistance with these instructions, please call **Pride Technical Service** at **1-877-800-1248**. **Canada providers**, phone **1-888-570-1113**.

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
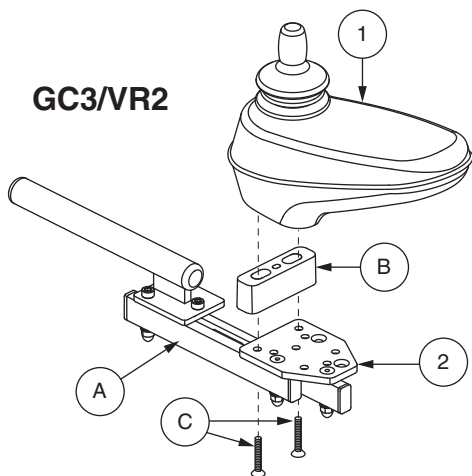
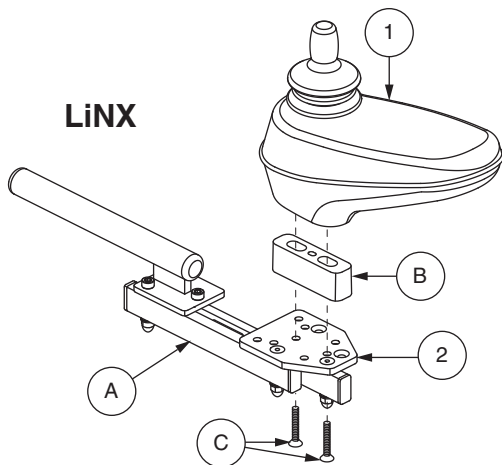
DYNAMIC

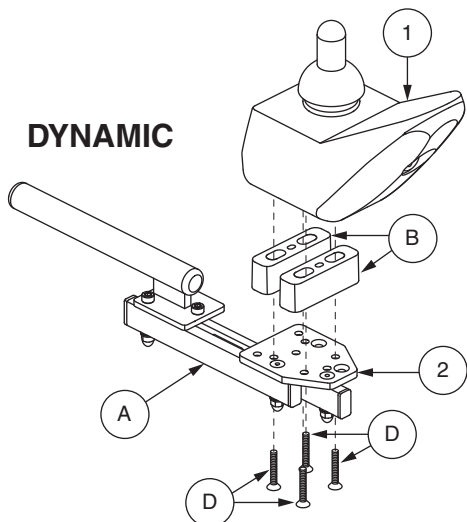To install the swing-away bracket (KIT2002912 or KIT2003310):

Turn off the power to the controller. Ensure the power chair is in drive mode. Loosen the setscrew(s) that secure the standard controller mounting bracket to the seat armrest and slide the bracket out. See owner's manual. Remove the screws from the standard controller bracket that holds the controller (1) in place.

Align the controller (1) and the spacer(s) (B) onto the new swing-away controller bracket (A) as shown, depending on your controller model. The holes through which you secure the controller (1) and spacer(s) (B) to the new bracket are determined by the controller that is being installed. Reference the images shown to determine which holes you must use to secure your controller (1) and spacer(s) (B) to the adapter plate (2). Once determined, use the correct bracket mounting screws (C or D) to secure the controller (1) and spacer(s) (B) to the new swing-away controller bracket (A). Place the fully assembled swing-away controller bracket into the seat arm and tighten the setscrew(s).

NOTE: Each controller uses a unique combination of spacer(s) and screws, which will result in leftover/unused hardware once installation is complete. Ensure you are using the correct screw type and number of spacers for your controller model.

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