
Sitting Pretty

POWER RECLINERS

325 lbs.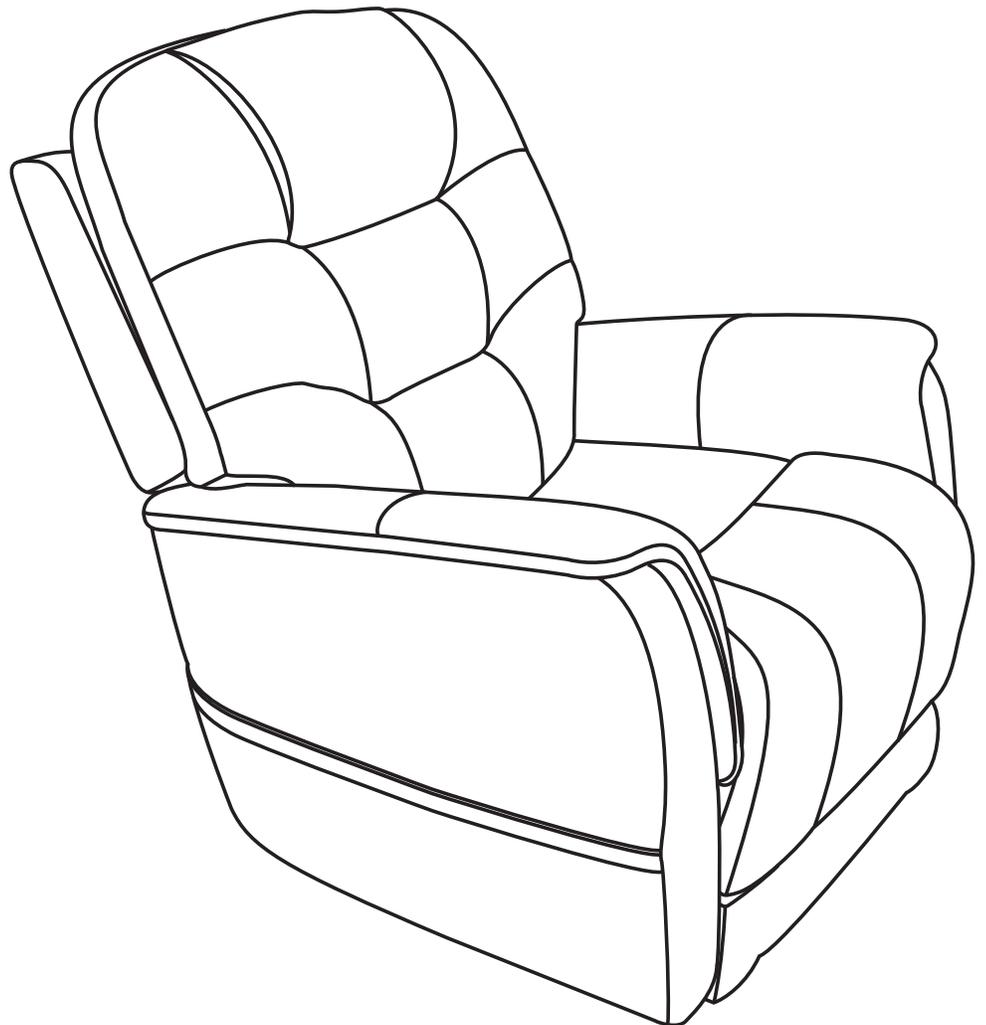

Included in this document: Hand Control Reference Guide, Fabric Care, Chair Assembly and Setup, Warranty, and Weight Capacity

ACN 088 609 661

CH REP

NOTE: This quick reference guide offers basic hand control functions, general care instructions, basic assembly information, warranty information, and weight capacity information for your chair. Before operating the chair, thoroughly read the owner's manual and all other supplemental documents for information on safety, basic operation, and care and maintenance.

The illustrations reflect those hand controls with the highest level of functionality.

The premier feature of the Sitting Pretty Power Recliner is that you have the ability to adjust your feet and back to achieve your perfect position. For example, the seatback can recline until it is horizontal to the floor.

HAND CONTROL IDENTIFICATION KEY

1. UP BUTTON

Press and hold to return all chair power options to the neutral position, while lifting you into an upright position.

2. DOWN BUTTON

Press and hold to lower chair to seated or reclined position.

Follow these steps to properly stand up from your chair:

1. Activate and hold the UP button (switch) to raise your chair.
2. Release the button (switch) when the chair reaches a height where you can stand up comfortably.

WARNING! Do not attempt to stand up until you can stand safely, are steady on your feet, and can bear weight. Pride recommends the use of assistance aids and/or an attendant for enhanced stability.

WARNING! Avoid putting all of your weight on the chair armrests when attempting to stand and do not use the armrests for weight bearing purposes.

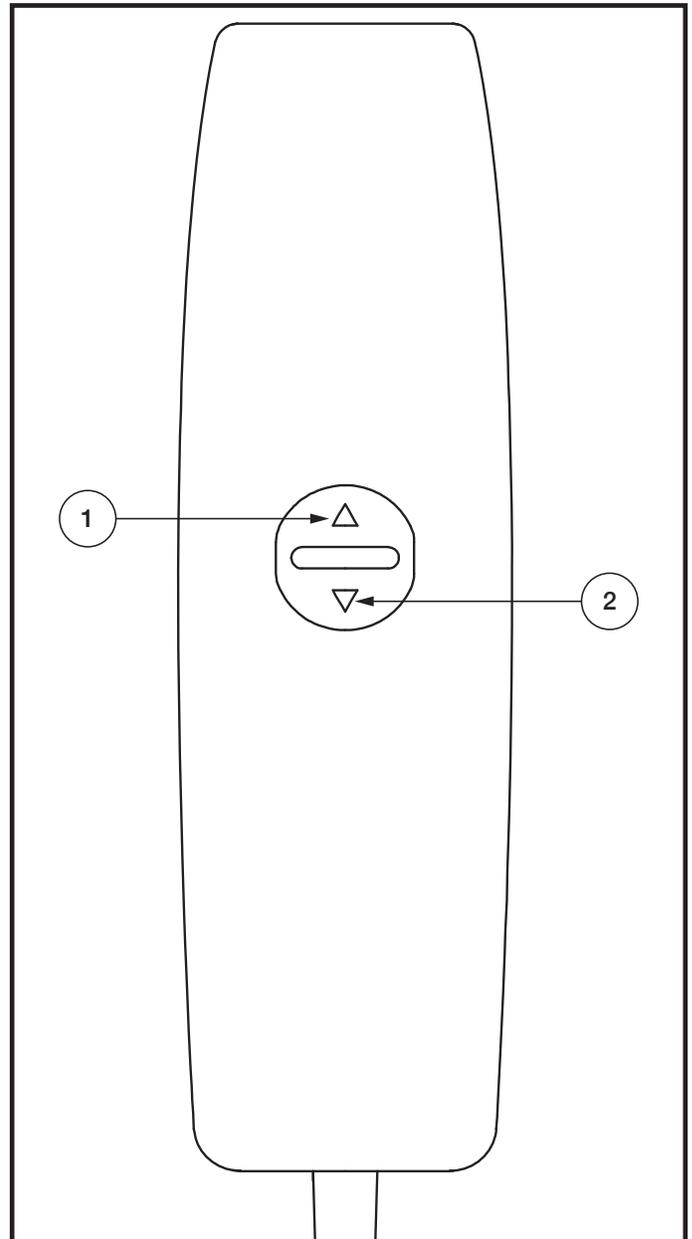
Figure 1. Hand Control

FABRIC CARE INSTRUCTIONS

NOTE: Refer to the Care and Maintenance section of the owner's manual supplied with your chair for complete fabric care, electronics care, and disposal and recycling information.

CHAIR ASSEMBLY AND SETUP

1

Unpack the chair, then lay the chair back, face down, on the chair base.

2

1. Check that the locking clip on the back of the chair is in the vertical position. Reposition if necessary.


NOTE: Depending on chair width, you may need to slide on one side of the back at a time. If this is the case, level the chair-back before pressing down to lock into position.

2. Push firmly on the top of the back to lock into position. You must hear a click from each side of the chair to ensure the back is locked in properly.

NOTE: Pull up on the chair-back to ensure that it is locked in place.

3. Plug in the lumbar and headrest connectors. If your model is Headrest Only, you only need to plug in the Headrest.

3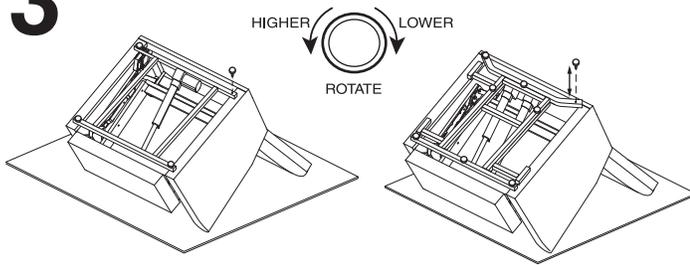

With the chair upside down, insert the leg levelers if not pre-installed, and tighten each leveler evenly. Return the chair to the upright position. If your chair is rocking, adjust the levelers as necessary (tighten to lower or loosen to raise) until the chair is stable.


NOTE: Depending on your kit and chair model, you may have four leg levelers or eight leg levelers.

4

TIP: To ensure quality delivery of your new chair, we have developed a customized packaging system to guard against possible damage that may occur during the transportation and handling of your product. As a result, the packaging components used to protect the product could cause the fibers in the cushioning to appear misshaped upon unpacking. In the event that you experience this, we recommend that you “fluff” the chair cushions to allow air into the cushion fibers. This process will return the cushions to their original shape.

5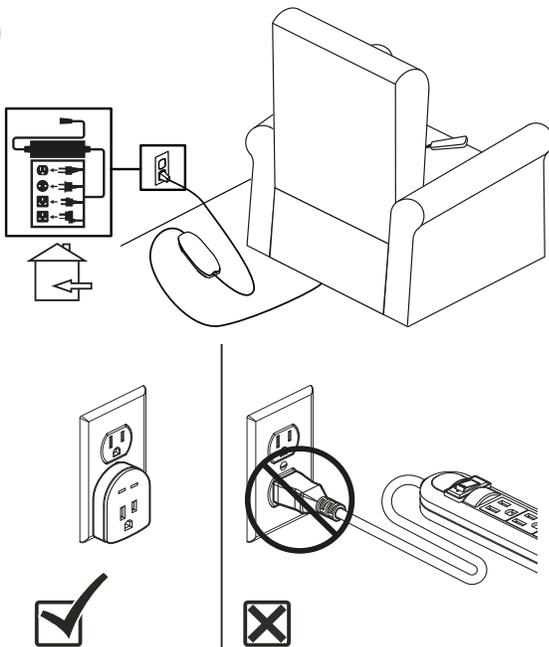

Plug the power cord leading from the external transformer directly into a standard electrical outlet. Route the power cord through the back of the chair. Do not use an extension cord.

WARNING! Do not connect an extension cord to the AC/DC converter or the battery charger. A UL Listed or UL Recognized single outlet plug-style surge protector with a minimum of 1000 joules of protection may be used. The surge protector **MUST** be plugged directly into an electrical outlet.

TWO-YEAR LIMITED WARRANTY

For two (2) years from the date of purchase, Pride will repair or replace at our option to the original purchaser any of the following parts found upon examination by an authorized representative of Pride to be defective in material and/or workmanship. Any warranty claims for electronic assemblies may become void if damage results from the use of parts/accessories that are not Pride original parts and are not purchased through an authorized Pride provider:

- Motor
- Hand Control
- Transformer/Control Boxes
- Scissor Mechanisms
- Lift Frame

Your chair was developed for your convenience. Your chair has a weight capacity of 325 lbs. (147 kg). Refer to the owner's manual for any additional information concerning your chair.

MANDATORY! Stay within the specified weight capacity of your lift chair. Exceeding the weight capacity voids your lift chair warranty.

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