If your single motor lift chair is equipped with a heat and massage hand control, please read the instructions below for operation. Refer to the owner's manual for any additional information on your lift chair.

Heat Buttons: Press the appropriate HEAT button(s) to activate the heat function in either the back, the base, or both. Press the HEAT button(s) again to cycle intensity through LOW (1), HIGH (2) and OFF (3). **See figure 1.**

Lock/Unlock Button: Press and hold the LOCK/UNLOCK button for approximately 5 seconds to lock the hand control. The center LOCK button will illuminate indicating the hand control is locked. Press and hold the LOCK/UNLOCK button for approximately 10 seconds to unlock the hand control. The hand control buttons/LEDs will all flash indicating it is no longer locked and the LOCK button will no longer be lit. **See figure 1.**

Massage Button: Press the MASSAGE button once to activate the massage function. This will automatically set the massage to the wave setting. Press the MASSAGE button again to go to the next of three available massage settings: wave, steady, or pulse. Each press of the button will take you to the next setting. Each setting is indicated by a colored LED. Press and hold the MASSAGE button for approximately 2 seconds to shut off the massage function, otherwise this function will shut off automatically after 20 minutes of continuous use. **See figure 1.**

Intensity Button: Press the INTENSITY button once to set the massage option to low. Press the INTENSITY button again to set the massage option to the next of three available settings: low, medium, or high. Each press of the button will take you to the next setting. Each setting is indicated by a colored LED. **See figure 1.**

Up Button: Press and hold the UP button to raise your lift chair from the seated position or to raise the chair back and lower the footrest from the reclined position. **See figure 1.** A green LED will indicate the operation is in process.

Down Button: Press and hold the DOWN button to lower the lift chair from the standing position or to recline the chair back and elevate the footrest from the seated position. **See figure 1.**

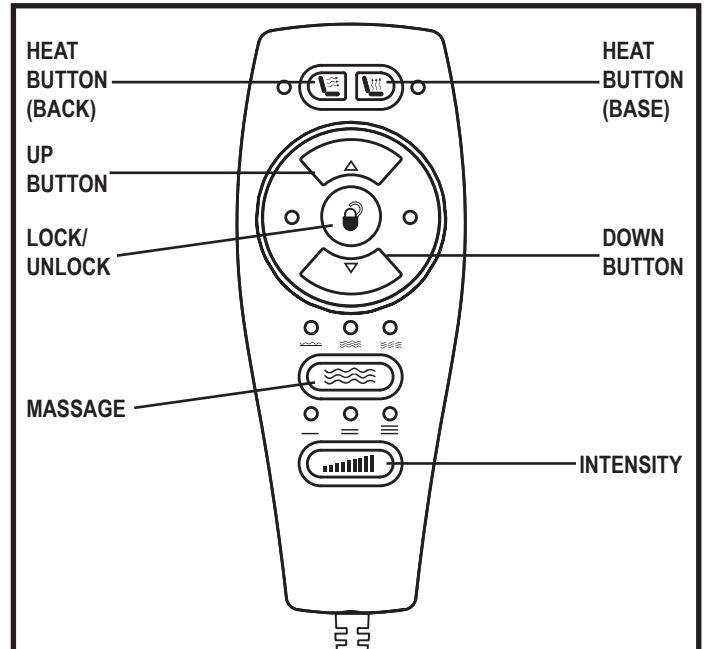
Figure 1. Single Motor Heat and Massage Hand Control

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