NOTE: The following is in addition to the information contained in the chair owner's manual. It explains how to initialize and program your memory button hand control prior to chair use. Read the owner's manual and all other supplemental information thoroughly before operating the chair for information on safety, basic operation, and care and maintenance.

Memory Button Hand Control Initialization

Your memory button hand control will need to be initialized when setting your chair up for the first time.

To initialize your hand control:

1. Assemble your chair. Refer to the "Assembly/Disassembly" section in your owner's manual.
2. Plug the power cord directly into an electrical outlet.
3. Locate the Recline Button and the Footrest Up Button on your memory button hand control. **See figure 1.**
4. Press and hold the Recline Button and the Footrest Up Button until your chair is in the full horizontal position. **See figure 2.**
5. Your memory button hand control and chair are now initialized for use.

Memory Button Hand Control Programming

Your memory button hand control can be set to four (4) individual settings.

1. Adjust your chair to the desired position.
2. Press and hold Memory Button 1 and Memory Button 2 for three (3) seconds. The LEDs on the memory buttons will flash to indicate that the hand control is in "set" mode.
3. Press and hold the desired memory button for three (3) seconds to set the memory position. The LEDs will stop flashing.
4. To go to a preset memory position, press and hold the desired memory button.

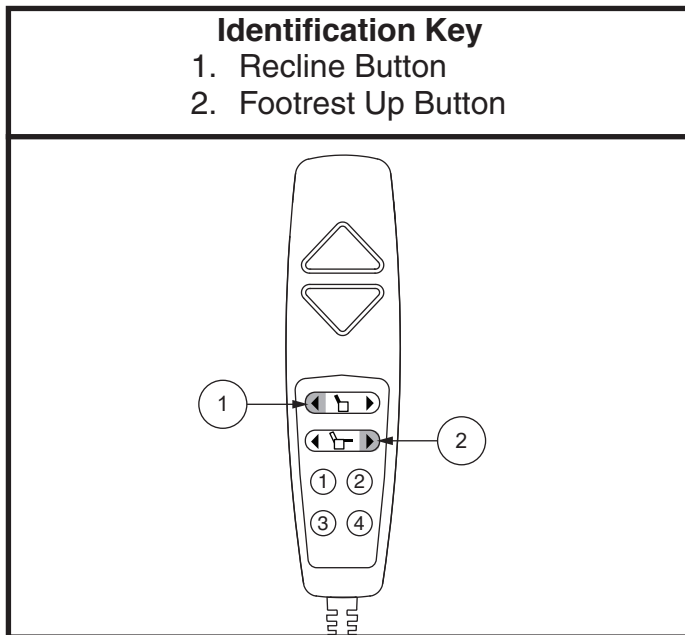
Figure 1. Memory Button Hand Control

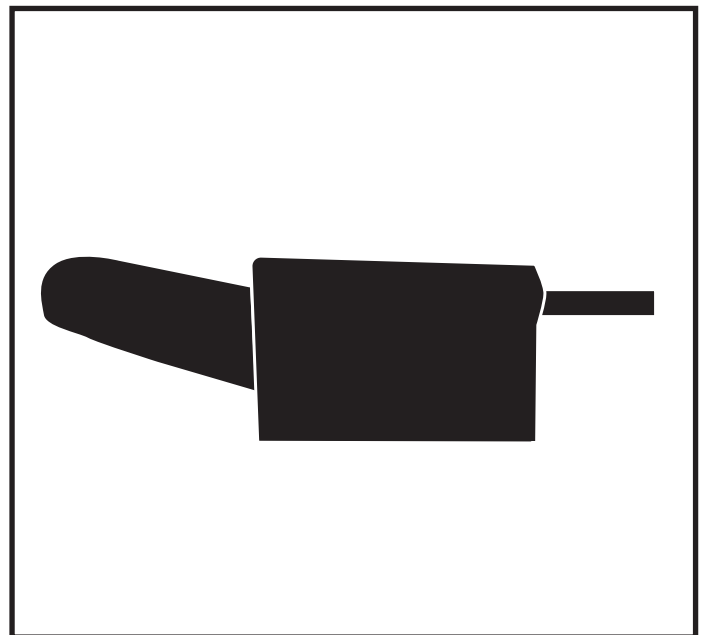
Figure 2. Lift Chair in Full Horizontal Position

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