NOTE: The following information is in addition to the information in the owner's manual provided with your power lift recliner. It explains how to use the wireless charger. You must read the owner's manual and all supplemental information before use. Please retain this information for your records, and contact your provider with any questions.

WIRELESS CHARGER

The wireless charger was developed to conveniently charge your smartphone. Ensure the screen of your smartphone is facing up when it's placed on the charger as shown in **figure 1**.

NOTE: The wireless charger is only compatible with smartphones which have wireless charging capabilities. Additionally, no other smart devices, (i.e., smartwatches) should be used with the wireless charger.

NOTE: Your smartphone must lie flat on the charging surface. You may need to remove your phone case in order to properly position your smartphone for charging.

WARNING! Your smartphone may shift or fall out of the charger when the chair is raised. Remove your smartphone from the wireless charger prior to elevating your chair above a seated position. Failure to do so may result in personal injury and/or property damage.

WARNING! Do not leave your smartphone or any other item in the retractable arm when closing it. Failure to remove items before closing the retractable arm may result in personal injury and/or product damage.

WARNING! NEVER place any item other than a smartphone on the wireless charger. NEVER place any liquid, food, ashtrays, or lit tobacco products in the wireless charger. Doing so may result in personal injury or product damage.

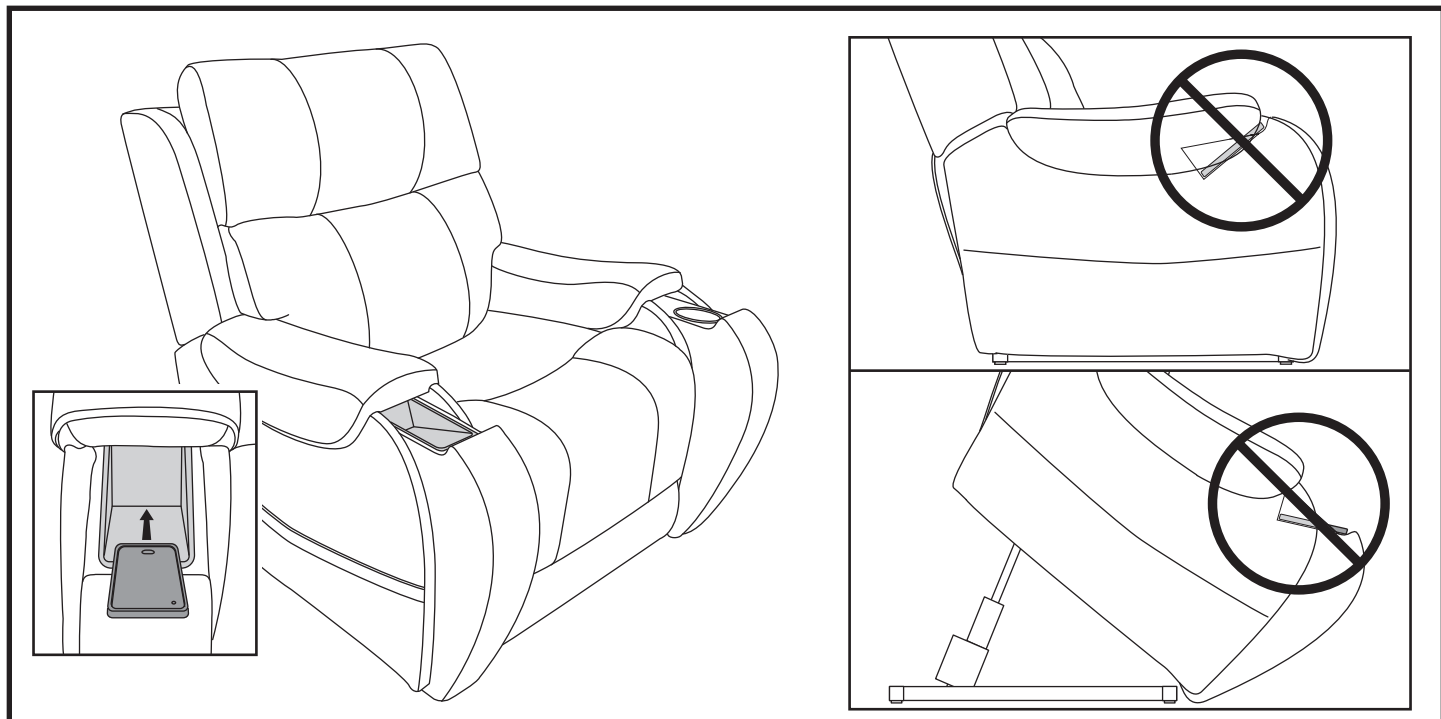
Figure 1. Charging your Smartphone on the Wireless Charger

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