NOTE: The following information replaces the information contained in the Scooter Series Owner's Manual provided with your scooter. It describes information on throttle control lever operation. Please retain this information with the scooter owner's manual and contact your authorized Pride Provider with any questions.

TILLER CONSOLE

The tiller console houses all of the controls needed to drive your scooter, including the speed adjustment dial, key switch, battery condition meter, horn buttons, turn signal buttons and the throttle control lever. Please note that some of the features listed in this section are optional items and may not appear on your scooter. Contact your authorized Provider for more information. See figure 1 or 2.

PROHIBITED! Do not expose the tiller console to moisture. In the event that the tiller console does become exposed to moisture, do not attempt to operate your scooter until the tiller console has dried thoroughly.

IDENTIFICATION KEY FOR FIGURES 1 AND 2

- | | |
|------------------------------|-----------------------------|
| 1. HORN BUTTON | 7. THROTTLE CONTROL LEVER |
| 2. *LEFT TURN SIGNAL BUTTON | 8. SPEED ADJUSTMENT DIAL |
| 3. *HAZARD LIGHTS SWITCH | 9. BATTERY CONDITION METER |
| 4. LIGHT SWITCH | 10. KEY SWITCH |
| 5. *RIGHT TURN SIGNAL BUTTON | 11. MIRROR AND MIRROR PLUGS |
| 6. *HI-LOW SWITCH | 12. *HANDBRAKE LEVER |

*OPTIONAL COMPONENTS AVAILABLE ON SOME MODELS

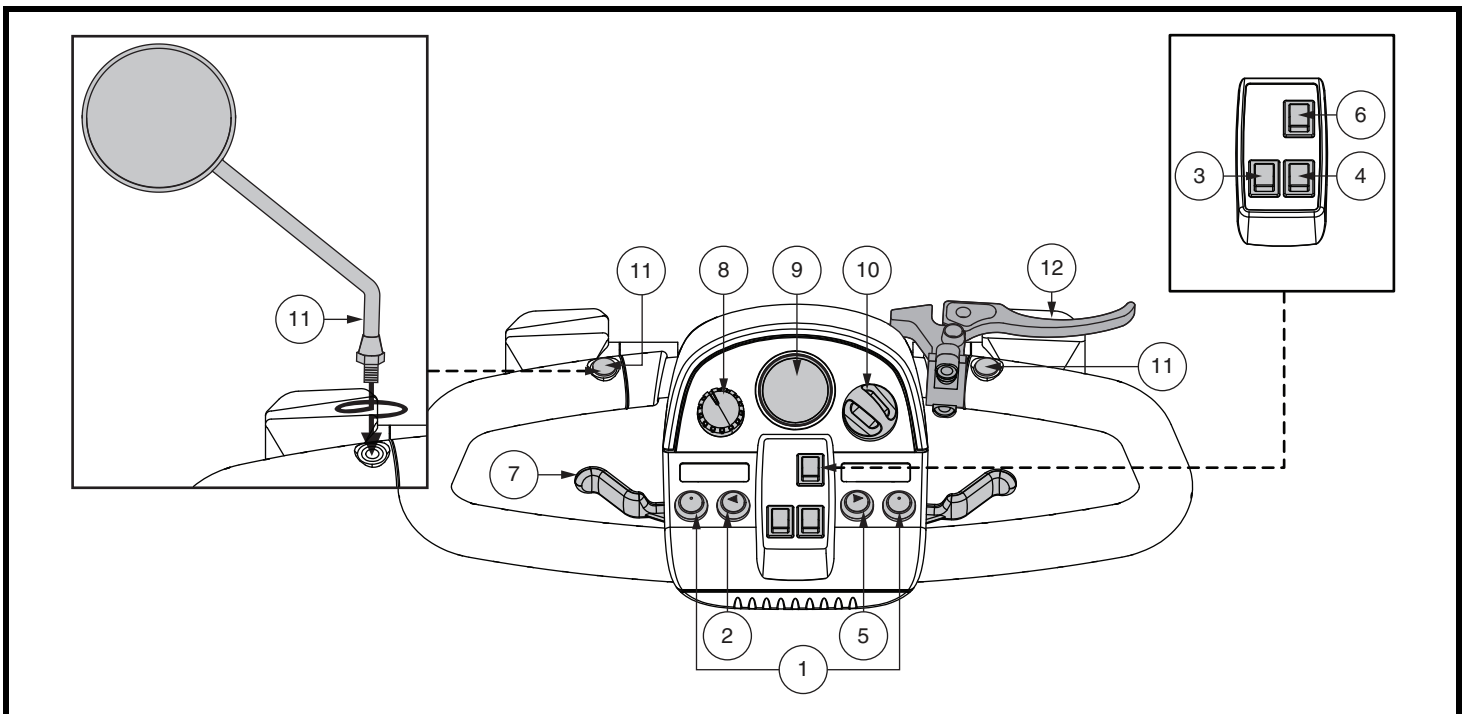
Figure 1. Tiller Console A

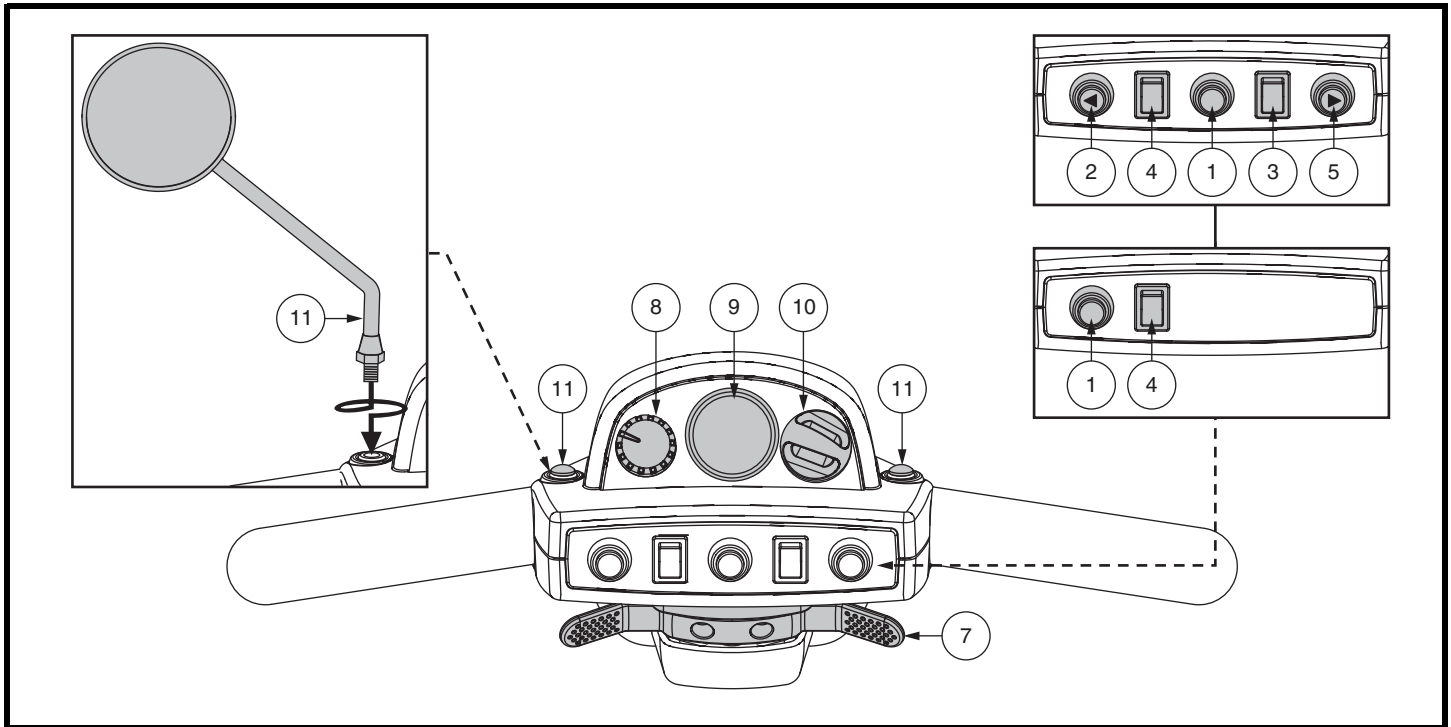
Figure 2. Tiller Console B

Throttle Control Lever

This lever allows you to control the forward speed and the reverse speed of your scooter up to the maximum speed you preset with the speed adjustment dial.

To use throttle control lever for models equipped with Tiller Console A:

To Move Forward, use either of the following:

- Use your left thumb to push the left side of the throttle control lever.
- Use your right hand fingers to pull back on the right side of the throttle control lever.

To Move Rearward, use either of the following:

- Use your right thumb to push the right side of the throttle control lever.
- Use your left hand fingers to pull back on the left side of the throttle control lever.

To use throttle control lever for models equipped with Tiller Console B:

- To move **Forward**, use your right thumb to push the right side of the throttle control lever.
- To move **Rearward**, use your left thumb to push the left side of the throttle control lever.

Release the throttle control lever and allow your scooter to come to a complete stop before engaging the other side of the lever. When the throttle control lever is completely released, it automatically returns to the center “stop” position and engages your scooter’s brakes.

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